



3M LBL 143 Littmann Core Stethoscope User Manual

[Home](#) » [3M](#) » 3M LBL 143 Littmann Core Stethoscope User Manual 

Contents

- [1 3M LBL 143 Littmann Core Stethoscope](#)
- [2 Product Usage Instructions](#)
- [3 Service and Support](#)
- [4 In the Box](#)
- [5 Notifications](#)
- [6 Recording](#)
- [7 Charging](#)
- [8 Service and Support](#)
- [9 Documents / Resources](#)
 - [9.1 References](#)
- [10 Related Posts](#)



3M LBL 143 Littmann Core Stethoscope



The CORE Stethoscope is a digital stethoscope made in the USA and assembled and distributed by 3M Company in collaboration with Eko Devices, Inc. The stethoscope comes with eartips, a micro USB cable, and a quick start guide.

Product Usage Instructions

Inserting Eartips

Insert the eartips in a forward direction.

Adjusting Volume

Use the (+) and (-) buttons to adjust volume. There are 7 volume steps.

Turning On and Off

When the CORE stethoscope is off, it functions as an unamplified stethoscope. To turn the CORE stethoscope on, press the power toggle in. The light will begin flashing. You will now hear amplified and filtered sound in the eartips.

Telemedicine and AI Features

Eko offers telemedicine and AI features that require a software subscription (where available, sold separately). You can setup a subscription online at ekohealth.com.

Active Noise Cancelling

The CORE stethoscope includes active noise cancelling to remove ambient noise from the surrounding environment. Active noise cancelling can be turned off or on through the Eko App.

Notifications

- **Blinking:** CORE stethoscope is on & seeking device
- **Yellow:** CORE stethoscope is on & connected
- **Yellow:** CORE stethoscope is off & charging
- **Green:** CORE stethoscope is fully charged
- **Blue:** CORE stethoscope is recording

Recording

When connected to the Eko App, press the (o) button to begin recording.

Charging

Connect the micro USB cable to the CORE Stethoscope and use a UL-certified or CE-marked 5V USB wall charger. The CORE stethoscope will take about 2 hours to fully charge. The light will turn green when it is fully charged.

User Manual

- Review the user manual before using your Littmann CORE
- Stethoscope. To view the manual visit: ekohealth.com/getstarted.
- For more information on 3M devices, see HCBGregulatory.3M.com.

Service and Support

In the USA, please visit littmann.com/service or call 1-800-228-3957. If you are outside of the USA, please visit littmann.com for the contact information of your local 3M office.


Download the Eko App from the Apple App Store® or Google Play Store®.
Turn on Bluetooth® on your phone or tablet, launch the app, and turn on the CORE stethoscope.

In the Box

- Eko CORE Digital Attachment and 3M™ Littmann® Cardiology IV™ Stethoscope
- Additional set of 3M Littmann Snap Tight Soft-Sealing Eartips (small)
- Nonchill Bell Sleeve
- USB Charging Cable
- Quick Start Guide

Eartips

Insert eartips in a forward direction.

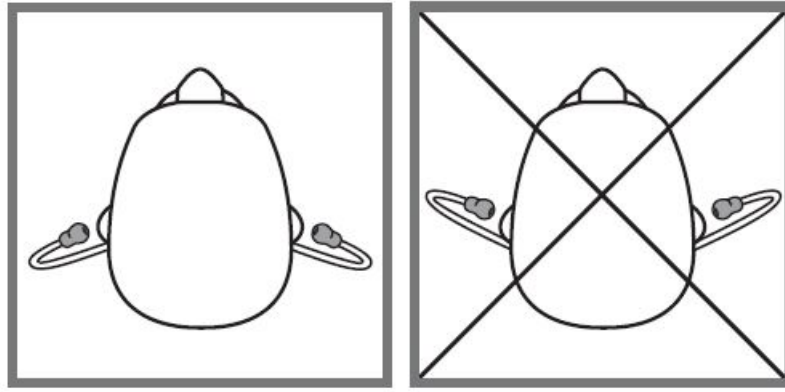

Volume

Use the (+) and (-) buttons to adjust volume. There are 7 volume steps.

Turn On and Off

- When the CORE stethoscope is off, it functions as an unamplified stethoscope.
- To turn the CORE stethoscope on, press the power toggle in. The light will begin flashing. You will now hear amplified and filtered sound in the eartips.

Telemedicine and AI Features

Eko offers telemedicine and AI features that require a software subscription (where available, sold separately). You can setup a subscription online at ekohealth.com

Active Noise Cancelling

The CORE stethoscope includes active noise cancelling to remove ambient noise from the surrounding environment. Active noise cancelling can be turned off or on through the Eko App.



Notifications

(🔊) (blinking)

CORE
stethoscope is
on & seeking
device

● (yellow)

CORE
stethoscope is
off & charging

(🔋) (yellow)

CORE
stethoscope is
low on battery

○

CORE
stethoscope is
on & connected

● (green)

CORE
stethoscope is
fully charged

(🔋) (blue)

CORE
stethoscope is
recording

Recording



When connected to the Eko App, press the (o) button to begin recording.

Charging

Connect the micro USB cable to the CORE Stethoscope and use a UL-certified or CE-marked 5V USB wall charger. The CORE stethoscope will take about 2 hours to fully charge. The light will turn green when it is fully charged

User Manual

Review the user manual before using your Littmann CORE Stethoscope. To view the manual visit:

ekohealth.com/getstarted

For more information on 3M devices, see HCBGregulatory.3M.com

Service and Support

In the USA, please visit littmann.com/service or call 1-800-228-3957. If you are outside of the USA, please visit littmann.com for the contact information of your local 3M office.

Apple App Store is a trademark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC. Bluetooth® is a registered trademark of Bluetooth SIG LBL 143 Rev A

Stethoscope Made in the USA

CORE Digital Attachment Made in Malaysia Assembled and distributed by 3M Company for -Eko Devices, Inc.
1212 Broadway, Suite 100 Oakland, CA 94612 USA 1-800-228-3957 (USA only) 1-844-356-3384

www.littmann.com

©2020 3M; Eko Devices Inc. All rights reserved.

3M, Littmann, the L Littmann logo, the L, and Cardiology IV are marks and/orV registered marks of 3M.


Unauthorized use is prohibited. Used under license in Canada.

Eko trademarks, including CORE, are marks and/or registered marks of Eko Devices Inc.





Issue Date: 2020-11

34-8726-5427-1

Documents / Resources

	<p>3M LBL 143 Littmann Core Stethoscope [pdf] User Manual LBL 143 Littmann Core Stethoscope, LBL 143, Littmann Core Stethoscope, Core Stethoscope, Stethoscope</p>
--	--

References

-  [Eko Digital Stethoscopes, AI Disease Detection & Telehealth](#)
-  [User Manuals – Help Center](#)
-  [eIFU Health Care](#)
-  [Littmann Stethoscopes | 3M United States](#)
-  [USA Warranty & Repairs for Littmann Stethoscopes | 3M](#)
-  [Littmann Stethoscopes | 3M United States](#)