



AbleNet 10000021 Hitch 2 Computer Switch Interface User Guide

[Home](#) » [AbleNet](#) » AbleNet 10000021 Hitch 2 Computer Switch Interface User Guide 

Contents

- [1 AbleNet 10000021 Hitch 2 Computer Switch Interface](#)
- [2 Product Registration](#)
- [3 Getting Started](#)
- [4 Documents / Resources](#)
 - [4.1 References](#)
- [5 Related Posts](#)

AbleNet[®]

AbleNet 10000021 Hitch 2 Computer Switch Interface



Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.



Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.



To get started:


1. Connect Hitch 2 to USB port on computer. One LED light on Hitch
2. will turn on when connected. Activate select mode button to select desired row of functions.
3. Attach switch(es) with 3.5-mm / 1/8-in TS (mono) plug or joystick with female D-sub 9 connector to Hitch 2.
4. Your Hitch 2 is now ready to use.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.

Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources. Scan the QR code below to download the free ableCARE app to your phone or tablet.



Documents / Resources

	AbleNet 10000021 Hitch 2 Computer Switch Interface [pdf] User Guide 10000021, Hitch 2 Computer Switch Interface
---	--

References

- [A Home - AbleNet](#)