



AbleNet 10000032 TalkingBrix 2 Multi-Message Speech Generating Device User Guide

[Home](#) » [AbleNet](#) » AbleNet 10000032 TalkingBrix 2 Multi-Message Speech Generating Device User Guide 

AbleNet[®]

10000032 TalkingBrix 2 Multi-Message Speech Generating Device User Guide

TalkingBrixTM 2

speech device

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details are available at www.ablenetinc.com.

AbleNet, Inc.

2625 Patton Road Roseville,

MN 55113

United States of America

(651) 294-3101

ablecare@ablenetinc.com

www.ablenetinc.com



Made with recycled content

Contents

- [1 Product Registration](#)
- [2 Getting Started](#)
- [3 Device Overview](#)
- [4 Documents / Resources](#)
 - [4.1 References](#)
- [5 Related Posts](#)

Product Registration

Registering your product provides you access to AppleCare, product updates, and resources for your product. Scan the QR code below to register your product.



<https://www.ablenetinc.com/product-registration/>

Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.



<https://ablenetinc.zendesk.com/hc/en-us/articles/360060500011>

To get started:

1. On the back of the device, move the switch to REC.
2. Press and hold colored switch top.
3. Begin speaking for up to 10-seconds when the light starts blinking.
4. Release colored switch top when finished.

5. On the back of the device, move the switch from REC to ON to start using.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.



ableCARE™

Download the AppleCare app to access live support from the AppleCare Product Success team, an online knowledge base filled with videos and getting started information, and other resources. Scan the QR code below to download the free AppleCare app to your phone or tablet.




Download on the
App Store



<https://apps.apple.com/us/app/ablecare/id1564779986?ign-mpt=uo%3D2>

<https://play.google.co>

Device Overview

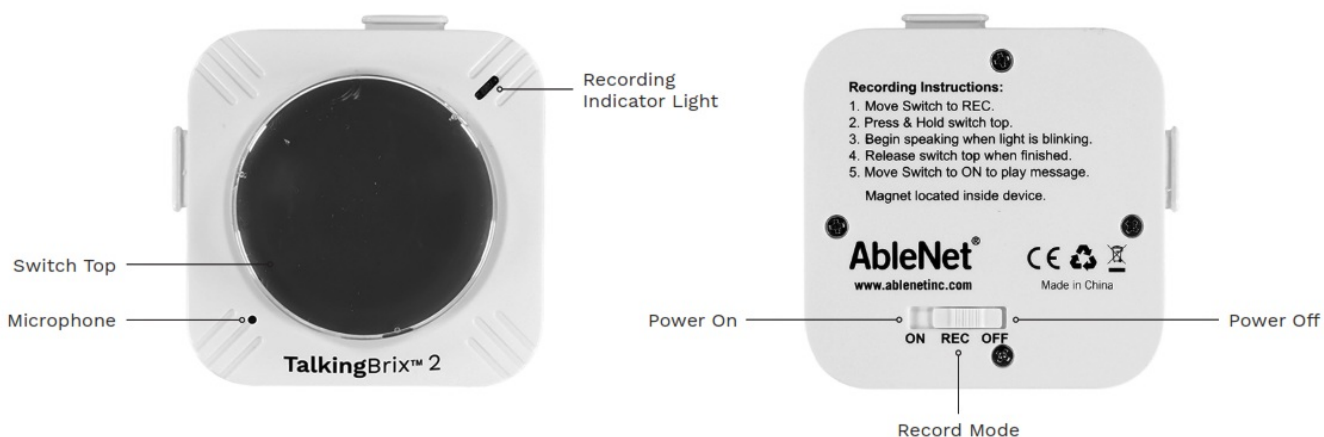


Create printable picture symbol overlays for your device with the FREE AbleNet Symbol Overlay Maker app. Scan the QR code to download today!



<https://apps.apple.com/us/app/symbol-overlay-maker/id1510255596>



Documents / Resources

	AbleNet 10000032 TalkingBrix 2 Multi-Message Speech Generating Device [pdf] User Guide e 10000032, TalkingBrix 2 Multi-Message Speech Generating Device
---	---

References

- [A Home - AbleNet](#)