

AbleNet[®] BIG TalkingBrix Speech Device



AbleNet BIG TalkingBrix Speech Device User Guide

[Home](#) » [AbleNet](#) » AbleNet BIG TalkingBrix Speech Device User Guide 

Contents

- [1 AbleNet BIG TalkingBrix Speech Device](#)
- [2 Product Information](#)
- [3 Product Usage Instructions](#)
- [4 FAQ](#)
- [5 To get started](#)
- [6 Device Overview](#)
- [7 Recording Instructions](#)
- [8 Documents / Resources](#)
 - [8.1 References](#)

AbleNet[®]

AbleNet BIG TalkingBrix Speech Device



Product Information

Specifications:

- Product Name: BIG TalkingBrix™ speech device
- Manufacturer: AbleNet, Inc.
- Warranty: 2-year limited warranty against defects in materials and manufacturing
- Website: www.ablenetinc.com

Product Usage Instructions

Product Registration

Register your product to access ableCARE, product updates, and resources. Scan the QR code provided for registration.

Getting Started

1. Move the switch on the back of the device to REC.
2. Press and hold the colored switch top.
3. Begin speaking for up to 10 seconds when the light starts blinking.

4. Release the colored switchtop when finished.
5. Move the switch from REC to ON on the back of the device to start using it.
6. Your device is now ready to use.

Device Overview

- Switch Top Microphone
- Recording Indicator Light
- Power On/Off switch
- Record Mode switch

This device supports more functionalities. Full instructions are available on the manufacturer's website. Download the ableCARE app for live support, videos, and resources.

FAQ

- **Q: How long is the warranty for the BIG TalkingBrix™ speech device?**
 - A: The device includes a 2-year limited warranty against defects in materials and manufacturing from the date of purchase.
- **Q: Where can I find the full instructions for use?**
 - A: You can find the full instructions for use on the AbleNet website at www.ablenetinc.com.
- **Q: How can I access live support for the product?**
 - A: Download the ableCARE app to access live support from the ableCARE Product Success team.

To get started

1. On back of the device, move the switch to REC.
2. Press and hold colored switch top.
3. Begin speaking for up to 10 seconds when light starts blinking.
4. Release colored switch top when finished.
5. On the back of the device, move the switch from REC to ON to start using it.
6. Your device is ready to use.

Device Overview



Recording Instructions

1. Move the switch to REC
2. Press and hold the switch top
3. Begin speaking when light is blinking
4. Release switch top when finished
5. Move the switch to ON to play the message Magnet located inside the device.

Documents / Resources

	AbleNet BIG TalkingBrix Speech Device [pdf] User Guide BIG TalkingBrix Speech Device, BIG TalkingBrix, Speech Device, Device
---	---

References

- [AbleNet, Inc.](#)
- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.