



ACCU-CHEK 3.15 ml Cartridge System Instruction Manual

[Home](#) » [ACCU-CHEK](#) » ACCU-CHEK 3.15 ml Cartridge System Instruction Manual 

ACCU-CHEK®

3.15 ml Cartridge System
Instruction Manual








Contents

[1 3.15 ml Cartridge System](#)


[2 Documents / Resources](#)

[2.1 References](#)

[3 Related Posts](#)

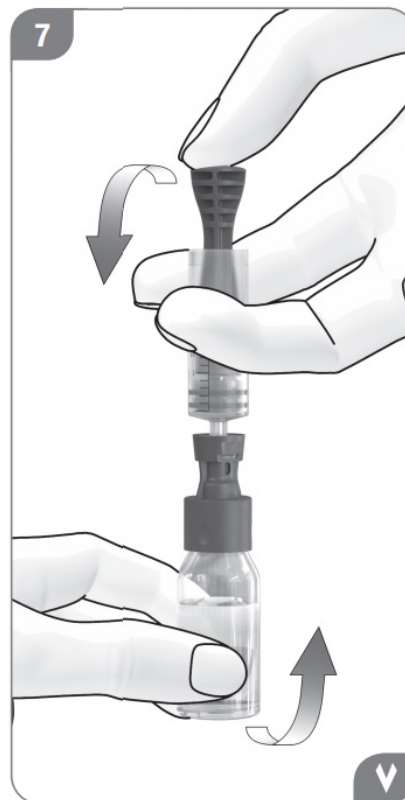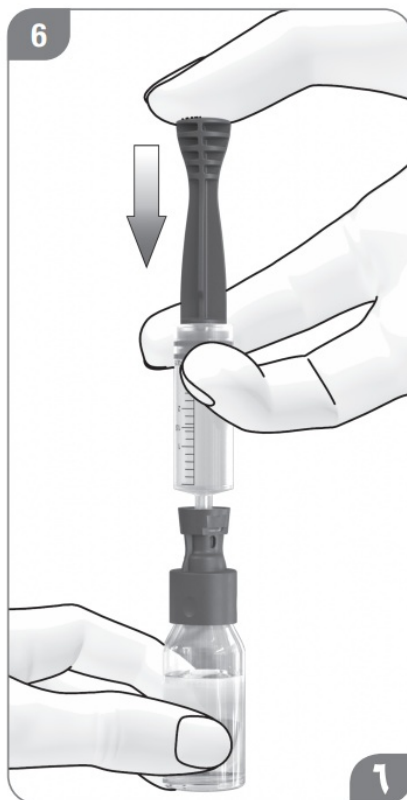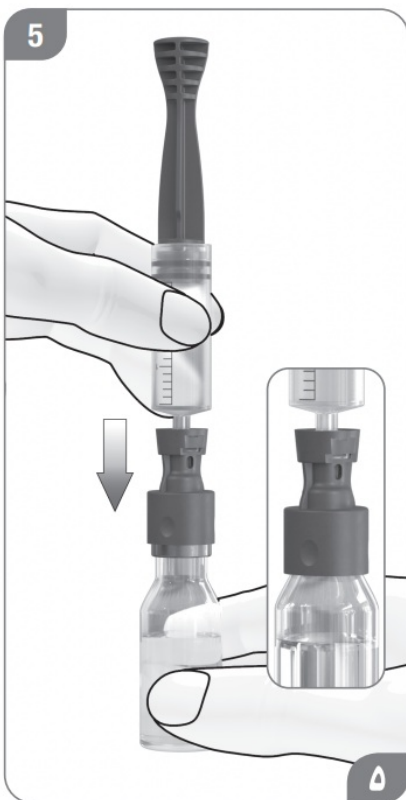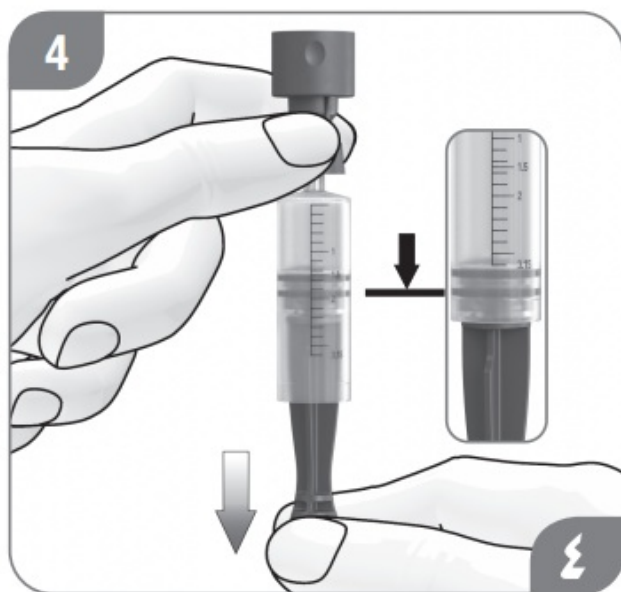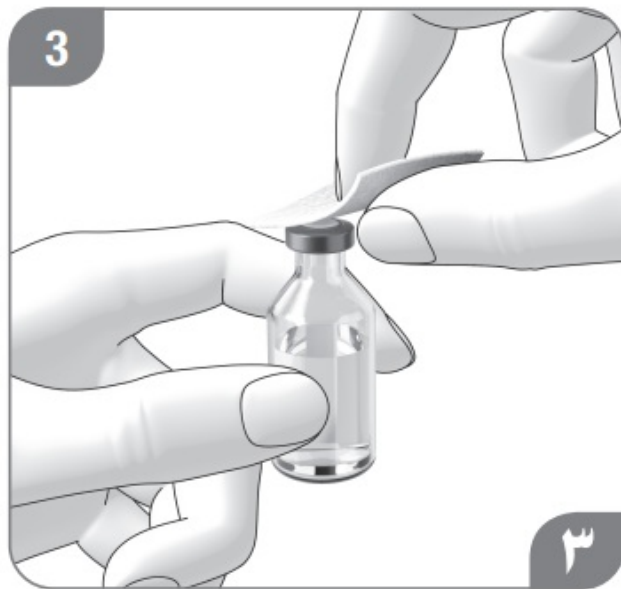


8



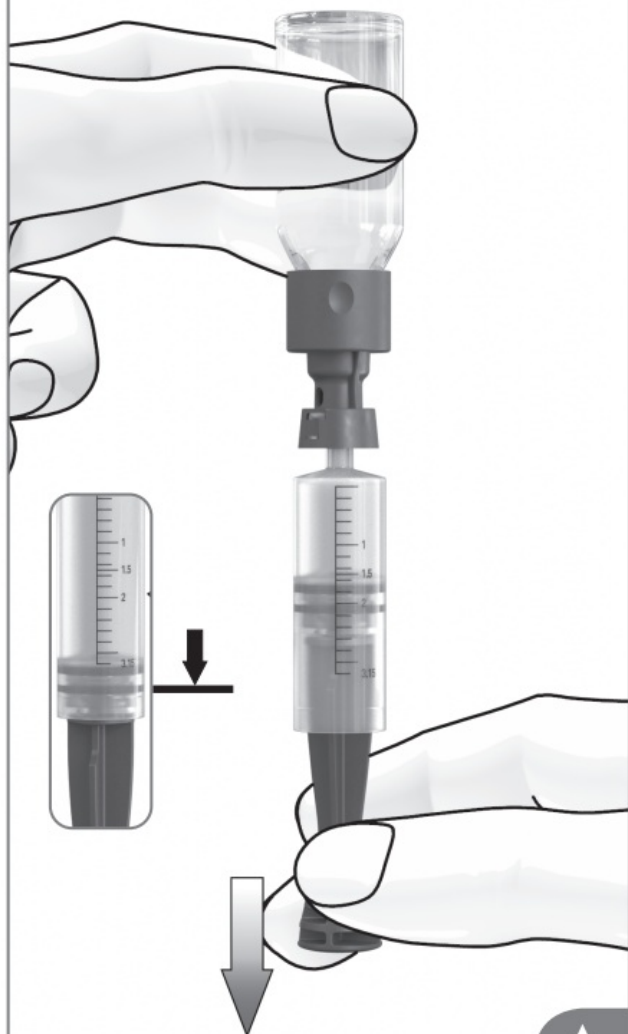


A

9

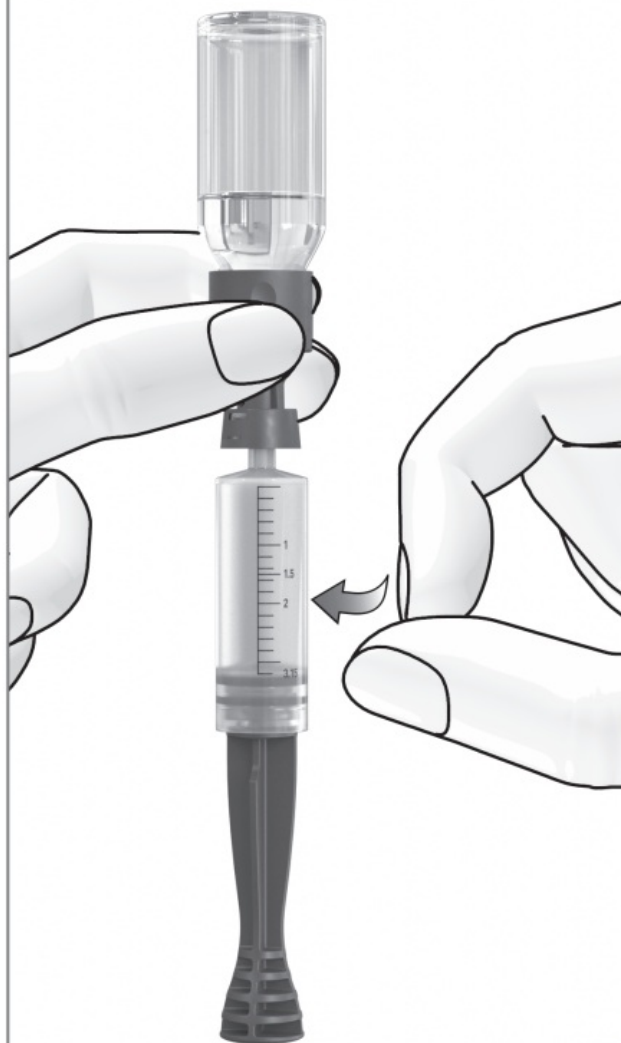

A

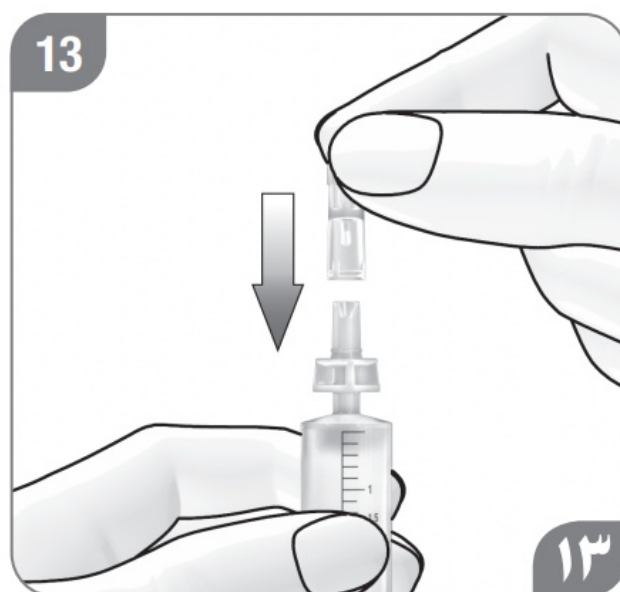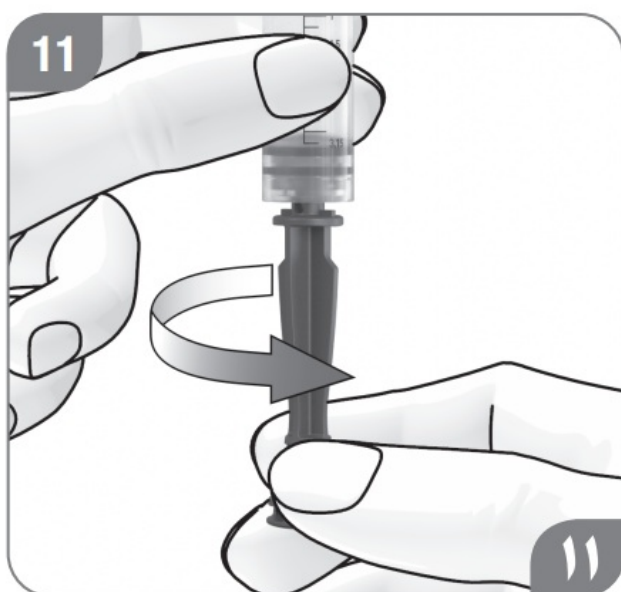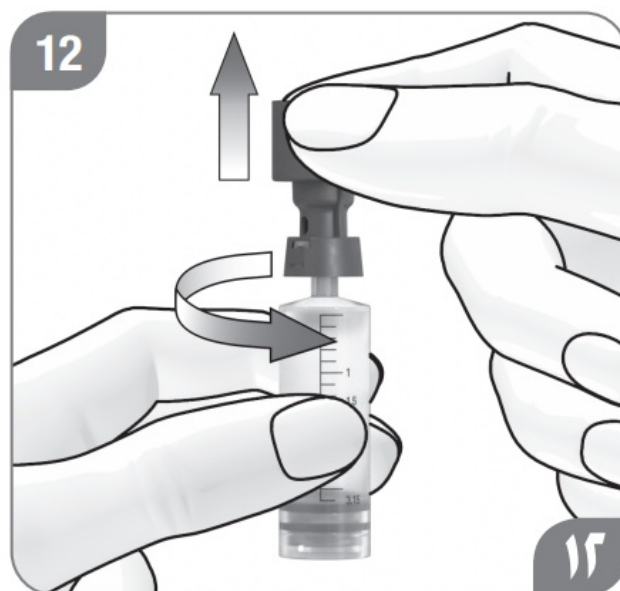

The Accu-Chek Spirit 3.15 ml cartridge system is an insulin reservoir for use with compatible Accu-Chek insulin pumps and Accu-Chek infusion sets with a standard luer-lock connector.

- Take a close look at these instructions for use before using the cartridge system. Also, read the User's Manual of the insulin pump and the instructions for use of the infusion set.
- The cartridge system contains small parts. There is a risk of suffocation if small parts are swallowed. Keep all parts of the cartridge system away from persons who may not be aware of the risks and may put small parts in their mouth, for example, small children.

Australia

Roche Diabetes Care Australia Pty. Limited

Insulin Pump Support: 1800 633 457

australia.insulinpumps@roche.com

www.accu-chek.com.au

Singapore

Accu-Chek ExtraCare line: 6272 9200

www.accu-chek.com.sg

Roche Diabetes Care South Africa (Pty) Ltd.

Hertford Office Park, 90 Bekker Road

Vorna Valley, 1686

South Africa

Tel: +27 (11) 504 4600

Email: info@accu-chek.co.za

www.accu-chek.co.za

Distributed in the United Kingdom by: Roche Diabetes Care Limited
Charles Avenue, Burgess Hill West Sussex, RH15 9RY

United Kingdom

Accu-Chek Pump Careline

1): UK Freephone number: 0800 731 22 91

ROI Freephone number: 1 800 88 23 51

1) calls may be recorded for training purposes

Some mobile operators may charge for calls to these numbers.

burgesshill.insulinpumps@roche.com

www.accu-chek.co.uk

www.accu-chek.ie

In US, distributed by:

Roche Diabetes Care, Inc.

Indianapolis, IN 46256, USA

Toll Free: 1-800-280-7801

Accu-Chek Customer Care Service Center: 1-800-688-4578

www.accu-chek.com

Canada

Accu-Chek Pump Support: 1-800-688-4578 (toll-free)

www.accu-chek.ca

	Caution, refer to safety-related notes in the instructions for use accompanying this product
	Do not use if the package is damaged
	Use only once
	Keep dry
	Keep away from sunlight
	Use by
	Sterilized using ethylene oxide
	Catalog number
	Batch code
	Global Trade Item Number
	Manufacturer
	This product fulfills the requirements of the European Directive 93/42/EEC on medical devices.

ACCU-CHEK®

Made in Poland

ACCU-CHEK, ACCU-CHEK SPIRIT,
ACCU-CHEK SPIRIT COMBO and AKKY-CHEK
are trademarks of Roche.

© 2021 Roche Diabetes Care
In the US distributed by:
Roche Diabetes Care Inc.
Indianapolis, IN 46256, USA
Toll Free 1-800-280-7801



Roche Diabetes Care GmbH
Sandhofer Strasse 116
68305 Mannheim, Germany
www.accu-chek.com



Documents / Resources



[ACCU-CHEK 3.15 ml Cartridge System](#) [pdf] Instruction Manual
3.15 ml Cartridge System, Cartridge System, System

References

- [CHECK.de - Kreditvergleich](#)
- [CHECK.de - Kreditvergleich](#)
- [Chek Personas - La única billetera que necesitas](#)
- [chek.co.uk | This domain is for sale!](#)
- [Chek Confecções : Chek Confecções](#)
- [Briella - Home](#)
- [Cronon GmbH - Professional IT Services IT-Services](#)
- [The domain name chek.fr is for sale](#)
- [chek.pl](#)
- [chek.ru](#)
- [מוצרים לחולי סוכרת | מגוון מוצרים לשמירה על רמת הסוכר - דין סטור](#)
- [Rochepumpa](#)
- [Surgitech](#)