

ACCU-CHEK Meter App User Guide

Home » ACCU-CHEK » ACCU-CHEK Meter App User Guide 12

Contents

- 1 ACCU-CHEK Meter App
- 2 How to connect your Accu-Chek meter to the mySugr app
- 3 Documents / Resources
 - 3.1 References

ACCU-CHEK®

ACCU-CHEK Meter App



The mySugr app automatically and wirelessly captures your readings so you get the full picture of what's affecting your blood glucose. No more paper logs.

Download the app here

It's easy. Scan the code with your phone to download, then use your email to create an account.



How to connect your Accu-Chek meter to the mySugr app









- 1. Make sure Bluetooth is turned on.
- 2. In the mySugr app, tap on "Connections" at the bottom of the screen.
- 3. Choose your device from the list.
- 4. Tap the Connect button to start the pairing.









- 5. Follow the on-screen instructions.
- 6. Find the PIN code displayed on your meter's screen.
- 7. When prompted, enter the PIN code to connect your device.
- 8. Success! Your device is now connected to the mySugr app.

Reference: 1. Debong F, Mayer H, and Kober J. Real-World Assessments of mySugr Mobile Health App. Diabetes Technol Ther. 2019;21(S2):S235–S240. MYSUGR, ACCU-CHEK and ACCU-CHEK GUIDE are registered trademarks of Roche. CA-1982

Documents / Resources



ACCU-CHEK Meter App [pdf] User Guide Meter App, Meter, App

References

User Manual

Manuals+, Privacy Policy

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.