



ACCU-CHEK Mobile Wireless Adapter Instruction Manual

[Home](#) » [ACCU-CHEK](#) » ACCU-CHEK Mobile Wireless Adapter Instruction Manual 

Contents

- 1 [ACCU-CHEK Mobile Wireless Adapter](#)
- 2 [Overview of wireless adapter](#)
- 3 [Attaching the wireless adapter](#)
- 4 [Pairing the wireless adapter](#)
- 5 [Transferring test results](#)
- 6 [Replacing the battery](#)
- 7 [Troubleshooting](#)
- 8 [Customer Support](#)
- 9 [Documents / Resources](#)
 - 9.1 [References](#)
- 10 [Related Posts](#)

ACCU-CHEK®


ACCU-CHEK Mobile Wireless Adapter




WARNING

- Choking hazard. Small parts. Keep away from children under the age of 3 years.
- Keep new and used batteries away from children. Ingestion or insertion into the body may cause chemical burns, perforation of soft tissues, and death. Severe burns may occur within 2 hours of swallowing. If you think a battery might have been swallowed or placed inside any part of the body, seek medical attention immediately.
- If the battery compartment does not close securely, stop using the product and keep it away from children. Contact Roche.


Overview of wireless adapter

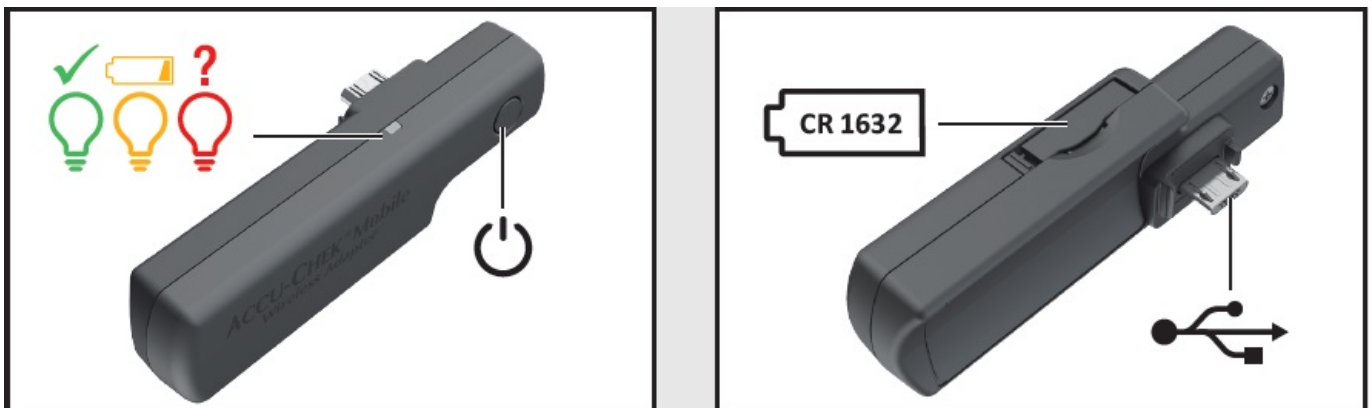



Attaching the wireless adapter

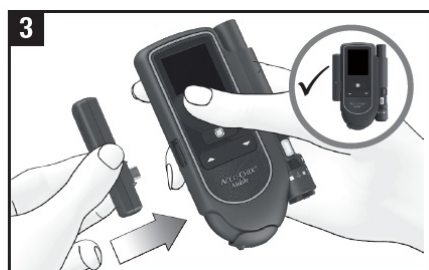



Pairing the wireless adapter

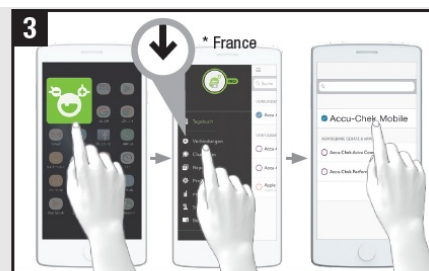

Transferring test results

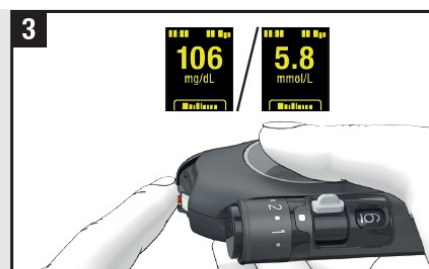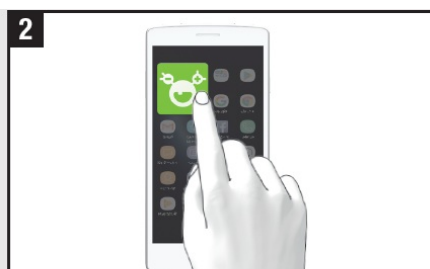

Replacing the battery

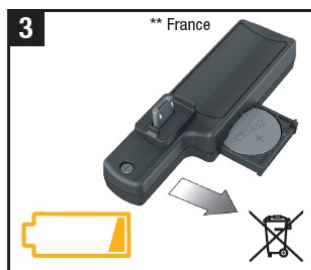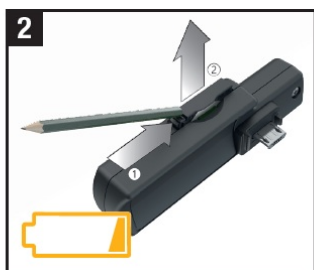

Troubleshooting



PRECAUTION

Used devices carry a risk of infection. Dispose of the used wireless adapter according to local regulations.

Customer Support

Belgium

- Tel: 0800-93626 (Roche Diabetes Service) www.accu-chek.be

Danmark

- Accu-Chek Kundeservice: Tlf. 80 82 84 71 www.accu-chek.dk

France

- Service après-vente : Numéro vert : 0800 27 26 93 www.accu-chek.fr

Finland

- Kundtjänsttelefon: 0800 92066 (kostnadsfri) www.accu-chek.fi

Italia

- Servizio Assistenza 800 822 189 www.accu-chek.it

Nederland

- Roche Diabetes Service Tel. 0800-022 05 85 www.accu-chek.nl

Norge

- Accu-Chek Kundesenter: 21 400 100

Sverige

- Accu-Chek Kundsupport:

- Telefon: 020-41 00 42
- E-post: info@accu-chek.se www.accu-chek.se

Switzerland

- Accu-Chek Kundenservice 0800 803 303 gebührenfrei
- Service clientèle Accu-Chek 0800 803 303 appel gratuit
- Servizio clienti Accu-Chek 0800 803 303 gratuito www.accu-chek.ch

Suomi

- Asiakaspalvelupuhelin: 0800 92066 (maksuton) www.accu-chek.fi

ACCU-CHEK and ACCU-CHEK MOBILE are trademarks of Roche

- The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Roche is under license.
- All other product names and trademarks are the property of their respective owners.
- © 2022 Roche Diabetes Care
- M Roche Diabetes Care GmbH
- Sandhofer Strasse 116
- 68305 Mannheim, Germany www.accu-chek.com

Documents / Resources

	ACCU-CHEK Mobile Wireless Adapter [pdf] Instruction Manual Mobile Wireless Adapter, Mobile, Wireless Adapter, Adapter
---	--

References

- [Willkommen bei Accu-Chek](#)
- [Accu-Chek® |](#)
- [Willkommen bei Accu-Chek | Switzerland](#)
- [Accu-Chek Diabetes Care Products Home | Accu-Chek](#)
- [Accu-Chek | Diabetes Management und Blutzuckerselbstkontrolle](#)
- [Velkommen til Min Accu-Chek | Danmark](#)
- [Tervetuloa Accu-Chek Clubiin](#)
- [Bienvenue sur Accu-Chek | France](#)
- [Benvenuti a Accu-Chek | Italy](#)

- [!\[\]\(2dc8cdc0c918df88cde61039ecf68682_img.jpg\) Welkom bij Accu-Chek | Nederland](#)
- [!\[\]\(793119bf0d613bd9b598fb8668922511_img.jpg\) Velkommen til Accu-Chek® | Accu-Chek](#)
- [!\[\]\(0a4819029e810ca9d2aba79260b63a4d_img.jpg\) Welcome to Accu-Chek | Accu-Chek Sweden](#)

Manuals+.