



---

[Home](#) » [adaptdefy](#) » **Adaptdefy LapStacker Flex Wheel Chair Installation Guide** 

## **Contents** [ [hide](#) ]

- [1 Adaptdefy LapStacker Flex Wheel Chair](#)
- [2 Specifications](#)
- [3 COMPONENTS](#)
- [4 HORIZONTAL INSTALLATION](#)
- [5 Installation instruction](#)
- [6 FAQs](#)
- [7 Documents / Resources](#)
  - [7.1 References](#)



## **Adaptdefy LapStacker Flex Wheel Chair**



## Specifications

- **Product Name:** LapStacker
- **Designed for:** Wheelchair use
- **Usage:** Aid for carrying items
- **Not Suitable for:** Securing people or animals

Congratulations and thank you for choosing to install a LapStacker on your wheelchair. Please read this installation guide carefully to ensure your LapStacker is installed and used correctly and safely.

By using your LapStacker you agree to the terms of our Warranty and Product Disclaimer which can be read online at <https://adaptdefy.com/pages/lapstacker-warranty-disclaimer>

## Warnings – Please read carefully

### Warnings – General

- LapStacker is not a seatbelt, safety device, or medical device.
- LapStacker is designed as an aid only.

- Do not use your LapStacker to secure people or animals.
- Carefully place and monitor items as they can still move, even when secured.
- Do not exceed any wheelchair manufacturer's guidelines for loading or attachment use.
- Do not apply excessive force to the LapStacker or its components. It is not designed to support body weight.
- Do not release the LapStacker straps and allow them to retract without control, as this may cause damage over time.
- In no event shall our company be liable for any direct, indirect, punitive, incidental, special, or consequential damages to property or life resulting from the use or misuse of our products.
- Do not lift or move the wheelchair by the LapStacker straps, as this may cause damage.

### **Warnings – Maintenance**

- Clean components using a damp cloth only. Avoid using chemical cleaners on plastic surfaces.
- To clean the straps, use water and a cloth. Fully extend the straps to dry, and ensure they are completely dry before retracting.

### **Warnings – Injury**

- Do not use your LapStacker to secure items that could cause injury, such as sharp, hot or heavy objects.
- Over-tightening the LapStacker straps may cause loss of circulation, pressure injury, or damage to items being carried.
- Avoid contact with your LapStacker during transfers to prevent injury or damage.
- Avoid extended use, which may cause loss of circulation or pressure injury.
- Do not stack items too high, as this may limit visibility and increase the risk of collision or injury.


### **Warnings – Installation**


- If moving existing wheelchair components (e.g. armrests, leg restraints), ensure they

are reinstalled securely to avoid failure and injury.

- The LapStacker Retractor is not designed for submersion. Water damage may void the Warranty.
- LapStacker is not designed for use in environments below -15°C (5°F) or above 45°C (113°F). Use outside this range may cause damage and may void the warranty.
- Do not over-tighten the tension straps during installation, as excessive force may damage the components and will void the warranty.

## COMPONENTS

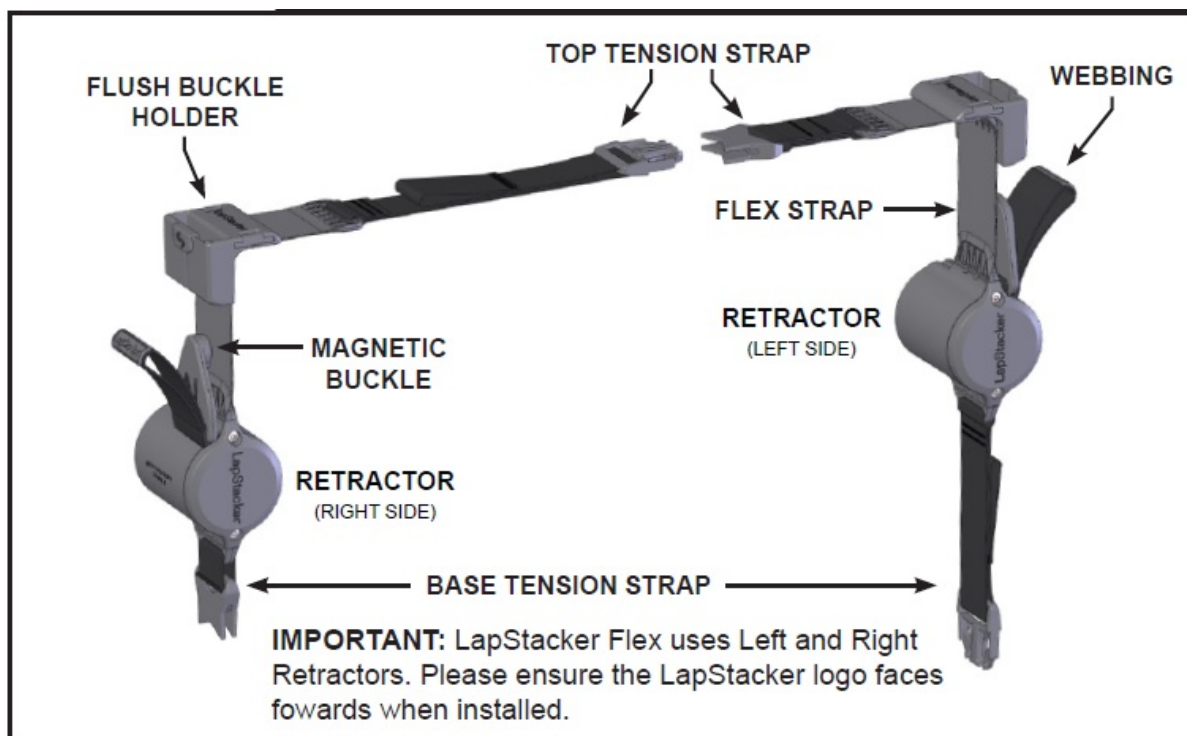

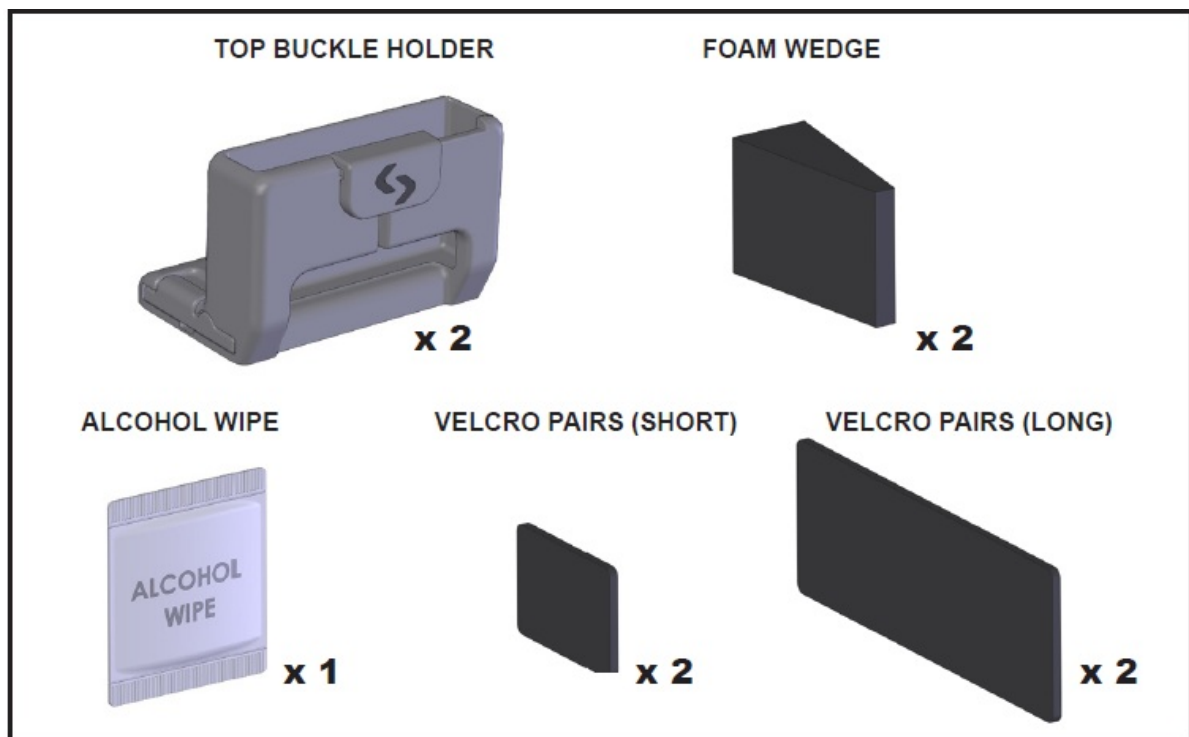

## 1. FIND OPTIMAL LOCATION

- LapStacker is best installed inline with the centre of where you will carry items, but will still work if forward or back of this.
- Locate this central point on your wheelchairs top tube.



## 2. SELECT INSTALL ORIENTATION

### **RIGID CHAIRS**

Single tube frames

HORIZONTAL INSTALL



Dual, non parallel tube frames

HORIZONTAL INSTALL



## **FOLDING CHAIRS**

### **FOLDING CHAIRS**


Single tube frames

HORIZONTAL INSTALL



Parallel dual tube frames

VERTICAL INSTALL

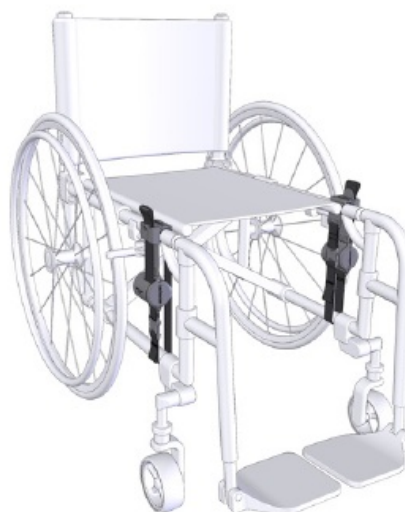

## **3. CHECK CLEARANCE – HORIZONTAL**

### **HOW IT WORKS**

The horizontal install works by fastening LapStacker across the chair frame. If you have a raising seat base, the top tension strap must pass under this.


### **TEST FIT:**

Check that your wheelchair components will not clash with LapStacker. It is possible to adjust some wheelchair components if clearance is an issue.

### **CLEARANCE CHECKLIST**

- Back of your legs

- Brakes
- Cross Bar
- Cushion
- Wheelguards
- Folding Mechanisms

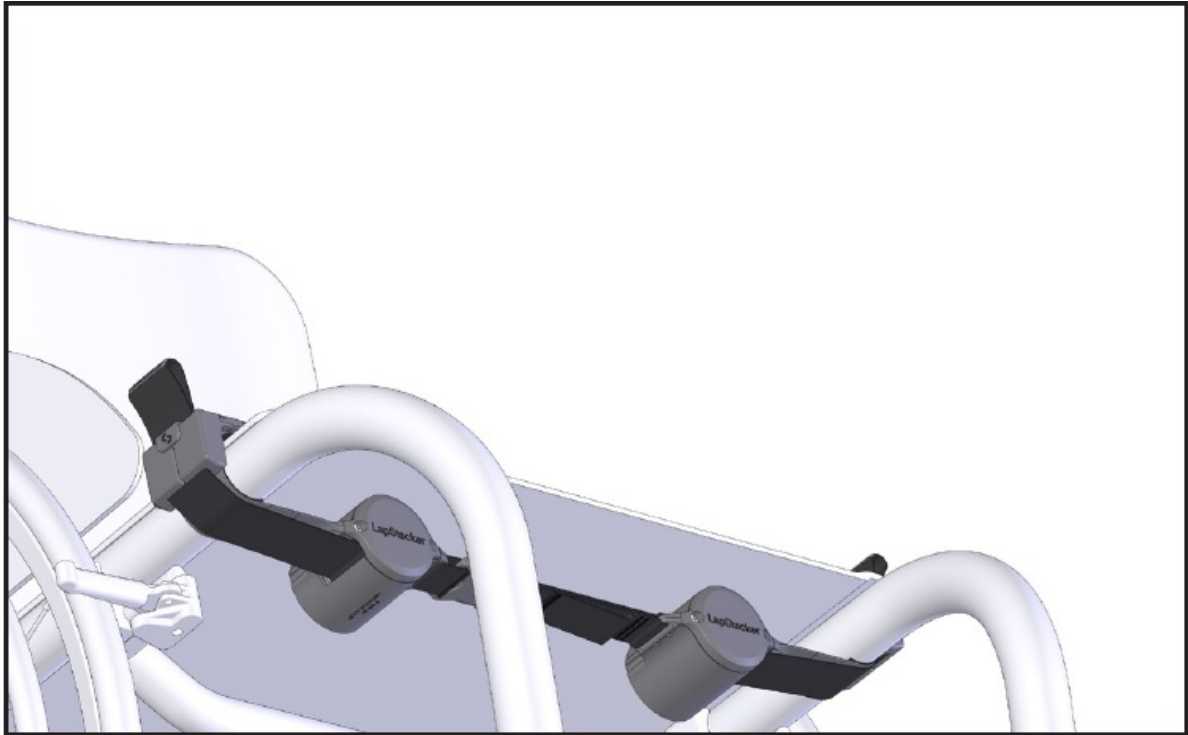

BUCKLE HOLDER NOT FITTING? GO TO PAGE 11.

## **VERTICAL**

### **HOW IT WORKS**

The vertical install works by fastening each half of LapStacker around the top and lower frame tubes.

### **TEST FIT:**


Check that your wheelchair components will not clash with LapStacker. It is possible to adjust some wheelchair components if clearance is an issue.

### **CLEARANCE CHECKLIST**

- Back of your legs
- Brakes
- Seat Frame



- Folding Mechanisms
- Cushion
- Wheelguards

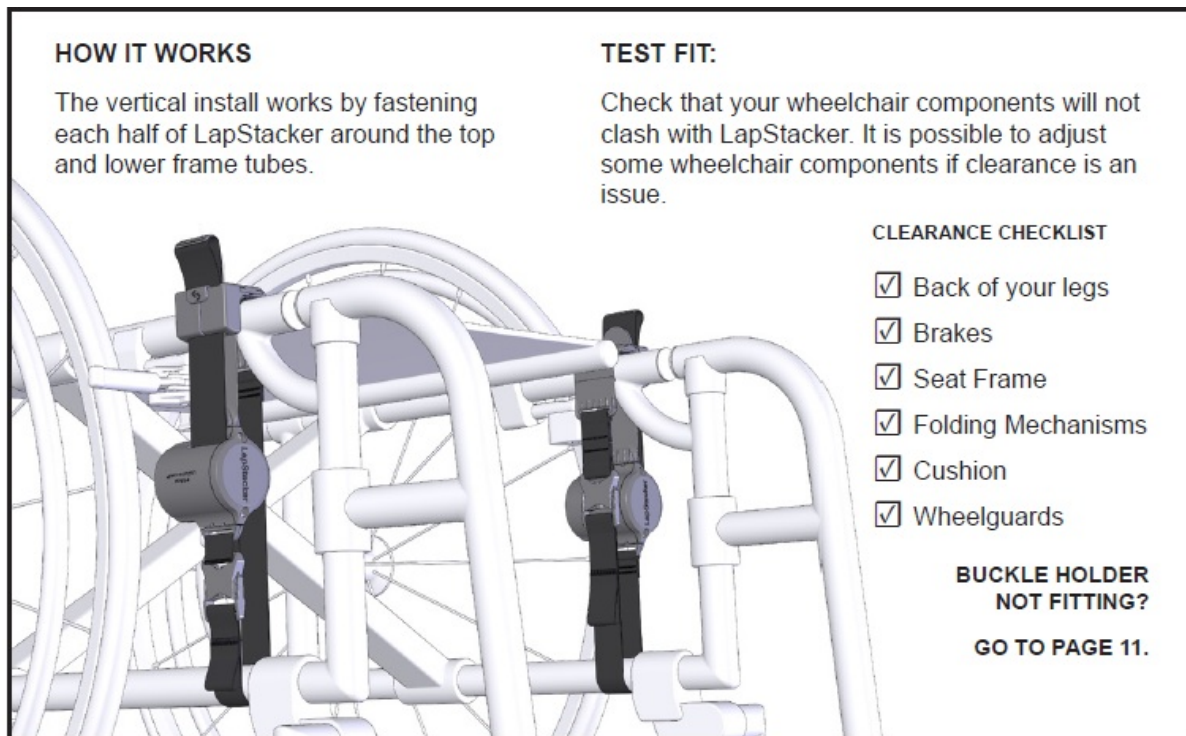

BUCKLE HOLDER NOT FITTING? GO TO PAGE 11.

## SELECT BUCKLE HOLDER

### FLUSH BUCKLE HOLDER

The Flush Buckle Holder comes installed as standard with your LapStacker and will suit 80% of installs. If clearance is an issue, you may consider using the Top Buckle Holder.




### **FLUSH BUCKLE HOLDER**

The Flush Buckle Holder comes installed as standard with your LapStacker and will suit 80% of installs.

If clearance is an issue, you may consider using the Top Buckle Holder.

#### **IMPORTANT**

The buckle holders are best rotated outwards slightly for clearance from cushion and to reduce friction.

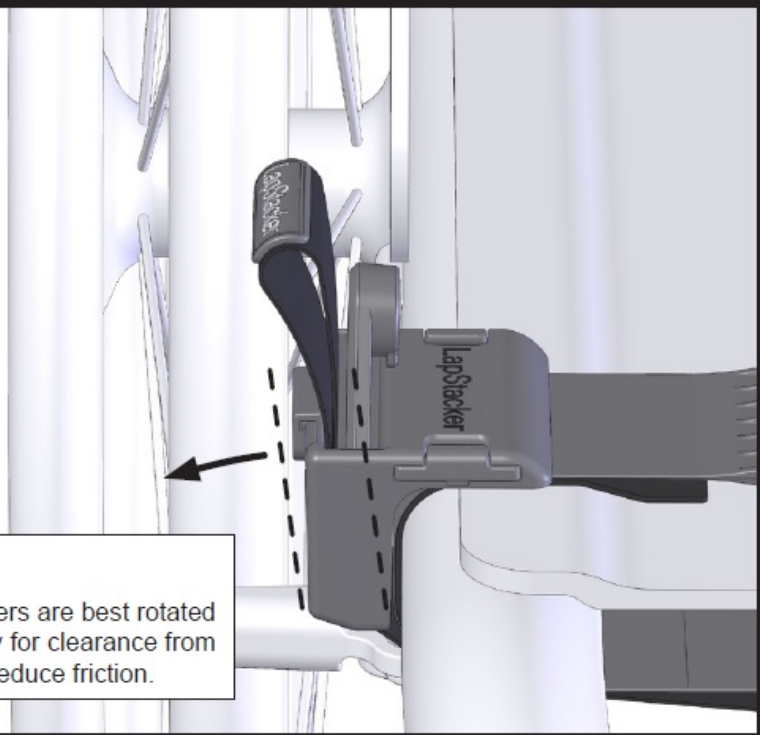


### **IMPORTANT**

The buckle holders are best rotated outwards slightly for clearance from cushion and to reduce friction.

### **TOP BUCKLE HOLDER**


- The Top Buckle Holder sits above the frame top tube and has three adjustment positions to shift the keeper left to right.
- The adjustment can be used to create clearance from your wheel guards, brakes, seat frame or cushion etc.

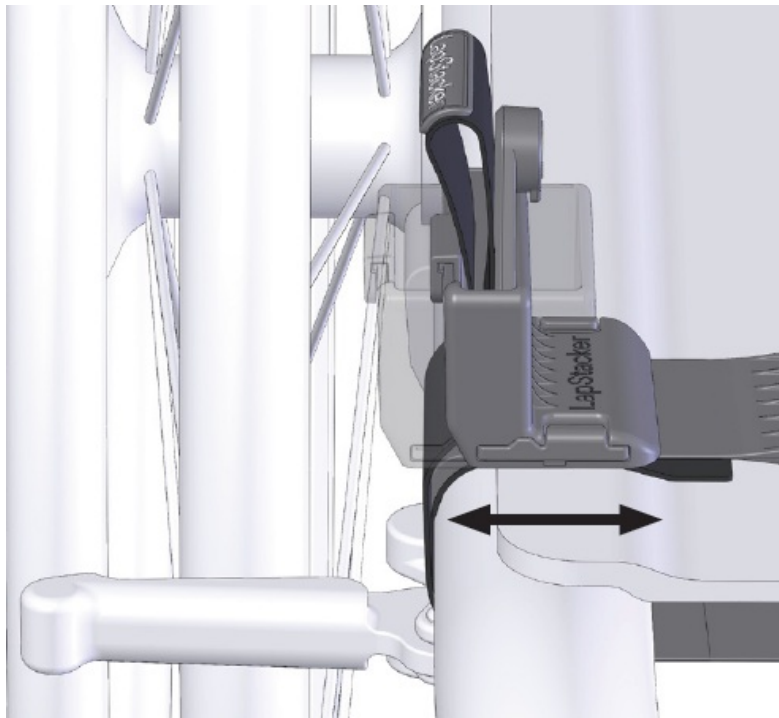

## TOP BUCKLE HOLDER (OPTIONAL INSTALLATION)


### 1. REMOVE FLUSH BUCKLE HOLDER

Unlock the Flush Buckle Holder by pulling both locking tabs out.

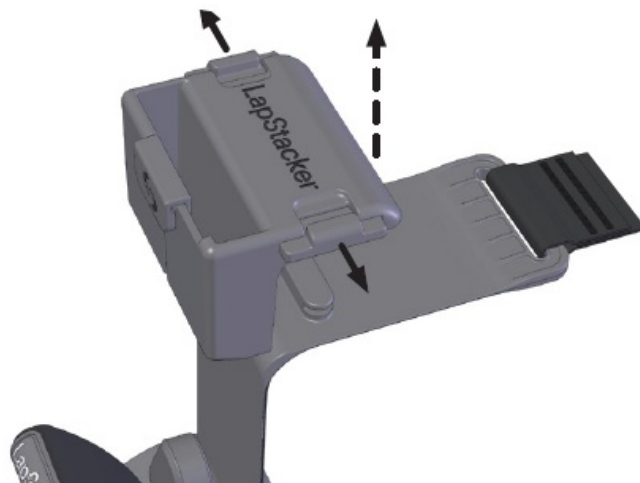


### 2. INSTALL TOP BUCKLE HOLDER

With both locking tabs open, place the Top Buckle Holder onto the Flex Strap in the desired position. Lock in place by pressing in both locking tabs.

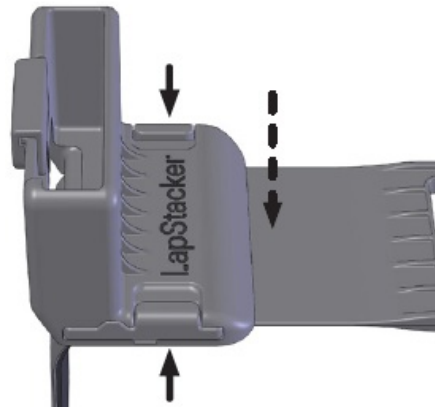


## HORIZONTAL INSTALLATION

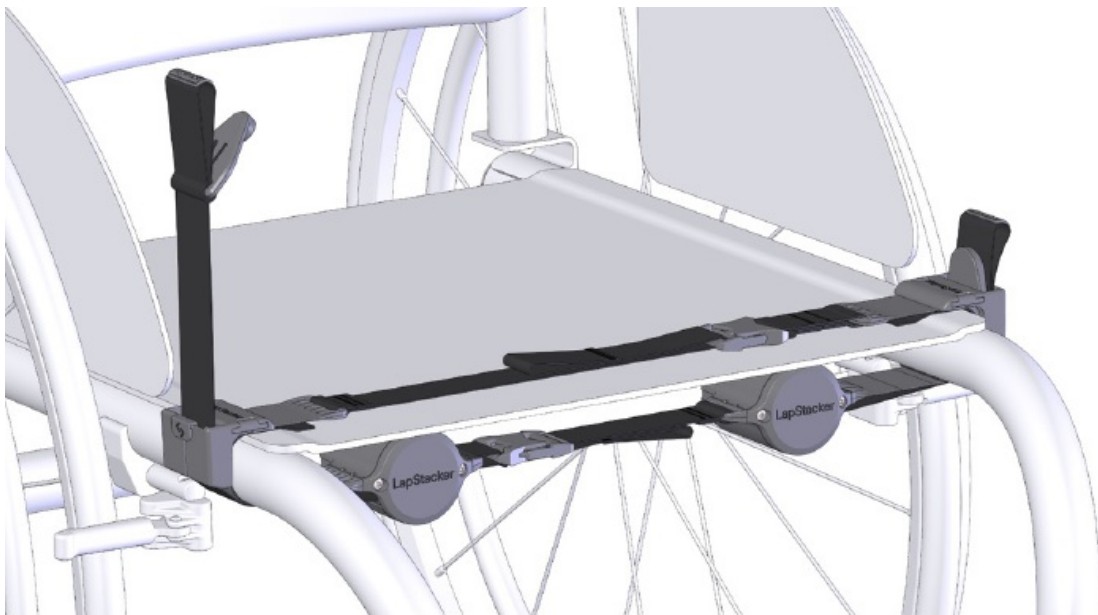

The wheelchairs depicted are for illustration purposes only. These instructions will work for whatever chair type you own.


## HORIZONTAL INSTALLATION

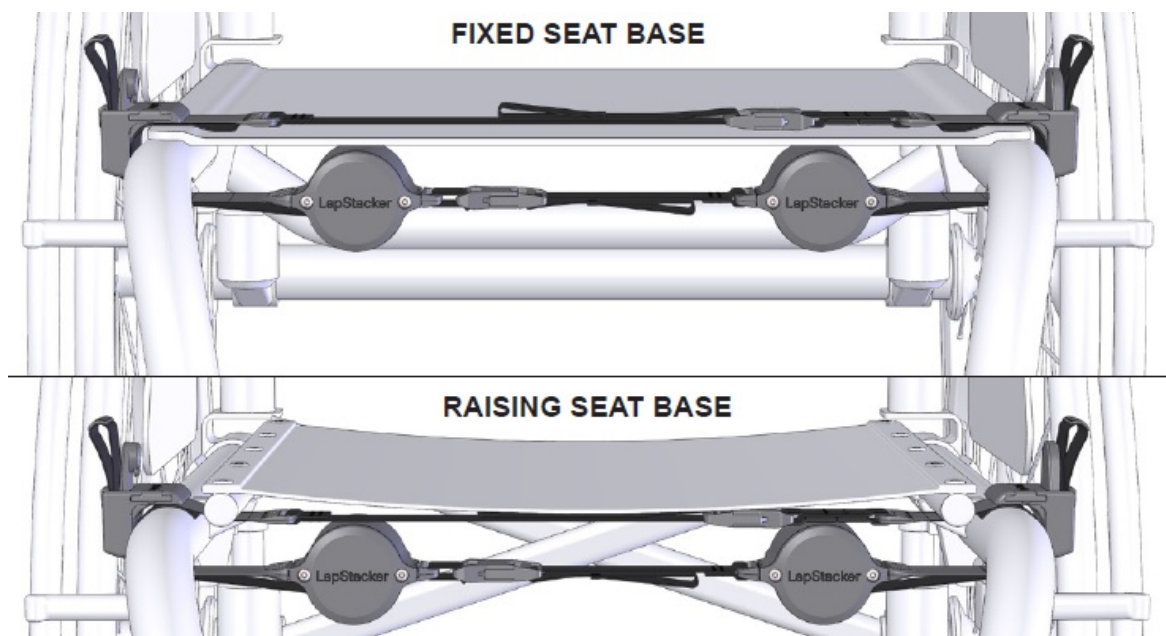


If you have a raising seat base, the LapStacker must pass under the seat base. Adapt install steps to suit.

## Installation instruction


1. Clean the frame and seat base where LapStacker will be located using the alcohol wipes provided.

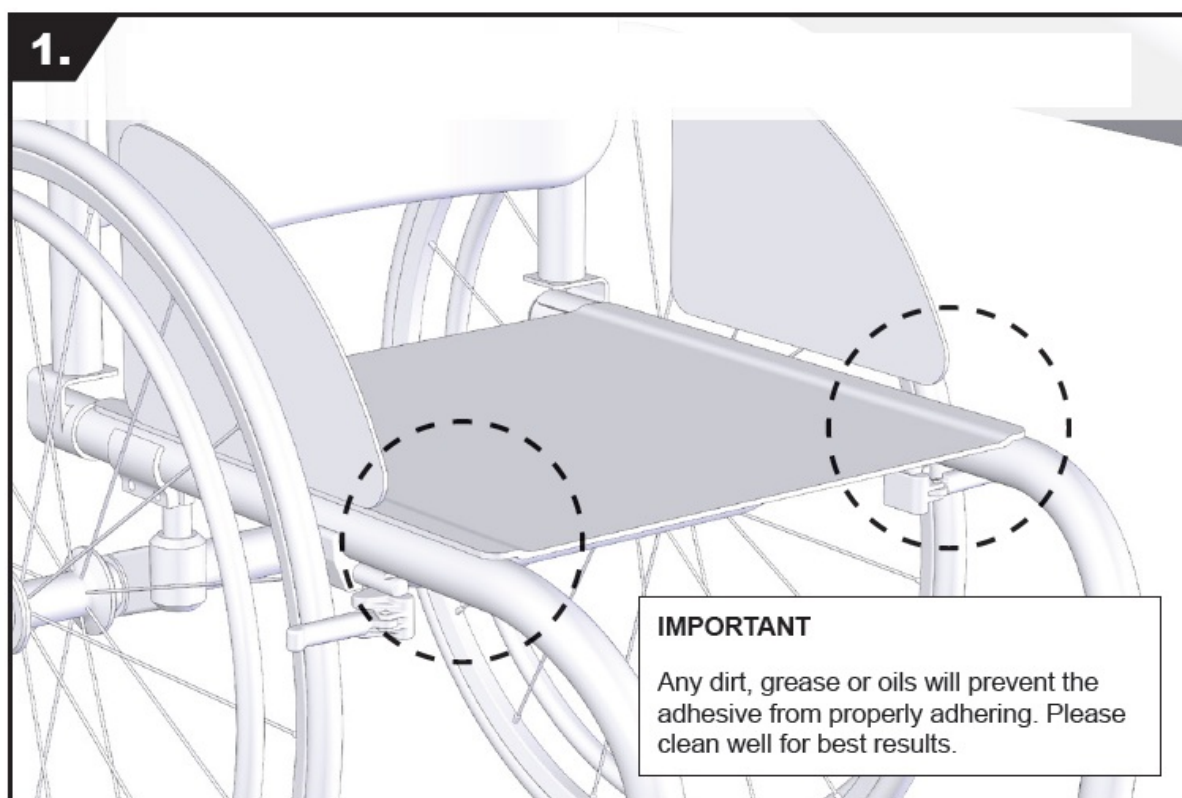

2. Adhere velcro to the frame and seat base where the Flex Straps will make contact. Repeat on each side.

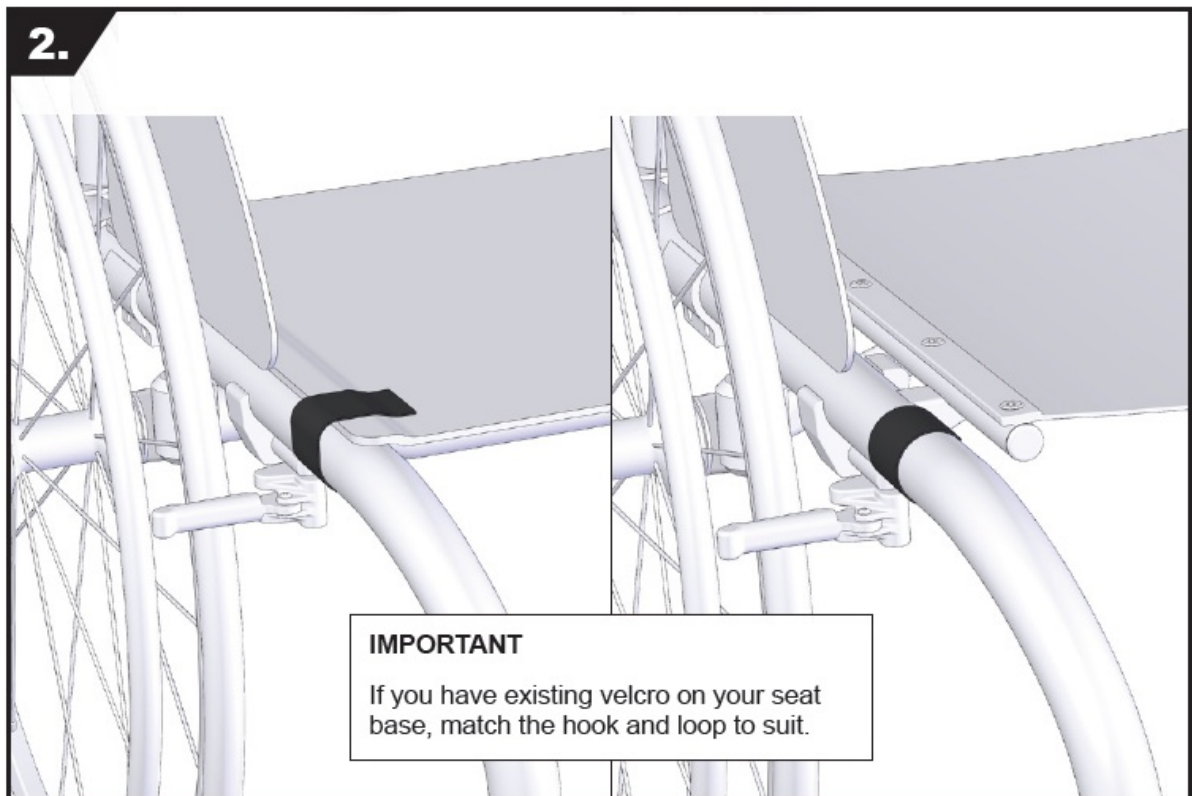


3. Adhere velcro to both LapStacker Flex Straps.

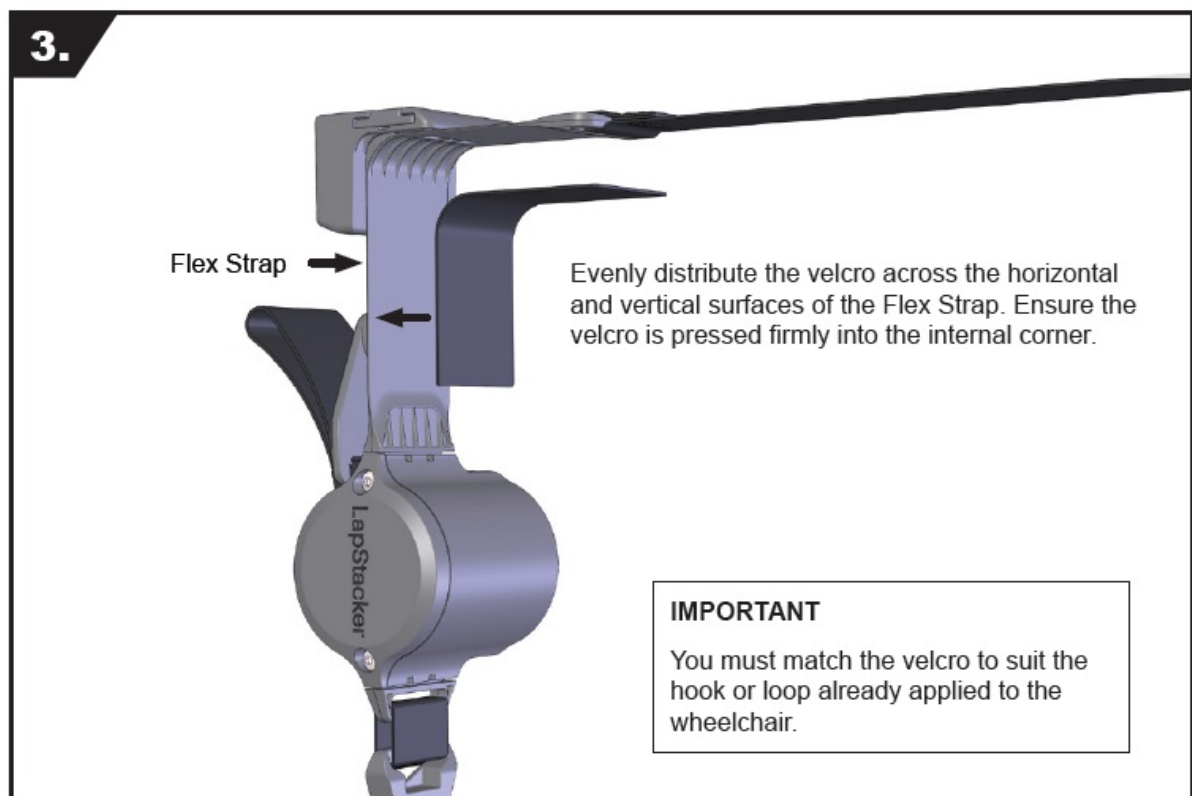


If the frame is not parallel at the optimal position, use the foam wedges to correct the angle of the Buckle Holders.




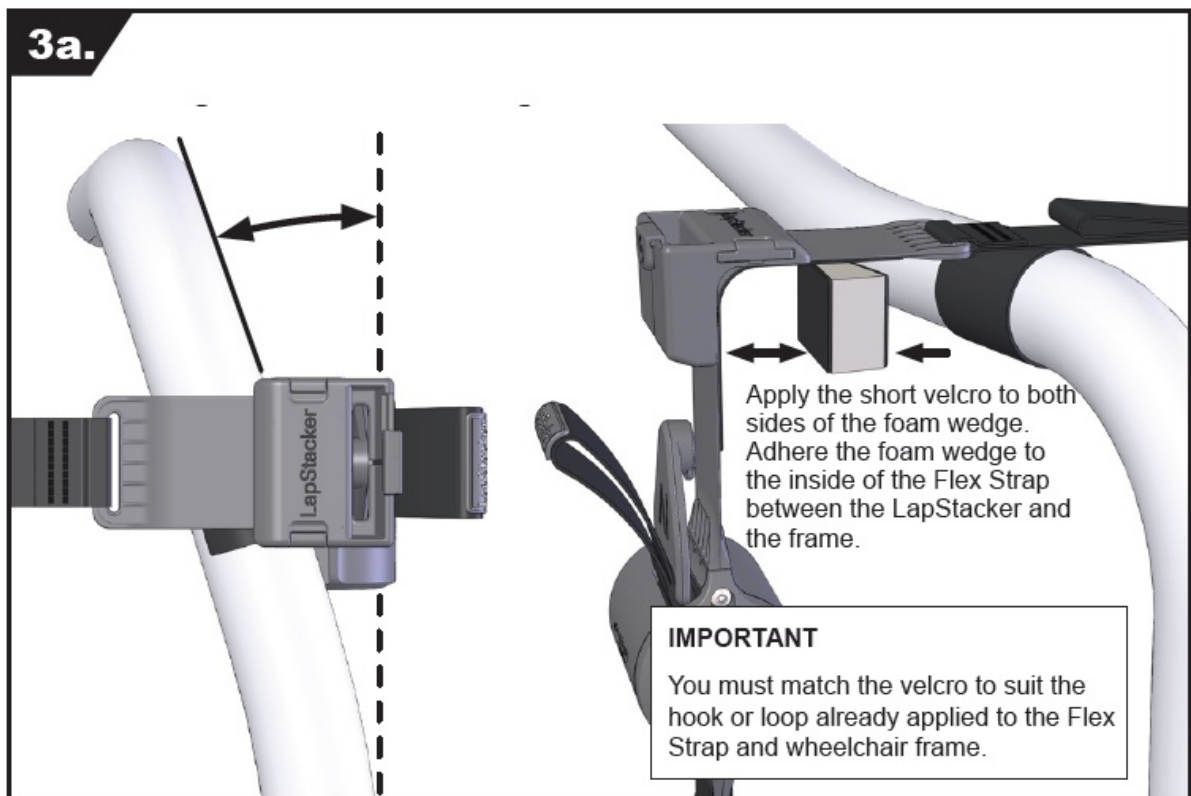


4. Attach LapStacker assembly to the frame.

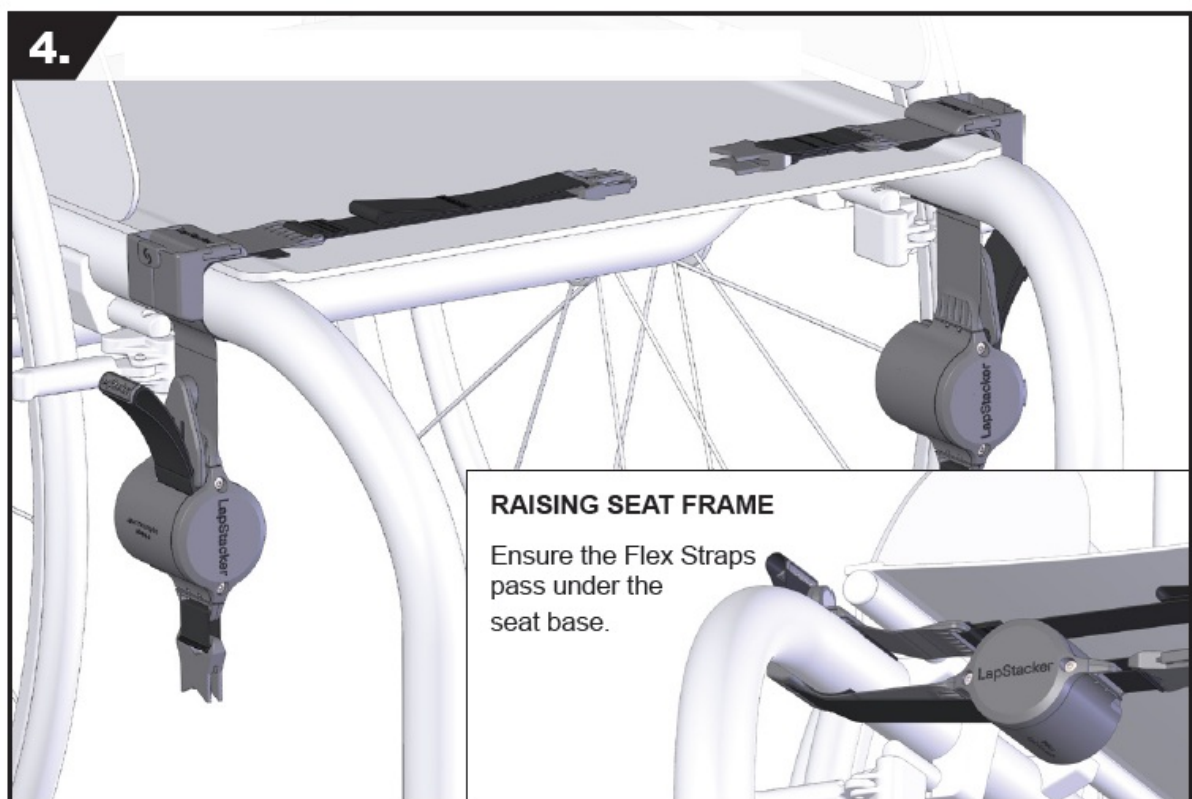

5. Clip the Base Tension Strap together and tighten gently.

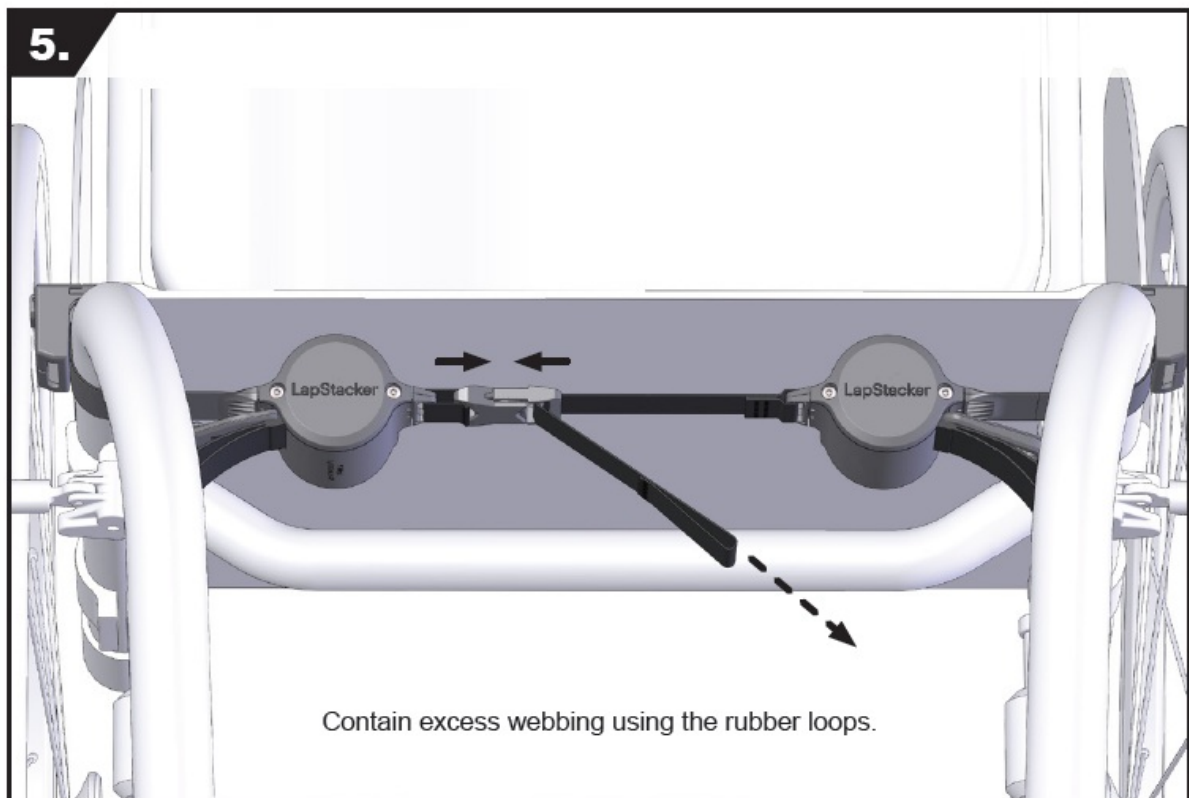


6. Clip the Top Tension Strap together and tighten gently.

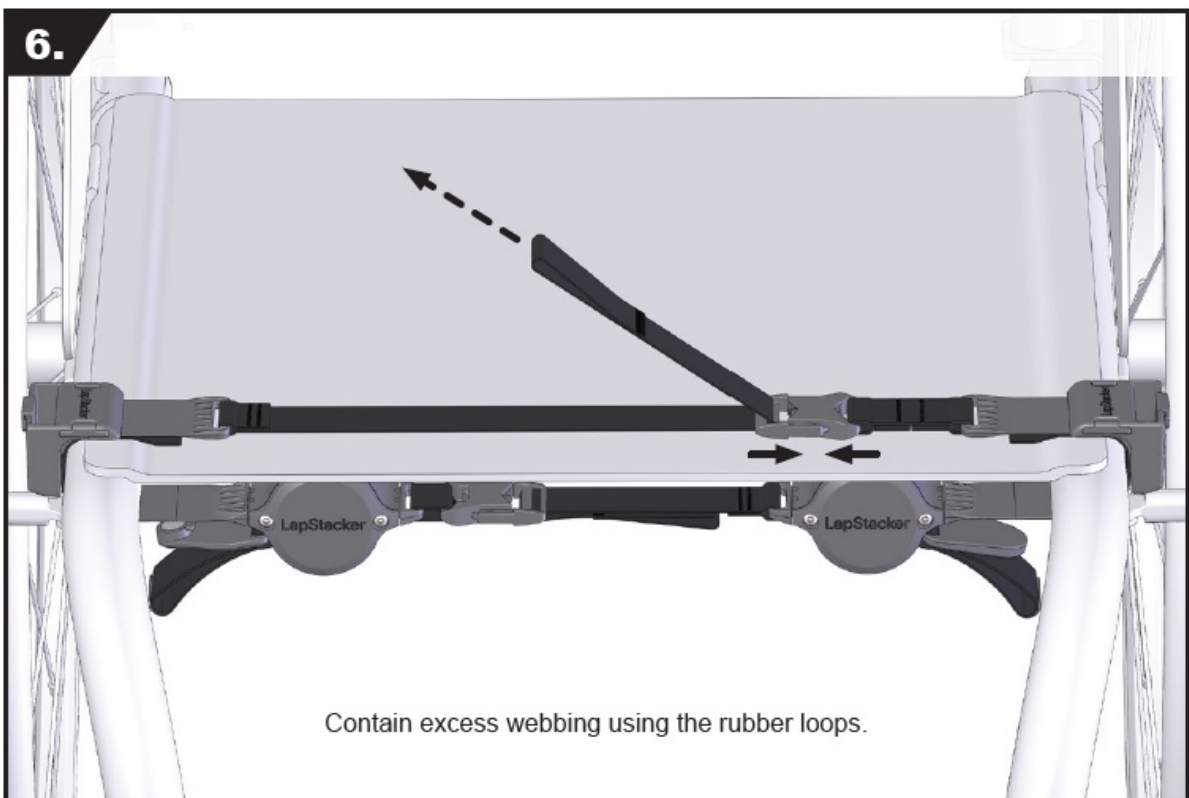


7. Pass the webbing through the Buckle Holder. Repeat for each side.



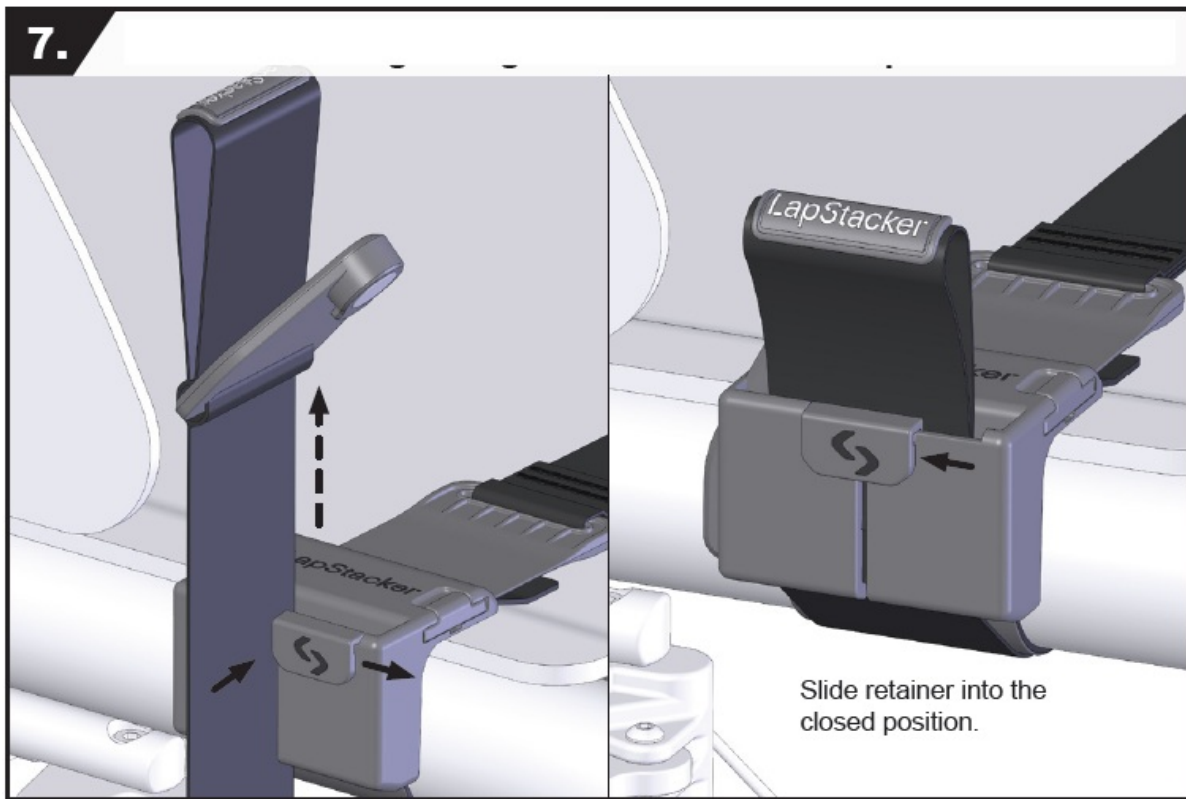


## VERTICAL INSTALLATION

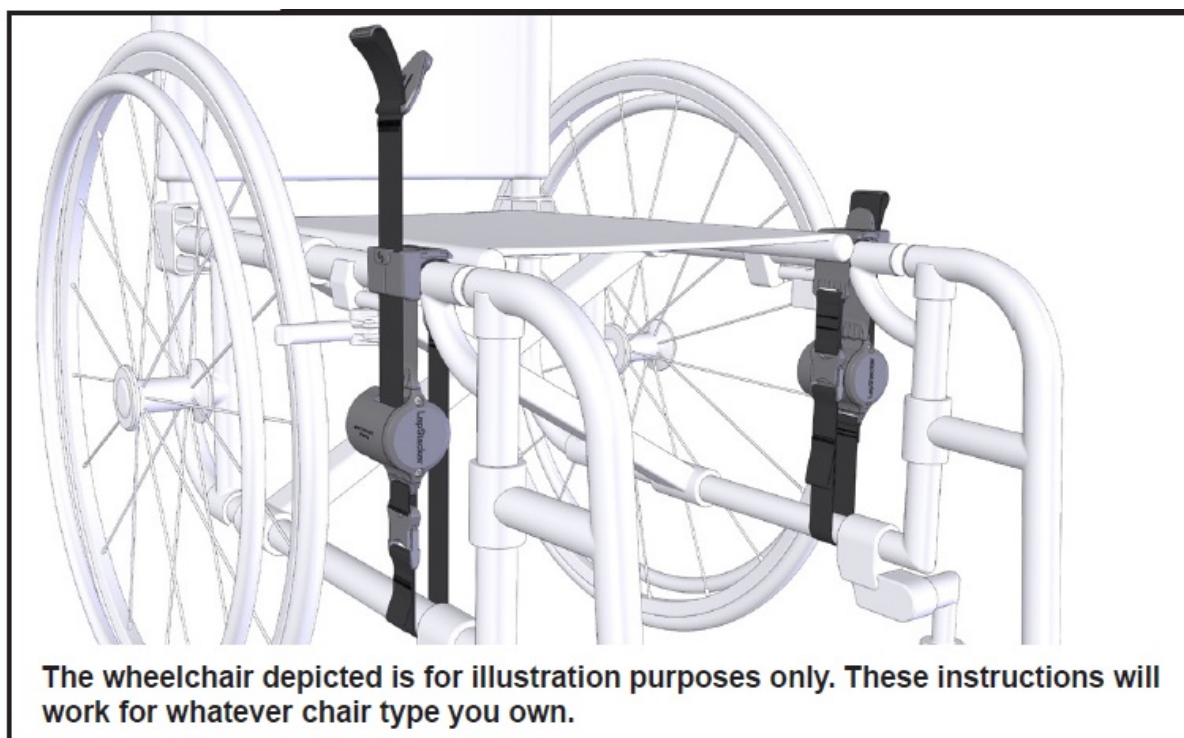


The wheelchair depicted is for illustration purposes only. These instructions will work for whatever chair type you own.

1. Using the alcohol wipes clean the frame where LapStacker will be located.

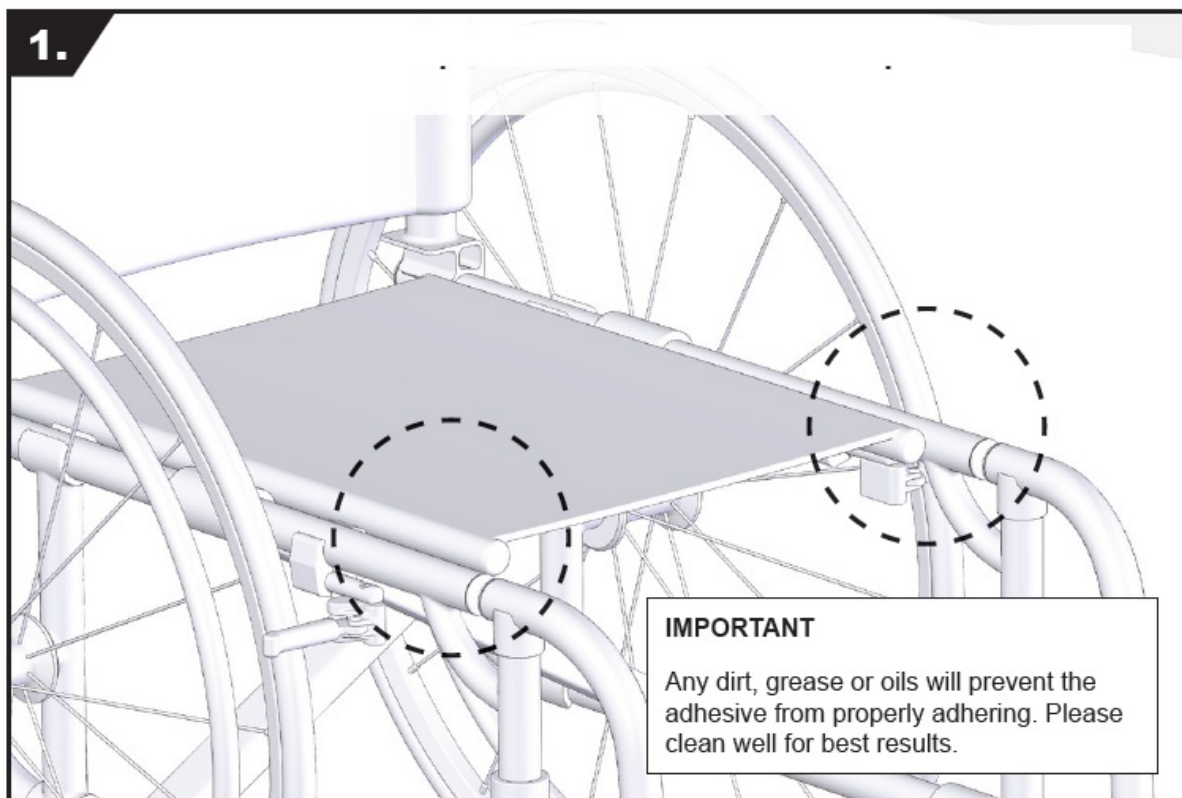


2. Adhere velcro to the frame on both sides where the Flex Straps will make contact.

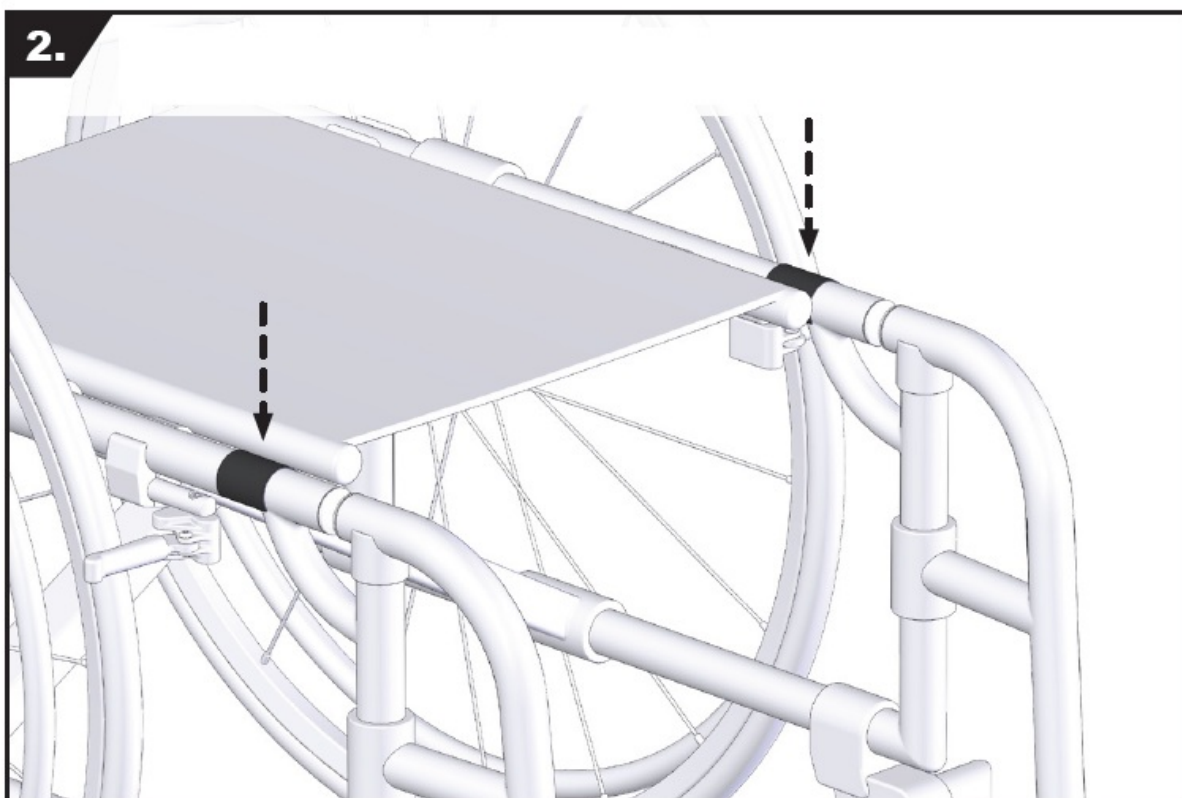

3. Adhere velcro to both LapStacker Flex Straps.


Evenly distribute the velcro across the horizontal and vertical surfaces of the Flex Strap. Ensure the velcro is pressed firmly into the internal corner.

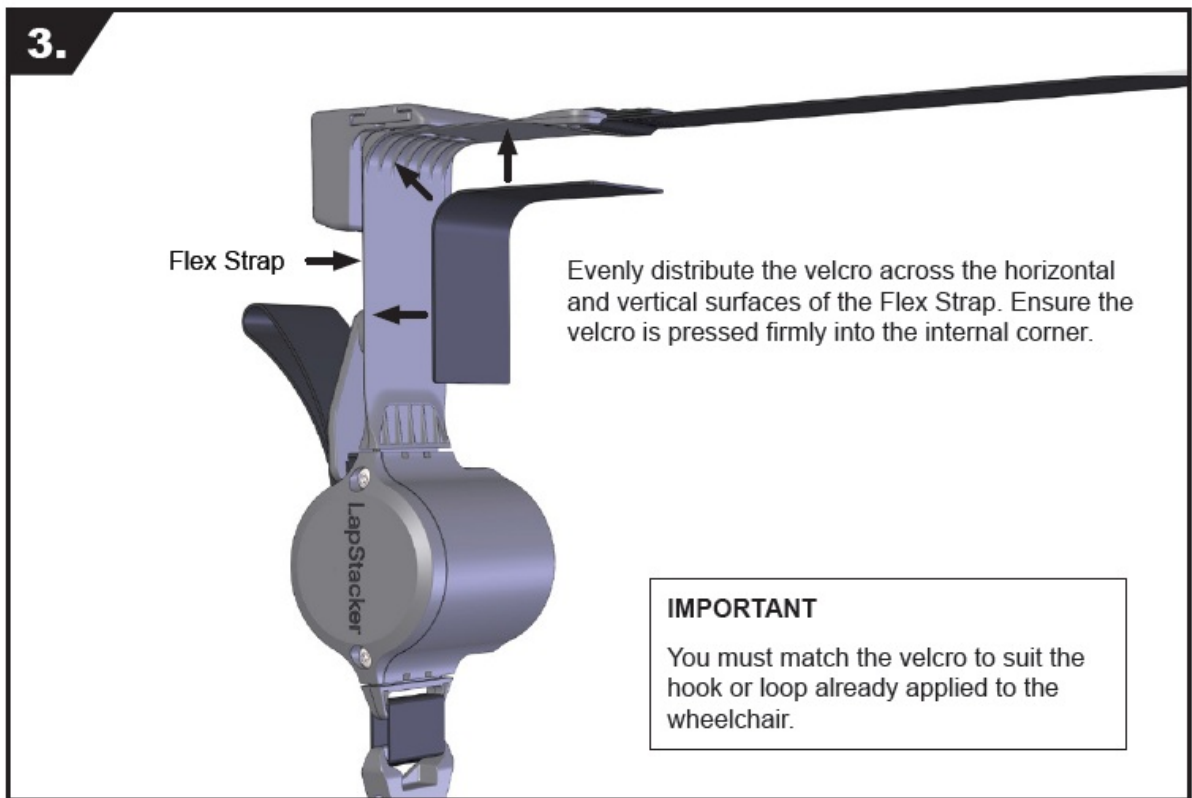

**IMPORTANT**


You must match the velcro to suit the hook or loop already applied to the wheelchair.

4. Attach LapStacker assembly to the frame.

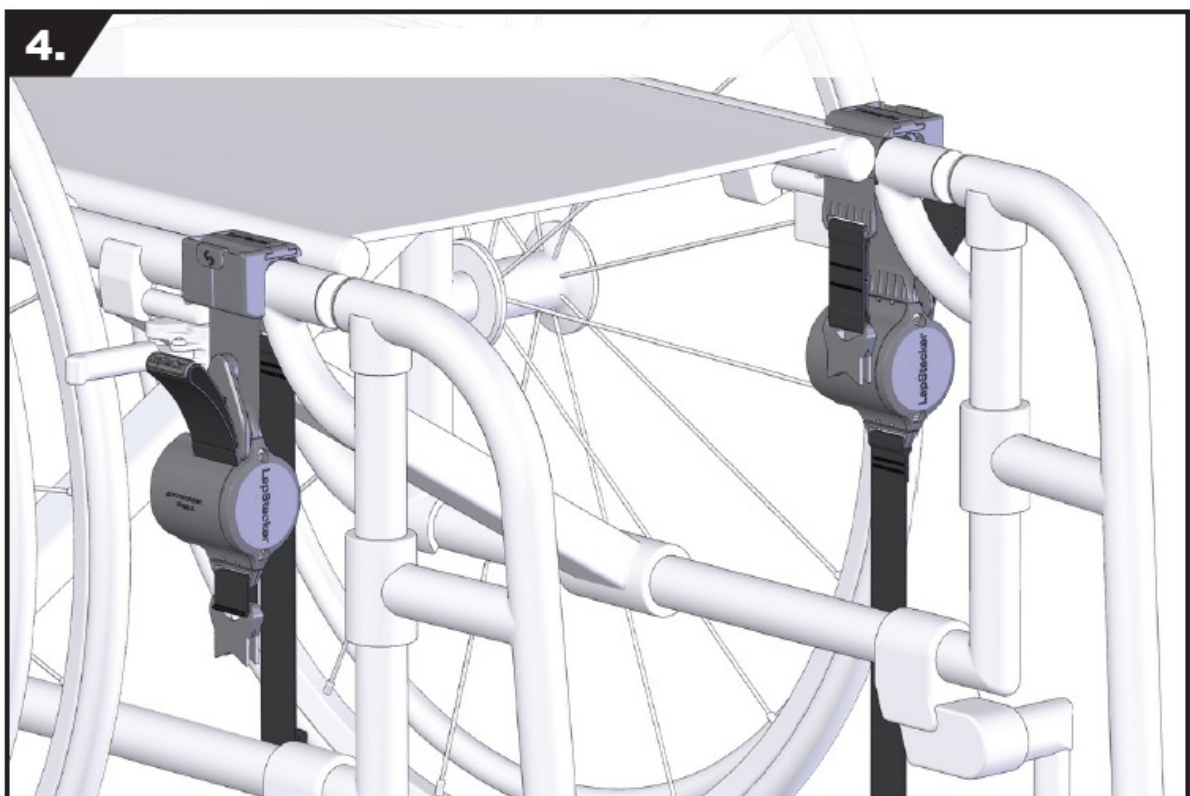


5. Pass Tension Strap around the lower tube, join buckles and tighten.




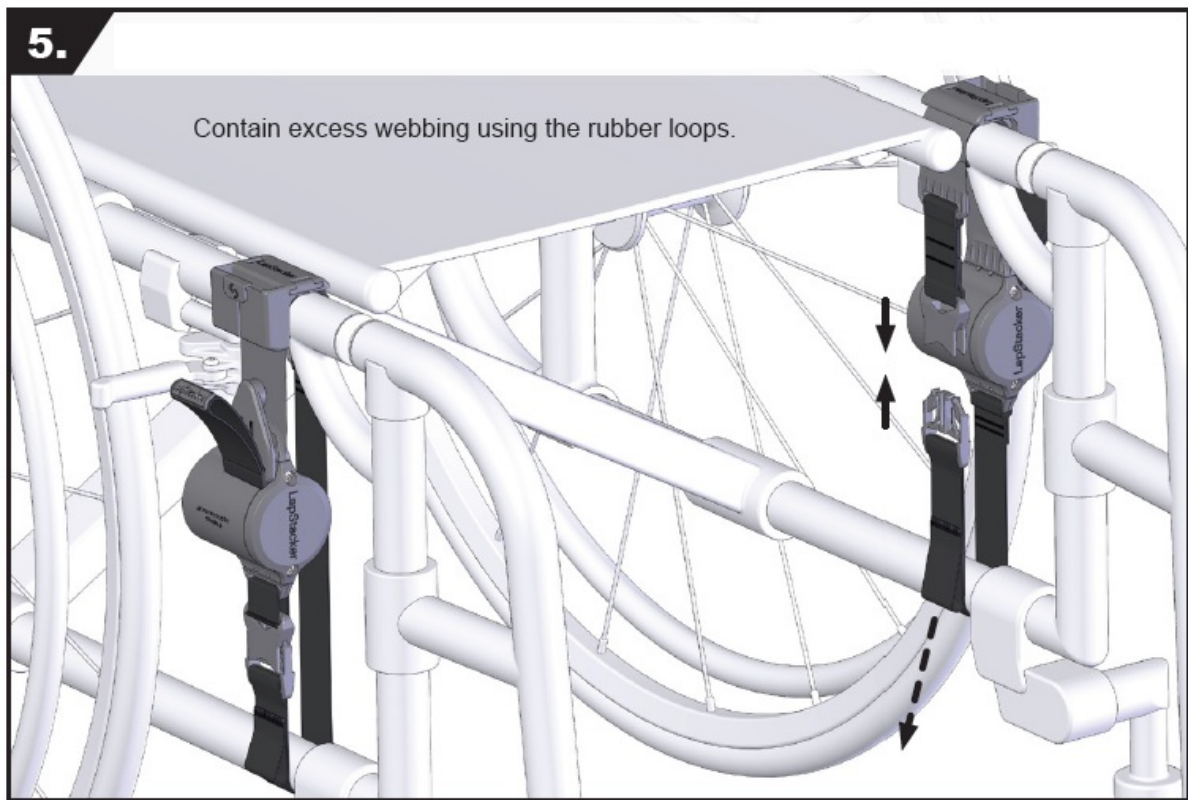

6. Pass the webbing through the Buckle Holder. Repeat for each side.

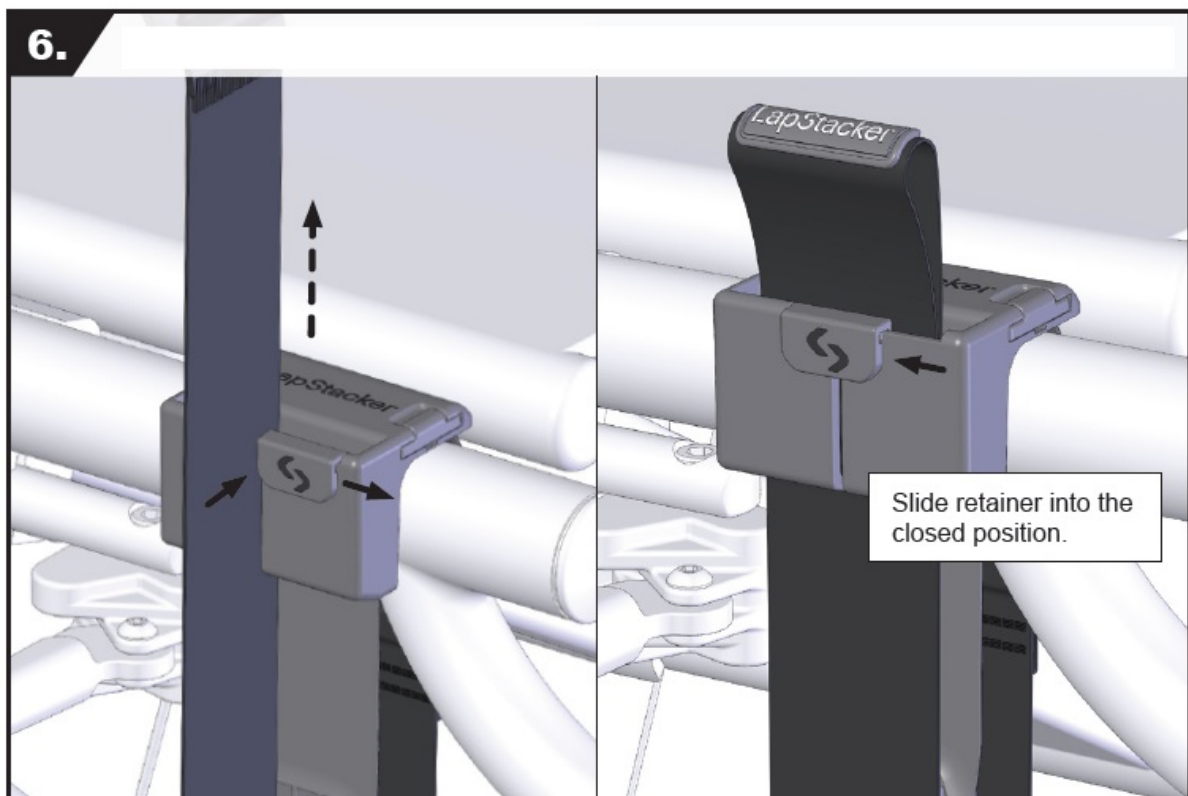

## Further Information

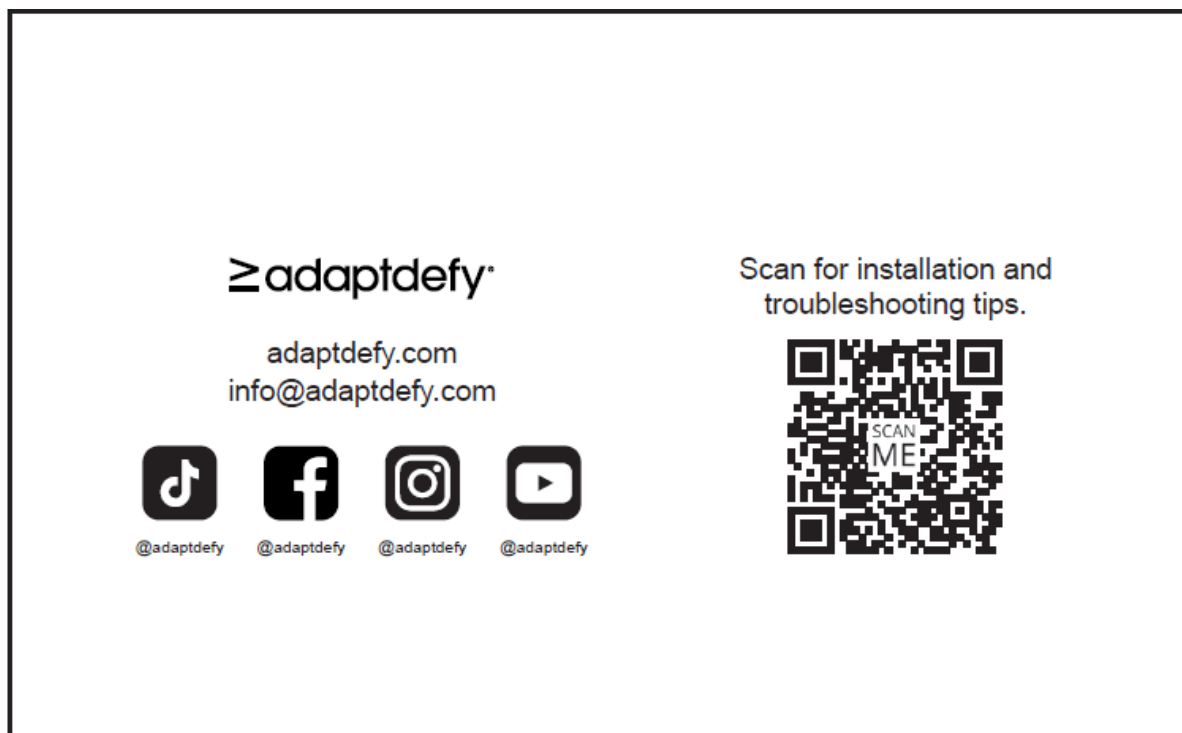

- adaptdefy.com info@adaptdefy.com
- Scan for installation and troubleshooting tips.

## FAQs

- **Q: Can LapStacker be used as a safety device?**

A: No, LapStacker is not a safety device but designed as an aid for carrying items on wheelchairs.


- **Q: How should I clean LapStacker components?**

A: Use a damp cloth for cleaning components and water with a cloth for straps. Ensure straps are fully dry before retracting.

- **Q: What should I do if the buckle holder does not fit?**

A: Refer to page 11 of the manual for further instructions if the buckle holder does not fit properly.

## Documents / Resources

 <p>LapStacker Flex</p> <p><small>Installation Guide</small></p>	<p><a href="#">Adaptdefy LapStacker Flex Wheel Chair [pdf]</a> Installation Guide</p> <p>V3-2025-03, LapStacker Flex Wheel Chair, LapStacker Flex, Wheel Chair, Chair</p>
---	---

## References

- [User Manual](#)

◆ adaptdefy, Chair, LapStacker Flex, LapStacker Flex Wheel Chair, V3-2025-03, Wheel

■ adaptdefy Chair

---

## Leave a comment

Your email address will not be published. Required fields are marked \*

Comment \*

Name

Email

Website

☐ Save my name, email, and website in this browser for the next time I comment.

**Post Comment**

**Search:**

e.g. whirlpool wrf535swhz

**Search**

[Manuals+](#) | [Upload](#) | [Deep Search](#) | [Privacy Policy](#) | [@manuals.plus](#) | [YouTube](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos

are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.