



**Aidapt VR253  
Deluxe  
Commode**



# Aidapt VR253 Deluxe Commode User Manual

[Home](#) » [Aidapt](#) » **Aidapt VR253 Deluxe Commode User Manual** 

## Contents

- [1 Aidapt VR253 Deluxe Commode](#)
- [2 Intended Use](#)
- [3 Before Use](#)
- [4 Important Safety Instructions](#)
- [5 List of Components](#)
- [6 User Instruction](#)
- [7 Product Dimensions](#)
- [8 Spare Parts](#)
- [9 Part Description](#)
- [10 CARE AND MAINTENANCE](#)
- [11 FAQ](#)
- [12 Documents / Resources](#)
  - [12.1 References](#)
- [13 Related Posts](#)



**Aidapt VR253 Deluxe Commode**

**Specifications:**

- Product: Deluxe Commode
- Model Variants: VR253 Latte, VR253A Pebble Grey
- Maximum User Weight: 250kg (39 stone)
- Manufacturer: Aidapt Bathrooms Limited
- Address: Lancots Lane Sutton Oak St Helens Merseyside WA9 3EX UK

This user manual contains essential information in relation to safe use and should be retained for future use.

**Intended Use**

The VR253\VR253A Deluxe Commode is a bedside commode designed with the intention of compensating for an injury, illness or overcoming a temporary or permanent disability. The commode provides access to bedside WC facilities where the user is unable to safely negotiate a move to the bathroom but is able to retain an assessed level of independence. Maximum user weight: 250kg (39 stone)

**Before Use**

- Carefully remove all packaging. Avoid using knives or other sharp instruments as this may damage the surface of the product.
- Check the product for visible damage. If you see any damage or suspect a fault do not use your product and contact your supplier for support.

**Important Safety Instructions**

When using the commode basic precautions should always be followed.

- Read this manual carefully, ensuring that all instructions are fully understood prior to use. Retain manual for future reference.
- Inspect all parts prior to use, do not use if any sign of damage is seen.
- User should have suitable upper and lower body strength (an assessment by a competent healthcare professional may be required).
- Floors should be clean and level.
- Do not stand on the commode.
- Bucket should only be emptied into a suitable waste receptacle (e.g plumbed WC).
- Do not exceed the maximum user weight: 250kg (39 stone).
- Remove from use should any damage be identified, do not attempt to repair as this may cause injury.

## List of Components

Please check that all of the items listed below are inside the box.

- Contact your supplier should any part be missing and do not attempt to use the commode.
- 1x Commode chair (fully assembled)
- 1x Commode bucket
- 1x Commode bucket lid



## User Instruction

The VR253\VR253A Deluxe Commode is provided fully assembled and ready for use. The commode bucket is situated under the padded seat. The padded seat is hinged to enable ease of use.

**To access the commode facility;**

1. Holding onto the handle at the front of the seat (fig 1)

Fig 1



2. Lift the seat (fig 2) and slide backwards until the seat rests at the rear of the seat as per fig 3.

Fig 2



Fig 3



3. Remove the lid from the commode bucket (fig 4).

Fig 4



#### **After use;**

1. Replace the commode bucket lid. Note: the soiled bucket should be emptied and cleaned as soon as possible.
2. Using the handle on the front of the seat (fig 5 & fig 6) pull the seat pad toward you until it sits securely in place.

Fig 5



Fig 6



Removing the commode bucket;

**The use of disposable gloves is recommended.**

1. Ensuring that the commode bucket lid is securely in place, locate the bucket handle (fig 7) and carefully lift the bucket straight up (fig 8).

Fig 7



Fig 8



2. Empty the bucket into a suitable waste outlet (eg a plumbed WC) and clean using a non-abrasive anti-bacterial cleaning solution.
3. Return the bucket and lid to the commode chair for next use.

## Product Dimensions



## Spare Parts

Contact our sales team on [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk) or on 01744 745020 to make a purchase.

## Part Description

Replacement commode bucket to suit VR253/VR253A

- Issue 1 November 2024
- Part Number
- VR253BK

Replace all worn, missing or damaged parts where necessary. If in doubt, please do not use, but immediately contact your supplier for service support.

## CARE AND MAINTENANCE

Please ensure safety checks are undertaken on a regular basis, this should include:

- Checking the condition of the chair it should be removed from use immediately if any cracking of the frame is observed.
- Ensuring that the backrest cushion is secure.
- Check the commode pan/bucket for signs of cracking
- The chair, cushion and pan can be cleaned using a non-abrasive anti-bacterial cleaner with a soft cloth.

## IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given. Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product. Please do not hesitate to contact the person who issued this product to you or the manufacturer if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Limited Lancots Lane St Helens Merseyside, WA9 3EX UK**

- Email: [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)
- Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)

**Aidapt B.V Algolweg 22 3821BH Amersfoort Netherlands**

- Email: [nederland@aidapt.eu](mailto:nederland@aidapt.eu)
- Web: [www.aidapt.eu](http://www.aidapt.eu)

## FAQ

- **Q: Can I purchase replacement parts for the commode?**


A: Yes, you can contact the sales team at [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk) or on 01744 745020 to purchase spare parts like the replacement commode bucket (Part Number: VR253BK).

- **Q: What is the intended use of the Deluxe Commode?**

A: The Deluxe Commode is designed to provide bedside WC facilities for individuals unable to safely move to

the bathroom while maintaining an assessed level of independence due to injury, illness, or disability.

## Documents / Resources

 <p>The cover of the user manual for the Aidapt VR253 Deluxe Commode. It features a photograph of the wooden commode chair. Text on the cover includes 'User Manual', 'Deluxe Commode', 'VR253 Left', 'VR253A Right', 'Maximum user weight: 250kg (55 stone)', the Aidapt logo, and CE and UKCA certification marks.</p>	<p><a href="#">Aidapt VR253 Deluxe Commode</a> [pdf] User Manual VR253, VR253A, VR253 Deluxe Commode, VR253, Deluxe Commode, Commode</p>
---	--

## References

- [Aidapt Mobility Aids & Independent Living Solutions - Buy Mobility Aids Online](#)
- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.