



Alcolizer OnSite Testing App User Guide

[Home](#) » [Alcolizer](#) » Alcolizer OnSite Testing App User Guide 

Alcolizer OnSite Testing App User Guide



Contents

- [1 Pairing Instructions Troubleshooting Quick Guide](#)
- [2 Testing Instructions](#)
- [3 Documents / Resources](#)
 - [3.1 References](#)
- [4 Related Posts](#)

Pairing Instructions Troubleshooting Quick Guide


Step 1

Access the iPad/iPhone Settings > Bluetooth menu.
Ensure BT is on.




Step 2

Access the Druglike Main Menu: press the \wedge & \vee buttons together on the idle screen and navigate to Configuration > Alco CONNECT > Configure

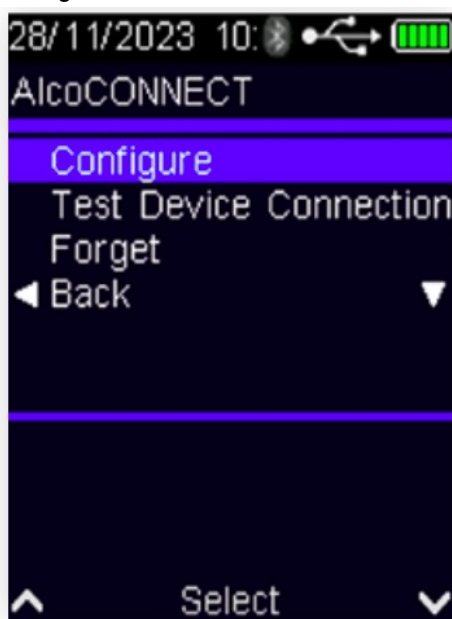

Step 3

Press the \vee button, ensure BT is enabled if not already. Press the \vee button again until page 5 is displayed. The screen will start to populate with devices.

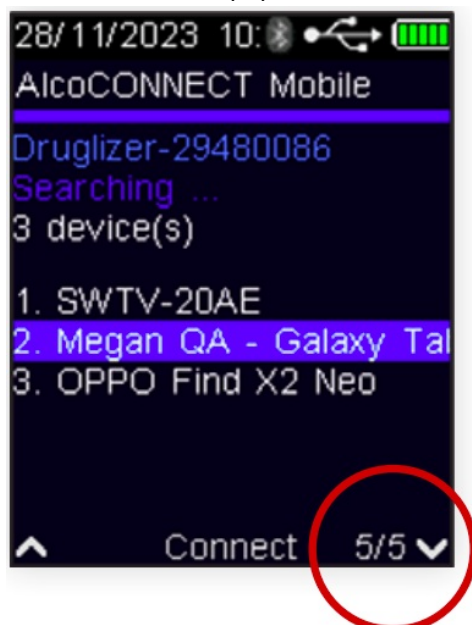

Step 4


DO NOT select a device from this screen. It will be done from the App.



Step 5


Open the App, navigate to Settings > Pairing > Scan The App will search for the Druglizer serial number. When it is displayed, select "Pair". The iPad/iPhone will request the pairing pin. Enter the pin displayed on the Druglizer. Select Pair.

The Druglizer will display "Paired".

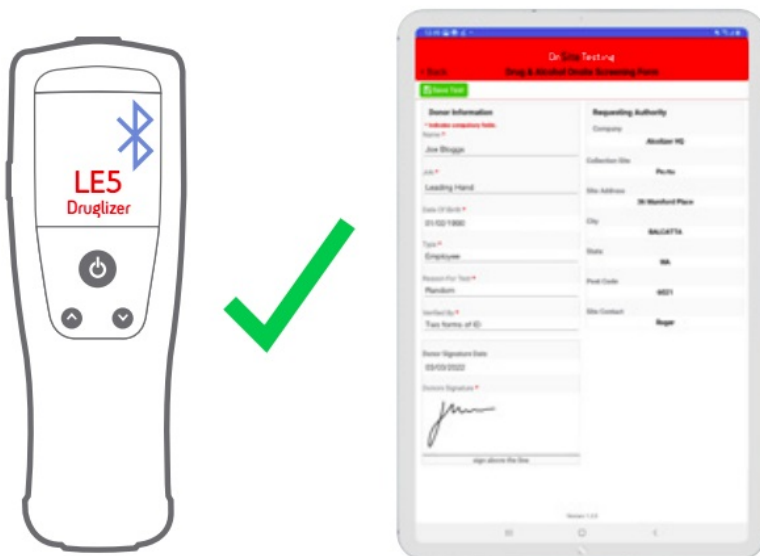


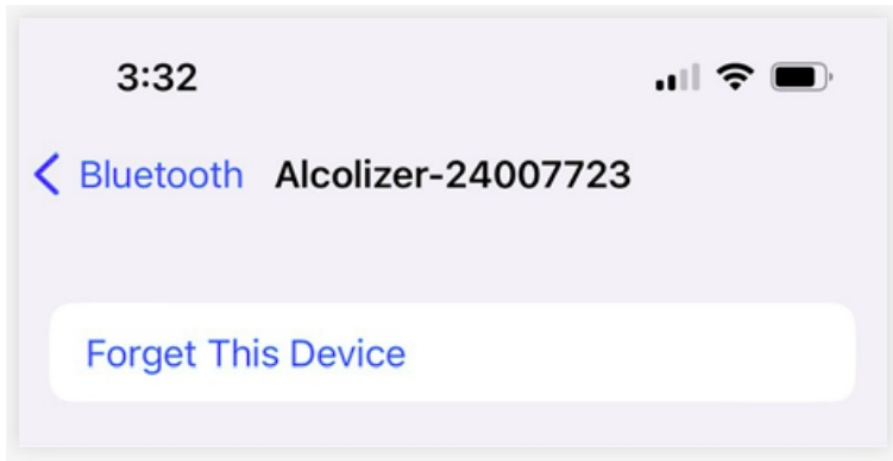
Step 6

For 'Full Test Mode', go to 'Settings', select the Druglike device, set expiry date and begin testing. *Quick Test Mode is currently not supported in Druglizer.*

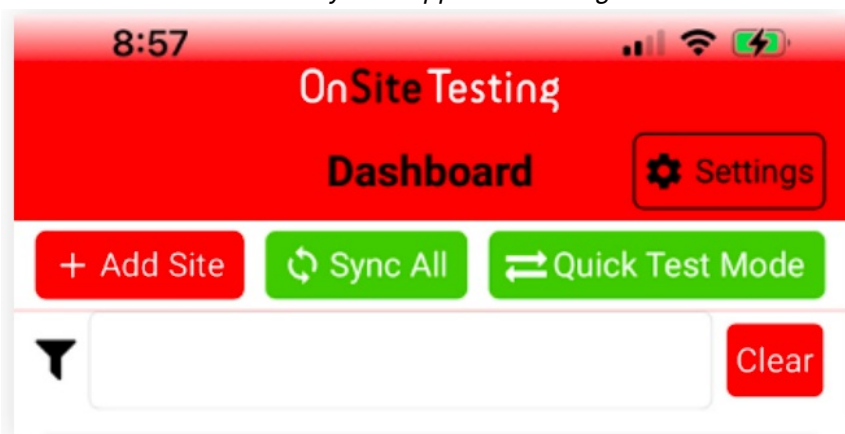

Step 1

On the iPad/iPhone, select Settings > Bluetooth. Select the  icon next to the Druglizer serial number and select 'Forget This Device'.



Step 2

Access the Druglizer Main Menu: press the Λ & V buttons together on the idle screen and navigate to Configuration > Alco CONNECT > Forget. Select Forget. The Druglizer and Android phone/tablet are now unpaired.



Step 3


Follow steps 1 – 6 of the **Pairing Instructions** (see left hand side) to re-establish a connection between devices.



Testing Instructions

Step 1

Select the appropriate 'Site' from the dashboard, listed in white boxes with the Site name.

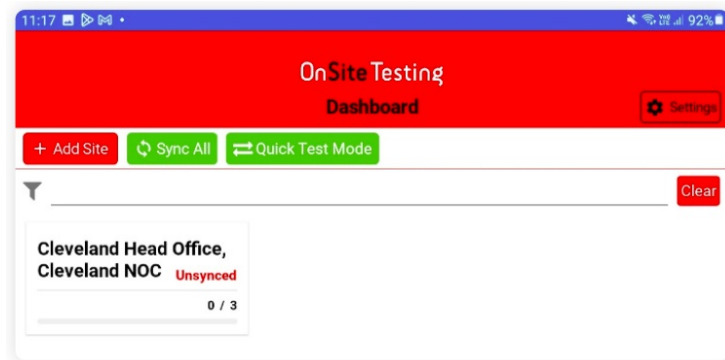


The screenshot shows the 'OnSite Testing Dashboard' with a red header. Below the header are three buttons: '+ Add Site', 'Sync All', and 'Quick Test Mode'. A search bar with a 'Clear' button is present. A list of sites is shown, with 'Cleveland Head Office, Cleveland NOC' selected and marked as 'Unsynced'. The count '0 / 3' is displayed at the bottom of the list.

Step 2

Select 'Add Test' on the top left.

Complete all donor information.

All fields are required.

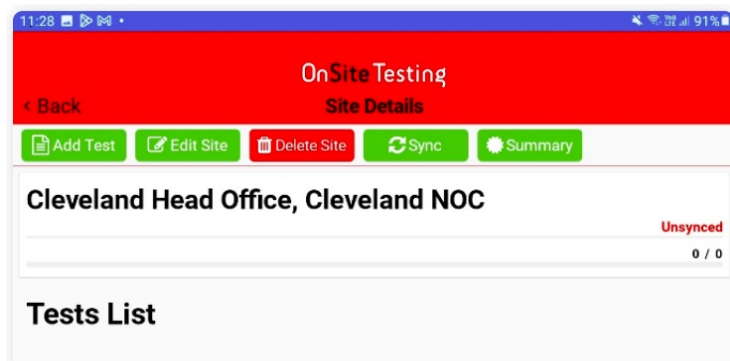

The screenshot shows the 'OnSite Testing Site Details' screen with a red header. Below the header are five buttons: 'Add Test', 'Edit Site', 'Delete Site', 'Sync', and 'Summary'. The site name 'Cleveland Head Office, Cleveland NOC' is displayed, marked as 'Unsynced'. Below this is a 'Tests List' section.

Step 3

Tap the Donor Signature section to select test type and donor medication if any. Donor must sign the Agreement.



The screenshot shows the 'Donor's Agreement' screen. It includes a 'Test Type' dropdown menu with 'Oral Fluid Drug Test' selected. Below this is a consent statement: 'I consent to the testing of my breath and oral fluid sample for alcohol and/or drugs. I acknowledge that the drug and alcohol test(s) will only be used for the purposes for which it was obtained as set out in my employers Drug and Alcohol Policy. I consent to the results of the drug and/or breath alcohol test(s) to be provided to the nominated representative of the requesting authority. I acknowledge the information I give is true and correct and I declare that I have taken the following medication, drugs or other non-prescribed agents in the last 48 hours.' Below the consent statement is a section for 'Medication/non prescribed agents used in last 48 hours by donor' with an '+ Add Medication' button. A 'Medication #1' field is present. The 'Donor's signature' section has a line for the signature and a 'sign above the line' instruction. A green 'COMPLETE' button is at the bottom right.

Step 4

Select Drug Test once donor information has been completed.

Complete the drug screening using the LE5 Druglizer.

12:47 OnSite Testing 83%

< Back Drug & Alcohol Onsite Screening Form

Save Test

Donor Info Alcohol Test Drug Test

Donor Information Requesting Authority

* Indicates compulsory fields.

Name *

Company

Step 5

The result will transfer to the app and populate the results screen.

11:03 OnSite Testing 46%

< Back Drug & Alcohol Onsite Screening Form

Save Test Declaration

Donor Info Alcohol Test Drug Test

Test Type: Oral Fluid Drug Test Device: LE5 Drugalyzer Batch: R14045 Expiry: 05/03/2030 Serial Number: 2

Drug Class Time Sample Taken

BZO Negative 2:53 PM

COMMENTS

OXY Negative

MET Negative

THC Negative

AMP Negative

COC Negative

CPH Negative

Step 6

The donor must sign the Declaration form.

The Certification form will then pop up for the Collector to sign.

1:47 OnSite Testing

< Back Drug & Alcohol Onsite Screening Form

Save Test Declaration

Donor Info Alcohol Test

Device: LE5 Breathalyzer Exp Date: 09/03/2030 Serial Number: 00000001

Step 7

The app will now display the Site section again and the test listed as completed in green.

Follow the above steps if completing more than one test.

Step 8

Once the testing session is complete, sign the Summary form. Both signatures are required. After signing, select 'back' on the top left of the form.

Step 9

Select 'Sync' on the top of the screen. Ensure you have internet connection. Access the Alco CONNECT platform to view results and reports.



Alcolizer LE5 Druglizer

Contact any of our nation-wide offices by calling 1300 789 908 or email sales@alcolizer.com



Documents / Resources



[Alcolizer OnSite Testing App](#) [pdf] User Guide
LE5, OnSite Testing App, Testing App, App

References

- [User Manual](#)

Manuals+, Privacy Policy

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.