

**allard
35202
Selection
Thumb Rigid**



allard 35202 Selection Thumb Rigid Instructions

[Home](#) » [allard](#) » allard 35202 Selection Thumb Rigid Instructions 

Contents

- [1 allard 35202 Selection Thumb Rigid](#)
- [2 Specifications](#)
- [3 FAQs](#)
- [4 PRODUCT INFORMATION](#)
- [5 USING INSTRUCTIONS](#)
- [6 SIZE](#)
- [7 CONTACT INFORMATION](#)
- [8 Documents / Resources](#)
 - [8.1 References](#)
- [9 Related Posts](#)

allard

allard 35202 Selection Thumb Rigid



Specifications

- Intended Use: Mild to moderate arthritis in the first CMC-joint, protects the CMC-joints and the first MCP-joint, rheumatic arthritis, after trauma, hypermobile thumb and wrist, after arthroplasty surgery, repetitive strain injuries
- Material: Polyester, nylon, elastane
- Flame Retardant: No

FAQs

Q: How should I adjust the product for the best fit?

A: Adjust the strap around the thumb to follow the indicated line on the surface and ensure it is comfortable.

Q: Can I wash the orthosis?

A: Please refer to the care instructions provided with the product for information on washing and maintenance.

Q: What should I do if I experience skin irritation while using the support?

A: Immediately remove the support if any sign of skin irritation occurs and consult with a medical professional if needed.

PRODUCT INFORMATION

Indications / Intended Use

Is intended for mild to moderate arthritis in the first CMC joint and protects the CMC joints and the first MCP joint. Rheumatic arthritis, after trauma, in case of hypermobile thumb and wrist, after arthroplasty surgery, and for repetitive strain injuries. Single Patient-multiple use.

Contraindications

No known

Adjustments

This product should only be altered by a medical professional person.

Warnings

Make sure that the straps are not too tight. If any signs of skin irritation occur, immediately remove the support. Check your skin on a daily basis.

Material Specifications

This orthosis is made from polyester, nylon, and elastane. This product is not flame retardant

SELECTION THUMB ORTHOSIS RIGID

Carefully read this instruction sheet before use!

Washing Instruction

Close all Velcro prior to laundering. Machine wash at 40°C (104°F), gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Disposal

Disposal in accordance with local legislation.

Information

Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.


Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

USING INSTRUCTIONS

1. Check that the support is complete and without damage.
2. Attach the thumb strap and place the opening over the thumb with the orthosis towards the back of the hand.

3. Pull the strap around the wrist and secure.
4. Adjust the strap around the thumb so it follows the line indicated on the surface.
5. Adjust for comfort.

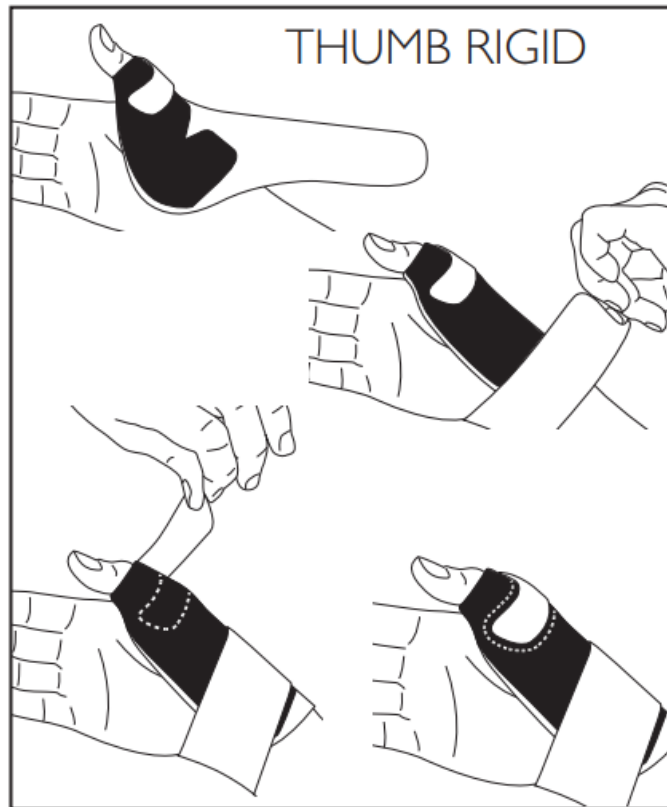

SIZE

Item No. RIGID/Black	Item No. RIGID/Beige	Side	Size	Wrist circumference
35202 1010	35203 1010	Left	X-Small	< 13 cm
35202 1011	35203 1011	Left	Small	13 – 15 cm
35202 1012	35203 1012	Left	Medium	15 – 17 cm
35202 1013	35203 1013	Left	Large	17 – 20 cm
35202 1014	35203 1014	Left	X-Large	> 20 cm
35202 2010	35203 2010	Right	X-Small	< 13 cm
35202 2011	35203 2011	Right	Small	13 – 15 cm
35202 2012	35203 2012	Right	Medium	15 – 17 cm
35202 2013	35203 2013	Right	Large	17 – 20 cm
35202 2014	35203 2014	Right	X-Large	> 20 cm

CONTACT INFORMATION

- Camp Scandinavia AB. Karbingatan 38
- SE-254 67 Helsingborg, Sweden
- Phone +46 42 25 27 01
- Sverige
- 042-25 27 01
- info@camp.se
- camp.se
- Danmark
- 43 96 66 99
- info@camp.dk
- camp.dk

UK/Ireland

- 42 932 8177
- info@allarduk.co.uk
- allarduk.co.uk


USA

- 888-678 6548
- info@allardusa.com
- allardusa.com

International

- 42 25 27 01
- info@allardint.com
- allardint.com

Documents / Resources

 <p>allard Instructions for Use Selection® Thumb Rigid AA No. 35202, 35203</p>	<p>allard 35202 Selection Thumb Rigid [pdf] Instructions 35202, 35203, 35202 Selection Thumb Rigid, 35202, Selection Thumb Rigid, Thumb Rigid, Rigid</p>
---	--

References

- [a Allard Support for Better Life! | Allard International](#)
- [a Allard Support for Better Life! | Allard UK](#)
- [a Allard Support for Better Life! | Allard USA](#)
- [Camp Scandinavia - Support for Better Life! | Camp Scandinavia](#)
- [Camp Scandinavia - Support for Better Life! | Camp Scandinavia](#)
- [Camp Scandinavia - Support for Better Life! | Camp Scandinavia](#)
- [Camp Scandinavia - Support for Better Life! | Camp Scandinavia](#)
- [User Manual](#)

Manuals+ Privacy Policy

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.