



Allflex RapIDMatic Evo Applicator User Guide


[Home](#) » [Allflex](#) » Allflex RapIDMatic Evo Applicator User Guide 



Contents

- [1 RapIDMatic Evo Applicator](#)
- [2 Documents / Resources](#)
 - [2.1 References](#)

RapIDMatic Evo Applicator

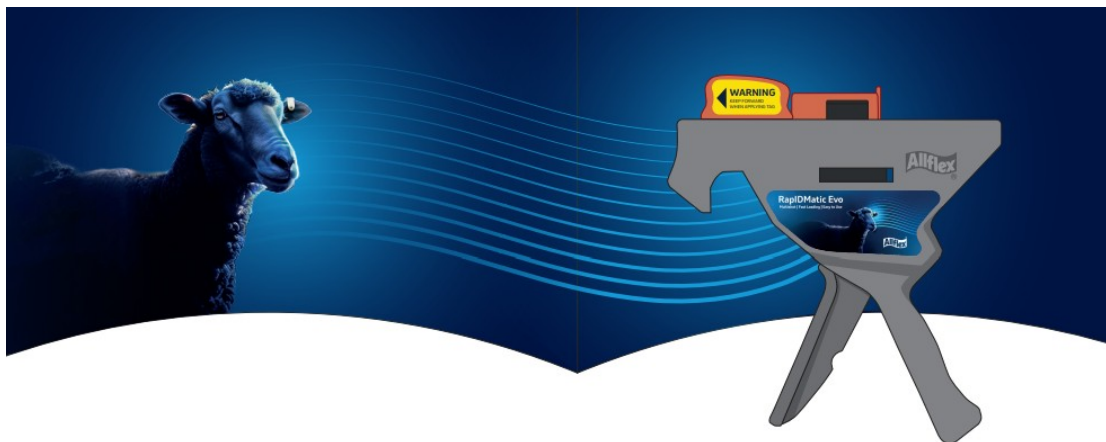

RapIDMatic Evo Applicator


For the application of Allflex RapID Evo tags only

For a translated user guide, please scan the QR code or visit the website www.allflex.global/rapidmatic-evo


Ensure loader/cutter is in the locked position (at front of applicator).




Roll RapID Evo Tags towards you firmly.
Bias the pins outwards to avoid jamming.

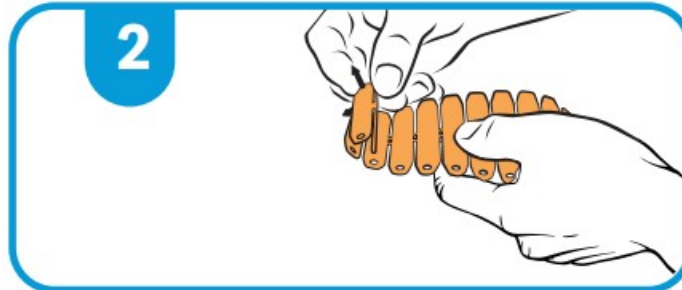

Leave the last two RapID Evo Tags Unrolled.

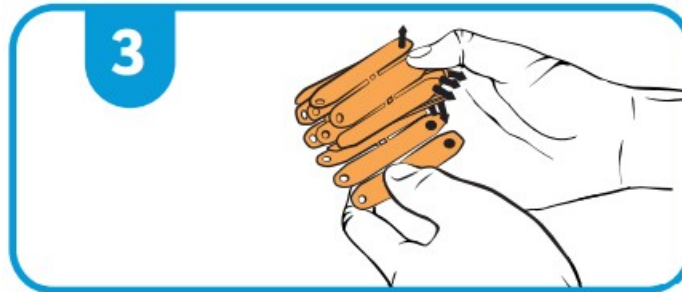

Load into drum.

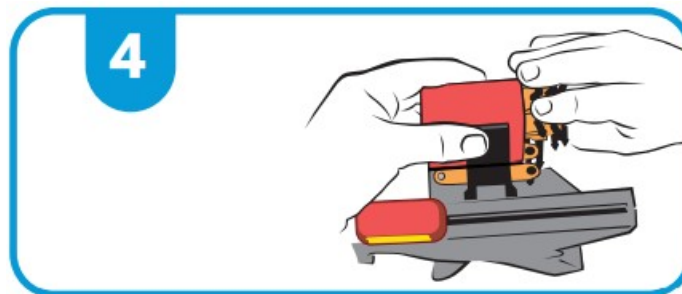


Depress the clamp (A) firmly. Slide tags out until location lugs are between first and second tags.




Pull trigger to move tag into the Load position.



 Ensure that the applicator is tilted to the left in the following steps

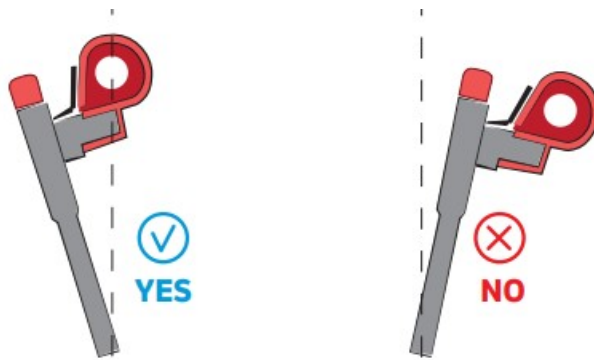


Pull topslide back to cut the tag.

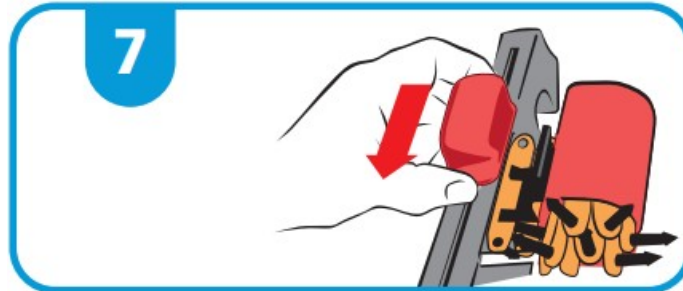

Push topslide forward to load into applicator jaw.



Check tag position.

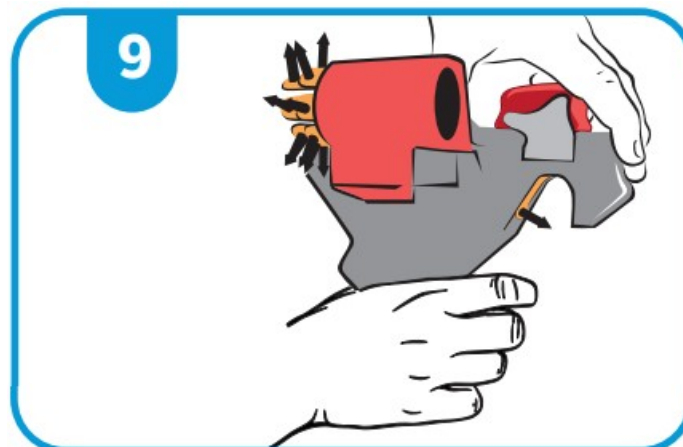


Avoid Iodine based disinfectants when sterilising tags. If irritation occurs after applying the identification tag, please consult your veterinarian for appropriate treatment.




Select optimum position on ear for tag.

Pull trigger to apply.

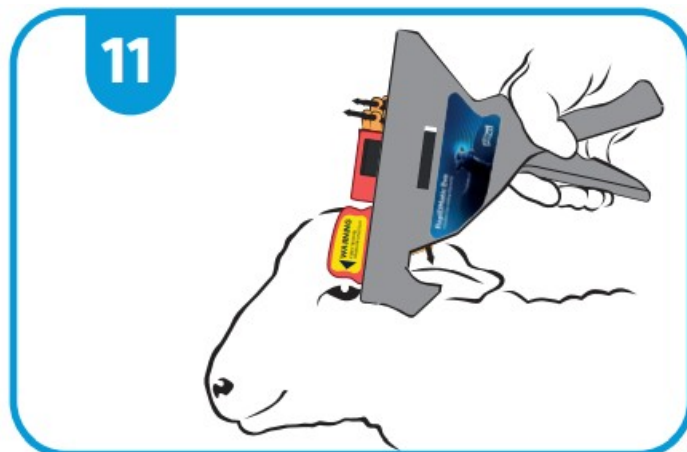


Tags should be located as close to the head as practically possible, allowing room for growth if required.

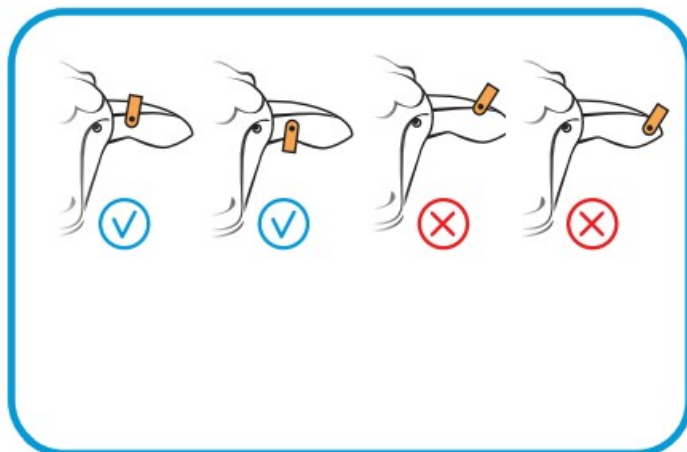

Keep your trusty Allflex Sheep Tag Applicator running at full speed by keeping it clean. All it takes is a quick rinse of water to loosen any dirt or grime that might have accumulated during tag application.

Remove the barrel to clean inside applicator.

For best results wash the applicator periodically during application sessions.

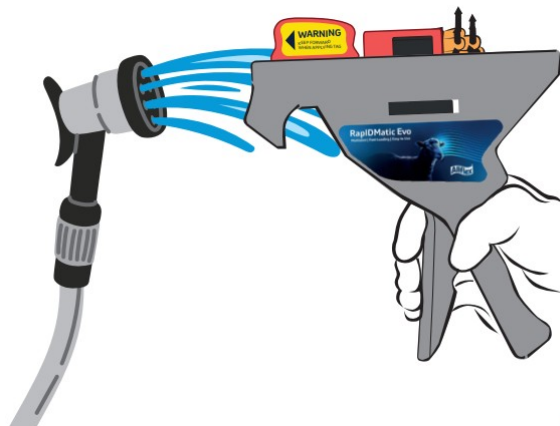

For a translated user guide, please scan the QR code or visit the website www.allflex.global/rapidmatic-evo



www.allflex.global/rapidmatic-evo

Allflex New Zealand Ltd
17 El Prado Drive
Palmerston North
06 3567199
www.allflex.co.nz
Allflex Australia Pty Ltd
185 Queensport Road
Murarrie QLD 4172
1300 138 247

www.allflex.com.au

Copyright © 2024 Merck & Co., Inc., Rahway, NJ, USA and its affiliates.
All rights reserved. AHI-NON-240600001



Documents / Resources



[Allflex RapIDMatic Evo Applicator](#) [pdf] User Guide
AHI-NON-240600001, RapIDMatic Evo Applicator, Evo Applicator, Applicator

References

- [Allflex Livestock Intelligence NZ | ID, Monitoring & Health](#)
- [Home - Allflex Australia](#)
- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.