



Amico LCD AIO Powered Cart with Advanced Interface Instruction Manual

[Home](#) » [Amico](#) » Amico LCD AIO Powered Cart with Advanced Interface Instruction Manual 



Powered Cart with Advanced Interface Quick Setup, Installation, and Adjustment Instructions

Contents

- [1 Automatic Height Adjustment](#)
- [2 Manual Height Adjustment](#)
- [3 Documents / Resources](#)
 - [3.1 References](#)
- [4 Related Posts](#)

Automatic Height Adjustment

Medical Facility Responsibility




WARNING: This Quick Setup guide is not a replacement for the full manual. It is the responsibility of the end user to ensure all aspects of the installation are covered by following the full manual.



NOTE: Inspect cart integrity prior to beginning setup.

1. First Time Initializing

- Briefly plug the cart into the power outlet the Advanced Interface should turn on (this may take up to 30 seconds).
- After initializing the cart, a “NO HOME” symbol () will appear in the top right hand corner of the Advanced Interface. This means the user must set the cart’s “HOME” (lowest) position.

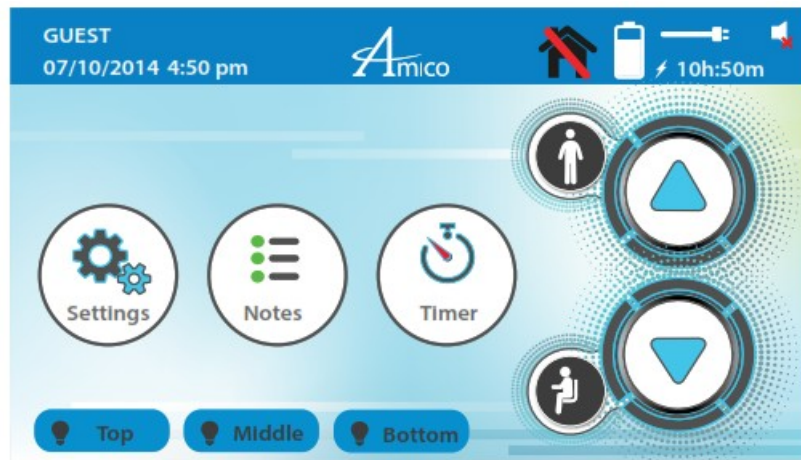


Figure 1

2. Setting Home Position


- Hold the DOWN arrow to lower the cart’s worksurface until the collision symbol () appears.
- Release the DOWN arrow, reverse direction to raise worksurface to desired position.



Figure 2

3. Setting Time and Date

- Tap on “GUEST” in top left corner of the interface. (Figure 1)
- Select “ADMIN” from drop down list and log in using password: AMICO. (Figure 2)
- Tap on “SETTINGS” icon.
- Select “ADMIN SETUP” (only available in the “ADMIN” profile).
- Select “SET CLOCK”.
- Using the arrow icons set the time and date in the desired format (NOTE: date and time formats must first be selected on right of interface).
- Tap “SAVE” to finish.



Figure 3

4. Creating a User Profile

- Tap on “ADMIN” in top left corner of interface and select “LOGOUT” when prompted.
- Tap “GUEST” in top left corner of interface, and select “NEW USER”.
- Enter desired user name, select “NEXT”.
- Enter desired password, select “NEXT” (re-enter password and select “NEXT”).
- Enter the user’s height. Select “NEXT” then “OK” when prompted.

5. Setting “SIT” Heights



NOTE: User’s forearms should be parallel to ground (90° bend at elbows) for ergonomic use.

- To input sit and stand positions, select “SETTINGS” and then “USER SETUP”. (Figure 3)
- Adjust the height using the UP/DOWN arrows. When height is at desired “SIT” position, select “SAVE” next to respective position icons.

6. Setting “STAND” Heights

- To input “SIT” and “STAND” positions, select “SETTINGS” and then “USER SETUP”. (Figure 3)
- Adjust the height using the UP/DOWN arrows. When height is at desired “STAND” position, select “SAVE” next to respective position icons.
- Tap “OK” twice to return to Main Menu.

For power system software download, please visit <http://www.amico.com/hummingbird-power-system>

Manual Height Adjustment

Medical Facility Responsibility



WARNING: This Quick Setup guide is not a replacement for the full manual. It is the responsibility of the end user to ensure all aspects of the installation are covered by following the full manual.



NOTE: Inspect cart integrity prior to beginning setup.

1. First Time Initializing

- Briefly plug cart into the power outlet the Advanced Interface should turn on (this may take up to 30 seconds).



Figure 1

2. Setting Time and Date

- Tap on "GUEST" in top left corner of the interface.
- Select "ADMIN" from the drop down list and log in using the password: AMICO. (Figure 1)
- Tap on the "SETTINGS" icon.
- Select "ADMIN SETUP" (only available in the "ADMIN" profile).
- Select "SET CLOCK".
- Using the arrow icons set the time and date in the desired format (NOTE: date and time formats must first be selected on right of the interface).
- Tap "SAVE" to finish.



Figure 2

3. Creating a User Profile:

- Tap on "ADMIN" in the top left corner of the interface and select "LOGOUT" when prompted.
- Tap "GUEST" in the top left corner of the interface, select "NEW USER".
- Enter desired user name, select "NEXT". (Figure 2)
- Enter desired password, select "NEXT" (re-enter password and select "NEXT").

4. Height Adjustment:

- Pull the lever and adjust to desired height. (Figure 3)

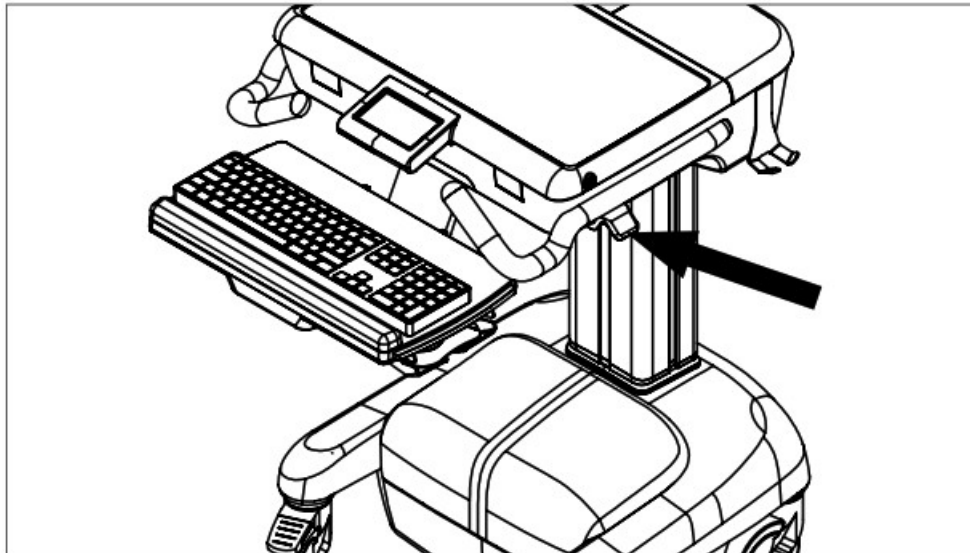



Figure 3

For power system software download, please visit: <http://www.amico.com/hummingbird-power-system>

Amico Accessories Inc. | 85 Fulton Way, Richmond Hill, ON L4B 2N4, Canada | www.amico.com
Toll Free Tel: 1.877.264.2697 | Tel: 905.763.7778 | Fax: 905.763.8587 | Email: info@amico-accessories.com
AA-QG-MOBILE-COMPUTER-WORKSTATION-HUMMINGBIRD-ADVANCED-INTERFACE 09.14.2020

Documents / Resources

	<p>Amico LCD AIO Powered Cart with Advanced Interface [pdf] Instruction Manual AA-QG-MOBILE-COMPUTER-WORKSTATION-HUMMINGBIRD-ADVANCED-INTERFACE, LCD AIO Powered Cart with Advanced Interface, AIO Powered Cart with Advanced Interface, Cart with Advanced Interface, Advanced Interface, Interface</p>
---	--

References

- [Top 10 Must-Have Accessories for Fashion, Home, Car, and Tech.](#)
- [Amico Group of Companies | Home Page](#)