



APEX WiZARD 320 Full Face Mask Installation Guide

[Home](#) » [Apex](#) » APEX WiZARD 320 Full Face Mask Installation Guide 



WiZARD 320 Full Face Mask Assembling Guide





Component

1. WiZARD 310/320 Headgear with Buckles
2. WiZARD 320 Elbow Connector with frame
3. WiZARD 320 Full Face Mask Cushion, Large
4. WiZARD 320 Full Face Mask Cushion, Medium
5. WiZARD 320 Full Face Mask Cushion, Small
6. Silicon Tubing

Caution!


Please confirm that all the components belong to WiZARD 320 Full Face Mask by checking the reference number before assembling.

 <p>1</p>	 <p>2</p>
<p>Connect the silicon tube to the frame directly</p>	<p>Align the arrow on cushion to the groove on frame and assemble them together directly</p>
 <p>3</p>	 <p>4</p>
<p>Lay flat the headgear with WiZARD label downwards</p>	<p>Place the frame on the center of headgear and hook on the buckles</p>
 <p>5</p>	 <p>6</p>
<p>Assemble the upper straps through the holes in front of forehead pad and attach the velcros</p>	<p>Assembly completed. (The WiZARD logo should be outwards on the back of headgear)</p>

Contents

- 1 Documents / Resources
- 2 Related Posts

Documents / Resources

	<p>APEX WiZARD 320 Full Face Mask [pdf] Installation Guide</p> <p>WiZARD 320 Full Face Mask</p>
---	---