

## Geemarc LH10\_ANT\_I

# Geemarc LH10 Sound Amplifier User Manual

Model: LH10\_ANT\_I

## INTRODUCTION

---

This manual provides comprehensive instructions for the safe and effective use of your Geemarc LH10 Sound Amplifier. The LH10 is designed to enhance sound for individuals with hearing difficulties, featuring adjustable tone and volume controls for a personalized listening experience. Please read this manual thoroughly before operating the device and retain it for future reference.

For digital hearing aid users, it is important to ensure your hearing aids are programmed for use with the "T" setting to utilize the full capabilities of the LH10.

## PRODUCT OVERVIEW

---

### Components Included

- LH10 Personal Listener Unit
- Bud type headset
- USB lead for charging (power adapter not supplied)
- Instruction Manual (this document)
- 24-month guarantee card with Helpline number

### Device Layout and Controls



**Figure 1: LH10 Sound Amplifier with Earbuds.** This image displays the main LH10 unit in grey/black, connected to the included bud-type headset. The device features visible controls for volume and tone, along with indicator lights.



**Figure 2: LH10 Top View.** A closer look at the top surface of the LH10 amplifier, highlighting the power button, battery indicator, and the rotary dial for volume adjustment. The integrated microphone is also visible at the top.



**Figure 3: LH10 Headset Port and Microphone.** This image provides a detailed view of the 3.5mm headset jack and the built-in microphone located on the device.



**Figure 4: LH10 in Use.** This image demonstrates the compact size and portability of the LH10, shown clipped to a person's belt, indicating its suitability for on-the-go use.

The LH10 features a compact design with clearly marked controls. On the main body, you will find the power button, a battery charge indicator, and a volume control dial. The device also includes a dedicated port for connecting the headset and an integrated microphone for capturing sound.

## SETUP

---

### 1. Battery Installation and Charging



**Figure 5: LH10 Battery Compartment.** The underside of the LH10 unit with the battery cover removed, revealing the rechargeable Lithium Ion battery and its connection.

1. Locate the battery compartment on the back of the LH10 unit.
2. Open the battery cover.
3. Insert the provided Lithium Ion battery, ensuring correct polarity as indicated inside the compartment.
4. Close the battery cover securely.
5. Connect the supplied USB charging lead to the LH10 unit's charging port.
6. Connect the other end of the USB lead to a standard USB power adapter (not supplied) or a computer USB port.
7. The battery charge indicator light will illuminate during charging and change color or turn off when fully charged.
8. Allow the device to fully charge before first use.

### 2. Connecting the Headset

1. Insert the 3.5mm jack of the bud-type headset firmly into the headset port on the LH10 unit.
2. Ensure the connection is secure to prevent audio interruptions.

## OPERATING INSTRUCTIONS

---

### Powering On/Off

- To power on the LH10, press and hold the power button until the indicator light illuminates.
- To power off, press and hold the power button again until the indicator light turns off.

### Adjusting Volume

- Rotate the volume control dial located on the side of the unit to increase or decrease the sound level.
- Start with a low volume and gradually increase it to a comfortable listening level.

### Adjusting Tone

- The LH10 features a tone control to adjust the frequency response for clearer sound.
- Experiment with the tone control to find the setting that best suits your hearing needs and the environment.

### Using with Hearing Aids (T-Setting)

- If you use digital hearing aids, ensure they are programmed to the "T" (Telecoil) setting.
- This allows the LH10 to transmit audio directly to your hearing aids, providing a clearer and more direct sound experience, especially in environments with background noise.

## MAINTENANCE

---

### Cleaning

- Wipe the LH10 unit with a soft, dry cloth.
- Do not use liquid cleaners or abrasive materials, as these can damage the device.
- Regularly clean the earbuds with a damp cloth and ensure they are dry before use.

### Battery Care

- To prolong battery life, avoid fully discharging the battery frequently.
- Charge the device regularly, even if not in constant use.
- If storing the device for an extended period, ensure the battery is partially charged (around 50%) and recharge every few months.

### Storage

- Store the LH10 in a cool, dry place, away from direct sunlight and extreme temperatures.
- Keep the device away from dust and moisture.

## TROUBLESHOOTING

---

Problem	Possible Cause	Solution
---------	----------------	----------

Problem	Possible Cause	Solution
No sound from headset	<ul style="list-style-type: none"><li>• Device is off</li><li>• Headset not properly connected</li><li>• Battery is depleted</li><li>• Volume is too low</li></ul>	<ul style="list-style-type: none"><li>• Turn on the device</li><li>• Ensure headset is fully plugged in</li><li>• Charge the battery</li><li>• Increase volume using the dial</li></ul>
Sound is distorted or unclear	<ul style="list-style-type: none"><li>• Volume set too high</li><li>• Tone setting is unsuitable</li><li>• Microphone obstructed</li><li>• Low battery</li></ul>	<ul style="list-style-type: none"><li>• Decrease volume</li><li>• Adjust tone control</li><li>• Ensure microphone is clear</li><li>• Charge the battery</li></ul>
Device not charging	<ul style="list-style-type: none"><li>• USB cable not connected properly</li><li>• Faulty USB cable or power adapter</li><li>• Battery issue</li></ul>	<ul style="list-style-type: none"><li>• Ensure USB cable is securely connected to both device and power source</li><li>• Try a different USB cable or power adapter</li><li>• Contact customer support if issue persists</li></ul>

## SPECIFICATIONS

---

Model Name	LH10
Item Model Number	LH10_ANT_I
Brand	Geemarc
Product Dimensions	3.94 x 2.17 x 1.18 inches
Item Weight	2.46 ounces
Battery Type	1 Lithium Ion battery required
Connectivity Technology	Wired
Special Features	Battery Charge Indicator
Color	Grey/Black
Material	Silicone Plastic
Manufacturer	Geemarc Telecom

## WARRANTY AND SUPPORT






---

Your Geemarc LH10 Sound Amplifier comes with a**24-month guarantee** from the date of purchase. Please refer to the guarantee card included in the box for full terms and conditions.

For technical assistance, troubleshooting beyond this manual, or warranty claims, please contact Geemarc customer support. The helpline number is provided on your guarantee card.

You can also visit the official Geemarc website for additional resources and support information.

## Related Documents - LH10\_ANT\_I

	<p><a href="#">Geemarc CL595 Corded Phone Quick Start Guide</a></p> <p>A quick start guide for the Geemarc CL595 corded phone, covering installation, base overview, menu settings, SOS functions, answering machine operation, and technical specifications.</p>
	<p><a href="#">Geemarc CL8360 Amplified Mobile Phone User Manual</a></p> <p>Comprehensive user guide for the Geemarc CL8360 amplified mobile phone. Features include dual SIM, camera, SOS button, large display, and amplified sound for seniors and those with hearing impairments. Covers setup, operation, SMS, phonebook, and troubleshooting.</p>
	<p><a href="#">Geemarc AMPLIDECT595 U.L.E Quick User Guide - Setup and Features</a></p> <p>Comprehensive quick user guide for the Geemarc AMPLIDECT595 U.L.E cordless phone. Learn about installation, setup, features like answering machine, SOS calls, phonebook management, and technical specifications.</p>
	<p><a href="#">Geemarc CL9000 4G Phone with SOS Emergency Call User Manual</a></p> <p>Explore the Geemarc CL9000, a 4G amplified phone with SOS emergency features and hearing aid compatibility. This user manual guides you through setup, operation, and safety for clear communication and peace of mind.</p>
	<p><a href="#">Geemarc CL9000 4G Telefoon met SOS Noodoproep Handleiding</a></p> <p>Handleiding voor de Geemarc CL9000 4G telefoon, speciaal ontworpen voor senioren en personen met gehoorproblemen. Bevat installatie-instructies, instellingen, gebruiksaanwijzingen, SOS-noodoproepfunctie en technische specificaties.</p>



### [Geemarc CL9000: 4G Telefon mit SOS-Notruffunktion – Offizielle Bedienungsanleitung](#)

Umfassende Bedienungsanleitung für das Geemarc CL9000 4G-Telefon. Entdecken Sie Funktionen wie SOS-Notruf, verbesserte Hörgerätekompabilität und einfache Bedienung. Ideal für Senioren und Benutzer mit Hörproblemen.