

[Manuals.plus](#) /

> [Aika](#) /

> AIKA AG-7901 Wireless Rechargeable Bud Hearing Aid Instruction Manual

Aika AG-7901

AIKA AG-7901 Wireless Rechargeable Bud Hearing Aid Instruction Manual

Model: AG-7901 | Brand: Aika

INTRODUCTION

This manual provides detailed instructions for the safe and effective use of your AIKA AG-7901 Wireless Rechargeable Bud Hearing Aid. Please read this manual thoroughly before using the device and keep it for future reference. This device is designed to enhance sound clarity and improve the hearing experience for individuals with up to 65% hearing loss.

WHAT'S IN THE BOX

Your AIKA AG-7901 package includes the following items:

- Hearing Aids (2 units for both ears)
- 1 Charging Case
- Multiple Earbuds (various sizes)
- Cleaning Brushes
- 1 Charging Cable (Type-C USB)
- User Guide
- Color Box
- Warranty Card


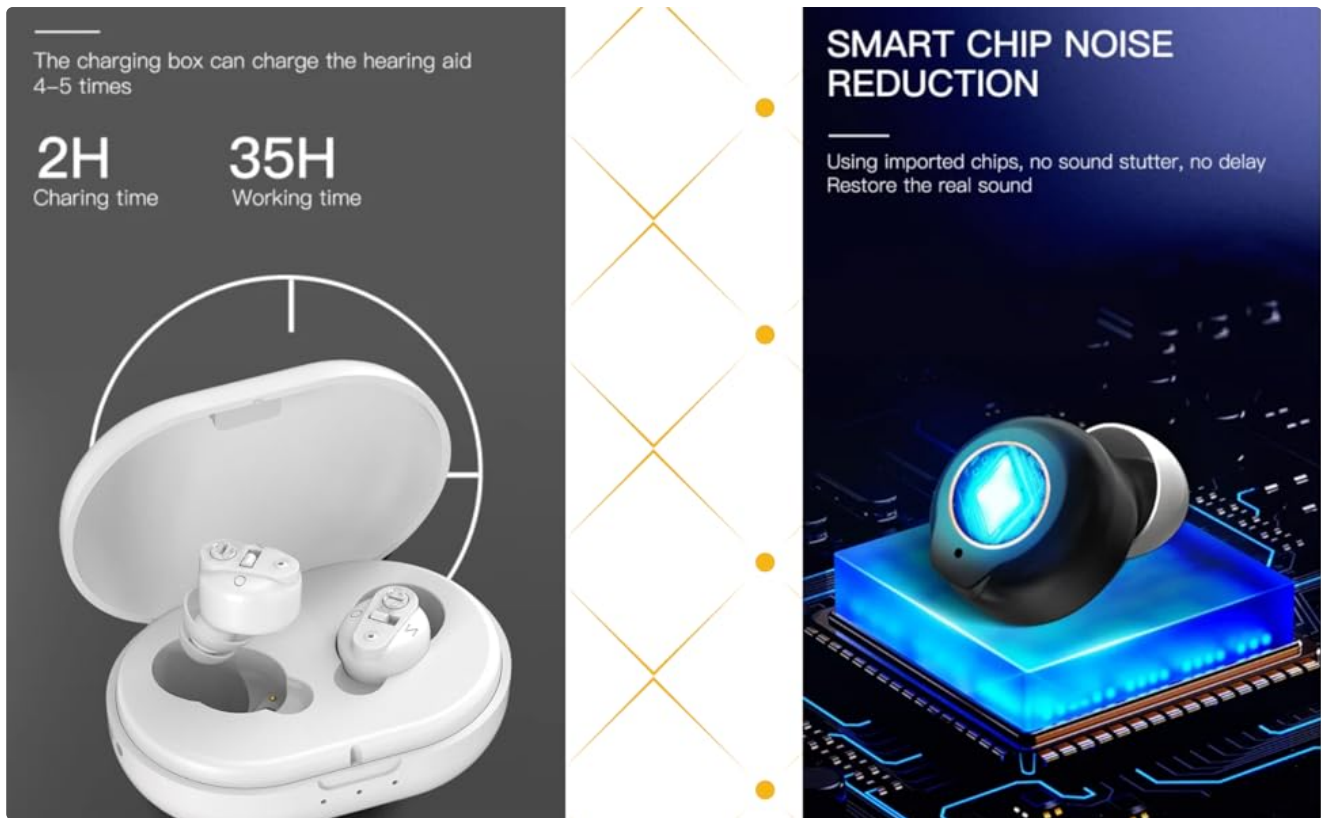


Figure 1: Package Contents - Hearing aids, charging case, various earplugs, cleaning brush, and charging cable.

PRODUCT FEATURES

- **Enhanced Sound Clarity:** Delivers clear and amplified sound for improved communication.
- **Wireless Convenience:** Cordless design for freedom and comfort.
- **Rechargeable Efficiency:** Built-in rechargeable battery for long-lasting use.
- **Discreet and Comfortable Fit:** Ergonomic bud design ensures a secure and comfortable fit.
- **Noise Cancellation:** Features active noise cancellation for clearer listening.
- **Volume Control:** Easy-to-use button control for volume adjustment.

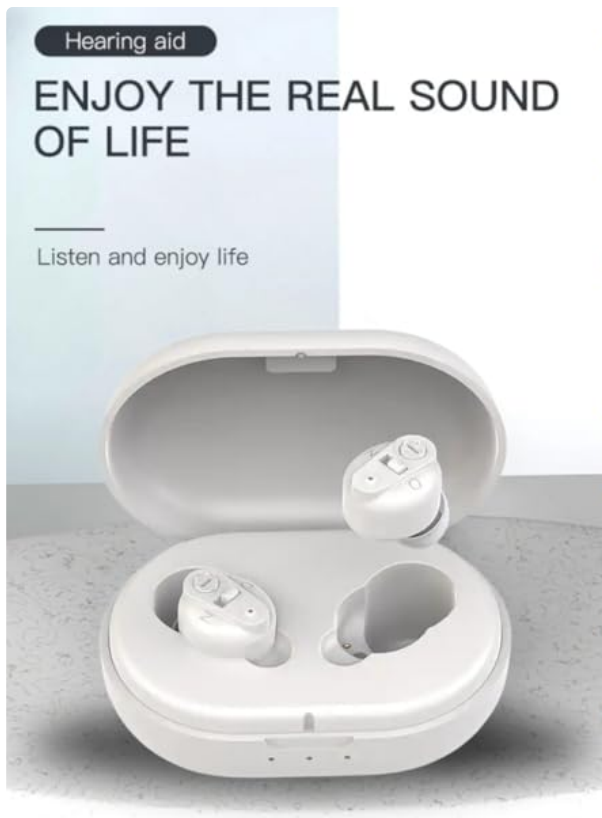

MAGNETIC CHARGING

- USB rechargeable, used in various scenarios



Power Bank



Charger



Computer

Product structure & Operation

Simple operation, Convenient and Fast



Figure 2: Discreet and Comfortable Fit.

SETUP AND FIRST USE

1. **Initial Charge:** Before first use, fully charge the hearing aid using the charging case for 12 hours.
2. **Clean Ears:** Use a cotton swab to clean your ear canal before wearing. Excessive earwax can block the sound outlet and affect performance.
3. **Select Ear Dome:** Choose the correct ear dome size for a secure and comfortable fit. The package includes multiple sizes.
4. **Insert Device:** Gently insert the device into your ear canal.
5. **Power On:** Locate the ON/OFF switch on the hearing aid and turn it to the "N" position to power it on.
6. **Adjust Volume:** Start with a low volume setting and gradually increase it to a comfortable level using the volume button.

The charging box can charge the hearing aid 4-5 times

2H
Charing time

35H
Working time



SMART CHIP NOISE REDUCTION

Using imported chips, no sound stutter, no delay
Restore the real sound




Figure 3: Components and Initial Operation.

AUTOMATIC NOISE REDUCTION

Clear sound, effectively reduce noise
Cut off howling, smarter sound processing



Before noise reduction After noise reduction

AUTOMATIC ADAPTATION

Strong ability to adapt to the environment
able to adapt to a variety of environmental changes



Daily communication



Indoor communication



Outdoor communication



Noisy environment

Figure 4: Volume Adjustment.

OPERATING INSTRUCTIONS

Power On/Off

- To turn on the hearing aid, slide the switch to the "N" position.

- To turn off the hearing aid, slide the switch to the "O" position.

Volume Adjustment

- The hearing aid features a volume button. Press the volume keys to adjust the volume.
- Clockwise rotation means to increase the volume.
- Counter-clockwise rotation means to decrease the volume.
- There are 6 sound levels of volume. When it reaches the lowest volume, the next press will change it to the highest volume.

Video: Operation Guide

Your browser does not support the video tag.

Video 1: Demonstrates unboxing, identifying components, and basic operation including power on/off and volume adjustment.

CHARGING INSTRUCTIONS

The AIKA AG-7901 hearing aid comes with a rechargeable charging case. Follow these steps to charge your device:

1. Ensure the insulation sticker is removed from the hearing aids before the first charge.
2. Place the hearing aids securely into their respective slots in the charging case.
3. Connect the supplied Type-C USB charging cable to the charging case and then to a power source (e.g., wall adapter, computer, power bank).
4. The charging case has indicators:
 - **Hearing Aids Charging Indicator:** Steady blue light indicates charging; light off means fully charged.
 - **Charging Case Charging Indicator:** Flashing green light indicates charging; steady green light means fully charged.
5. A full charge for the hearing aids takes approximately 3-4 hours and provides up to 35 hours of use. The charging case can fully charge the hearing aids 4-5 times.



PRODUCT USE

For the first time wearing, please fully charge the hearing aid



1. Choose the right earplugs



2. Product is fully charged



3. Wear



4. Adjust the volume

Figure 5: Fast Charge and Larger Capacity.

Video: Charging Instructions

Your browser does not support the video tag.

Video 2: Demonstrates how to connect the charging cable to the case and the indicators during charging.

MAINTENANCE

- **Daily Cleaning:** After each use, turn off the device and gently clean the ear domes and the device surface with the provided cleaning brush to remove any earwax or debris.
- **Moisture Protection:** Place the hearing aids in the provided dry kit overnight or at least every alternate night to protect them from moisture and ensure long-lasting performance.
- **Storage:** Store the hearing aids and charging case in a cool, dry place away from direct sunlight and extreme temperatures.
- **Avoid Liquids:** Do not immerse the hearing aids or charging case in water or any other liquids.

TROUBLESHOOTING

Problem	Possible Cause	Solution
---------	----------------	----------

Problem	Possible Cause	Solution
No Sound or Low Sound	<ul style="list-style-type: none"> ◦ Device is off. ◦ Low battery. ◦ Earwax blockage in ear dome. ◦ Incorrect ear dome size. ◦ Volume too low. 	<ul style="list-style-type: none"> ◦ Turn on the device. ◦ Charge the device fully. ◦ Clean the ear dome and sound outlet. ◦ Replace with a correctly sized ear dome. ◦ Increase the volume.
Whistling/Feedback Sound	<ul style="list-style-type: none"> ◦ Improper fit in ear canal. ◦ Earwax buildup. ◦ Volume too high. 	<ul style="list-style-type: none"> ◦ Re-insert the device for a snug fit. ◦ Ensure correct ear dome size. ◦ Clean ear canal and ear dome. ◦ Reduce the volume.
Device Not Charging	<ul style="list-style-type: none"> ◦ Charging cable not properly connected. ◦ Faulty power source. ◦ Insulation sticker not removed (first use). 	<ul style="list-style-type: none"> ◦ Check cable connection to case and power source. ◦ Try a different power outlet or USB port. ◦ Ensure insulation sticker is removed.

SPECIFICATIONS

Brand	Aika
Model Name	901-White (AG-7901)
Colour	White
Style	For Both Ear
Power Source	Battery Powered (Lithium-Ion, 3.7 Volts)
Special Features	Noise Cancellation, Rechargeable, Sweatproof, Volume Control
Fit Type	In-Ear
Control Type	Button Control
Noise Control	Active Noise Cancellation
Audio Sensitivity	120 Decibels
Item Dimensions (L x W)	5L x 5W Centimeters
Item Weight	10 Grams

Country of Origin	India
-------------------	-------

WARRANTY AND SUPPORT

Your Aika Hearing Aids come with a **1-year warranty**. For online purchases, a **6-month warranty** is provided by default. To extend your warranty to **1 year**, please register your product within **15 days from the date of delivery** by following the instructions on the warranty card included in the package.

For customer care, warranty registration, or any support inquiries, please contact:

- **Email:** aikatrading20@gmail.com
- **Mobile:** +91-7678452043
- **Timing:** 11 AM - 6 PM (Monday to Saturday)
- **Office and Service Center Address:** Aika Trading Co, C-18 Basement, Sec 49 Noida 20130 UP
- **Website:** www.theaika.com