

iBstone Nova

iBstone Nova Hearing Aids User Manual

Model: Nova

1. INTRODUCTION

This manual provides detailed instructions for the safe and effective use of your iBstone Nova Hearing Aids.

Please read this manual thoroughly before using your device and retain it for future reference.

The iBstone Nova Hearing Aids are designed to provide clear sound with advanced noise-cancelling technology, offering a comfortable listening experience for individuals with mild to severe hearing loss.



Image: iBstone Nova Hearing Aids (blue and red) with their charging case and packaging. This image illustrates the complete product package.

2. WHAT'S INCLUDED

Your iBstone Nova Hearing Aids package contains the following items:

What Included

Full Package • Real-human Support • Operation Video



Charging Case



Hearing Aids



Anti-lost Strap



Cleaning Brush



Wax Guard Filters




USB Cable



User Manual



Silicone Ear Domes

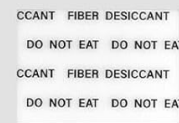

Dry Tablets

Image: A visual representation of all items included in the iBstone Nova Hearing Aids package, arranged neatly.

- Charging Case
- iBstone Nova Hearing Aids (Left and Right)
- Anti-lost Strap
- Cleaning Brush
- Wax Guard Filters
- USB Charging Cable
- User Manual (this document)
- Assorted Silicone Ear Domes
- Dry Tablets (for moisture-proof compartment)

3. SETUP AND FIRST USE

3.1 Charging the Device

Before initial use, fully charge the hearing aids and the charging case.

1. Place the hearing aids into their respective slots in the charging case. Ensure the blue aid is in the 'L' slot

and the red aid is in the 'R' slot.

2. Connect the USB charging cable to the charging case and a power source.
3. The charging case features a clear remaining battery display. Lights will indicate charging status. A full charge is indicated when all lights are off.
4. A full charge takes approximately 1.5 hours and provides up to 80 hours of use with the charging case.



Image: Illustration of the charging case with hearing aids, showing charging indicators and battery duration information. It highlights the 1.5-hour fast charge for 80 hours of use.

3.2 Selecting and Attaching Ear Domes

Properly fitted ear domes are crucial for comfort and optimal sound quality, preventing feedback (whistling).

1. Choose the ear dome size that feels most comfortable and creates a good seal in your ear canal. The package includes various sizes and styles.
2. Gently attach the selected ear dome to the hearing aid.

3.3 Wearing the Hearing Aids

The iBStone Nova hearing aids are designed for a slim, in-ear fit.

1. Identify the Left (L) and Right (R) hearing aids. They are typically color-coded (blue for left, red for right) and marked.
2. Gently insert the hearing aid into your ear canal, ensuring the upward direction marking is correctly oriented.
3. Adjust the hearing aid until it feels secure and comfortable.



Image: A step-by-step guide illustrating how to choose ear domes, place the hearing aid in the ear canal, adjust for comfort, and use the function button. It shows a person comfortably wearing the device.

4. OPERATING INSTRUCTIONS

4.1 Power On/Off

- **Automatic On/Off:** The hearing aids automatically turn on when removed from the charging case and turn off when placed back into the case.

4.2 Adjusting Volume and Modes

The iBstone Nova features a single function button for all adjustments.



One Button, All Functions

Auto On/Off • 9 Volume Levels • 3 Listening Modes Memory Function

Function button

-  **Auto turn ON/OFF**
when take out/put into the case
-  **Quick press to adjust volume**
9 volumes levels for different hearing loss
-  **2-second press to adjust**
Quiet / Noisy / Outdoor modes for different scenes
-  **Auto memory function to keep the setting**
Don't need to adjust next time

Image: Diagram illustrating the single function button on the hearing aid and its various operations: auto on/off, quick press for volume adjustment, 2-second press for mode adjustment, and auto memory function.

- **Volume Adjustment:** A quick press of the function button cycles through 9 volume levels. Select the level that provides optimal clarity without discomfort.
- **Mode Selection:** Press and hold the function button for 2 seconds to switch between the 3 hearing modes:

- **Quiet Mode:** Optimized for calm environments.
- **Noisy Mode:** Enhances speech in environments with background noise.
- **Outdoor Mode:** Designed for open-air settings, reducing wind noise.
- **Memory Function:** The device remembers your last used volume and mode settings.

5. MAINTENANCE

Regular maintenance ensures the longevity and optimal performance of your iBstone Nova Hearing Aids.

5.1 Daily Cleaning

- After each use, gently wipe the hearing aids with a soft, dry cloth.
- Use the provided cleaning brush to remove any earwax or debris from the ear domes and sound outlets.
- Do not use water or cleaning solutions directly on the hearing aids.

5.2 Wax Guard Filters

Periodically check and replace the wax guard filters as needed to prevent earwax from entering the hearing aid and affecting sound quality.

- Refer to the specific instructions included with your wax guard filters for replacement procedures.

5.3 Moisture Control

The charging case includes a built-in moisture-proof compartment to help keep your hearing aids dry.

- Place the provided dry tablets into this compartment.
- Store your hearing aids in the charging case with the dry tablets when not in use to extend their lifespan.

Extra Design Features



Anti-Loss Strap
Never worry about losing



Built-in Dehumidifying Capsule
Extent the lifetime of your devices



Portable Charging Case
Small case, big power



Abundant Ear domes
Get your best fit

Image: A collage showcasing the anti-lost strap, the built-in dehumidifying capsule in the charging case, the portability of the charging case, and the variety of ear domes for a custom fit.

6. TROUBLESHOOTING

If you encounter any issues with your iBstone Nova Hearing Aids, please refer to the following common solutions:

Problem	Possible Cause	Solution
---------	----------------	----------

Problem	Possible Cause	Solution
No Sound or Weak Sound	<ul style="list-style-type: none"> • Low battery • Ear dome blocked by earwax • Incorrect volume setting 	<ul style="list-style-type: none"> • Charge the hearing aids and charging case. • Clean the ear dome and sound outlet with the cleaning brush. Replace wax guard if necessary. • Increase the volume using the function button.
Whistling or Feedback	<ul style="list-style-type: none"> • Improperly seated ear dome • Ear dome too small • Hearing aid not fully inserted 	<ul style="list-style-type: none"> • Re-insert the hearing aid firmly into the ear canal. • Try a larger ear dome size to ensure a better seal. • Ensure the hearing aid is fully inserted and snug.
Hearing Aid Not Charging	<ul style="list-style-type: none"> • Poor contact with charging pins • Charging cable or adapter issue 	<ul style="list-style-type: none"> • Ensure hearing aids are correctly seated in the charging case. Clean charging contacts if necessary. • Try a different USB cable or power adapter.

If the problem persists, please contact customer support.

7. SPECIFICATIONS

- **Model Number:** Nova
- **Manufacturer:** iBstone
- **Battery Type:** Rechargeable (1 A battery included per aid)
- **Charging Time:** Approximately 1.5 hours
- **Usage Time:** Up to 80 hours with charging case
- **Hearing Modes:** Quiet, Noisy, Outdoor (3 modes)
- **Volume Levels:** 9 levels
- **Noise Reduction:** Up to 26dB AI Noise Reduction
- **Package Dimensions:** 6.1 x 3.9 x 1.8 inches
- **Item Weight:** 8.4 ounces

8. WARRANTY AND SUPPORT

iBstone stands behind its products and is committed to user satisfaction.

- **Customer Support:** U.S.-based customer phone support is available to assist with any after-sales problems.
- **Warranty Information:** Please refer to the product packaging or contact iBstone customer service for specific warranty details.
- **Contact:** For support, please refer to the contact information provided on the iBstone official website or product packaging.

Lifetime Support from iBstone Team

Any question about our hearing aids, contact us by text, email, message or phone call









-  Real-human Instant Service
-  US Based Replacement & Repairment
-  Comprehensive Manual & Video Instruction



Image: A group of professionals in lab coats, representing the iBstone support team, emphasizing lifetime support and customer service.

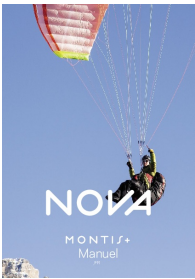Related Documents - Nova

	<p>NOVA MONTIS+: Manuel pour Harnais de Parapente Hike & Fly</p> <p>Découvrez le harnais de parapente NOVA MONTIS+, idéal pour le hike & fly. Ce manuel détaillé couvre les caractéristiques, l'utilisation, la sécurité et l'entretien de votre équipement NOVA.</p>
	<p>Project Guide: How to Turn a Champagne Flute with NOVA Tools</p> <p>Learn how to craft an elegant champagne flute with this step-by-step woodworking project guide. Ideal for honing spindle turning techniques, this project emphasizes precision and balance, resulting in a functional and beautifully crafted piece perfect for celebrations. Features NOVA tools and expert instructions by Jeff Hornung.</p>
	<p>Manuale d'Uso e Manutenzione: Nova 15-22 Spezzatrice Manuale</p> <p>Scopri il manuale d'uso e manutenzione per la Nova 15-22, una spezzatrice manuale professionale progettata per panetterie, pasticcerie e pizzerie. Guida completa per l'installazione, l'uso sicuro e la manutenzione.</p>
	<p>NOVA Eco-NOVA Bidet Seat User Manual and Installation Guide</p> <p>This document provides comprehensive instructions for the NOVA Eco-NOVA Bidet Seat, including installation requirements, operation details, maintenance tips, troubleshooting guide, and warranty information.</p>

	<p>Nova Folding Cup Holder Assembly and Warranty Information</p> <p>Detailed assembly instructions and warranty information for the Nova Folding Cup Holder (Item # CH-1101), designed for Nova Medical rollators. Learn how to attach the cup holder to Express and Star series rollators.</p>
	<p>NOVA Traveler Rolling Walker User Guide</p> <p>Comprehensive user guide for the NOVA Traveler Rolling Walker (Model 4900). Includes essential safety warnings, step-by-step assembly instructions, guidance on attaching accessories, operating the walker, adjusting hand brakes, and details on the product warranty.</p>

Documents - iBstone – Nova

no relevant documents