


 **aspire**  
MWS449872 Document  
Camera



# aspire MWS449872 Document Camera Instructions

[Home](#) » [aspire](#) » **aspire MWS449872 Document Camera Instructions** 

## Contents

- [1 aspire MWS449872 Document Camera](#)
- [2 Specifications](#)
- [3 Product Information](#)
- [4 Product Usage Instructions](#)
- [5 Frequently Asked Questions](#)
- [6 INTRODUCTION](#)
- [7 MATERIAL](#)
- [8 MODELS](#)
- [9 Accessories](#)
- [10 USING INSTRUCTION](#)
- [11 SPECIFICATIONS](#)
- [12 ABOUT COMPANY](#)
- [13 Documents / Resources](#)
  - [13.1 References](#)
- [14 Related Posts](#)



**aspire MWS449872 Document Camera**



## Specifications

- **Material:** Magnesium
- **Weight:** 6.7kg
- **Weight Capacity:** 113kg (18st)
- **Frame Warranty:** 5 years
- **Components Warranty:** 1 year
- **Upholstery:** Teal Mesh
- **Frame Finish:** Gloss or Matte

## Product Information

- The VID A MG wheelchair is constructed from Magnesium, making it ultra lightweight at only 6.7kg. It offers exceptional comfort with an ergonomic contoured seat shape that supports posture and prevents sliding forward, ensuring independence for users.
- The wheelchair comes with various accessories including anti-tippers, lap belt, removable leg rests for compact storage, attendant-operated hand brakes with a parking lock, and a rear storage pocket for essentials.

## Product Usage Instructions

### Assembly and Setup

1. Unfold the wheelchair frame until it locks into place securely.
2. Attach any additional accessories as needed (e.g., leg rests, lap belt).

### Operating the Wheelchair

1. Sit in the wheelchair ensuring proper posture and comfort.
2. Use the hand brakes for control when necessary, and engage the parking lock when stationary.

## Folding and Transportation

1. To fold the wheelchair, release any locking mechanisms and fold the frame carefully.
2. Ensure all accessories are secured and fold the backrest if needed for compact storage.

## Frequently Asked Questions

- **What is the weight capacity of the VIDA MG wheelchair?**

The wheelchair has a weight capacity of 113kg (18st).

- **Can the backrest be adjusted on the wheelchair?**

The backrest is not adjustable but can be folded for transportation and storage.


- **Are there different color options available for the upholstery?**

The VIDA MG wheelchair comes with Teal Mesh upholstery as the standard option.

## INTRODUCTION

- The VIDA MG is made from Magnesium and weighs only 6.7kg making it the lightest and easiest wheelchair to lift and manoeuvre in its class.
- The ergonomic contoured seat shape maximizes comfort and supports posture by preventing users from sliding forward in the seat, making it an ideal solution for maintaining independence.

## MATERIAL

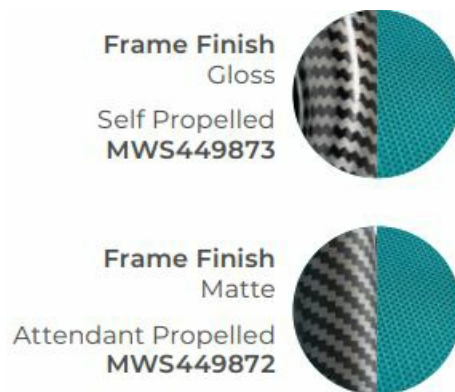

## MODELS



Self Propelled - Gloss  
MWS449873

Attendant Propelled - Matte  
MWS449872

## Accessories



Anti-tippers  
WCA848275



Lap belt  
WCA848300

## USING INSTRUCTION

1. Removable leg rests for compact storage
2. Attendant operated hand brakes with parking lock
3. Rear storage pocket for essentials
4. Intuitive folding frame and hinged backrest make transportation compact and easy!




## SPECIFICATIONS

VIDA MG WHEELCHAIR	Overall Length x Width x Height mm/ in	Seat Depth x Width mm/ in	Anterior x Posterior Seat Height mm/ in	Backrest Height mm/ in	Armrest Height mm/ in	Minimum Chair Weight kg/ lbs	Total Chair Weight kg/ lbs	Rear Wheel in
<b>Attendant Propelled</b> MWS449872	995 x 650 x 930 / 39 x 25.5 x 36.5	420 x 450 / 16.5 x 18	500 x 460 / 19.5 x 18	430 / 17	200 / 8	6.7 / 15	9 / 20	13
<b>Self Propelled</b> MWS449873	1040 x 690 x 950 / 41 x 27 x 37.4	420 x 450 / 16.5 x 18	500 x 460 / 19.5 x 18	430 / 17	200 / 8	6.7 / 15	11 / 24	22

## ABOUT COMPANY

- **Australia**
  - 1300 133 120
  - [aspirecare.com.au](https://aspirecare.com.au)
- **United Kingdom**
  - 01622 541235
  - [aidacare.co.uk](https://aidacare.co.uk)

## Documents / Resources

	<a href="#">aspire MWS449872 Document Camera</a> [pdf] Instructions MWS449872, MWS449873, MWS449872 Document Camera, MWS449872, Document Camera, Camera
---	--

## References

- [Home | Aidacare](#)
- [aspirecare.com](https://aspirecare.com)
- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.