



## B BRUN 131182 Intrafix Handling Poster Instructions

[Home](#) » [B BRUN](#) » B BRUN 131182 Intrafix Handling Poster Instructions 

### Contents

- [1 B BRUN 131182 Intrafix Handling Poster](#)
- [2 Product Information](#)
- [3 Product Usage Instructions](#)
- [4 INSTALLATION](#)
- [5 Contact](#)
- [6 Documents / Resources](#)
  - [6.1 References](#)



**B BRUN 131182 Intrafix Handling Poster**



## Product Information

- The product is an IV-set used for infusion therapy.
- Manufacturer: B. Braun Melsungen AG
- Product Website: [www.safeinfusiontherapy.com](http://www.safeinfusiontherapy.com)
- Manufacturer's Website: [www.bbraun.com](http://www.bbraun.com)
- Manufacturer's Address: 34209 Melsungen, Germany
- Manufacturer's Contact: Tel +49 56 61 71-0
- Date of last revision: March 2020


## Product Usage Instructions

1. Unwrap the IV-set and visually inspect it.
2. Check if the green air vent cap is closed.
3. Check if the roller clamp is open.
4. Spike the semirigid container or bag with the IV-set.
5. Close the roller clamp.
6. Turn the IV-system upside down.
7. Squeeze the drip chamber impulsively to bring the fluid level up to the ring of the drip chamber.
8. Open the roller clamp and prime the line until the end.
9. Close the roller clamp again.
10. Squeeze the drip chamber impulsively once again.
11. Remove the cap from the patient connector and connect it to the patient.


Best practice recommendation to prime Intrafix® Primeline and Intrafix® SafeSet in combination with semirigid container/ bags.

## INSTALLATION

1. Unwrap the IV-set and check whether the green air vent cap is closed and the roller clamp is open.
2. Spike the semirigid container / bag.
3. Close the roller clamp.

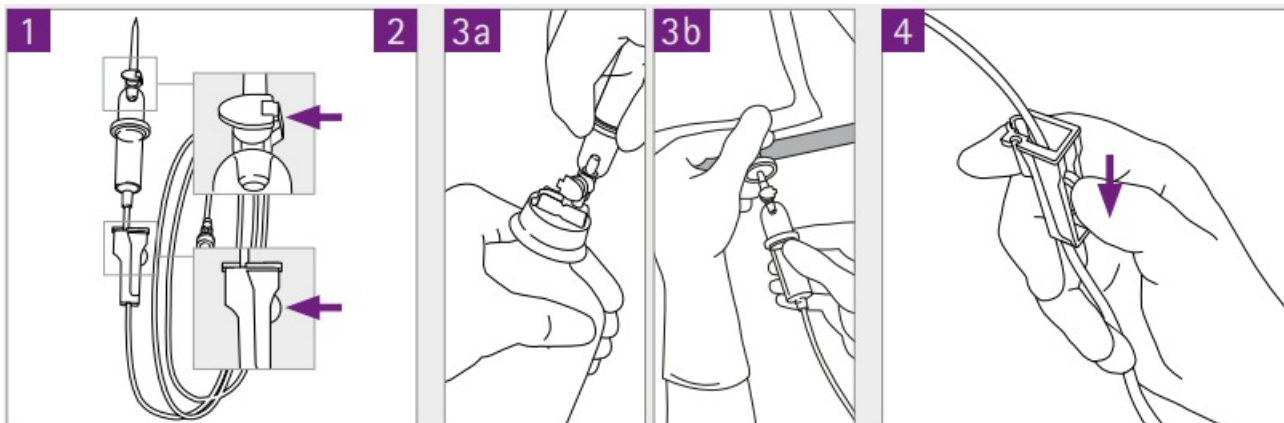

4. Turn the IV-system upside down. Squeeze the drip chamber impulsively and bring fluid level up to the ring of the drip chamber.
5. Open roller clamp. Prime the line until the end. Afterwards close the roller clamp again.
6. Squeeze the drip chamber impulsively once again. Remove cap and connect it to the patient connector.

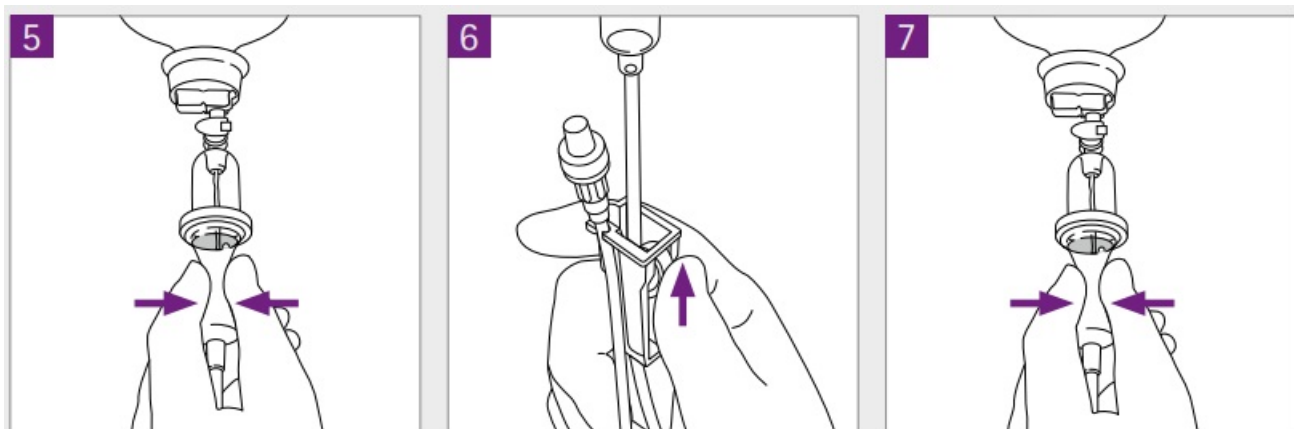

## Contact

- [www.safeinfusiontherapy.com](http://www.safeinfusiontherapy.com)
- B. Braun Melsungen AG
- Hospital Care
- 34209 Melsungen
- Germany
- Tel +49 56 61 71-0
- [www.bbraun.com](http://www.bbraun.com).
- Date of last revision: 03.2020

|                                                                                  |                                                                                                                                                                                    |
|----------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  | <p><a href="#">B BRUN 131182 Intrafix Handling Poster</a> [pdf] Instructions</p> <p>131182, 131182 Intrafix Handling Poster, Intrafix Handling Poster, Handling Poster, Poster</p> |
|----------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|

## References

- [B B. Braun: A leading medical technology company](#)
- [B. Braun for Safety](#)
- [Intrafix® Primeline Comfort Infusion Set | Buy Now](#)
- [User Manual](#)

Manuals+.