



BEMIS 303 3-Gallon Sharps Container with Large Opening Lid Instruction Manual

[Home](#) » [BEMIS](#) » BEMIS 303 3-Gallon Sharps Container with Large Opening Lid Instruction Manual 

BEMIS 303 3-Gallon Sharps Container with Large Opening Lid Instruction Manual



Contents

[1 See instructions for use on back.](#)

[2 INSTRUCTIONS FOR USE](#)

[3 Documents / Resources](#)

[3.1 References](#)

[4 Related Posts](#)

See instructions for use on back.

ATTENTION

Please review these instructions carefully before using this container

- For use in supervised and controlled areas
- Please check that container has not been damaged before placing into use.
- Never hold sharps container below recommended fill line.
- Examine autoclaved parts before handling.
- If container is dropped, inspect container for damage before handling.
- Do not fill sharps container above recommended fill line.
- For single use only.
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

INSTRUCTIONS FOR USE

1. To assemble

- a. Align front of lid (lid opening) to front of base (biohazard label).
- b. Align tabs to holes in base. If they do not align, make sure lid and base are both facing forward.
- c. Press lid to base until all tabs snap into slots and are secure.

2. To mount in bracket

NOTE: This container must be wall-mounted during use.


Mount on wall using wall mounting bracket #423 000 (non-locking) or #425 000 (locking). Gently slide container down into bracket.

3. To use

- a. Using one hand, drop syringe horizontally into opening.
 - b. Check fill level regularly. Replace container when fill level is reached.
4. Temporary & final closure
 - a. Keep lid down when not in use.
 - b. When full, insert tabs on lid into slots and press down firmly.
 - c. Remove container from bracket.

4. Disposal

Dispose of container according to facility policy. May be autoclaved or incinerated

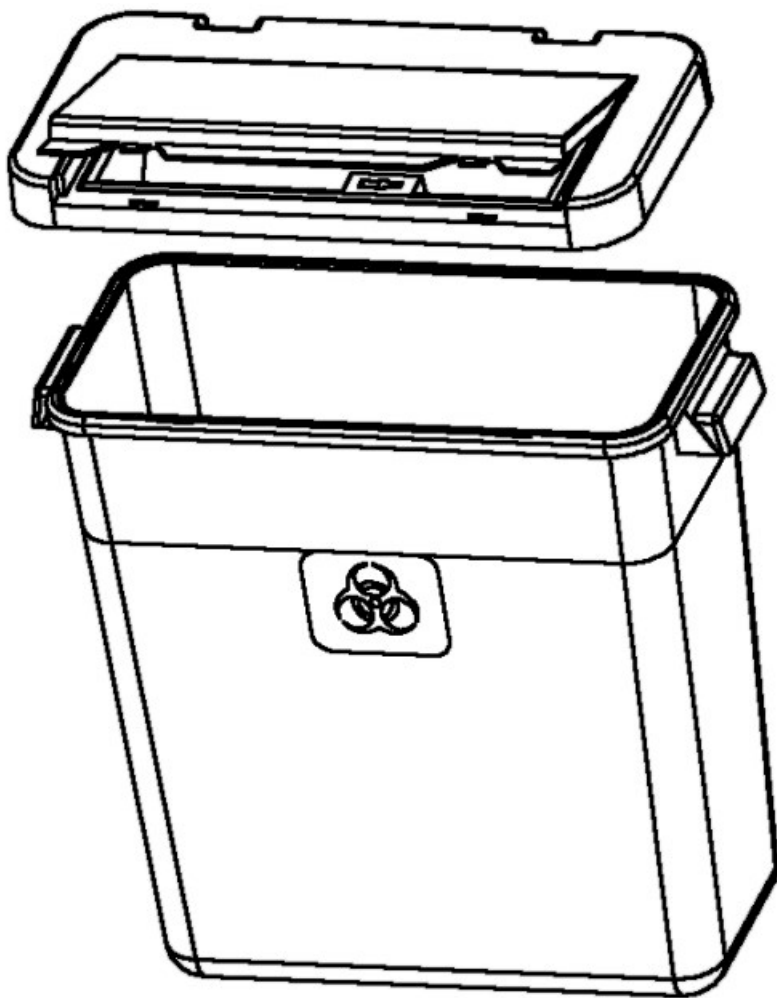


Bemis Manufacturing Co., 300 Mill Street, Sheboygan Falls, WI 53085
920-467-4621 • 800-558-7651 • Fax 800-499-8160
E-mail: HCG@BemisMfg.com • www.bemishealthcare.com

Bemis de Mexico S.A de C.V, Ave. Texas #200, Parque Industrial
Nacional, Cienega de Flores, Nuevo Leon. Mexico C.P. 65550
(52) 81-81545500 • Fax (52) 81-83197884
E-mail: latinamerica@BemisMfg.com



Documents / Resources

	<p>BEMIS 303 3-Gallon Sharps Container with Large Opening Lid [pdf] Instruction Manual 303 3-Gallon Sharps Container with Large Opening Lid, 303, 3-Gallon Sharps Container with Large Opening Lid, 3-Gallon Sharps Container, Sharps Container, Container</p>
--	--

References

- [BEMIS Bemis Health Care](#)