



bleep Eclipse CPAP Interface Starter Kit Instructions

[Home](#) » [Bleep](#) » bleep Eclipse CPAP Interface Starter Kit Instructions 

Contents

- [1 bleep Eclipse CPAP Interface Starter Kit](#)
- [2 Product Information](#)
- [3 Product Usage Instructions](#)
- [4 Features & Benefits](#)
- [5 Instructions](#)
- [6 Contact](#)
- [7 Documents / Resources](#)
 - [7.1 References](#)



bleep Eclipse CPAP Interface Starter Kit



Product Information

The Eclipse™ PAP Solution is a revolutionary product designed to provide a comfortable and effective solution for individuals using CPAP machines. It features the following benefits:

- **Eclipse™ Halos Means No Headgear:** The innovative design eliminates the need for traditional headgear, providing a more comfortable and convenient experience.
- **Eclipse™ Frame – Magnetic No Leak Seal:** The frame of the Eclipse™ PAP Solution utilizes magnetic technology to create a secure seal, preventing leaks and ensuring effective therapy.
- **Improved Arousal Index:** By eliminating the discomfort and restrictions of headgear, the Eclipse™ PAP Solution helps improve the arousal index, resulting in a better quality of sleep.
- **Eliminates Vanity Lines:** With no headgear to cause marks or lines on the face, users can enjoy a more aesthetically pleasing appearance.
- **No Hair Matting/Balding:** The adhesive seals of the Eclipse™ PAP Solution are applied outside the nares, reducing the risk of hair matting or balding.
- **Reduces Nosocomial Infection:** The design of the Eclipse™ PAP Solution minimizes the risk of nosocomial infections, providing a safer and healthier sleep environment.
- **No In-Night Adjustments to Correct Leak:** Users can enjoy uninterrupted sleep as the Eclipse™ PAP Solution eliminates the need for in-night adjustments to correct leaks.
- **No CPAP Caused Dry Eye/Puffy Eye:** Unlike traditional headgear, the Eclipse™ PAP Solution does not cause

dry eye or puffy eye symptoms associated with CPAP therapy.

- Reduces Skin and Bridge Breakdown: The Eclipse™ PAP Solution is designed to reduce skin irritation and breakdown, ensuring long-term comfort and usage.
- Reduces Neck/Facial Pain from Headgear: By eliminating headgear, the Eclipse™ PAP Solution helps alleviate neck and facial pain often caused by traditional CPAP masks.
- Consumer Satisfaction Rating of >80%: The Eclipse™ PAP Solution has received a high consumer satisfaction rating, demonstrating its effectiveness and user satisfaction.

Product Usage Instructions

1. Clean and dry your face before using the Eclipse™ PAP Solution.
2. Gently peel off the adhesive seals from the packaging.
3. Place the adhesive seals on the outside of your nares, ensuring a secure and comfortable fit.
4. Attach the Eclipse™ Frame to the adhesive seals, aligning the magnetic connectors.
5. Adjust the position of the frame to achieve a proper seal and comfortable fit.
6. Connect the tubing from your CPAP machine to the designated port on the Eclipse™ Frame.
7. Turn on your CPAP machine and adjust the settings according to your prescribed therapy.
8. Ensure that the Eclipse™ PAP Solution is securely in place before going to sleep.
9. After waking up, gently remove the adhesive seals and dispose of them properly.
10. Clean the Eclipse™ Frame and tubing according to the manufacturer's instructions.

Please note that it is essential to consult the complete user manual for detailed instructions and safety information before using the Eclipse™ PAP Solution.

Features & Benefits

Eclipse™ Halos Means No Headgear

- Adhesive seals outside of the nares
- Eliminates Vanity Lines
- No hair matting/balding
- Reduces Nosocomial Infection
- Reduces Skin and Bridge Breakdown
- Reduces Neck/Facial Pain from Headgear
- Single night use – clean application nightly

Eclipse™ One Size Fits All

- Inventory/SKU Reduction
- Single Night Use/Forced Replenishment
- Perfect fit out of the box
- For Use with Facial Hair

Eclipse™ Frame – Magnetic No Leak Seal

- Improved Arousal Index
- Improved Quality of Sleep
- Improved Compliance
- No In-Night Adjustments to Correct Leak
- No CPAP caused Dry Eye/Puffy Eye
- Consumer satisfaction rating of >80% vs. <20% for standard CPAP mask


Light Weight/Low Profile Design

- Increased Comfort
- Reduces Claustrophobia
- Vent Diffuser – Super Quiet
- Smallest In Market
- Super Flexible Tube
- Quiet, Leak Free means no waking bed partner


Instructions

“Eclipse innovation is expected to be a game changer for patients,”

Mary Susan Esther, MD, past president of the American Academy of Sleep Medicine and Data Safety Monitoring Board Chairperson, stated in a release.

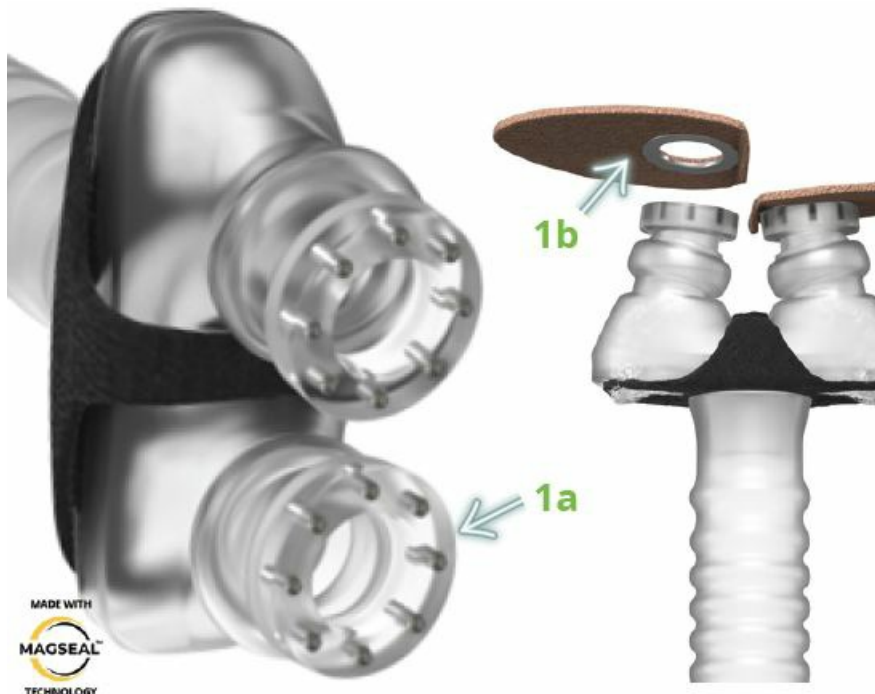

- Eclipse uses MagSeal™ Magnetic technology (each port has a 65 mT magnetic strength) to ensure a clinically proven leak free seal. Bleep received an innovation grant from the National Institute of Health for Eclipse. The Eclipse Frame has 2 nostril ports. Each port contains 8 magnets (1a) which adhere to stainless steel rings (1b) on each Halo (nightly disposable nasal pillows).

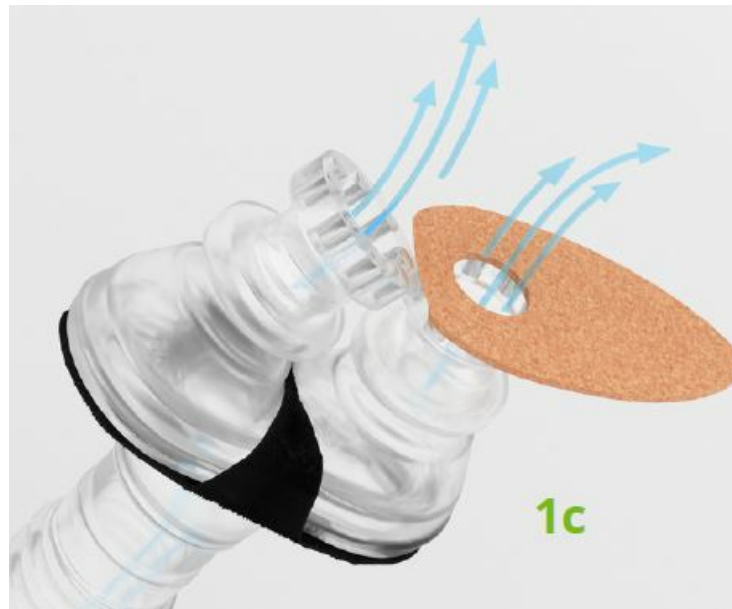

- Nothing enters the nares. Gentle, surgical grade adhesive creates a seal on the nose, allowing unrestricted airflow. (1c)

“Forced Replenishment, better patient comfort, a win-win solution”

Reimbursement Codes

- Eclipse Frame bills as A7034 Mask and is reimbursable 1x per quarter = 4/year
- Halo disposable nasal pillows bill as A7033 and are reimbursable 6x per quarter = 24 annually

Magnets used in the Eclipse™ are within ICNIRP guidelines for general public use. Some active medical implants – such as pacemaker or defibrillator – may be affected by localized magnetic fields. The Eclipse magnets in this mask should be kept at least 2 in. (50 mm) away from any active medical implant

Contact

- 11312 US 15-501 North, Suite 107-206, Chapel Hill, NC 27517
- (919) 619-7170
- bleepsleep.com

©2017 Bleep. All rights reserved.

Documents / Resources

