



Contents [[hide](#)]

- [1 CareCo DL02030008 Task Alert Clock](#)
- [2 Product Information: Task Alert Clock](#)
- [3 GETTING STARTED](#)
- [4 SAFETY GUIDELINES](#)
- [5 SETTING UP YOUR TASK ALERT CLOCK](#)
- [6 HOW TO USE YOUR TASK ALERT CLOCK](#)
- [7 BUTTON FUNCTIONS](#)
- [8 SETTING ALARM\(S\)](#)
- [9 ALARM OPTIONS \(EXAMPLE ALARM TITLES\)](#)
- [10 SETTING UP THE DIGITAL PHOTO FRAME](#)
- [11 SPECIFICATIONS](#)
- [12 WARRANTY](#)
- [13 FAQ](#)
- [14 Documents / Resources](#)
 - [14.1 References](#)



CareCo DL02030008 Task Alert Clock



Product Information: Task Alert Clock

Specifications

- Product Code: DL02030008
- Power Supply: USB C
- Time Formats: 12-Hour, 24-Hour
- Date Formats: Day-Month-Year, Month-Day-Year, Year-Month-Day
- Text Types: Lower Case, Upper Case

CareCo (UK) Limited, 1 Turing Court, Great Notley, Braintree, CM77 7AT.

- Tel: 0333 015 5000
- www.careco.co.uk

Thank you for choosing CareCo.

This manual can be viewed in PDF format at the following URL: www.careco.co.uk

GETTING STARTED

CARTON CONTENTS

ITEM NUMBER	DESCRIPTION	QUANTITY
1	Dementia Hub – Task Alert Clock	1
2	USB C Power Supply	1
3	User Manual	1

Contact CareCo immediately if any of the listed contents are missing.



We strongly recommend that you keep the box and internal packaging for the duration of your warranty period.

SAFETY GUIDELINES

- Keep product away from children.
- Do not use the clock near a magnetic/electric field.
- Do not place the clock in direct sunlight or extreme heat for extended periods of time.
- Do not use the clock if it has been damaged or modified.
- Do not place the clock in water.
- Do not attempt to disassemble, repair or open the clock.



SETTING UP YOUR TASK ALERT CLOCK

1. Remove all components from the outer packaging and place them onto a level surface.
2. Connect the power supply to the clock, then insert the 3-pin plug into a wall socket.

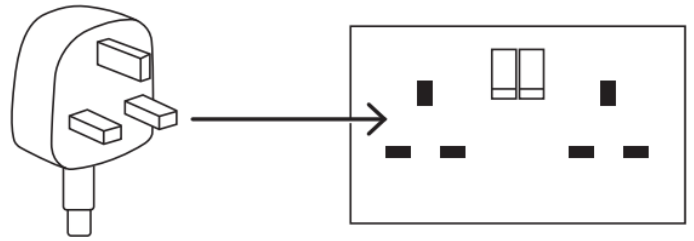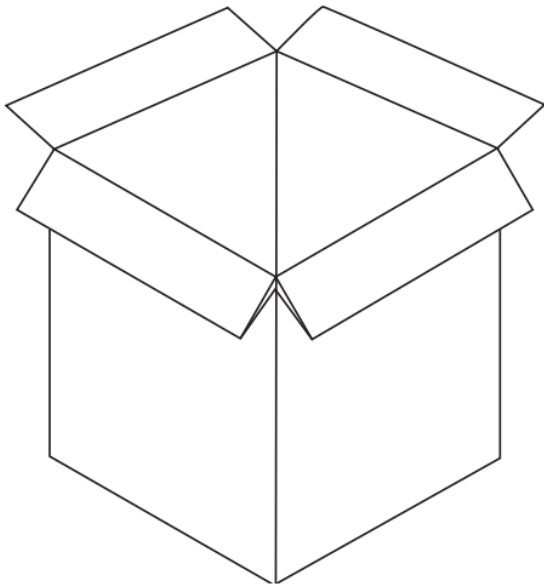

POSITIONING YOUR TASK ALERT CLOCK

- (Optional) Unfold the kickstand, then place the clock in the desired position.
- (Optional) Mount the clock onto a wall using suitable fixings (not included).

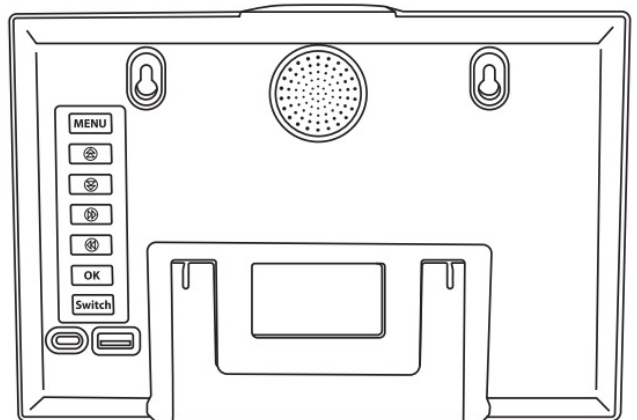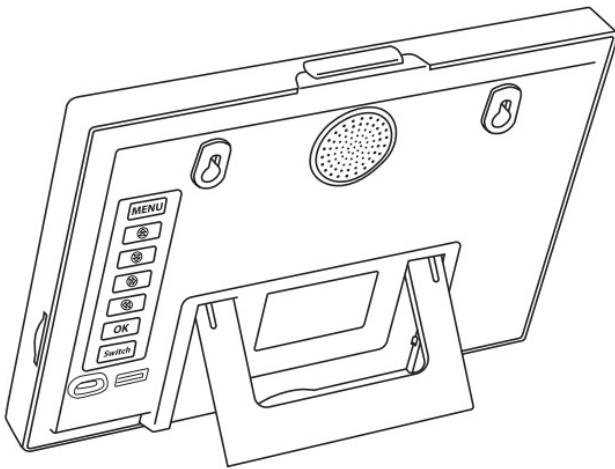

HOW TO USE YOUR TASK ALERT CLOCK


MENU OPTIONS

Language Set	<	English	>
Time Set	<	09:09:53	>
Date Set	<	01-01-2023	>
Time Mode	<	24 Hour	>
Date Mode	<	Day-Month-Year	>
Auto Dimming	<	Auto	>
Volume Set	<	28	>
Text Type	<	Lower Case	>
Alarm Set	<	OK	>
Medicine Reminder	<	OK	>
Daylight Saving Time	<	Off	>
Factory Reset	<		>

LANGUAGE SET 1. English 2. Deutsch 3. Francais 4. Espanol 5. Italiano 6. Portugues 7. Polski 8. Nederlands 9. Finnish 10. Magyar	DATE MODE 1. Day-Month-Year 2. Month-Day-Year 3. Year-Month-Day	ALARM SET 21 Mode Alarm Clock
TIME SET Current Time FORMAT: 0 0:00:00	AUTO DIMMING 1. Off 2. Auto 3. Mid 4. Low	MEDICINE REMINDER 4 Interval Medicine Reminder Alarm

DATE SET Current date • FORMAT: DD/MM/YYYY	VOLUME SET 1. Min – 00 2. Max – 30	DAYLIGHT SAVING TIME 1. OFF 2. UK 3. USA 4. EU
TIME MODE 1. 12-Hour 2. 24-Hour	TEXT TYPE 1. Lower Case 2. Upper Case	FACTORY RESET

BUTTON FUNCTIONS

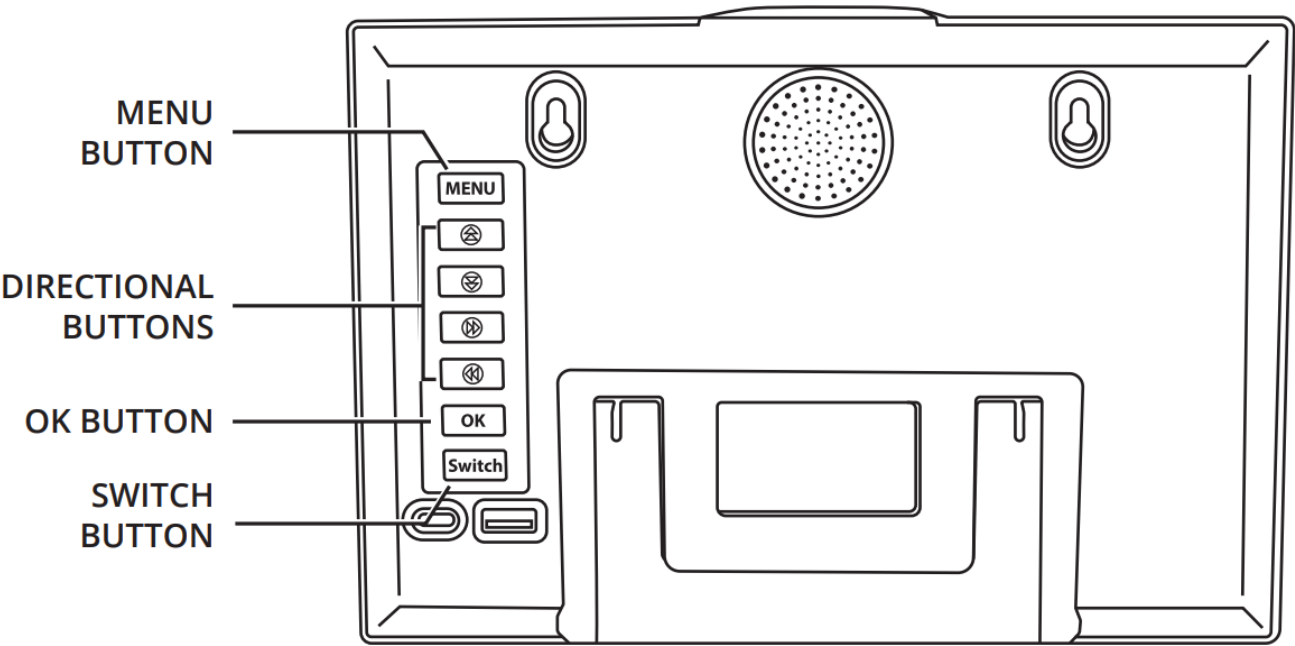

MENU

MENU BUTTON

Press to navigate to the Menu Screen and return to the Home screen.



DIRECTIONAL BUTTONS

Press to navigate within the Menu screen and change values.

OK

OK BUTTON

Press to select and save values within the Menu screen.


Switch

SWITCH BUTTON

Press to cycle between different Home screen options.

CHANGING THE SCREEN LAYOUT

Press the [SWITCH] button to cycle between four different screen layouts.

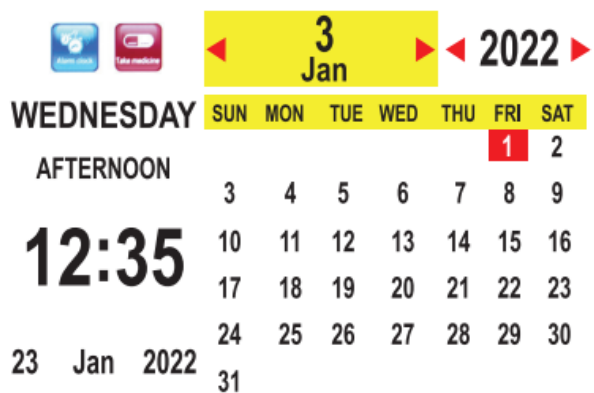

SETTING ALARM(S)

INSTRUCTIONS

1. Press the [MENU] button to access the clock Menu.
2. Press the [DIRECTIONAL] buttons to navigate to ALARM SET within the clock Menu.
3. Press the [OK] button to edit.

The alarm menu will appear.

4. Press the [DIRECTIONAL] buttons to navigate the alarm menu.

5. Press the [OK] button to edit.

The item that is currently being edited will be highlighted.

4. 6. Press the [DIRECTIONAL] buttons to cycle between HOURS/MINUTES and ALARM OFF/ON settings.

7. Press the [OK] button to save the settings.

Repeat the above steps to add additional alarms.

ALARM OFF/ON SETTINGS

- OFF (Never)
- Monday to Sunday (Everyday)
- Saturday to Sunday (Weekends ONLY)
- Monday to Friday (Weekdays ONLY)

ALARM OPTIONS (EXAMPLE ALARM TITLES)

1. Alarm 1
2. Alarm 2
3. Alarm 3
4. Alarm 4
5. Alarm 5
6. Time for Breakfast
7. Time for Lunch
8. Time for Dinner
9. Wake Up
10. Doctor's Appointment Today
11. Time for a glass of water
12. Time for a cup of coffee
13. Walk the dog
14. Blood sugar test
15. Time for bed
16. Feed the dog

17. Insulin Management time
18. Measure your Blood Pressure
19. Go to church
20. Water your plants
21. Haircut Appointment

TURNING OFF THE ALARM

Press any button to turn [OFF] the alarm OR wait for 20 seconds for the alarm to turn [OFF] automatically.

SETTING MEDICINE REMINDER(S)

INSTRUCTIONS

1. Press the [MENU] button to access the clock Menu.
2. Press the [DIRECTIONAL] buttons to navigate to MEDICINE REMINDERS within the clock Menu.
3. Press the [OK] button to edit.
The Medicine Reminder menu will appear.
4. Press the [DIRECTIONAL] buttons to navigate the medicine reminder menu.
5. Press the [OK] button to edit the desired reminder time.
6. Press the [DIRECTIONAL] buttons to cycle between HOURS/MINUTES, OFF/ON and MODE SETTINGS.
7. Press the [OK] button to save the settings.

MEDICINE REMINDER SETTINGS

- OFF (Never)
- Monday to Sunday (Everyday)
- Saturday to Sunday (Weekends ONLY)
- Monday to Friday (Weekdays ONLY)

SETTING UP THE DIGITAL PHOTO FRAME

IMAGE FORMAT	VIDEO FORMAT
--------------	--------------

<ul style="list-style-type: none"> • JPG • JPEG 		<ul style="list-style-type: none"> • MPG • MPEG • DAT • VOB • AVI • MP4 	<ul style="list-style-type: none"> • MKV • MOV • 3GP • FLV • RMVB
---	--	---	--

1. Copy photo/video to a USB Flash Drive/SD Card (Not included)
2. Insert the USB Flash Drive/ SD Card into the port on the clock to begin playing the media.

Return to the chosen screen layout at any time by pressing the [MENU] button.

SPECIFICATIONS

SCREEN SIZE (IN)	8
SCREEN RESOLUTION	800×600
POWER CONSUMPTION	8W
USB OUTPUT	5V / 1A
SUPPORTED MEDIA	PHOTO/VIDEO
SPEAKERS	DUAL 2W
INSTALLATION OPTIONS	DESKTOP
WEIGHT (KG)	0.37KG

RECYCLING & DISPOSAL

Recycling or disposal of the clock should be completed through an approved distributor or disposal facility. (Local disposal & recycling regulations may apply).

Contact CareCo if further advice is needed regarding recycling & disposal.

WARRANTY

If a component requires repair or replacement due to a specific manufacturing or material defect within its warranty period, it will be repaired or replaced free of charge by CareCo.

1-YEAR LIMITED WARRANTY

CareCo is not responsible if the clock requires repair or replacement as a direct result of:

- Changes or modifications made to the product that fall outside of the manufacturer's specifications.
- The product not being serviced or maintained according to the manufacturer's recommendations contained in this user manual.
- Negligence, Accidental Damage, and Improper use.
- Repairs attempted before CareCo have been notified.
- The product being purposely damaged or mistreated.

In the unlikely event that this product is recalled, CareCo will notify the user. Please ensure that your contact information is up to date.

Alternatively, please visit our website for any product alerts: www.careco.co.uk

CareCo (UK) Limited, 1 Turing Court, Great Notley, Braintree, CM77 7AT

Tel: 0333 015 5000

www.careco.co.uk


FAQ

How do I set the alarm on my Task Alert Clock?

Follow these steps to set the alarm:

1. Press the MENU button to access the clock Menu.
2. Navigate to ALARM SET using the DIRECTIONAL buttons.
3. Edit the alarm by pressing OK.
4. Use the DIRECTIONAL buttons to adjust the settings.

Documents / Resources

	<p>CareCo DL02030008 Task Alert Clock [pdf] User Manual</p> <p>DL02030008 Task Alert Clock, DL02030008, Task Alert Clock, Alert Clock , Clock</p>
---	---

References

- [User Manual](#)

🔍 Alert Clock, CareCo, Clock, DL02030008, DL02030008 Task Alert Clock, Task Alert

📁 CareCo Clock

—Previous Post

[CareCo Osmo BA01020008 Independent Bathlift Owner’s Manual](#)

—

Leave a comment

Your email address will not be published. Required fields are marked *

Comment *

Name

Email

Website

☐ Save my name, email, and website in this browser for the next time I comment.

Post Comment

[Manuals+](#), [Privacy Policy](#) | [@manuals.plus](#) | [YouTube](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.