



careium Enzo Long Range Radio Trigger Instructions

[Home](#) » [careium](#) » careium Enzo Long Range Radio Trigger Instructions 



**Long Range Radio Trigger
Instructions**




Contents

- [1 Enzo Long Range Radio Trigger](#)
- [2 Safety information](#)
- [3 Environmental information](#)
- [4 Documents / Resources](#)
 - [4.1 References](#)
- [5 Related Posts](#)


Enzo Long Range Radio Trigger

Battery replacement guide


1

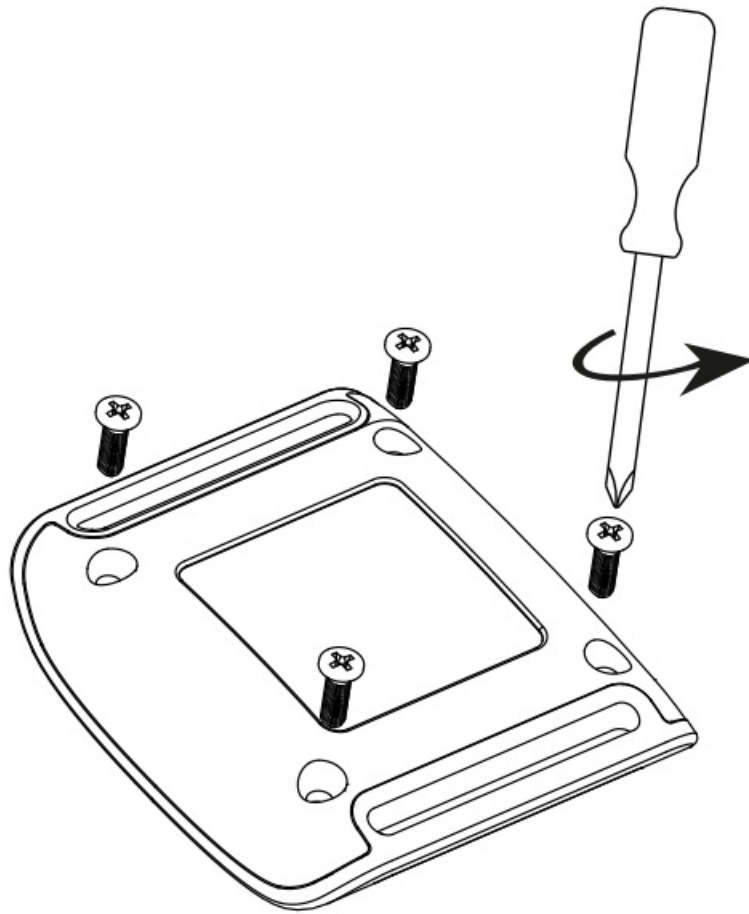

2

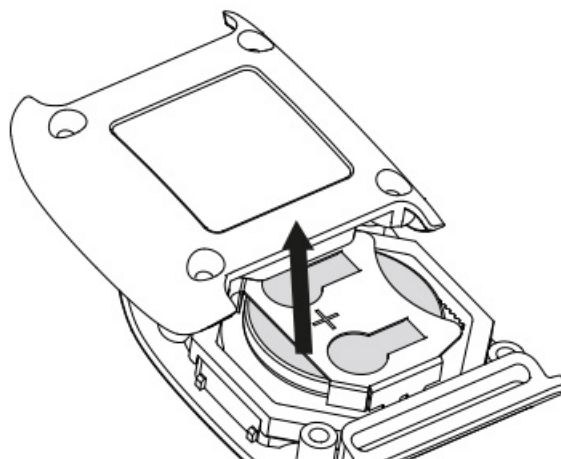

3

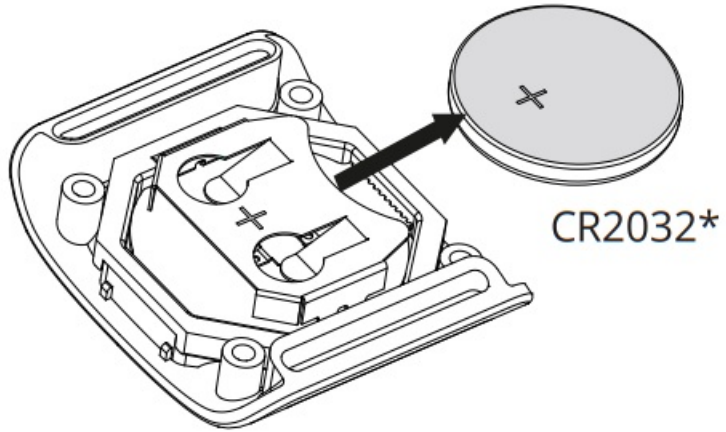


4




60 s.


5

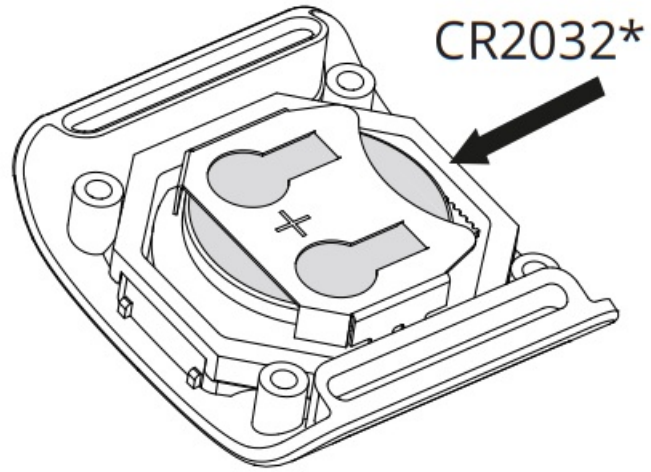


6

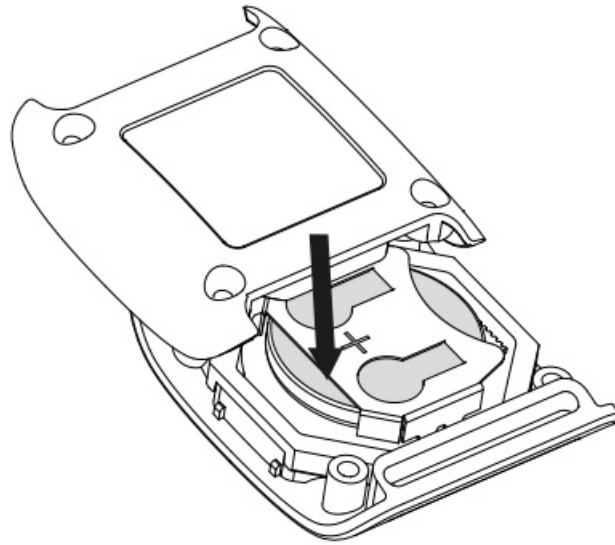

7

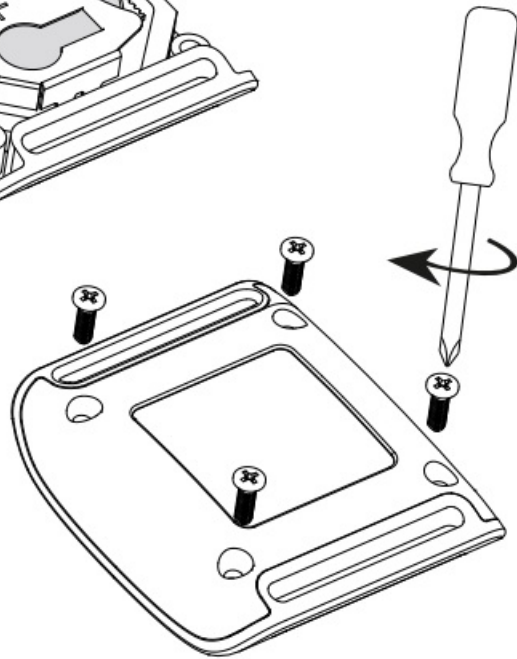

8

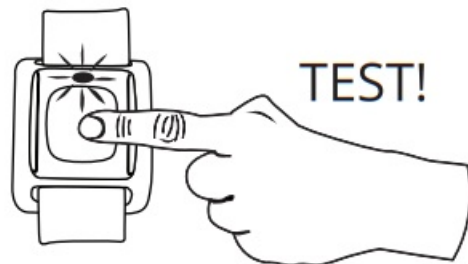

Only use a high-quality battery, CR2032, provided by Cardium.

Safety information



Battery replacement may be performed in accordance with the instructions. Only use recommended

battery type as stated in this manual (CR2032*). CAUTION – Risk of explosion if the battery is replaced by an incorrect type.

Environmental information

This product complies with the requirements of the EU directive 2006/66/EC (Batteries) and 2012/19/EU (WEEE) and should not be disposed of with other household waste. Always recycle your used electronic products, batteries, and packaging materials to dedicated collection points.



Enzo, battery replacement

Version 3.0

©2020. All rights reserved

www.careium.com

Documents / Resources

A thumbnail image showing a series of numbered steps (1-8) for battery replacement. It includes diagrams of the device's back cover being removed and a CR2032 battery being inserted into the compartment.	<p>careium Enzo Long Range Radio Trigger [pdf] Instructions Enzo, Long Range Radio Trigger, Range Radio Trigger, Radio Trigger, Enzo, Trigger</p>
A thumbnail image showing a series of numbered steps (1-6) for using the device. It includes diagrams of hands holding the device, pressing buttons, and the device emitting a signal.	<p>careium Enzo Long Range Radio Trigger [pdf] User Manual Enzo Long Range Radio Trigger, Enzo, Enzo Radio Trigger, Long Range Radio Trigger, Radio Trigger, Trigger</p>

References

- [Careium, formerly Doro Care](#)
- [DofC](#)