



ConvaTec ESteem Flex Convex Instructions

[Home](#) » [ConvaTec](#) » ConvaTec ESteem Flex Convex Instructions 

Contents


- [1 ConvaTec ESteem Flex Convex](#)
- [2 HOW TO USE THE CLIPLESS TAIL CLOSURE](#)
- [3 Documents / Resources](#)
- [4 Related Posts](#)



ConvaTec ESteem Flex Convex



HOW TO USE THE CLIPLESS TAIL CLOSURE

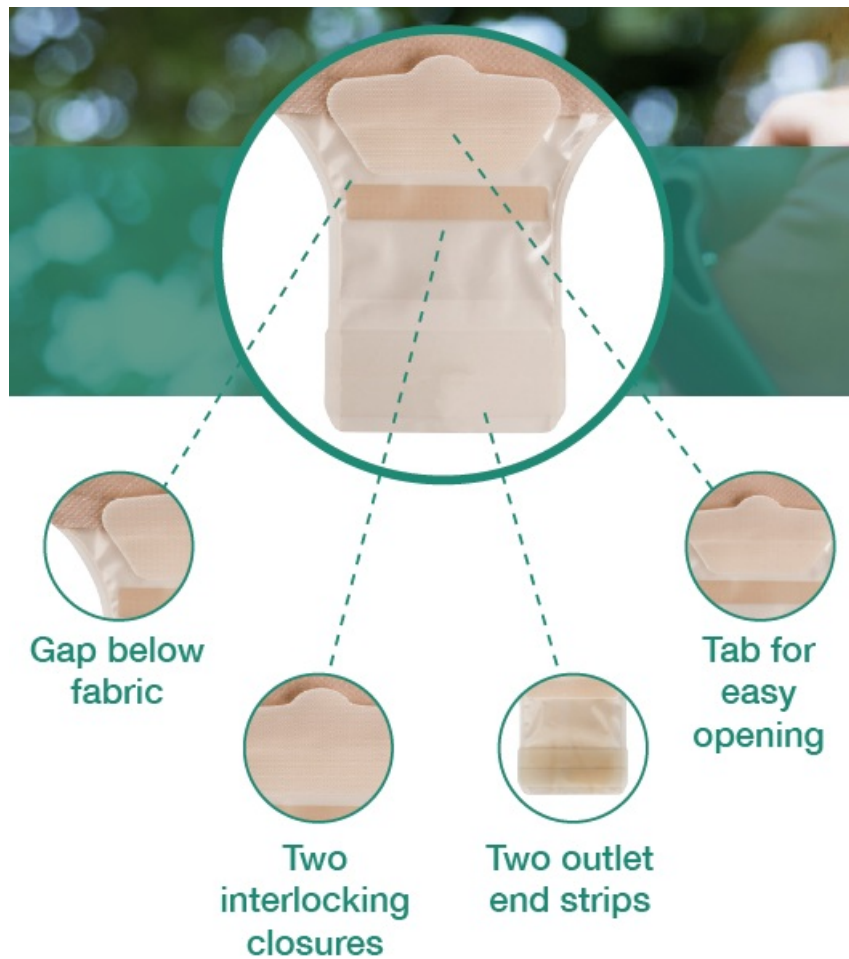


CLOSING THE POUCH


- **Step 1:** Holding the outlet end strips, fold the end up towards you until the interlocking closures line up.



- **Step 2:** Press the interlocking closures together across the width of the tail. Press and pinch across the closure until you feel them close.

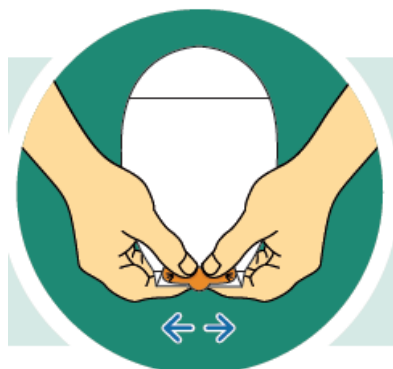

- **Step 3:** Place finger under the gap below the fabric and tuck in the closure, if desired.

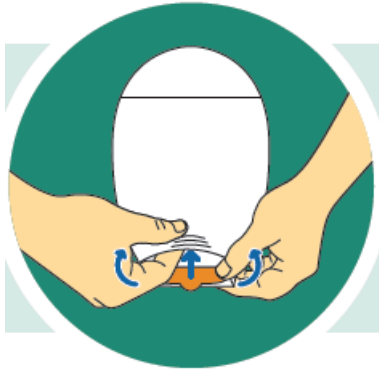

EMPTYING AND CLEANING THE POUCH

- **Step 1:** Remove closure from the gap below the fabric.



- **Step 2:** To empty the pouch, tilt the tail up towards your body. Peel open the closure and unfold the tail.



- **Step 3:** Holding the pouch tail with one hand, open the tail by pushing in both ends of the outlet end strips with the other hand to drain the pouch.



Tip: You can pull up on the tab to help open tail.


- **Step 4:** Clean the inside surface of the pouch tail with toilet tissue or a wipe. The pouch tail can also be cleaned gently with pH neutral soap and water.



For more information, call us on – 0800 282 254

Please refer to the package insert for complete instructions for use. TM ® indicates trademarks of ConvaTec Inc.
© 2016 ConvaTec Inc. AP-016789-MM

Documents / Resources

	<p>ConvaTec ESTeem Flex Convex [pdf] Instructions ESTeem Flex Convex, ESTeem, ESTeem Convex, Flex Convex</p>
--	--