

## coxo DB-686 Nano 001 Curing Light User Manual

[Home](#) » [coxo](#) » coxo DB-686 Nano 001 Curing Light User Manual 

### Contents




- 1 coxo DB-686 Nano 001 Curing Light
  - 1.1 Apex Locator Mode
- 2 Battery and Charging
  - 2.1 Battery Power
  - 2.2 Charging
  - 2.3 Replacement Battery
- 3 EMR (Electric Measurement of Root Canal Length)
- 4 Maintenance
  - 4.1 Check with Tester
  - 4.2 Check Canal Measurement Function
- 5 Documents / Resources
- 6 Related Posts

# COXO<sup>®</sup>

**Professional Dental Manufacturer**

**coxo DB-686 Nano 001 Curing Light**

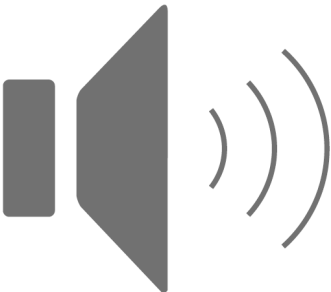

D2		Clockwise/Counterclockwise rotary
E		Control unit power
F		Control unit sound

### File Selection


The instrument presents a library of files of major brands. User can choose to use without setting.

#### 1. Manufacturer and NITI system


- Press icon to enter the NITI system, choose manufacturer and systems by pressing NITI system name

according to your needs.

- Press Direction keys to switch pages and press OK key to exit selection.

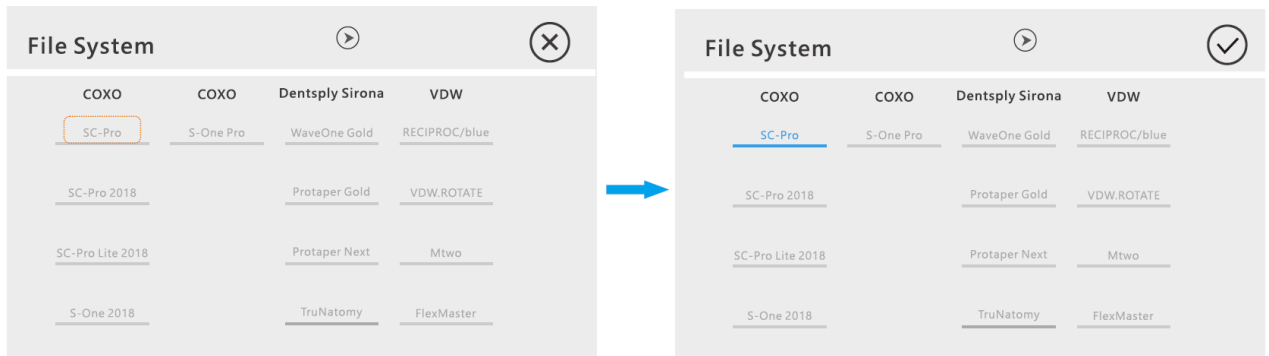


- After the selection is completed, the status bar of the main page displays the selected NITI system.

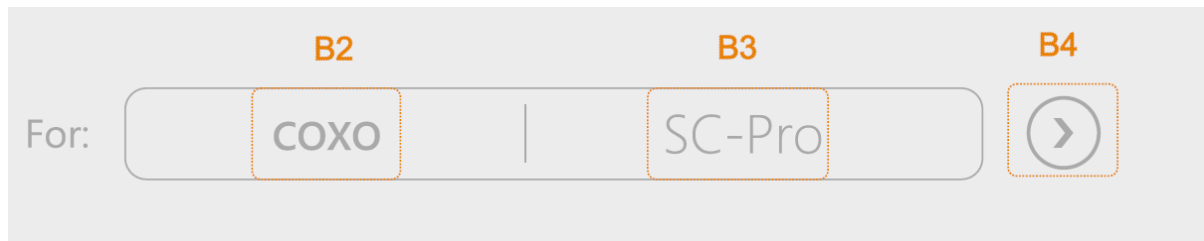

- B2 File manufacturer
- B3 NITI system name
- User can also press Right Direction key (B4) to select different NITI system under the same manufacturer.

## 2. File Selection

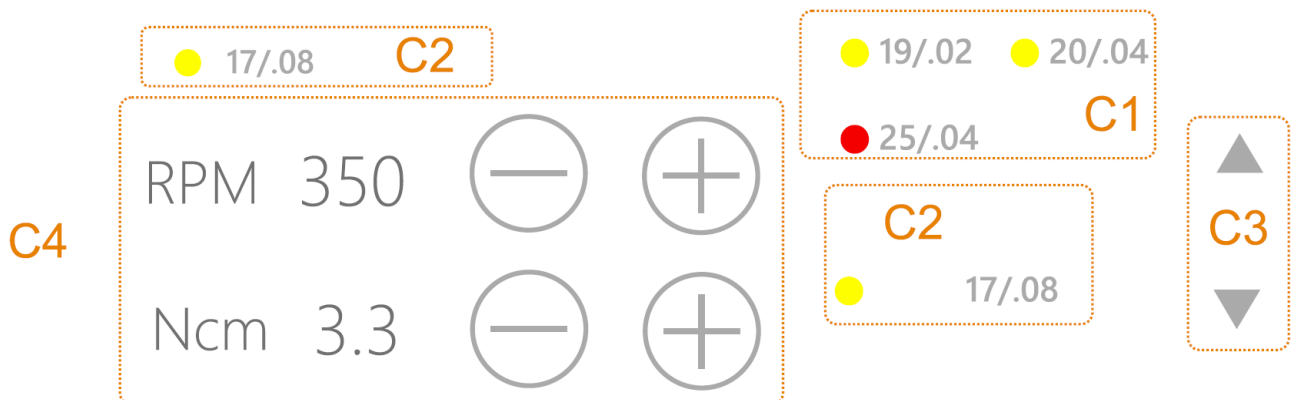

- C1 File list in the current NITI system
- C2 File(s) selected for use
- User can press UP and DOWN keys (C3) to select file(s)
- C4 Speed and torque value of the currently selected file(s), change speed and torque.

## Handpiece LED Light

Press Settings to enter setup state.



1. Press High Brightness icon to change the LED status as below.

When icon is disabled the LED is off, otherwise the LED will light up when the motor is running

2. Press button as below to select delay time.



0s delay



1s delay



3s delay




5s delay

After the motor stops running, the LED will continue to light up until the delay time expires.


### Torque Reverse

1. Press OFF to turn on torque reverse.


During operation, when load reaches preset torque limit value, motor will automatically reverse. When the load is reduced, the motor will automatically return to forward rotation.

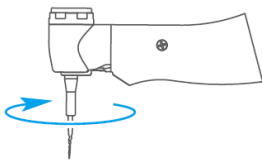

Load within the  
torque limit value.

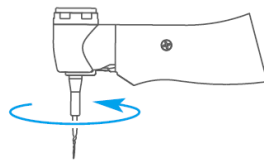

Load beyond the  
torque limit value.

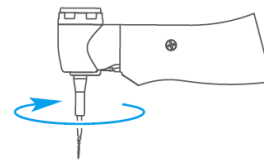

When the load is reduced, the  
motor is automatically restored.

2. Press ON to turn off torque reverse.

- During operation, when load reaches preset torque limit value, motor will stop.
- Restart handpiece to restart the motor.

### Rotary Direction

- Before working, press Clockwise / Anti-Clockwise buttons on the screen or long press icon on the handpiece to change rotary direction.
- During working, user can only change the direction by long press icon on the handpiece.

### Start Working

Press icon, motor rotates according to the set speed and rotary direction, press again to stop it.

● 19/.02 ● 20/.04 ● 25/.04

RPM 350

Ncm 3.0

Torque reverse on



● 17/.08

● 19/.02 ● 20/.04

● 25/.04

● 25/.06

● 35/.04




When the torque exceeds the set limit, the motor will stop or automatically reverse.

### Change Speed and Torque

Usually, the default settings do not need to be adjusted; however, user can modify the default speed and torque values.

- Press – and + on the screen to adjust speed or torque.
- When the setting is changed, Rpm (Ncm) will become Rpm (Ncm).

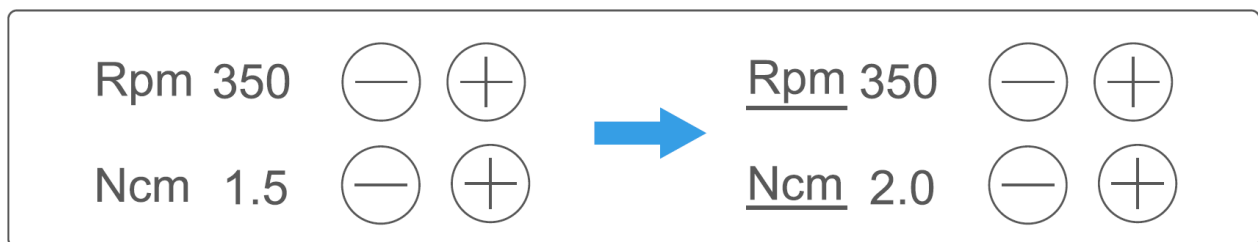

### User System


The instrument contains two user's systems: Rotary File and Recipro File.

#### 1. Rotary File

There are 5 continuous rotary files in the system and user can set speed and torque as required.

#### 2. Recipro File

There is a reciprocation rotary file in the system and user can set speed and rotation angle as required.

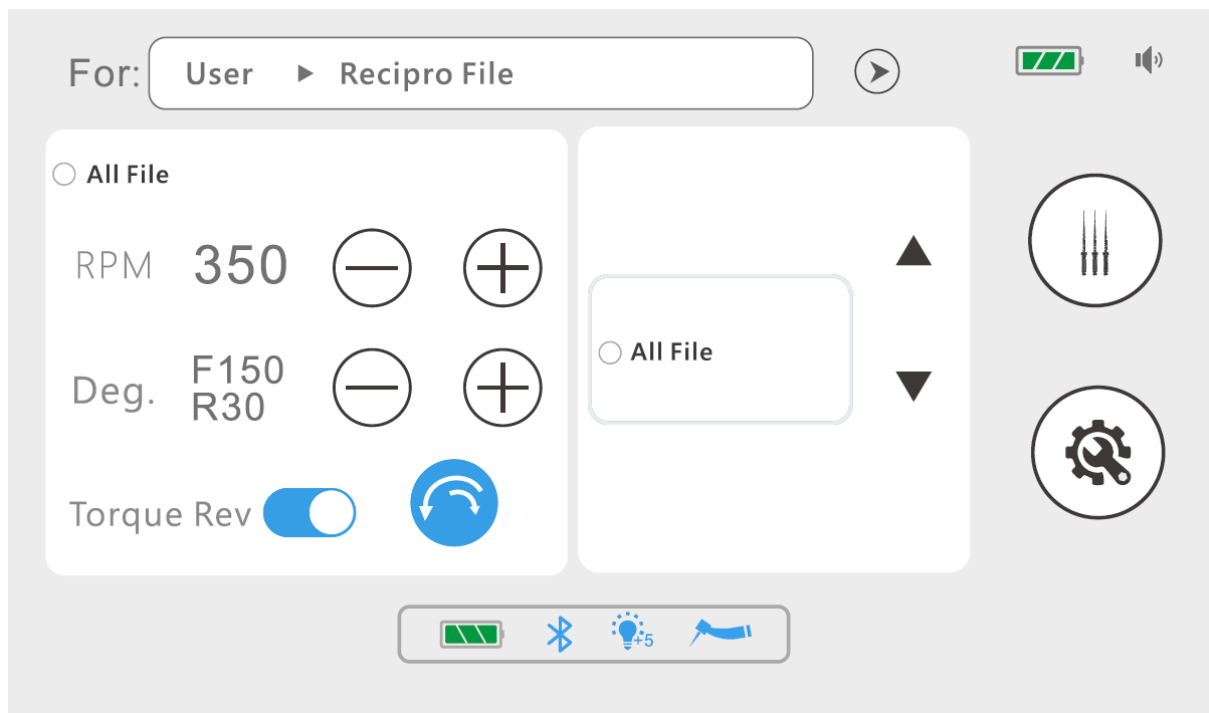


### Apex Locator Mode



If the root apex test wire is connected to hand piece, it enters the Apex Locator Mode automatically.






### Connection Accessories

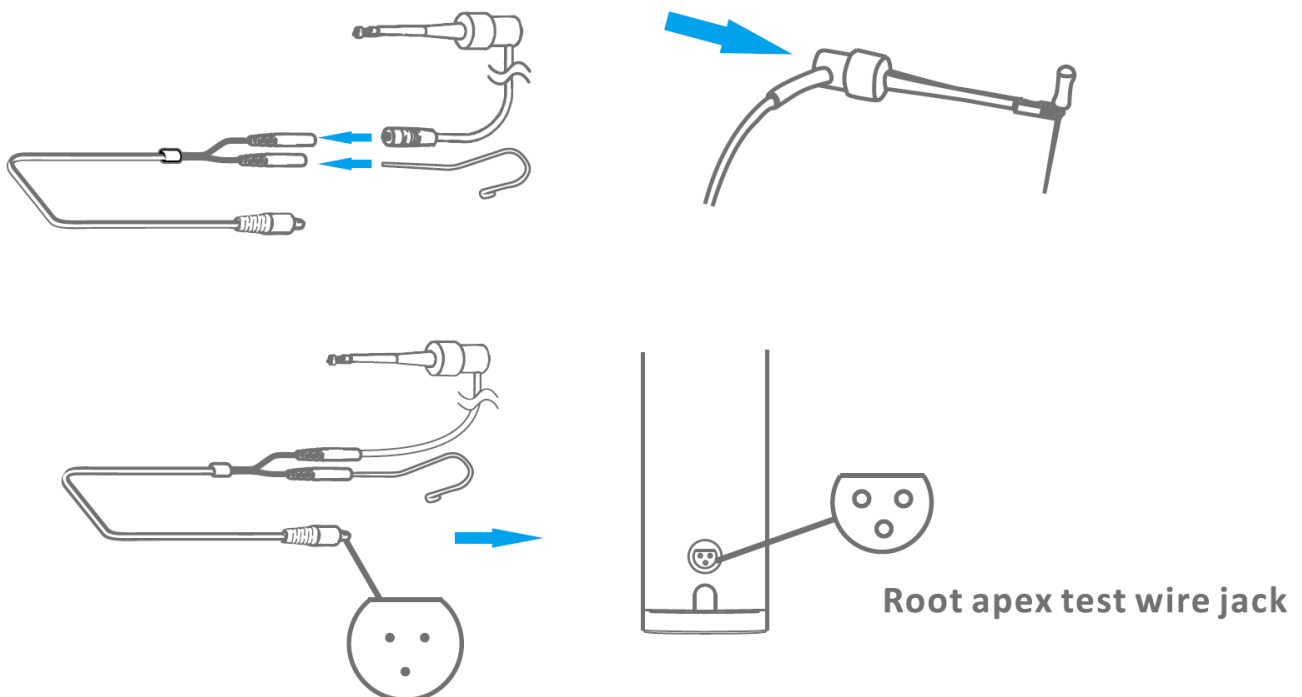Connect the components: lip Hook, file clip, root apex test wire and handpiece.

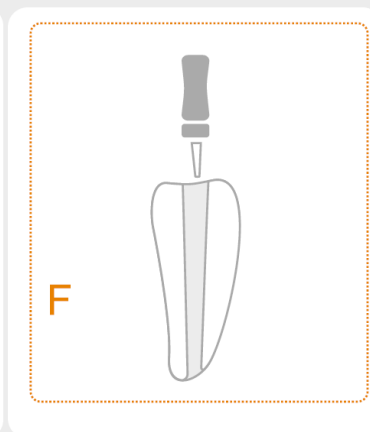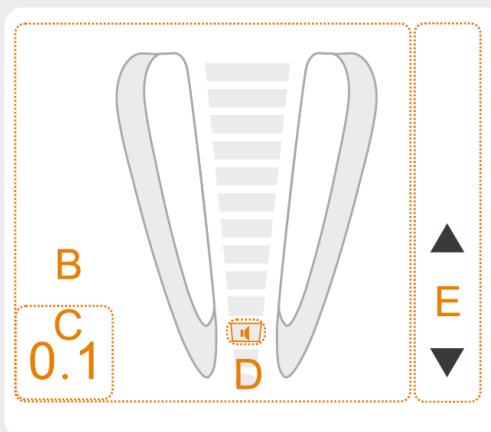

### Interface Overview

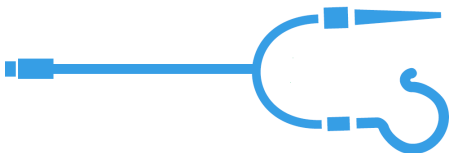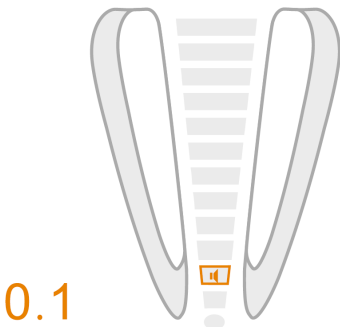

A		Apex Locator Function
B		Root apex area, shows the root apex area and indicate where file was reached
C, D		Apical reference position (The number represents the relative position of file tip and apical hole, the smaller the value, the closer to the apical hole)
E		Set the DR's CHOICE apex position
F		Complete root canal image Shows file entering the root canal opening

### Set the DR's CHOICE Apex Position

Press UP or DOWN on the screen to set the DR's CHOICE apex position, and then clear visual and audio indication is given that file has reached the pre-selected position.


### Operation Check

It is recommended to check the performance of the apex locator once a week, the detailed methods refer to 8.1 and 8.2.

### Measurement




- Hook the lip hook in the corner of the patient's mouth and slowly insert the file into the canal.
- The file movement in the canal is shown on the full canal image as below.



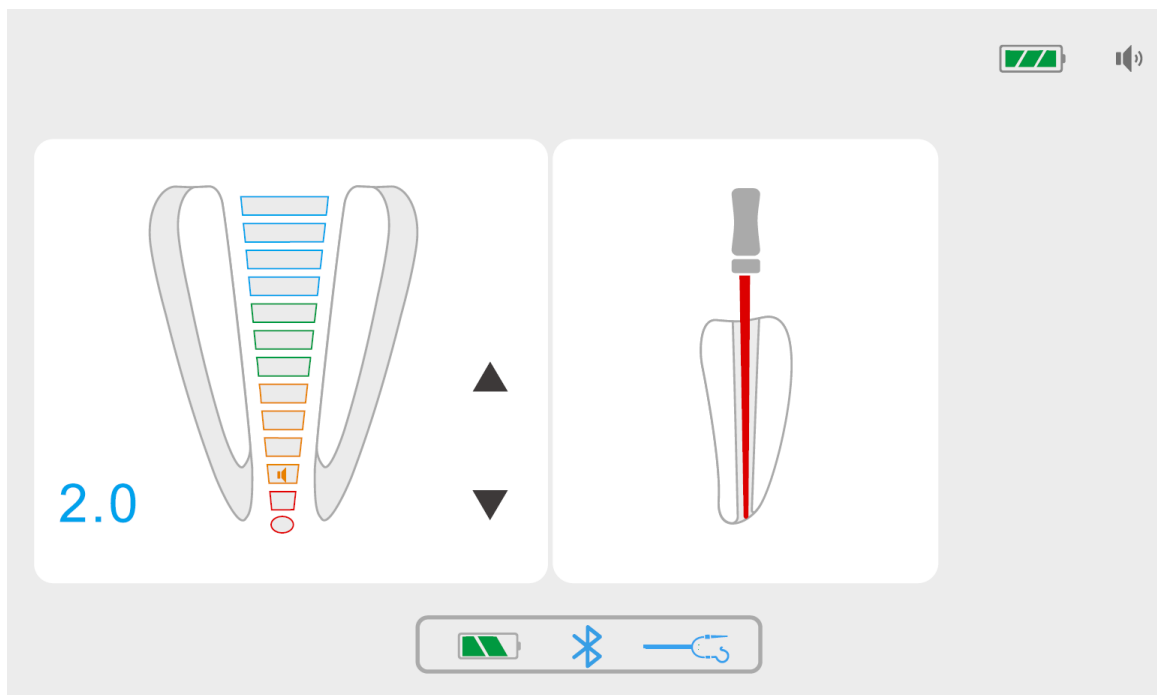

### Apex Enlargement Area:



- Indicators indicate the exact position and change accordingly from blue to green and then to yellow as shown below.

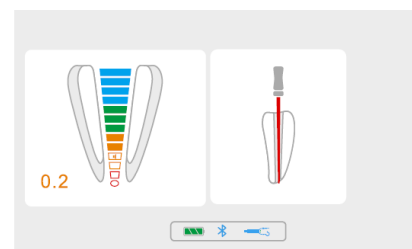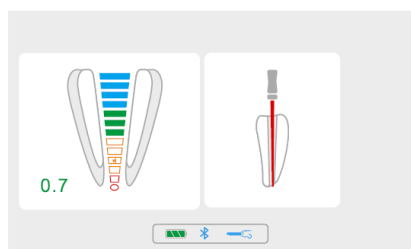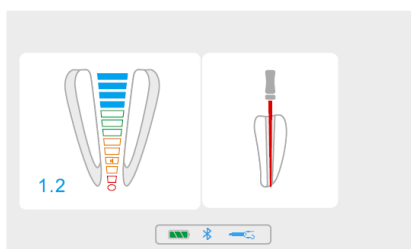

Blue (Close to Apex Area)

Green to yellow (Reach Apex Area)


- File movement is accompanied by audio signals as additional indication of file position. The shorter the sound interval, the closer the distance between file and apex.
- When file reaches apical foramen indicator is marked red and a constant sound is emitted.

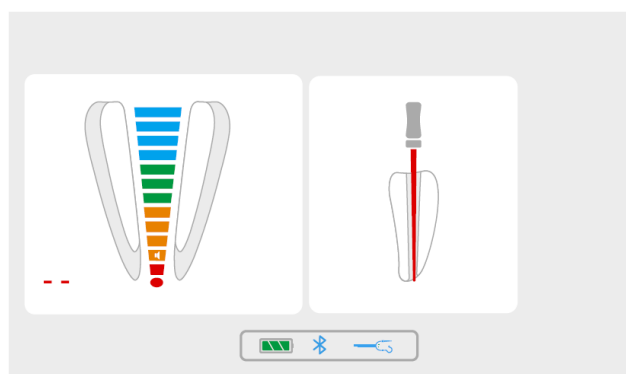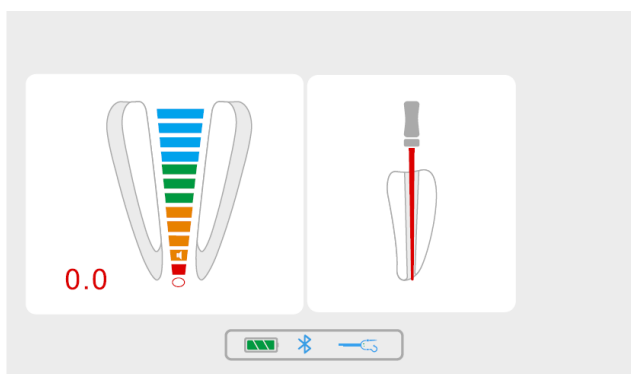

Red (Located in apex foramen)

Red ( Beyond apex foramen)

- During measurement, the color of Apex locator indicator is displayed in synchronization with indicator bar's as shown below:

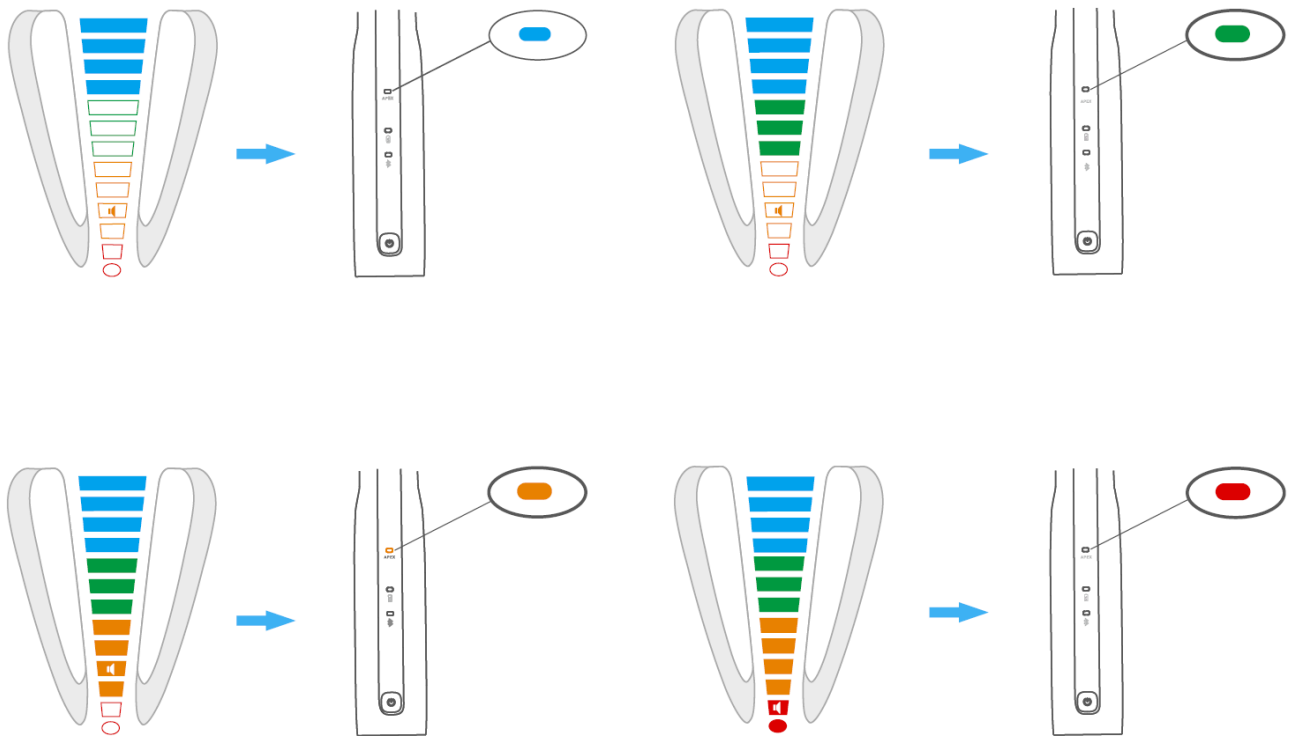

## Electric Measurement of Root Canal Length


### Multi-Function Mode

If lip hook wire is connected to handpiece, it enters the Multi-Function Mode automatically.



### Connection Accessories

Connect the components: handpiece, contra-angle, file, lip hook, lip hook wire.

- Connect contra-angle
- Connect file
- Connect lip hook and lip hook wire

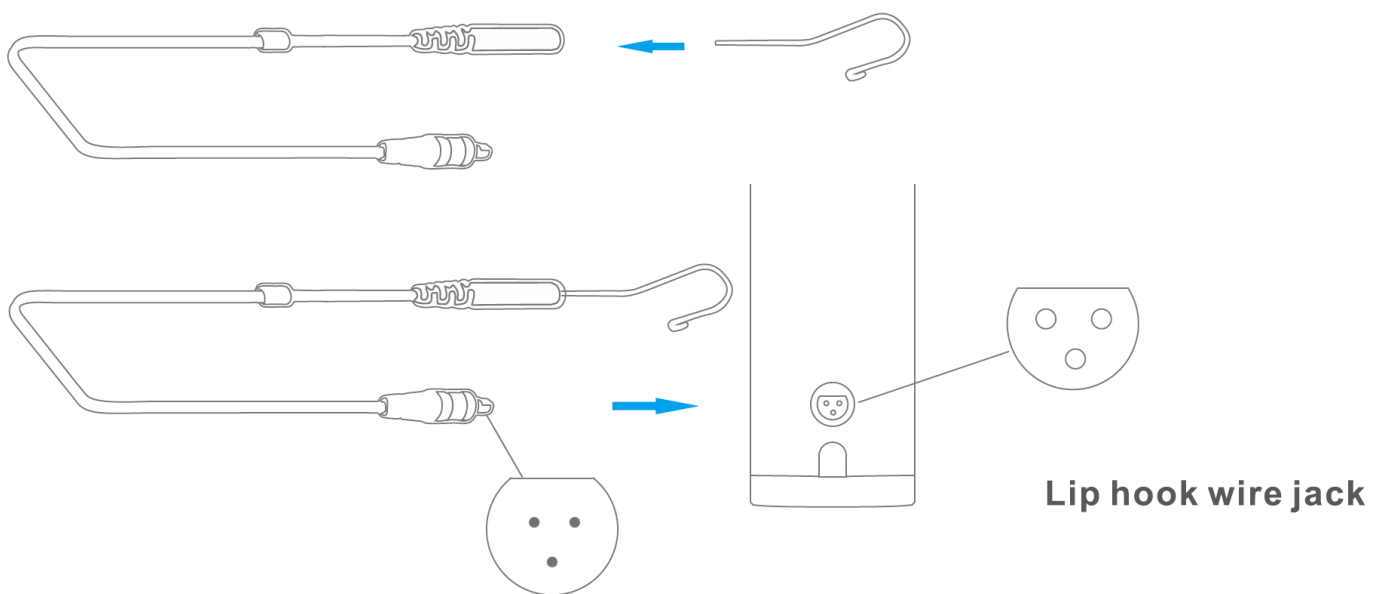


## Interface Overview

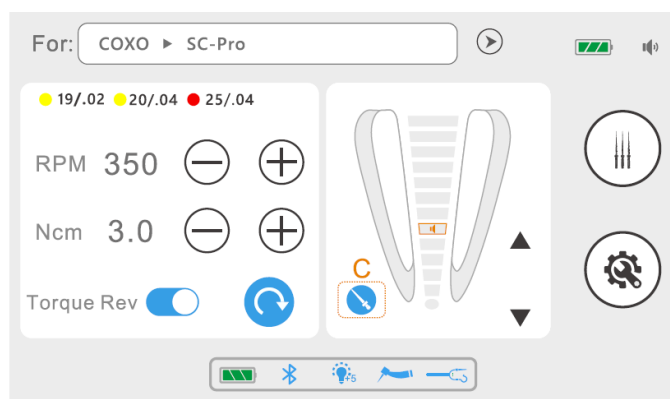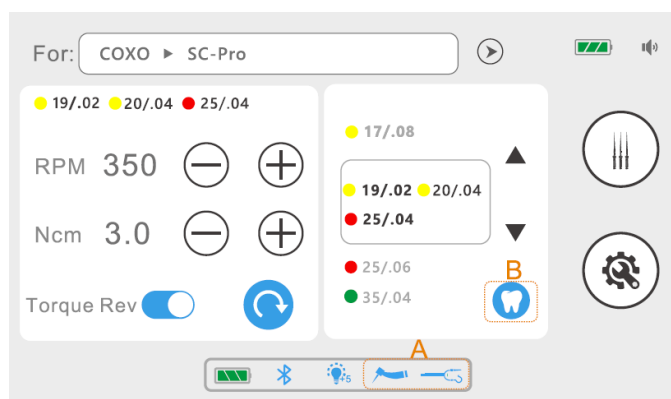

A		Endo Motor and Apex Locator Function
B		Press to enter root apex display interface
C		Press to enter file selection interface


## Start Working

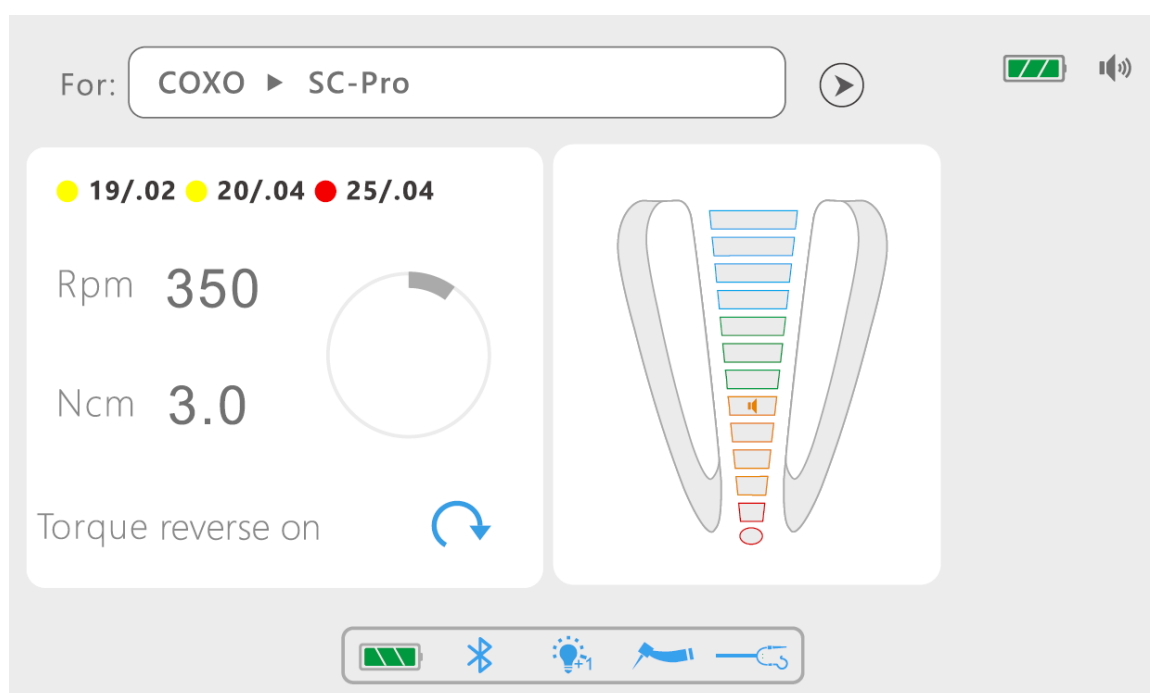

- In multi-function mode. When the file is contact with root canal, length is measured. The movement of the file is displayed on the right side of the screen, accompanied by different prompt sounds.

## Battery and Charging

### Battery Power

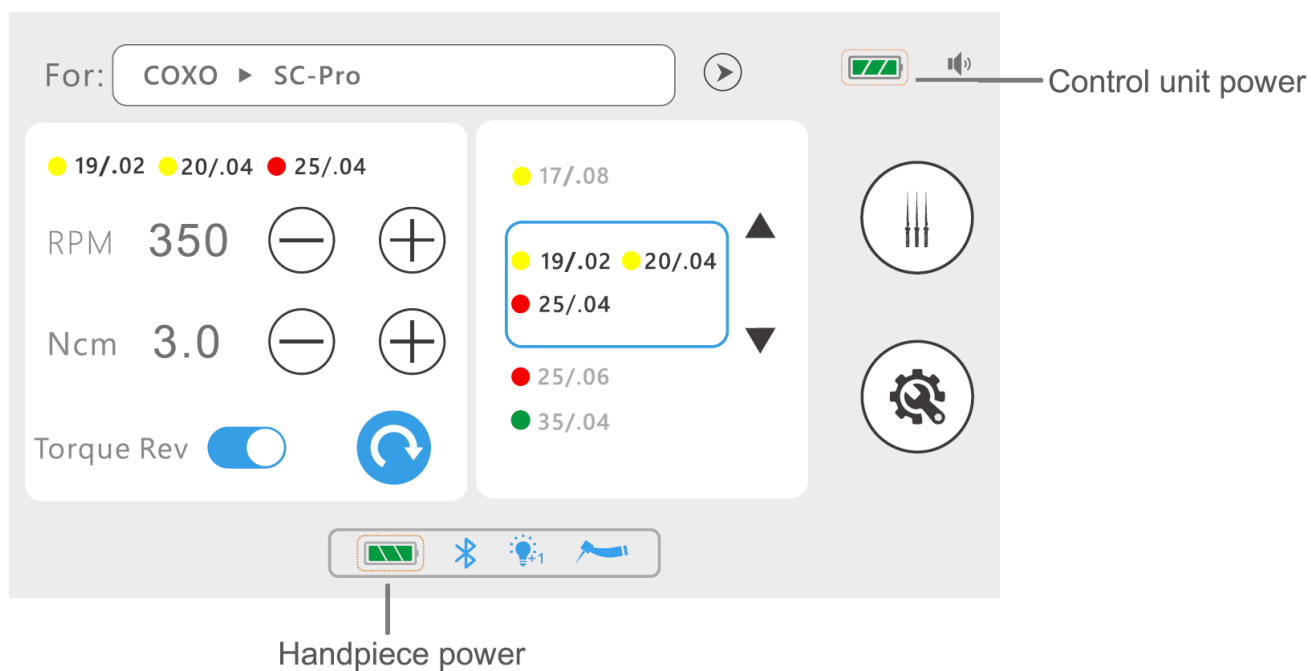

### Battery:


- >70% power



- 20-70% power



- <20% power, low battery level, please charge it in time.

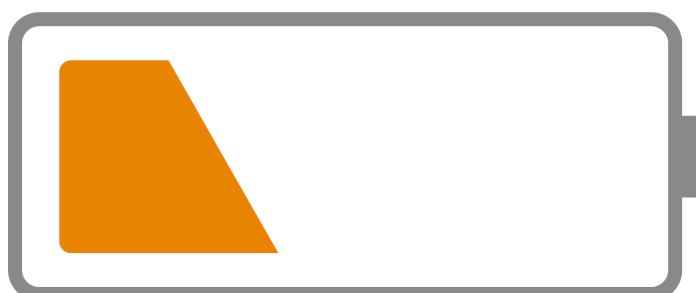


- <5% power, very low battery level.



## Charging

1. Control Unit :Connect hand piece and adapter to charge the handpiece, the charging status is shown as follows:

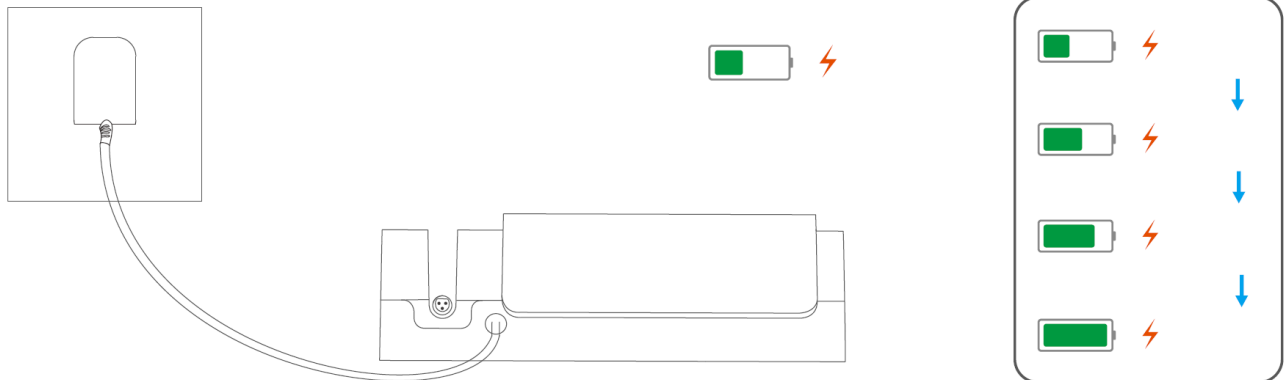


2. Handpiece:

External charging: Connect control unit and adapter, and place handpiece on handpiece charger to charge;

Internal charging: Control unit can charges handpiece without external input.


- Press setup icon to enter setup state and then press Off / On to turn on/ off internal charging.

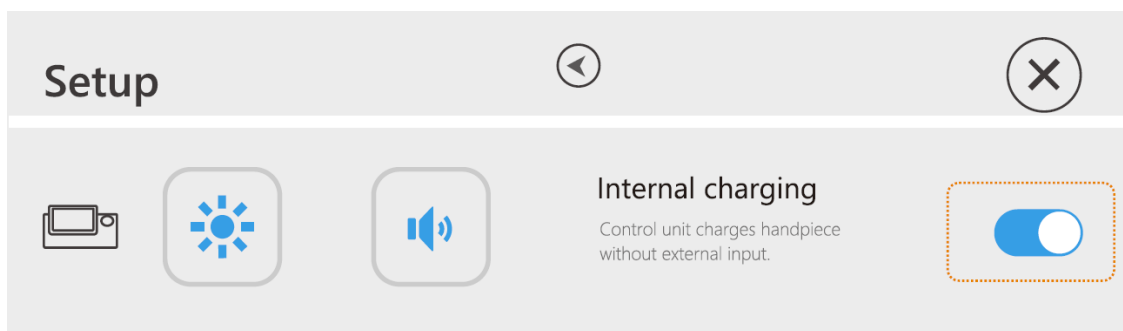


- The function turns off when control unit 'battery level is less than 20%.

## Replacement Battery

1. Open the rubber cover or remove the screw;
2. Remove the battery cover as shown in the illustration;
3. Remove the old battery ;
4. Connect the new battery ;
5. Install the cover and its screw or rubber cover.

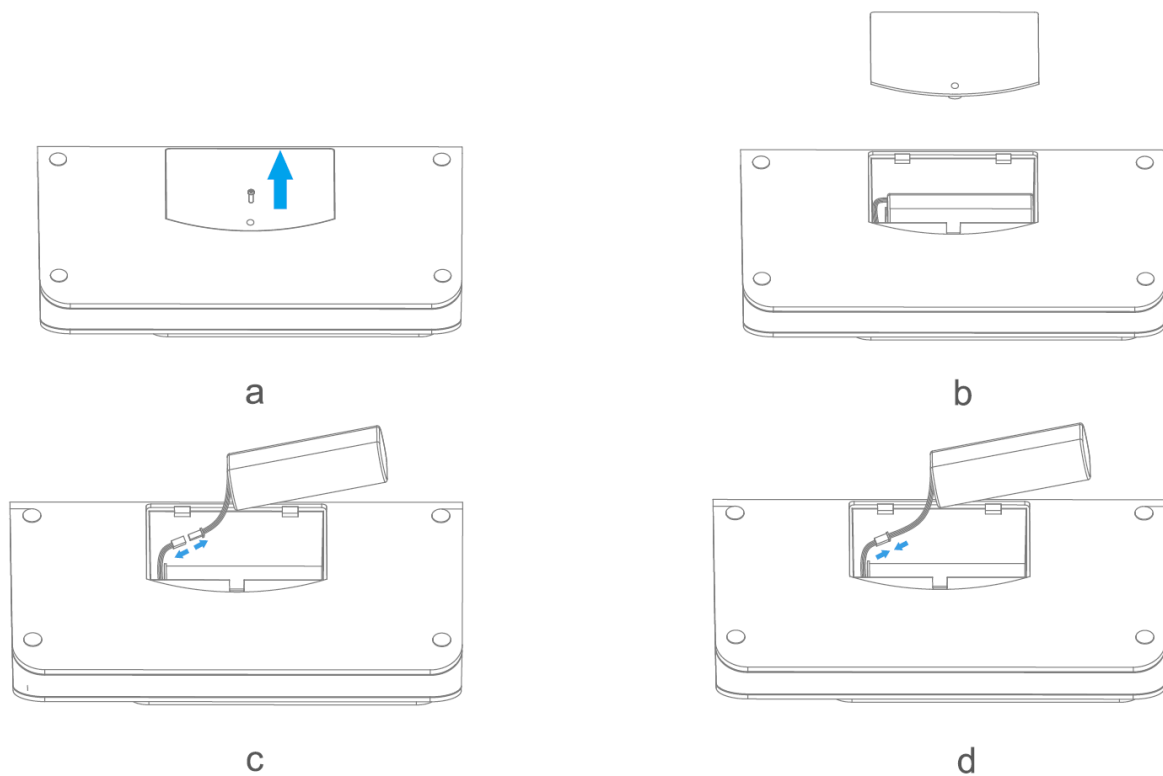

**Control Unit**

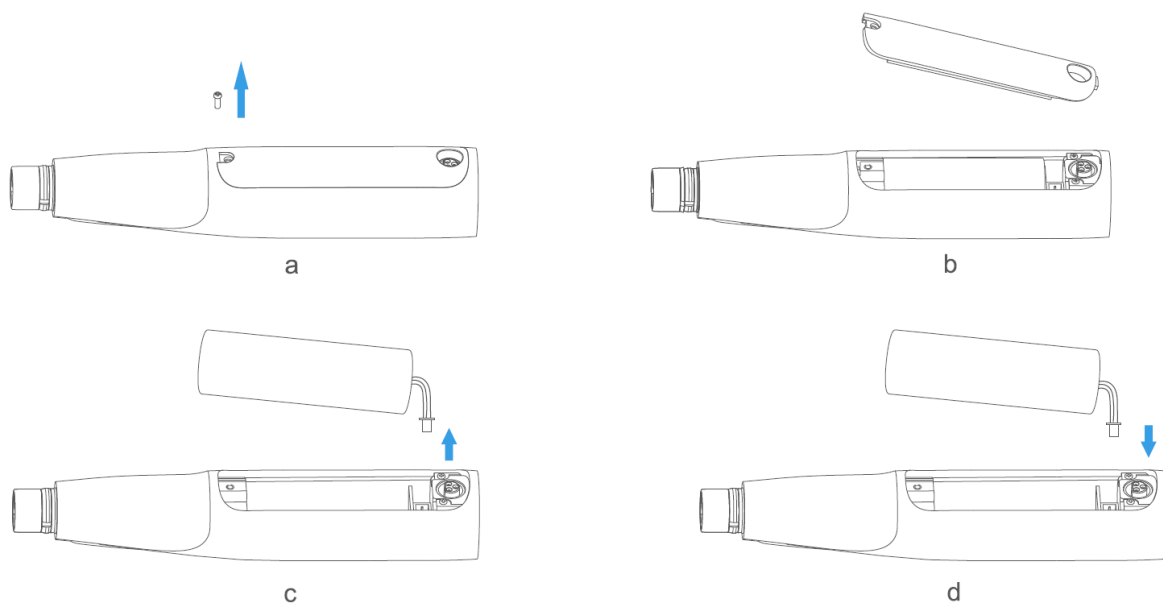

**Hand piece**

## EMR (Electric Measurement of Root Canal Length)

Accurate measurement cannot be obtained with the root canal conditions shown below.

### A huge apical hole

Root canal that has an exceptionally large apical foramen due to a lesion or incomplete development cannot be accurately measured.

Root canal with blood overflowing from the opening

If the blood overflows and contact with the gums, it will cause electrical leakage, and accurate measurement results cannot be obtained. After the bleeding has completely stopped, thoroughly clean the inside of the root canal and the opening, and then perform the measurement.

Root canal with a chemical solution overflowing from the opening

An accurate measurement cannot be obtained if chemical solution is overflowing from the canal opening. In this case, clean the canal and its opening. It is important to get rid of the solution overflowing the opening.

### Broken crown

If the crown is broken and a section of the gingival tissue intrudes into the cavity surrounding the canal opening, contact between the gingival tissue and the file will result in electrical leakage and an accurate measurement cannot be obtained. In this case, build up the tooth with a suitable material to insulate the gingival tissue.

### Fractured tooth

Leakage through a branch canal

Fractured tooth will cause electrical leakage and an accurate measurement cannot be obtained.

A branch canal will also cause electrical leakage.

### Re-treatment of a root filled with gutta-percha

The gutta-percha must be completely removed to eliminate its insulating effect. After removing the gutta-percha, pass a small file all the way through the apical foramen and then put a little saline in the canal, but do not let it overflow the canal opening.

Crown or metal prosthesis touching gingival tissue

Accurate measurement cannot be obtained if the file touches a metal prosthesis that is touching gingival tissue. In this case, widen the opening at the top of the crown so that the file will not touch the metal prosthesis before taking a measurement.

Cutting debris on tooth

### Pulp inside canal

Thoroughly remove all cutting debris on the tooth.

Thoroughly remove all the pulp inside the canal. Otherwise an accurate measurement cannot be obtained.

Caries touching the gums

In this case, electrical leakage through the caries infected area to the gums will make it impossible to obtain an accurate measurement.

### Blocked canal

The meter will not move if the canal is blocked.

Open the canal all the way to the apical constriction to measure it.



### Extremely dry canal

If the canal is extremely dry, the meter may not move until it is quite close to the apex. In this case, try moistening the canal with oxydolor saline.

## Maintenance

### Check with Tester



- Connect tester to handpiece;
- Confirm whether indicators are lit between 0.1-0.5.

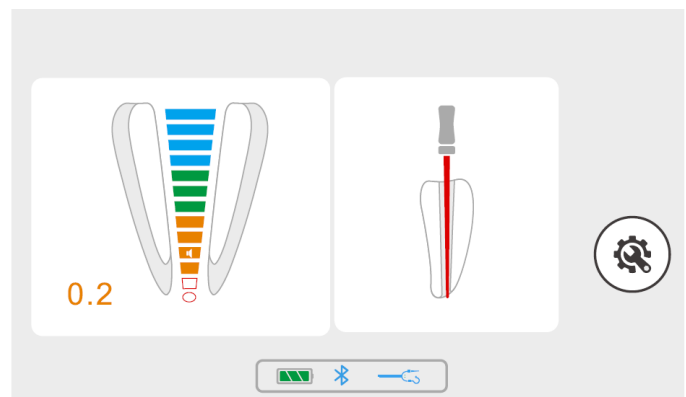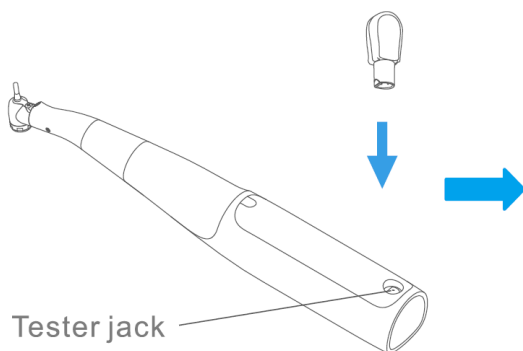

### Check Canal Measurement Function



### Check Test Root Apex Test Wire

Check the test root apex test wire and confirm whether indicators are lit.

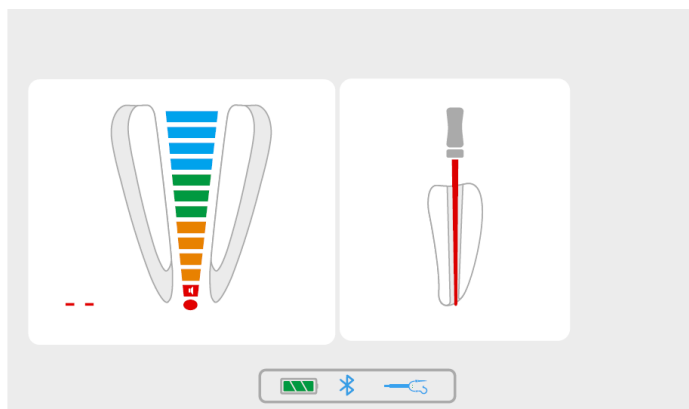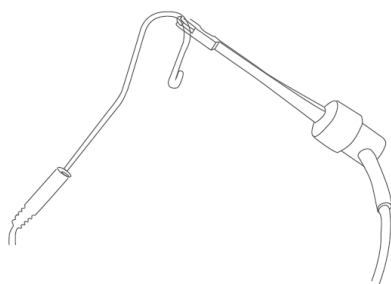


### Check Lip Hook wire

Check the lip hook wire and confirm whether indicators are lit.

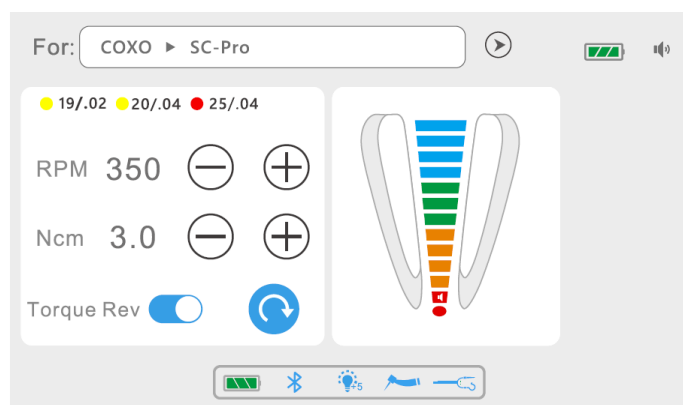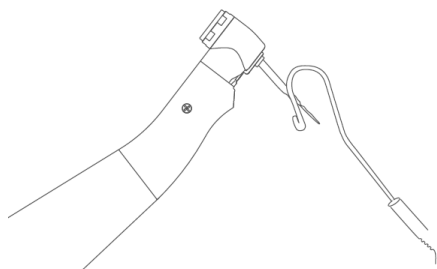

### Documents / Resources

	<a href="#">coxo DB-686 Nano 001 Curing Light</a> [pdf] User Manual 001, 2A2C6-001, 2A2C6001, 002, 2A2C6-002, 2A2C6002, DB-686 Nano 001 Curing Light, Nano 001 Curing Light, Curing Light
---	--