

Dexcom G7 Continuous Glucose Monitoring System



Dexcom G7 Continuous Glucose Monitoring System User Guide

[Home](#) » [Dexcom](#) » Dexcom G7 Continuous Glucose Monitoring System User Guide 

Contents

- 1 [Dexcom G7 Continuous Glucose Monitoring System](#)
- 2 [Product Information](#)
- 3 [Frequently Asked Questions](#)
- 4 [COMPONENTS](#)
- 5 [GETTING STARTED](#)
- 6 [Need help getting started](#)
- 7 [Need further assistance](#)
- 8 [Documents / Resources](#)
 - 8.1 [References](#)

Dexcom®

Dexcom G7 Continuous Glucose Monitoring System



Specifications:

- **Product:** Dexcom G7 Continuous Glucose Monitoring (CGM) System
- **Wear Time:** Up to 10 days

Product Information

Welcome to the Dexcom G7 Continuous Glucose Monitoring (CGM) System! The Dexcom G7 app or receiver will guide you step-by-step on how to set up your system and insert your sensor. It's simple, accurate, and effective.

Components:

Applicator with built-in sensor

Getting Started:

- Compatible Smart Device or Dexcom G7 Receiver
- Check smart device compatibility online at: dexcom.com/compatibility
- Download the Dexcom G7 app using a compatible smart device*
- Follow the on-screen instructions

Training Resources:

For training videos, guides, FAQs, and more, scan the QR code or visit: dexcom.com/en-ca/training

Need Help?

Contact Dexcom CARE for personalized support at 1-[844-832-1810](tel:844-832-1810) (option 4). Monday – Friday | 9:00 am – 5:30 pm EST.

Apps for Dexcom G7:

- **Dexcom Clarity:** Discover trends and insights to share with your healthcare provider.

- **Dexcom Follow:** Allow friends and family to see your glucose levels.

Frequently Asked Questions

- **How long can I wear the sensor?**

The sensor can be worn for up to 10 days.

- **How do I check smart device compatibility?**

You can check compatibility online at dexcom.com/compatibility.


- **What should I do if I have trouble with the setup?**

Contact Dexcom CARE at 1-[844-832-1810](tel:844-832-1810) (option 4) for personalized support.

ready to get started with Dexcom G7?

Welcome to the Dexcom G7 Continuous Glucose Monitoring (CGM) System! The Dexcom G7 app or receiver will guide you step-by-step on how to set up your system and insert your sensor. It's that simple!

COMPONENTS

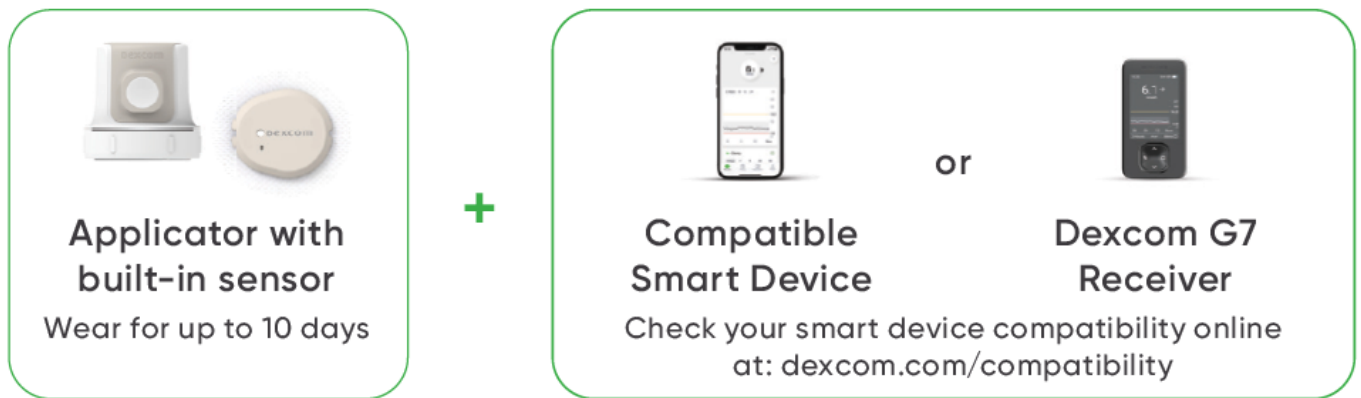


GETTING STARTED

1. Download the Dexcom G7 app using a compatible smart device*



2. Follow the on-screen instructions

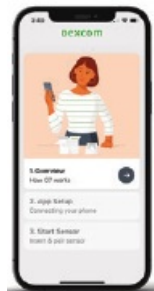

Need help getting started

- Our Dexcom CARE team of certified diabetes experts can provide training and assistance throughout your entire Dexcom CGM experience.

- Contact Dexcom CARE for personalized support at 1-[844-832-1810](tel:844-832-1810) (option 4).
- Monday – Friday | 9:00 am – 5:30 pm EST.†

Get the most out of your Dexcom G7 by using the following apps:



- **Dexcom Clarity**

Discover trends and insights that can be shared with your healthcare provider.



- **Dexcom Follow+**

Allow friends and family to see your glucose levels.

Need further assistance

- **Need further assistance?**

Call 1-[844-832-1810](tel:844-832-1810)

- **General inquiries:**

Select option 1

- **Insurance questions:** Select option 2

- **Product replacements & troubleshooting:**

Select option 3

- **New user training & support:**

Select option 4

1. Compatible smart devices sold separately: dexcom.com/compatibility.
2. Hours are subject to change and exclude holidays.
3. Separate Follow app and internet connection are required. Users should always confirm readings on the Dexcom G7 app or receiver before making treatment decisions.
4. Dexcom, data on file, 2023.

Dexcom, Dexcom G7, Dexcom Follow, Dexcom Share, and Dexcom Clarity are registered trademarks of Dexcom Inc. in the United States and may be registered in other countries. © 2023 Dexcom Canada, Co. All rights reserved. MAT-0305 V1.0

TRAINING RESOURCES

For training videos, handy guides, FAQs and more, scan the QR code or visit dexcom.com/en-ca/training.





[Dexcom G7 Continuous Glucose Monitoring System](#) [pdf] User Guide
G7 Continuous Glucose Monitoring System, G7, Continuous Glucose Monitoring System, Glucose Monitoring System, Monitoring System, System

References

-  [Phones that work with Dexcom apps | Dexcom](#)
- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.