



Dexcom G7 Receiver User Manual

[Home](#) » [Dexcom](#) » Dexcom G7 Receiver User Manual 

Contents

- [1 Dexcom G7 Receiver](#)
- [2 G7 Basics](#)
- [3 Receiver setup](#)
- [4 Use up to 3 display devices](#)
- [5 Documents / Resources](#)
 - [5.1 References](#)
- [6 Related Posts](#)


Dexcom

Dexcom G7 Receiver



G7 Basics

G7 shows your glucose information on your display devices

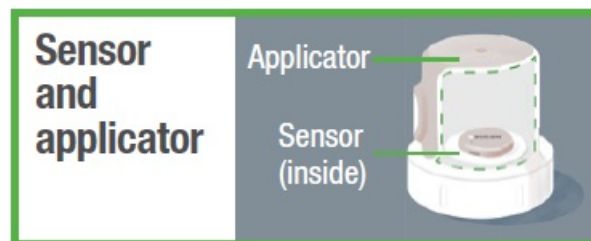

- During setup, we'll show you how to use the applicator to insert the built-in sensor under your skin
- The sensor sends a glucose reading to your receiver or other display device every 5 minutes
- The sensor lasts up to 10 days with a 12-hour grace period



After you insert the sensor, you must apply the overpatch to keep the sensor on your body.

Receiver setup

1. Insert sensor

1. You must insert your sensor before pairing to the receiver.
2. For instructions go to Inserting Sensor in the sensor box.

2. Set up receiver

1. Press the Select button for 3-5 seconds to turn on receiver. (Go to Receiver navigation.)
2. Follow receiver instructions to set up the receiver. Insert sensor before pairing.
3. After inserting and pairing, follow local guidelines for throwing out the applicator and recycling the Dexcom packaging.

3. Sensor warmup

Sensor warmup timer tells you when you'll start getting readings and alerts.



4. Your sensor session

Go to Using Your Receiver in the receiver box and the online User Guide at dexcom.com/guides to find out more.

The receiver screen tells you which button to use.

- To scroll faster hold down the Scroll button
- To move to next field use the Select button



Use up to 3 display devices


Get your glucose information on your phone, Apple watch, and the Dexcom receiver. You can set up the app, the

receiver, or both, in either order. Review Safety Information in product instructions. All graphics are representational. Your product may look different.

2022 Dexcom, Inc. All rights reserved. Covered by patents dexcom.com/patents. Dexcom, Dexcom Share, Share, Dexcom Follow, and Dexcom Clarity are registered trademarks of Dexcom, Inc. in the United States and/or other countries. Bluetooth is a registered trademark owned by Bluetooth SIG. Apple is a trademark of Apple Inc., registered in the U.S. and other countries. Android is a trademark of Google LLC. All other marks are property of their respective owners.

Dexcom, Inc. 6340 Sequence Drive San Diego, CA 92121 USA 1.888.738.3646 dexcom.com

Documents / Resources

	Dexcom G7 Receiver [pdf] User Manual G7 Receiver, G7, Receiver
---	---

References

-  [International | Dexcom](#)
-  [Dexcom G7 and G6 CGM Quick Start User Guides | Dexcom](#)
-  [Dexcom Patents](#)