



# DigiKuber E88P Smart Watch ECG User Manual

[Home](#) » [DIGIKUBER](#) » DigiKuber E88P Smart Watch ECG User Manual 




## DigiKuber

Watch quick instructions

### Contents


- [1 Parts introduction](#)
- [2 Install watch APP](#)
- [3 Main function interface](#)
- [4 Basic parameters](#)
- [5 Special note](#)
- [6 Documents / Resources](#)
- [6.1 References](#)

## Parts introduction

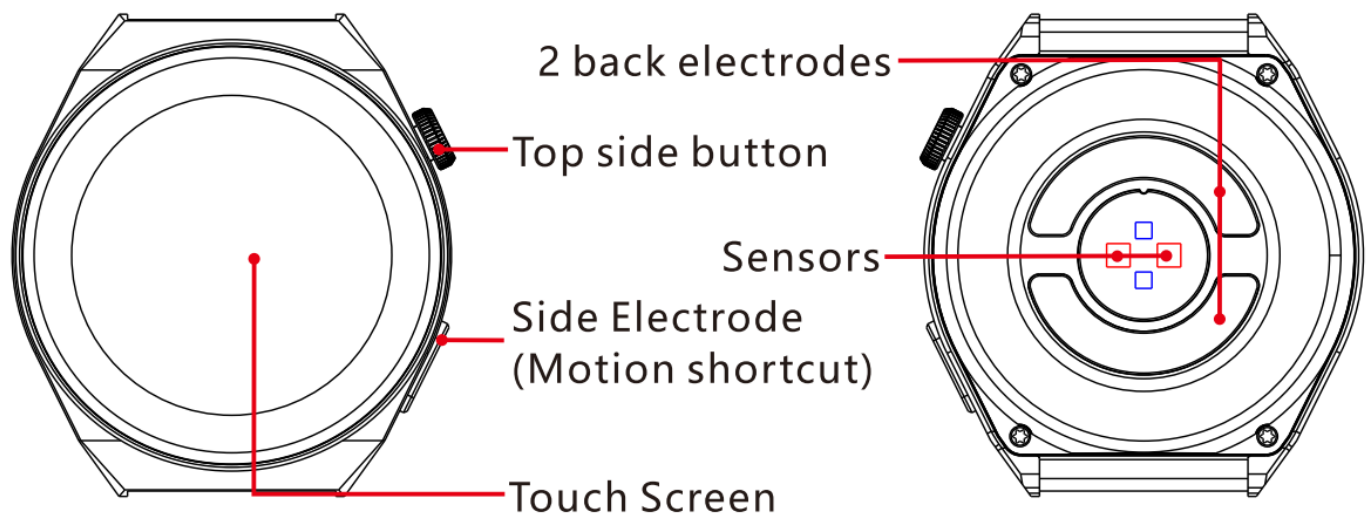

Long press the side button to turn on the watch

### Instruction for Charging

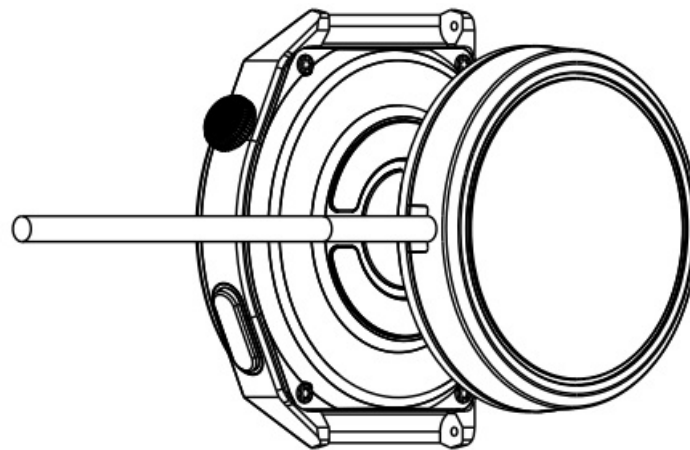

Need to charge and activate before using the watch for the first time. Use the equipped magnetic wireless charger to attach to the metal ring on the back of the watch, the other charging cable connect to USB charging head or USB interface. Can

### Install watch APP

Scan the following QR code or go to major application markets to download and install DeepFit



Device requirements iOS 9.0 and above; Android 4.4 and above Bluetooth 4.0 is supported.

Open the mobile client to set up personal information



Switch to the "Devices" page and click "Add New Device"



Click on your device in the list of scanned devices



Binding completed

## Main function interface

### Main interface/dial

On the main interface, long press the touch screen or in the APP menu, click "theme selection" to enter Switch the main interface/dial



On the main interface, press the side button or swipe up to enter the function page, and then press the side button or Swipe down to return to the main interface.



### Pedometer/Distance/Calories

View steps, distance, calories burn in real time It can synchronize apps in time to view motion data



### Body temperature


Switch to the temperature test interface to enter the real-time temperature detection, the monitoring data can be updated to the APP synchronously, and there is a test report.



**Note** When testing body temperature, the bottom temperature must be in full contact with the skin (tested after 10 minutes of wearing) The body temperature test will be affected by the temperature difference of the environment, which only serves as a warming reminder For details, please refer to medical professional equipment and doctor diagnosis.


### Heart rate monitoring

Switch to the heart rate monitor interface, heart rate monitoring. in real-time The measurement data can be synchronized with the APP in real time with a test report

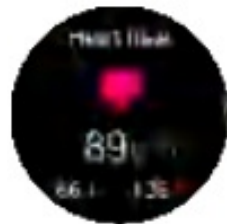

### ECG

Long press the front electrode on the ECG test interface, the three electrodes are in full contact Skin, test, test report needs to be viewed in APP.

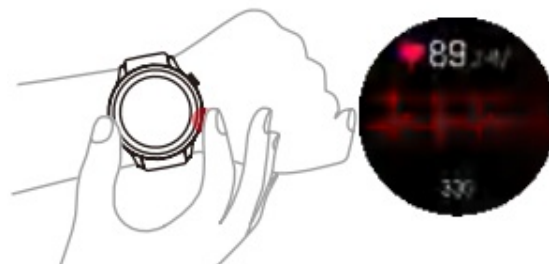

### Blood pressure monitoring

Switch to the blood pressure monitor interface, to enter real-time monitoring. The measurement data can be synchronized with the APP in real time with a test report

**Note** : When measuring blood pressure, please measure by the correct posture. The test result is for reference only, Cannot be used as medical data.



### Blood oxygen Interface

Switch to the blood oxygen test interface to enter the blood oxygen real-time detection and measurement data It can be updated to the APP synchronously, and there is a test report.



### Weather information

The weather page displays current weather, air quality information, and tomorrow's condition.

Weather information needs to be connected to the client before data can be obtained. If you disconnect for a long time, the weather information will not be updated.

**Note:** Phone settings need to be turned on for positioning.



### Sleep monitoring function

When you fall asleep, your watch automatically determines that you're in sleep detection mode, automatically detects how long you've been asleep/light sleeping/sleeping all night, calculates the quality of your sleep, and updates your sleep data to the APP synchronously.

**Note:** Wear your watch to sleep before you have sleep data.



### Stopwatch

Tap the stopwatch to enter the stopwatch function, and swipe right to exit.




### Music

When connected, tap music to control the music player.



### Shortcut function

Swipe down on the main interlace enter the shortcut function interface, select "Do not disturb Lock Screen About Settings" and other options, click the icon to You can enter the corresponding function.

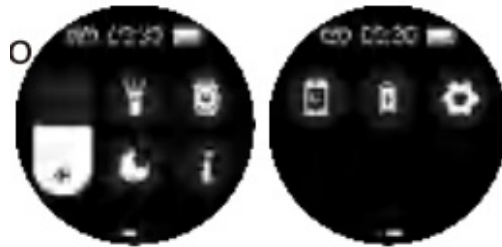

### Information

Click on the message to view the content of the message, scroll up and down to turn pages Swipe right to exit.



### About

Click the about icon to display the machine model and last four digits of the Bluetooth address

### Brightness

Click the brightness icon to freely select the screen brightness

### Raise your wrist

Tap the wrist lift icon to enter, you can choose to open or close the wrist lift

### Reset

Click the reset icon to enter, you can restore the factory settings of the watch,

### Shutdown

Click the shutdown icon to enter, you can shutdown, Other feature reminders.

The reminder function needs to set the on reminder switch on the App guide and keep the phone and watch connected successfully in Bluetooth to the message is a vibration reminder.

### Basic parameters

Connection mode: Bluetooth connection

Input: 5V,500mA MAX

Device requirements. Android 4.4or iOS9.0 and above

### Caution

1. Prohibit the use of an adapter with charging voltage greater than 5V, charging current greater than 2A, and charging time of 2-3 hours

2. Prohibit charging in the presence of water damage.
3. Do not disassemble the case of this product without permission, the product will be damaged and lose warranty eligibility

### Special note

1. This product is not a medical device, the measurement results are for reference only, not for any medical purposes and basis. Please follow your doctor's instructions and do not rely on the test results for self-diagnosis and treatment
2. This manual is based on existing information, in the spirit of continuous improvement, the principle of continuous development, the company reserves the right to change product specifications and functions, and any product described in this manual to modify and improve.



DigiKuber

---

### Documents / Resources

	<a href="#">DigiKuber E88P Smart Watch ECG</a> [pdf] User Manual E88P Smart Watch ECG, E88P, Smart Watch ECG, Watch ECG
--	----------------------------------------------------------------------------------------------------------------------------

### References

- [User Manual](#)