



EARASERS Pro-Kit with Two Filter User Manual

[Home](#) » [EARASERS](#) » EARASERS Pro-Kit with Two Filter User Manual 

Contents

- [1 EARASERS Pro-Kit with Two Filter](#)
- [2 Inserting Earasers](#)
- [3 Removing Earasers from ear](#)
- [4 Cleaning and Maintenance](#)
- [5 WARNINGS](#)
- [6 HEARING PROTECTION THAT FOCUSES ON HEARING™](#)
- [7 Documents / Resources](#)
- [8 Related Posts](#)



EARASERS Pro-Kit with Two Filter



Inserting Earasers

1. Make sure your ears and Earasers are clean.
2. Blue is for the Left ear and red is for the Right. The filter inside has a colored indicator (Fig. A).
3. Hold string with index finger and thumb. (Fig. B). Raise earplug up so the line guide is parallel to the floor at eye level. Bring the earplug straight back and guide into your ear canal. (Fig. C)


4. Let go of the string. With your pointer finger, gently push into your ear canal while wiggling your finger in an up/down motion. Push on the silicone until you feel a firm seal. Talk out loud using a strong voice. Your own voice must sound louder to you, especially the low frequencies. If it feels loose or there is little difference in the sound of your voice, select a larger size Earaser. If it does not fit or feels too tight, select a smaller size.

A deep seal will offer the best results.

Tip: Moistening the ear tip may ease insertion. Opening and closing your jaw may also help.

Removing Earasers from ear


Simply hold the string with your thumb and index finger and pull outward.

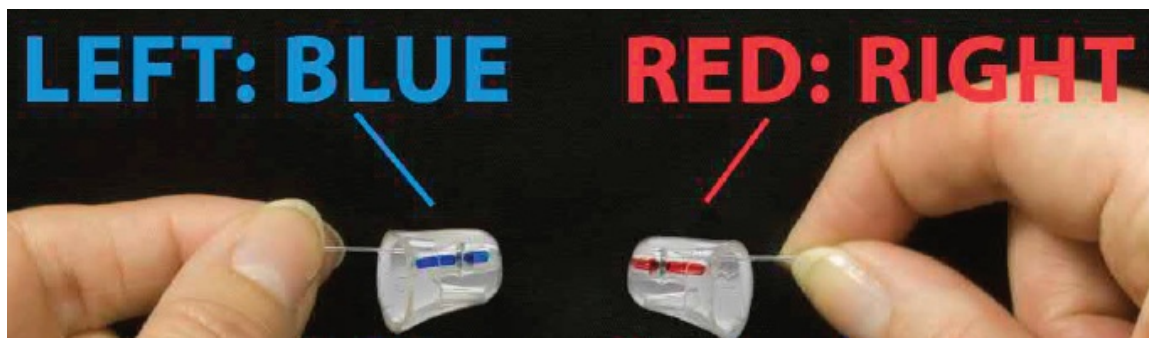

Right



Back | Front

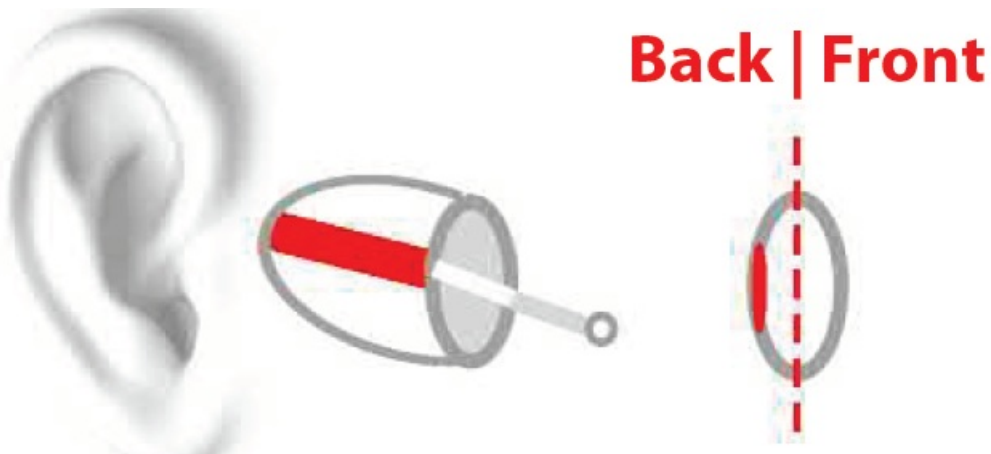

Side of Earasers with color line always faces towards the back of your head. The ear canal is shaped like an oval, standing upright and vertical. Earasers are also shaped like an oval. Insert the Earasers so the long axis remains upright.

Cleaning and Maintenance

We recommend regular cleaning of your Earasers earplugs to ensure they work as effectively as possible. To remove ear wax from the earplug, use a soft bristle brush such as a toothbrush. Cleaning tools, wax guards and germicidal cleaning wipes are available on our website or at participating stores. A soft, damp cloth may be used to clean around the outside of your earplugs. Do not use any chemicals as these can have a detrimental effect on the product. Do not put any items into filters as this may cause damage.

WARNINGS

- Keep out of reach of young children or pets. Product and packaging contain small parts, which can be swallowed.
- If intended use is for any OSHA noise abatement program or Industrial, Military, or commercial exposure, close coordination with a company approved safety officer should be maintained to ensure compatibility with your

environment.

- If you experience ANY ringing or muffled feeling in your ears after use, please choose an earplug with a higher reduction rating.
- Get annual hearing checkups from an Audiologist or Hearing Specialist if you use earplugs.
- Not to be used with any firearms.

HEARING PROTECTION THAT FOCUSES ON HEARING™

- World's most comfortable earplug
- Smart Seal™ ensures a perfect fit
- Wear comfortably for hours
- No plugged up feeling
- Replaces custom earplugs
- Will not sweat, tickle or itch
- Use your cell phone while wearing
- Helps prevent ringing in the ears
- Replaceable sleeves save money!

Documents / Resources



[EARSERS Pro-Kit with Two Filter](#) [pdf] User Manual
Pro-Kit with Two Filter