



ECG247 Heart Sensor User Guide

[Home](#) » [ECG](#) » ECG247 Heart Sensor User Guide 

Contents

- [1 ECG247 Heart Sensor](#)
- [2 RESPONSIBILITY](#)
- [3 ENVIRONMENT](#)
- [4 Documents / Resources](#)
 - [4.1 References](#)



ECG247 Heart Sensor



1. Download and install the ECG247 app on your mobile phone



2. Click the link in received SMS from your physician or open the app and follow the instructions



3. Enter your personal information in the app



4. Place the sensor on the electrode Do not remove the green foil over the battery Rotate clockwise



5. Connect the sensor to the phone Press the center of the sensor until the blue light comes on



6. Clean the skin



7. Attach the sensor on the upper part of sternum



8. The heart rhythm is recorded and analyzed



9. Use the sensor for at least 3 days. The test is stopped by pressing

Stop test

- «Stop test»
- Stop test

10. Order a review of the test results from our heart specialists or share the results with your physician

PRIVACY

Your privacy is important to us. We protect your data and do not share your information. Read more at ecg247.com

RESPONSIBILITY

The heart signals have different shapes and sizes in different people. We cannot guarantee that this test finds all different rhythm disorders in everyone. The sensor cannot look for signs of myocardial infarction. Any rhythm disturbances must always be confirmed by a doctor. Read more at ecg247.com Always contact a physician if you have symptoms with your heart.

WARNING

The sensor should not be used in children, pregnant women, significantly overweight (BMI>40), patients with pacemaker/ICD and in situations requiring external defibrillator use. If the use causes itching or rash, the sensor should be removed. Do not swallow. The sensor must be kept out of reach of children. The battery cannot be charged.

ENVIRONMENT

Remove battery from the electrode for proper recycling. You can shower with the sensor, but avoid bathing. Complete instructions at ECG247.com Order new sensors and electrodes at ECG247.com

Documents / Resources



[ECG ECG247 Heart Sensor](#) [pdf] User Guide
ECG247 Heart Sensor, ECG247, Heart Sensor, Sensor

References

-  [Forside - ECG247](#)
-  [Forside - ECG247](#)

