

Contents [[hide](#)]

- [1 eko COR501-CHP-CHP Digital Stethoscope](#)
- [2 Product Information](#)
- [3 Instructions for use](#)
- [4 Button Functions](#)
- [5 Connection with App](#)
- [6 Understand CORE 500™ Screen](#)
- [7 Tips to Improve ECG Signal](#)
- [8 Frequently Asked Questions](#)
- [9 Documents / Resources](#)
 - [9.1 References](#)

eko

eko COR501-CHP-CHP Digital Stethoscope



Product Information

Specifications

- Product Name: CORE 500™ Digital Stethoscope
- Features: Bluetooth connectivity, earpiece attachment, sleep mode, Eko App integration

Instructions for use

Step 1: Download the Eko App to Complete CORE 500™ Setup




Step 2: Set Up CORE 500™

Attach earpiece

- Plug the earpiece into the CORE 500™. The mark on the CORE 500™ socket should align with the right-side mark on the earpiece.
- Twist the earpiece clockwise with a quarter turn until it locks. The mark on the CORE 500™ socket should align with the left-side mark on the earpiece.

Detach earpiece


- Twist the earpiece counterclockwise with a quarter turn until it unlocks.
- Remove the earpiece from the CORE 500™.

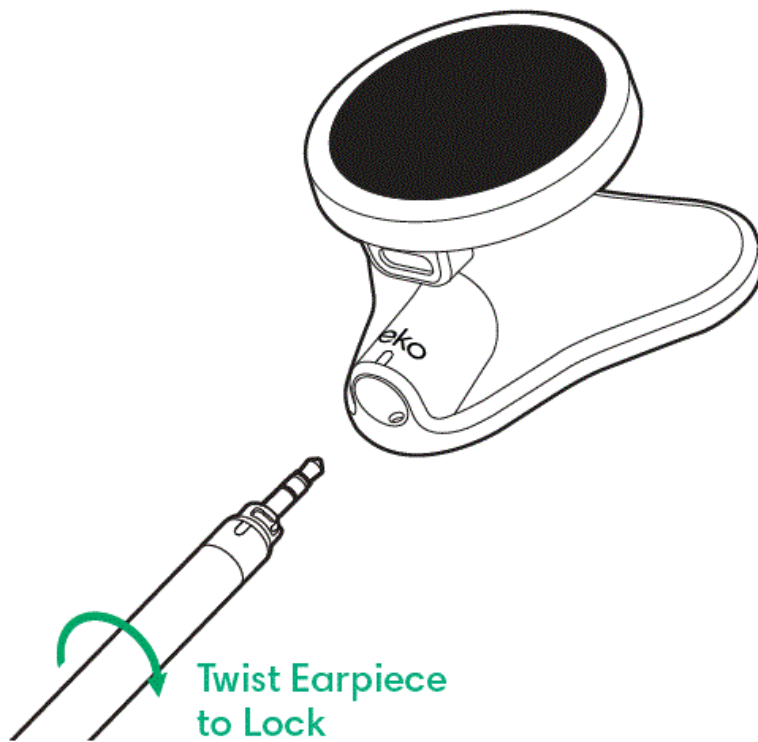

Button Functions

- **Turn on**

Press the top button.

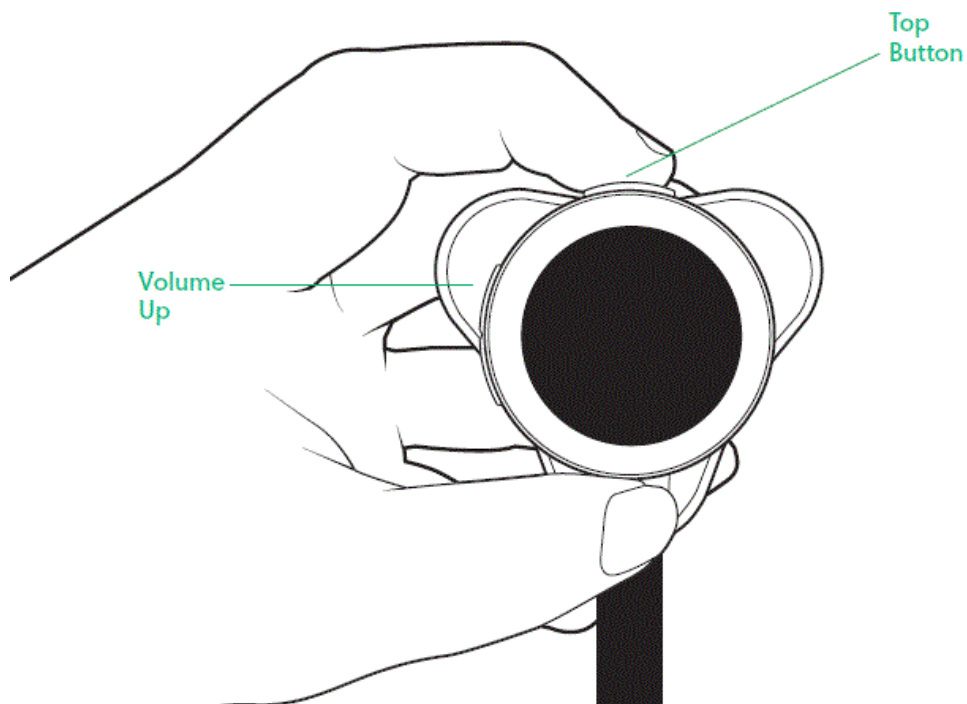

- **Turn off**

Hold the top button while pressing the volume-up button once to turn it off. Once you see the confirmation screen, press the volume-up button again to confirm.


- **Sleep**

The CORE 500™ automatically goes to sleep after a short period of not being used. Pick it up on the neck of the chestpiece to wake it up. You can turn off this setting in the Eko App.

Connection with App

Step 3: Connect CORE 500™ with Eko App

1. Open the Eko App and sign in.
2. Follow the onscreen instructions for pairing your device.
3. Press the volume up button to confirm the pairing if pairing for the first time.
4. The Bluetooth® icon shows on the CORE 500™ screen when connected.

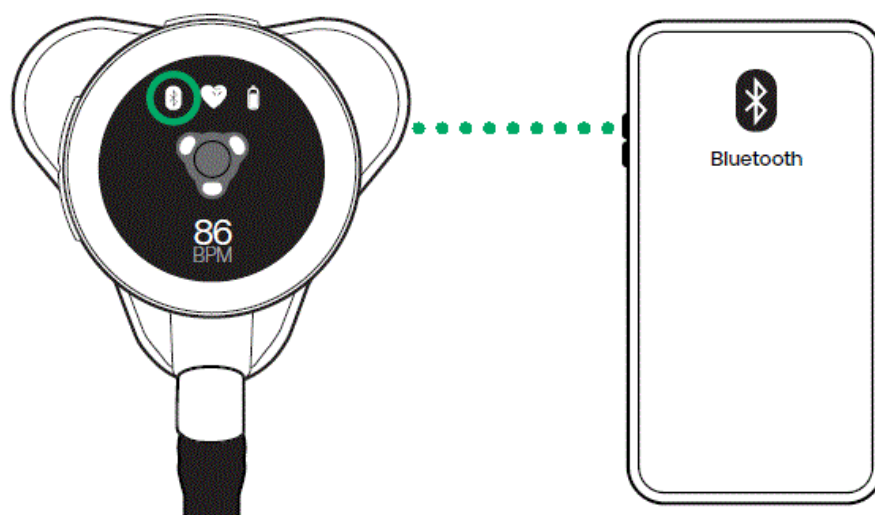

Understand CORE 500™ Screen


Resting state

1. **Current audio filter:**
 - Wide (for all use cases)
 - Cardiac (with ECG)
 - Pulmonary (with lung sounds)
2. **Top button:**
 - **Change filter:**
= press for <1 second
 - **Start recording:**
= press for 2 seconds
3. Battery level

4. Bluetooth connected
5. Volume up button
6. Volume down button
7. Real-time heart rate in beats per minute


8. **Leads indicator:**

- O= No skin contact detected
- ✓ = Skin contact detected

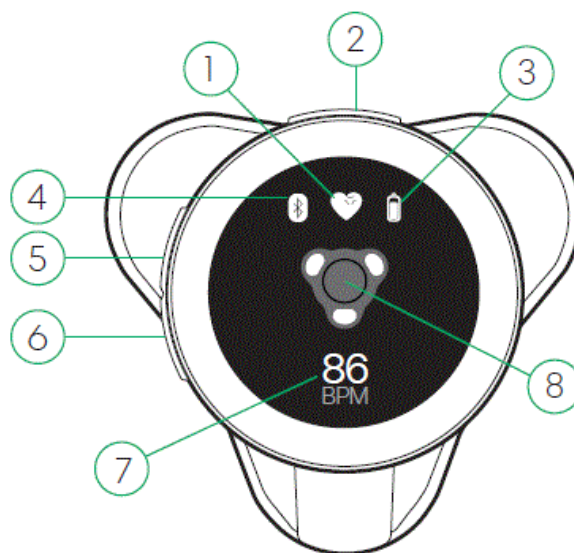

Active state

9. Current audio filter
10. Recording in progress
11. ECG waveform (shown in cardiac and wide filter)
12. Real-time heart rate in beats per minute
13. 1-second intervals
14. 0.2-second intervals

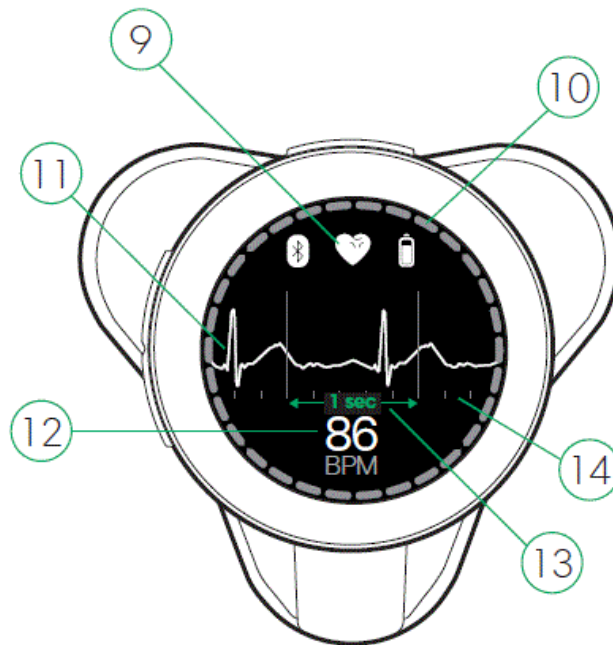

Tips to Improve ECG Signal

Placement


Place the CORE 500™ firmly on the patient's chest to get a clean ECG signal. When the CORE 500™ is held vertically, the ECG shown on screen corresponds to a modified lead I.

Interpret in Eko App

Use the Eko App to read and interpret the 3-lead ECG. The ECG tracing on the CORE 500™ is the same as the 1st channel shown in the Eko App.

Skin preparation

- Clean the skin and/or electrodes with 70% IPA wipes to get a better ECG signal. Some wipes are included in the product box.
- ECG gel is recommended for patients with dry skin or significant body hair.

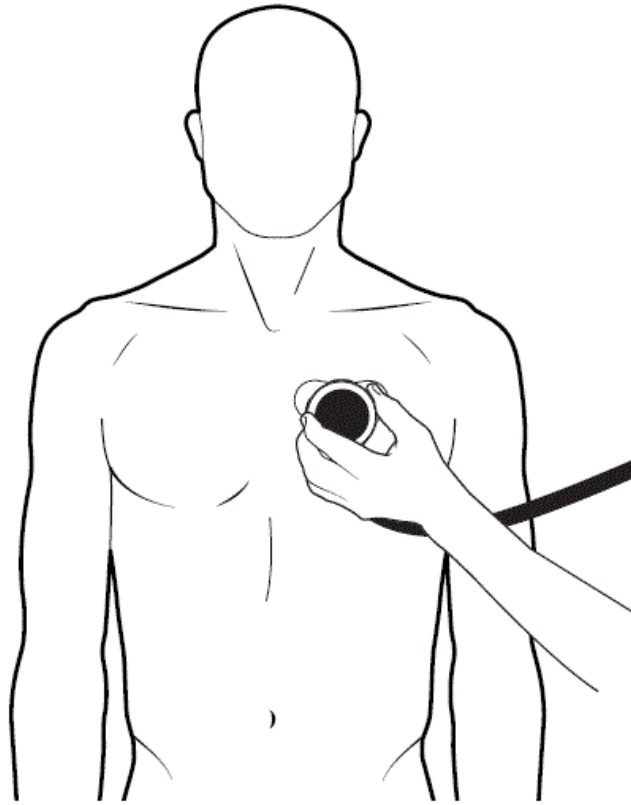

Learn More

- To view the CORE 500™ Instructions for Use, or to visit our Eko Help Center, go to ekohealth.com/ifu.
- Enterprise software users can contact their IT partner and/or Eko representative with onboarding questions.

Scan here to watch a getting started video:



- Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries.
- Google Play and the Google Play logo are trademarks of Google LLC. The Bluetooth®

word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Eko is under license.

Eko Health, Inc.


- 2100 Powell St, Suite 300
- Emeryville, CA 94608 USA
- ekohealth.com
- ©2023 Eko Health, Inc. LBL391
- Rev 2.0 May 2023

Frequently Asked Questions

Q: How do I change the filter on the CORE 500TM?


A: Press the top button to cycle through different filter settings.


Documents / Resources

	eko COR501-CHP-CHP Digital Stethoscope [pdf] User Guide COR501-CHP-CHP Digital Stethoscope, COR501-CHP-CHP, Digital Stethoscope, Stethoscope
---	---

References

- [User Manual](#)

 COR501-CHP-CHP, COR501-CHP-CHP Digital Stethoscope, Digital Stethoscope, EKO,

 EKO Stethoscope

Leave a comment

Your email address will not be published. Required fields are marked *

Comment *

Name

Email

Website

☐ Save my name, email, and website in this browser for the next time I comment.

Post Comment

Search:

e.g. whirlpool wrf535swhz

Search

[Manuals+](#) | [Upload](#) | [Deep Search](#) | [Privacy Policy](#) | [@manuals.plus](#) | [YouTube](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.