

# enabling devices 3211 Boppin Beaver User Guide

Home » enabling devices » enabling devices 3211 Boppin Beaver User Guide

#### **Contents**

- 1 enabling devices 3211 Boppin **Beaver**
- 2 Operation
- 3 Troubleshooting
- 4 Care of the Unit
- **5 Documents / Resources** 
  - **5.1 References**
- **6 Related Posts**



enabling devices 3211 Boppin Beaver



For Technical Support: Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697 <a href="mailto:customer\_support@enablingdevices.com">customer\_support@enablingdevices.com</a>

50 Broadway Hawthorne, NY 10532 Tel. 914.747.3070 / Fax 914.747.3480 Toll Free 800.832.8697

www.enablingdevices.com

# Helps with:

- Developing Cognitive Skills
- Increasing Auditory Development
- · Increasing Visual Attention
- Teaching Cause & Effect

## Operation

- 1. Boppin' Beaver requires 3 AA batteries. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types of batteries together.
- 2. The battery compartment is located under the toy. Carefully remove the screw from the cover using a Phillips head screwdriver, and lift the cover off. Observing proper (+) & (-) battery polarity, install 3 AA size batteries into the holders. Replace the cover and screw. Next set the ON/OFF switch to ON.
- 3. Boppin' Beaver has an external switch input jack so you can use your own capability switch if needed. To use an external switch, plug your switch into the jack. Make sure your switch is plugged in all the way. There should

be no gaps. Activating your switch will replicate pushing down on Boppin' Beaver's head.

4. Press his head and he lights up, swishes his tail and sings. Or use an external capability switch to control the music and lights.

Each touch of his head or switch changes the song. Boppin' Beaver also interacts with Cool Beats Penguin (#7578) and Smooth Moves Sloth (#7579). Works with or without a switch.

For more detailed information on all Boppin' Beaver features and functions please read the original manufacturer's user's guide.

# **Troubleshooting**

Problem: Boppin' Beaver does not work.

**Action #1**: Rotate (spin) the batteries within the battery compartment to ensure a good connection. Check that all batteries are installed properly.

**Action #2**: Check all batteries for freshness, if weak or dead replace with new batteries. Use an alkaline battery (E.G. Duracell or Energizer brand).

Problem: Boppin' Beaver does not work with a switch.

Action: Try a different switch with Boppin' Beaver to rule out that Boppin' Beaver is the source of the problem.


#### Care of the Unit

Boppin' Beaver can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non-toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.

#### **Documents / Resources**

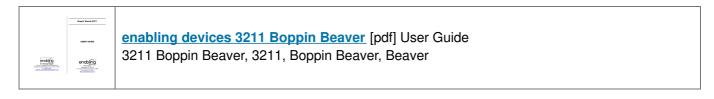

### References

Products For People With Special Needs | Enabling Devices

Manuals+,