



enabling devices 3212 Peter the Penguin Switch User Guide

Home » enabling devices » enabling devices 3212 Peter the Penguin Switch User Guide 🖫

Contents

- 1 enabling devices 3212 Peter the Penguin Switch
- 2 Operation
- 3 Troubleshooting
- **4 Specifications**
- **5 Frequently Asked Questions (FAQ)**
- 6 Documents / Resources
 - **6.1 References**
- **7 Related Posts**



enabling devices 3212 Peter the Penguin Switch



Cute and colorful!

Everyone is drawn to this colorful Penguin. The slightest touch rewards the user with a choice of lights, vibration, and/or music. This item has a special control that allows the user to select any or all of the rewards. Size: 9"L x 61/4"W x 4"H. Requires 2 AA Batteries. Weight: 1 lb.

Operation

- 1. The Peter the Penguin Switch requires two AA batteries (Not included). The battery compartment is located under the base of the unit. Carefully turn the switch over, and then remove the battery cover with a small Philips screwdriver. Install new batteries, being careful to observe proper battery polarity. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries or different brands or types together.
- 2. Replace the battery case cover and secure it with a small screw. Do not over-tighten.
- 3. To turn on or off the desired stimuli (1 vibration; 2 music; 3- lights), push the small white rocker switches to the on or (open position). Open position means off. Please see Fig 1.

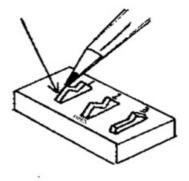


Figure 1.

- 4. Press anywhere on Peter the Penguin's back and you will be rewarded with the selected stimuli vibration, lights, or music.
- 5. To operate as a switch connect an external toy or device via the double-ended male 1/8-inch to 1/8-inch cord,

one end plugs into the 1/8-inch female jack on the side of the Peter the Penguin.

- 6. Plug the other end into your toy/device. If you need to use a ¼" adaptor, it must be a mono adaptor, not stereo. Press anywhere on Peter the Penguin's back to activate your toy or device. Once you release pressure from the switch your toy or device will turn off.
- 7. When using as a switch you can turn On or Off any of the three stimuli.

Troubleshooting

- Problem: Peter the Penguin does not operate properly.
- Action # 1: Make sure the batteries are in the battery compartment properly, and making good contact.
- Action # 2: Make sure you have the desired stimuli selected as explained in operation No.3
- Action # 3: If the unit still does not work properly, replace the batteries.
- **Problem**: Peter the Penguin does not activate a connected toy/device.
- Action #1: Make sure all connections are tight.
- Action #2: Check the batteries in the toy/device, replace if weak or dead.

Care of Unit

The Peter the Penguin can be wiped clean with any household multipurpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner. Do not submerge the unit, as it will damage the contents of the top and the electrical components.

For Technical Support:

Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697 customer_support@enablingdevices.com

Specifications

• Product Name: Peter the Penguin Switch #3212

• Features: Lights, Music & Vibration

Manufacturer: Enabling Devices

• Technical Support: 1-800-832-8697 / customer_support@enablingdevices.com

Address: 50 Broadway Hawthorne, NY 10532

• Website: www.enablingdevices.com

Frequently Asked Questions (FAQ)

Q: Can I use rechargeable batteries with Peter the Penguin?

A: Yes, you can use rechargeable batteries as long as they are compatible with the product specifications.

Q: How do I adjust the volume of the music on Peter the Penguin?

A: The volume of the music cannot be adjusted on Peter the Penguin; it plays at a standard level.

Documents / Resources



enabling devices 3212 Peter the Penguin Switch [pdf] User Guide3212 Peter the Penguin Switch, 3212, Peter the Penguin Switch, Penguin Switch, Switch

References

• User Manual

Manuals+, Privacy Policy

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.