



enabling devices 4532 Switch Spinner User Guide

[Home](#) » [enabling devices](#) » enabling devices 4532 Switch Spinner User Guide 

Contents

- [1 enabling devices 4532 Switch Spinner](#)
- [2 Product Information](#)
- [3 Product Usage Instructions](#)
- [4 Operation](#)
- [5 Troubleshooting](#)
- [6 Care of Unit](#)
- [7 Documents / Resources](#)
 - [7.1 References](#)
- [8 Related Posts](#)



enabling devices 4532 Switch Spinner



For Technical Support: Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697

customer_support@enablingdevices.com 50 Broadway Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Product Information

The Switch Spinner #4532 is a device designed to be used with a switch. When the switch is activated, the spinner will turn and stop on one of the locations.

Product Usage Instructions

1. Plug your switch into the jack, ensuring there is no gap in the connection.
2. Activate your switch to make the spinner turn. Release the switch to stop the spinner on one of the locations.
3. Print out the spinner blank template from the Switch Spinner product page on our website. Legal size paper is required for printing. You can find the product page at <https://enablingdevices.com/product/switchspinner> or by entering the item No. 4532 into the search field on our site.

Troubleshooting:

Problem: The Switch Spinner does not activate by your switch.

1. Make sure that the connection between the Switch Spinner and your switch is tight, with no gaps. This is a common error and can be easily fixed.
2. Ensure the batteries are properly placed in the battery compartment and making good contact. Replace them if

weak or dead.

3. Try using a different switch with the Switch Spinner to rule out the switch as the source of the problem.
4. Check for any debris or small objects that may be blocking the movement of the spinner.

Care of Unit:

The Switch Spinner can be wiped clean with any household multi-purpose cleaner and disinfectant. Do not submerge the unit, as it will damage the contents and electrical components. Avoid using abrasive cleaners, as they may scratch the surface of the unit.

Great for games

Our switch activated spinner is perfect for playing games, matching, rolling dice or making random selections. Perfect for teaching turn taking and visual tracking. Just activate your capability switch and you're set to play. You can create your own overlays to use on the spinner. Size: 13"L x 4"W x 14½"H. Requires 2 AA Batteries. Weight: 2 lbs.

Operation

1. The Switch Spinner requires two AA batteries. The battery compartment is located on the back of the unit.
Carefully turn the Switch Spinner over, and then remove the battery cover with a small Philips screwdriver. Install new batteries, being careful to observe proper battery polarity Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Plug your switch into the jack, making sure there is no gap in the connection. Activating your switch will make the spinner turn, once you release your switch the spinner it will stop on one of the locations.
3. Please visit the Switch Spinner product page on our website to print out your spinner blank template. Please Note: Legal size paper is required to print the blank template out:
<https://enablingdevices.com/product/switch-spinner> or entering the item No. 4532 into the search field on our site.

Troubleshooting

Problem: The Switch Spinner does not activate by your switch.

- Action #1: Make sure that the connection between the Switch Spin and your switch is tight. There should be no gaps. This is a common error and an easy fix.
- Action #2: Make sure the batteries are in the battery compartment properly, and making good contact. Replace if weak or dead.
- Action 3: Try a different switch with the Switch Spinner to rule out this as the source of the problem.
- Action #4: Check that no debris or small objects are blocking the movement of the spinner.

Care of Unit


- The Switch Spinner can be wiped clean with any household multi-purpose cleaner and disinfectant.

- Do not submerge the unit, as it will damage the contents and the electrical components.
- Do not use abrasive cleaners, as they will scratch the surface of the unit

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480 Toll Free 800.832.8697
www.enablingdevices.com

For Technical Support:
Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com

Documents / Resources

	<p>enabling devices 4532 Switch Spinner [pdf] User Guide 4532, 4532 Switch Spinner, Switch Spinner, Spinner</p>
---	---

References

- [enabling Products For People With Special Needs | Enabling Devices](#)
- [enabling Switch Assessment Kits | Enabling Devices](#)

Manuals+