



enabling devices 718 Saucer Switch User Guide

[Home](#) » [enabling devices](#) » enabling devices 718 Saucer Switch User Guide 

enabling devices

devices 718 Saucer Switch User Guide

Contents

- [1 718 Saucer Switch](#)
- [2 Operation:](#)
- [3 Troubleshooting:](#)
- [4 Documents / Resources](#)
 - [4.1 References](#)
- [5 Related Posts](#)

718 Saucer Switch

Ideal for those with limited motor abilities!

These handy switches are specially designed for individuals who are unable to control or sustain the hand or wrist movements necessary to operate traditional plate switches.

Weight: ½ lb.

Operation:

1. The Saucer Switch does not require any batteries to operate.
2. Plug the Saucer Switch into the jack on your switch-adapted toy/device.
3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
4. Make sure that the connection between the Saucer Switch and your toy/device is tight. There should be no gaps.
5. Simply press anywhere on the Saucer Switch, to activate your toy/device.
6. The toy/device will only remain activated while the Saucer Switch continues to be actuated. Once you release

the switch, the toy/device will turn off.

Troubleshooting:

Problem: The Saucer Switch does not activate your toy/device.

Action #1: Make sure that the Saucer Switch is on a flat surface (not tilted or vertical). This provides for optimal functioning.

Action #2: Make sure that the connection between the Saucer Switch and your toy/device is tight. There should be no gaps.
This is a common error and an easy fix.

Action #3: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #4: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Saucer Switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the switch.



Documents / Resources

	enabling devices 718 Saucer Switch [pdf] User Guide 718, Saucer Switch, 718 Saucer Switch, Switch, 718 Switch
---	--

References

- [enabling devices | Products For People With Special Needs | Enabling Devices](#)