



## enabling devices 7301 4 In 1 Joystick Switch User Guide

[Home](#) » [enabling devices](#) » enabling devices 7301 4 In 1 Joystick Switch User Guide 

### enabling devices 7301 4 In 1 Joystick Switch User Guide



Need to activate more than one switch? Our joystick is great for the user who needs to access multiple switch adapted devices or a single device with multiple switch inputs such as our adapted TV remote module (#5150). Can also be used to teach directional — left, right, up, down. Size: 53/ 4"D x 4 1/2"H. Weight: 34/ lb.

1. Connect up to four cords to external switch enabled toys or devices via the four 1/8-inch jacks on the side of the Joystick. If you need to use 1/4- to 1/8-inch adapters, they must be mono adapters, not stereo.
2. To operate a toy or device connected to the first jack, push the Joystick in the corresponding direction for that toy/device. To activate any additional connected toys/device repeat as before.
3. The toy/device will only remain activated while the Joystick is being actuated in one of the four directions. Once you release the Joystick, the toy/device will turn off.

## Contents

### 1 Troubleshooting

### 2 Documents / Resources

#### 2.1 References

## Troubleshooting

**Problem:** The 4-in-1 Joystick switch does not activate your toy/device.

**Action #1:** Make sure that the 4-in-1 Joystick switch is on a flat surface (not tilted or vertical). This provides for optimal functioning.

**Action #2:** Make sure that the connection between the 4-in-1 Joystick switch and your toy/device is plugged in all the way. There should be no gaps. This is a common error and an easy fix.

**Action #3:** Try a different switch with your toy/device to rule out the 4-in-1 Joystick switch as the source of the problem.

**Action #4:** Try a different adapter (if applicable) to rule this out as the source of the problem.

### Care of Unit:

The 4-in-1 Joystick switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners,** as they will scratch the surface of the unit.

### For Technical Support:

Call our Technical Service Department

Monday through Friday, 9 a.m. to 5 p.m. (EST)

**1-800-832-8697**

customer [support@enablingdevices.com](mailto:support@enablingdevices.com)

### 50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free **800.832.8697**

[www.enablingdevices.com](http://www.enablingdevices.com)



	<a href="#">enabling devices 7301 4 In 1 Joystick Switch</a> [pdf] User Guide 7301 4 In 1 Joystick Switch, 7301, 4 In 1 Joystick Switch, Joystick Switch
---	---

References

- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.