



enabling devices 8209 Easy Flex Dual Ultimate Switch Instruction Manual

[Home](#) » [enabling devices](#) » enabling devices 8209 Easy Flex Dual Ultimate Switch Instruction Manual 

Contents

- 1 [enabling devices 8209 Easy Flex Dual Ultimate Switch](#)
- 2 [Product Information](#)
- 3 [Product Usage Instructions](#)
- 4 [OPERATION](#)
- 5 [Troubleshooting](#)
- 6 [Care Instructions](#)
- 7 [Documents / Resources](#)
 - 7.1 [References](#)



enabling devices 8209 Easy Flex Dual Ultimate Switch



Product Information

- Product Name: Easy Flex Dual Ultimate Switch #8209
- Manufacturer: Enabling Devices
- Address: 50 Broadway Hawthorne, NY 10532
- Contact: Tel. 914-478-0960 / Fax 914-478-7030
- Website: www.enablingdevices.com

Product Usage Instructions

Easy Flex Dual Ultimate Switch Two Ultimate switches, one mount!

Easily activate two devices – access has never been more convenient. The flexible tubing can be positioned to meet the needs of most users, while the Super Clamp mounts securely to any wheelchair, table or bedrail. Arm sizes: 25"L x 3/4"D. Weight: 2½ lbs.

OPERATION

1. The Easy Flex Dual Ultimate Switch can be mounted to most tubular and flat surfaces as shown in the photos below. When mounting to a flat surface use the removable wedge insert inside the jaws of the clamp.



2. You can also remove one of the Easy Flex Dual Ultimate Switches from the clamp if only one switch is needed. To do this loosen the set knob on the clamp, then depress the spring loaded pin and lift up on the Flex arm to remove the switch from the clamp. See photos below.



Mounting the Switch

- Place the Easy Flex Dual Ultimate Switch on flat surfaces as shown in the photos.
- If mounting to a flat surface, use the removable wedge insert inside the jaws of the clamp.

Removing a Switch

- If only one switch is needed, loosen the set knob on the clamp.
- Depress the spring-loaded pin and lift up on the Flex arm to remove the switch from the clamp.

Activating the Switch

- Press and hold the Ultimate switch to turn on your toy/device.
- The toy/device will remain activated as long as you hold the Ultimate switch in the actuated position.
- Releasing pressure on the Ultimate switch will turn off the toy/device.


Troubleshooting

1. Problem: The toy/device is not activating.
 - Action: Make sure the connection to the toy/device is plugged in securely with no gaps.
 - Action: Use a MONO adapter, if applicable.
 - Action: Try using your toy/device with a different switch to rule out the toy/device as the source of the problem.
 - Action: Check and replace the batteries in the toy/device if necessary.

Care Instructions

- Wipe clean the Ultimate Switch with any household multi-purpose cleaner and disinfectant.
- Recommended cleaner: Simple Green (non-toxic biodegradable all-purpose cleaner).
- Do not submerge the unit as it will damage the electrical components.
- Avoid using abrasive cleaners that may scratch the surface of the unit.

Documents / Resources

	<p>enabling devices 8209 Easy Flex Dual Ultimate Switch [pdf] Instruction Manual 8209, 8209 Easy Flex Dual Ultimate Switch, Easy Flex Dual Ultimate Switch, Flex Dual Ultimate Switch, Dual Ultimate Switch, Ultimate Switch, Switch</p>
---	--

References

- [enabling devices](#) [Products For People With Special Needs | Enabling Devices](#)