


enabling 682 Vibrating Switch

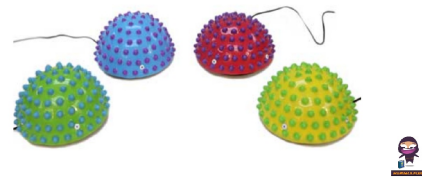

enabling 682 Vibrating Switch User Guide

[Home](#) » [enabling](#) » enabling 682 Vibrating Switch User Guide 

Contents

- 1 [enabling 682 Vibrating Switch](#)
- 2 [Product Information](#)
- 3 [Product Usage Instructions](#)
- 4 [FAQs](#)
- 5 [Operation](#)
- 6 [Troubleshooting](#)
- 7 [Documents / Resources](#)
 - 7.1 [References](#)



enabling 682 Vibrating Switch



Product Information

• Specifications

- **Product Name:** Senso Dot Vibrating Switch #682
- **Power Source:** 2 AAA batteries (Not included)
- **Battery Type:** Alkaline batteries (e.g. Duracell or Energizer brand)
- **Usage:** Switch-adapted toy/device activation
- **Cleaning:** Wipe clean with household multipurpose cleaner and disinfectant

Product Usage Instructions

1. Battery Installation:

- Use 2 AAA alkaline batteries.
- Do not mix old and new batteries or different brands/types.
- Install batteries observing proper polarity.
- Replace the battery compartment cover securely.

2. Usage:

- Plug the Senso Dot switch into the jack of your switch-adapted toy/device.
- Ensure a tight connection with no gaps.
- Press anywhere on the textured top to activate the toy/device.
- The toy/device remains activated while the switch is pressed.

3. Cleaning:

- Wipe clean with household multipurpose cleaner and disinfectant.
- Avoid submerging the unit or using abrasive cleaners.

4. Maintenance:

- Turn off the switch when not in use to prevent battery drain.
- Check battery freshness regularly for proper functioning.

FAQs

• Q: What should I do if the Senso Dot switch does not activate my toy/device?

- **A:** Follow these troubleshooting steps:
 - Ensure a tight connection between the switch and the toy/device without any gaps.
 - Try using a different switch to rule out toy/device issues.
 - If applicable, test with a different adapter to eliminate adapter problems.
 - Check and replace batteries if needed for proper functionality.

• Q: How should I care for the Senso Dot switch?

- **A:** Wipe clean with a household multipurpose cleaner and disinfectant. Avoid submerging the unit or using abrasive cleaners to maintain its condition.

Heightens sensory awareness!

- Our tactile and visually stimulating switch provides a wonderful somatosensory experience when activating any toy or device.

- The slightest push on any part of the dome will activate a device while you feel a soft vibration.
- The colorful tops can easily be turned round and round.
 - **Size:** 6"D x 2¾"H.
 - **Requires** 2 AAA Batteries.
 - **Weight:** 1 lb.

Operation

1. The Senso Dot switch requires 2 AAA batteries to operate (Not included). Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries or different brands or types.
2. Gently turn the unit over to face the black battery compartment cover. Carefully remove the small screw from the battery compartment cover using a Phillips head screwdriver, and lift the cover off, it may be necessary to use the end of the screwdriver to lift one edge of the cover. Observing proper (+) & (-) battery polarity, install 2 AAA-size batteries into the holder. Replace the battery compartment cover and screw. Set the On/Off switch located on the side of the switch to On.
3. Plug the Senso Dot switch into the jack on your switch-adapted toy/device.
4. If you must use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
5. Make sure that the connection between the Senso Dot switch and your toy/device is tight. There should be no gaps.
6. Simply press anywhere on the Senso Dot Switch's textured top to activate your toy/device.
7. The toy/device will only remain activated while the Senso Dot switch continues to be activated. Once you release the switch, the toy/device will turn off. The user will also feel a gentle vibration from the switch as long as the switch is being activated.

Please Note: Switch must be turned OFF when not in use to prevent battery drain . Failure to do so will cause the batteries to drain.


Troubleshooting

- **Problem:** The Senso Dot switch does not activate your toy/device.
- **Action #1:** Make sure that the connection between the Senso Dot switch and your toy/device is tight. There should be no gaps. This is a common error and an easy fix.
- **Action #2:** Try a different switch with your toy/device to rule out the toy/device as the source of the problem.
- **Action #3:** Try a different adapter (if applicable) to rule this out as the source of the problem.
- **Action #4:** Make sure the batteries are fresh; this switch will not work without batteries.

Care of Unit:

- The Senso Dot switch can be wiped clean with any household multipurpose cleaner and disinfectant. We recommend Simple Green, which is a nontoxic biodegradable all-purpose cleaner.
- Do not submerge the unit, as it will damage the contents and the electrical components.
- Do not use abrasive cleaners, as they will scratch the surface of the unit.

Documents / Resources
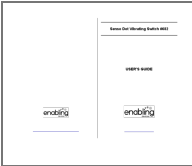
| | |
|---|---|
|  | enabling 682 Vibrating Switch [pdf] User Guide 682 Vibrating Switch, 682, Vibrating Switch, Switch |
|---|---|

References

- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.