



enabling 729T Turtle Switch User Manual

[Home](#) » [enabling](#) » enabling 729T Turtle Switch User Manual 

Contents

- 1 [enabling 729T Turtle Switch](#)
- 2 [Product Information: Turtle Switch #729T](#)
- 3 [Contact Information](#)
- 4 [Product Usage Instructions](#)
- 5 [Care Instructions](#)
- 6 [Operation](#)
- 7 [Troubleshooting](#)
- 8 [For Technical Support:](#)
- 9 [Documents / Resources](#)
 - 9.1 [References](#)



enabling 729T Turtle Switch



Product Information: Turtle Switch #729T

The Turtle Switch #729T is a versatile device designed to provide various stimuli for sensory stimulation. It features lights, music, and vibrations to engage and entertain users. The switch can also be used as a switch interface to operate external toys or devices.

Key Features:

- Lights, music, and vibration stimuli
- Switch interface capability
- Easy to operate
- Durable construction

Contact Information

For Technical Support:

- Call: 1-800-832-8697 (Monday through Friday, 9 a.m. to 5 p.m. EST)
- Email: customer_support@enablingdevices.com

Enabling Devices

50 Broadway
Hawthorne, NY 10532
Tel: 914.747.3070 / Fax: 914.747.3480
Toll-Free: 800.832.8697
Website: www.enablingdevices.com

Product Usage Instructions

1. Ensure the battery compartment is accessible.
2. Insert the required batteries (refer to the user manual for battery specifications) into the battery compartment.
3. Replace the battery case cover and securely fasten it with the provided small screw. Avoid over-tightening.
4. To activate or deactivate specific stimuli, use the small white rocker switches located on the device. Push the switches to the “on” position for the desired stimuli (1 for vibration, 2 for music, 3 for lights). The “open” position indicates that the stimuli is turned off (see Fig 1).
5. Press anywhere on the Turtle’s back to trigger the selected stimuli, such as vibration, lights, or music.
6. To use the Turtle Switch as a switch interface, connect an external toy or device using the provided double-ended male 1/8-inch to 1/8-inch cord. Plug one end into the 1/8-inch female jack on the side of the Turtle.
7. When using the Turtle Switch as a switch interface, you can turn any of the three stimuli (vibration, music, lights) on or off.

Troubleshooting

Problem: The Turtle Switch does not operate properly.

1. Ensure the batteries are correctly placed in the battery compartment and making good contact.
2. Verify that the desired stimuli are selected according to the operation instructions (No.3).
3. If the unit still does not work properly, replace the batteries with new ones.

Problem: The Turtle Switch does not activate a connected toy/device.

1. Check that all connections are fully plugged in without any gaps.
2. Inspect the batteries in the toy/device and replace them if weak or dead.

Care Instructions

To clean the Turtle Switch, wipe it with a household multipurpose cleaner and disinfectant. We recommend using Simple Green, a non-toxic biodegradable all-purpose cleaner. Avoid submerging the unit as it can damage the top and electrical components.

Revision Date: 3/15/2022

This turtle has triple the rewards!

It activates with a light touch—choose from music, lights, vibration, or all three! Makes a great companion to our Ladybug switch (#729). Size: 9"L x 6¼"W x 4"H. 2 AA Batteries. Weight: 1 lb.

Operation

1. The Turtle Switch requires two AA batteries (Not included). The battery compartment is located under the base of the unit. Carefully turn the switch over, and then remove the battery cover with a small Philips screwdriver. Install new batteries, being careful to observe proper battery polarity. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use Heavy Duty, Super Duty or rechargeable batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Replace the battery case cover and secure it with the small screw. Do not over-tighten.
3. To turn on or off the desired stimuli (1-vibration; 2-music; 3-lights), push the small white rocker switches to the on or (open position). Open position means off. Please see Fig 1.

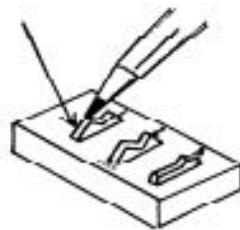


Figure 1.

4. Press anywhere on the Turtle's back and you will be rewarded with the selected stimuli vibration, lights or music.
5. To operate as a switch connect an external toy or device via the double ended male 1/8-inch to 1/8-inch cord , one end plugs into the 1/8-inch female jack on the side of the Turtle. Plug the other end into your toy/device. If you need to use a ¼" adaptor, it must be a mono adaptor, not stereo. Make sure both ends of the cable is plugged in all the way. There should be no gaps. Press anywhere on the Turtle's back to activate your toy or

- device. Once you release pressure from the switch, your toy or device will turn off.
6. When using as a switch you can turn On or Off any of the three stimuli.

Troubleshooting

Problem: The Turtle Switch does not operate properly.

Action # 1: Make sure the batteries are in the battery compartment properly, and making good contact.

Action # 2: Make sure you have the desired stimuli selected as explained in operation No.3

Action # 3: If the unit still does not work properly, replace the batteries.

Problem: The Turtle Switch does not activate a connected toy/device.

Action #1: Make sure connections are plugged in all the way. There should be no gaps.

Action #2: Check batteries in toy/device, replace if weak or dead.

Care of Unit:

The Turtle Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non-toxic biodegradable all-purpose cleaner.


Do not submerge the unit, as it will damage the contents of the top and the electrical components.

For Technical Support:

- Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697
- customer_support@enablingdevices.com

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480 Toll Free 800.832.8697
www.enablingdevices.com

Documents / Resources

| | |
|---|--|
|  | enabling 729T Turtle Switch [pdf] User Manual 729T Turtle Switch, 729T, Turtle Switch, Switch |
|---|--|

References

- [enabling Products For People With Special Needs | Enabling Devices](#)