

ErgoPack PR-34063-HPSM Pressure Injectable Three-Lumen PICC Instruction Manual

[Home](#) » [ErgoPack](#) » ErgoPack PR-34063-HPSM Pressure Injectable Three-Lumen PICC Instruction Manual 

ErgoPack®

ARROW



REF CDC-34063-HPK1A

6 Fr. **3** Lumen **40** cm catheter length **.018** inch dia. spring-wire guide

Contents

- [1 Pressure Injectable Three-Lumen PICC](#)
- [2 Documents / Resources](#)
- [3 Related Posts](#)

Pressure Injectable Three-Lumen PICC

Contents:

- 1: Three Lumen TaperFree® Catheter: 6 Fr. (2.14 mm OD) x 40 cm, Pressure Injectable, T-Port Connector, Blue FlexTip®, Contamination Guard and Placement Wire
- 1: GlideThrum Peel-Away Sheath: 6 Fr. x 2-3/4" (7 cm) Radiopaque over 6 Fr. Dilator

1: Spring-Wire Guide, Nitinol, Marked: .018" (0.46 mm) dia. x 17-3/4" (45 cm) (Straight Soft Tip on One End – Straight Stiff Tip on Other) with Arrow Advancer
 1: Injection Needle: SafetyGlide™1 25 Ga. x 5/8" (1.60 cm) RW
 1: Safety Introducer Needle: Echogenic 21 Ga. x 2-3/4" (7 cm) TW
 1: Blunt Fill Needle: 18 Ga. x 1-1/2" (3.81 cm)
 1: Introducer Needle: Echogenic 21 Ga. x 2-3/4" (7 cm) TW
 1: Syringe: 10 mL Luer-Lock 1: Syringe: 3 mL Luer-Lock 2: 10 mL Luer-Lock Pre-Filled Saline Syringe 1: Skin Protectant Prep
 1: 5 mL 1% Lidocaine HCl Solution and Alcohol Prep
 1: 3 mL Applicator 2% CHG and 70% IPA Chloraprep®2 One-Step Solution
 1: 3 mL Applicator 2% CHG and 70% IPA Chloraprep®2 One-Step Solution with Hi-Lite Orange™ Tint
 3: Dust Cap: Non-Vented
 1: SecondSite™ Adjustable Hub: Fastener 1: SecondSite™ Adjustable Hub: Catheter Clamp
 1: SharpsAway® Disposal Cup
 1: SharpsAway® II Locking Disposal Cup
 1: Catheter Trimmer 1: Towel: 24" x 36" (61 cm x 91 cm)
 1: Maximal Barrier Drape TM with two 13 cm (5") x 15 cm (6") fenestrations
 1: Towel
 1: Safety Scalpel: #11
 1: Sterile Procedure Sign
 1: Patient ID Card
 1: Chart Sticker
 1: Patient Information Booklet
 1: Medication Label: 1% Lidocaine
 1: Checklist/CLIP Sheet
 2: Paper Tape Measure
 1: Tourniquet 5: Gauze Pad: 2" x 2" (5 cm x 5 cm)
 10: Gauze Pad: 4" x 4" (10 cm x 10 cm)
 1: Dressing: Tegaderm 15.5 cm x 10 cm
 1: Surgical Apparel: Impervious Gown
 1: Dressing: BIOPATCH®4
 1: Dressing: STATLOCK®5 Catheter Stabilization Device
 1: Surgical Apparel: Mask with Eye Shield
 1: Surgical Apparel: Bouffant Cap
 1: Surgical Apparel: Mask
 1: Tape: Steri-Strip®3
 1: HemoHopper® Fluid Receptacle
 1A trademark of Becton, Dickinson and Company.
 2A registered trademark of CareFusion or one of its subsidiaries.
 3A registered trademark of 3M Company.
 4A registered trademark of Johnson & Johnson Corporation.
 5A registered trademark of C. R. Bard, Inc.


Not made with natural rubber latex.

Store between 20 — 25°C (68 — 77°F).

Arrow International provides the enclosed medication label(s) for your convenience. Please ensure that these labels are applied to the correct syringe and corresponding medication.

Contraindications: The Pressure Injectable P1CC is contraindicated wherever there is the presence of device-related infections or the presence of thrombosis in the intended insertion vessel or catheter pathway. A clinical assessment of the patient must be completed to ensure no contraindications exist.



 Lumen	Priming Volume* (mL)	Gravity Flow Rated (mL/hr)	Pump Flow Rate (mL/hr)	MAX Pressure Injection Flow Rate** (mL/sec)
Distal (17 Ga.) Medial (18 Ga.) Proximal (18 Ga.)	0.55 0.44 0.42	1187 343 363	6800 2820 2920	6 not rated _ not rated


* Priming volumes are approximate and are done without accessories. † Flow rate values are approximate and are determined using deionized water at 100 cm head height.

++Pump flow rates are determined at a maximum pump pressure of 10 psig and represent approximate flow capabilities.

**Pressure injection flow rates are determined at the injector pressure setting of 300 psi maximum using media of 11.8 centipoise viscosity, with 152 cm pressure tubing.

LBL056585 R01 (2022-04)

Documents / Resources

	ErgoPack PR-34063-HPISM Pressure Injectable Three-Lumen PICC [pdf] Instruction Manual PR-34063-HPISM, Pressure Injectable Three-Lumen PICC, PR-34063-HPISM Pressure Injectable Three-Lumen PICC, Injectable Three-Lumen PICC, Three-Lumen PICC
---	--