



ergotron StyleView Primary Storage Drawer User Guide

[Home](#) » [ERGOTRON](#) » [ergotron StyleView Primary Storage Drawer User Guide](#) 

Contents

- [1 ergotron StyleView Primary Storage Drawer](#)
- [2 Product Information](#)
- [3 Product Usage Instructions](#)
- [4 Safety](#)
- [5 Features & Specifications](#)
- [6 Tools Needed](#)
- [7 Documents / Resources](#)
 - [7.1 References](#)
- [8 Related Posts](#)

ergotron®

ergotron StyleView Primary Storage Drawer



Product Information

The product is a cart with drawers and a worksurface designed for use in various settings. The cart has a weight capacity of 2 lbs (1 kg) per drawer and a maximum listed weight capacity that should not be exceeded to avoid serious injury or property damage. The cart also has moving parts that can crush and cut, posing a risk for serious personal injury or property damage if not handled properly. The product comes with a user's guide that provides important information on safety hazards and how to avoid them. The user's guide also contains contact information for Ergotron Customer Care for further assistance.

Product Usage Instructions

Before using the cart, it is important to review the safety hazards and symbols provided in the user's guide to avoid accidents and injuries. Children who are not able to recognize and respond appropriately to Safety Alerts should not use this product without adult supervision.

When using the cart, it is important to:





- Close the worksurface before opening the drawers.
- Open only one drawer at a time.
- Do not push the cart when drawers or worksurface are open to avoid instability.
- Avoid exceeding the maximum listed weight capacity to avoid serious injury or property damage.

For more information and instructions on using the product, refer to the product guide at <http://4support.ergotron.com> or contact Ergotron Customer Care at 1-800-888-8458.


Hazard Symbols Review

These symbols alert users of a safety condition that demands attention. All users should be able to recognize and

understand the significance of the following Safety Hazards if encountered on the product or within the documentation. Children who are not able to recognize and respond appropriately to Safety Alerts should not use this product without adult supervision!

Symbol	Signal Word	Level of Hazard
	NOTE	A NOTE indicates important information that helps you make better use of this product.
	CAUTION	A CAUTION indicates either potential damage to hardware or loss of data and tells you how to avoid the problem.
	WARNING	A WARNING indicates either potential for property damage, personal injury, or death.
	ELECTRICAL	An Electrical indicates an impending electrical hazard which, if not avoided, may result in personal injury, fire and/or death.

Safety

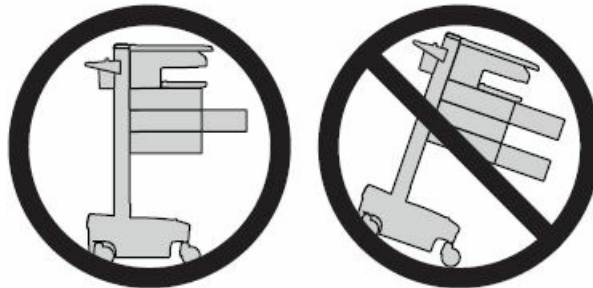



CAUTION: Close the worksurface before opening drawers. Open only one drawer at a time. Do Not push cart when drawers or worksurface are open. Failure to follow these instructions may cause the cart to be unstable.









Features & Specifications

CAUTION: DO NOT EXCEED THE MAXIMUM LISTED WEIGHT CAPACITY. SERIOUS INJURY OR PROPERTY DAMAGE MAY OCCUR!

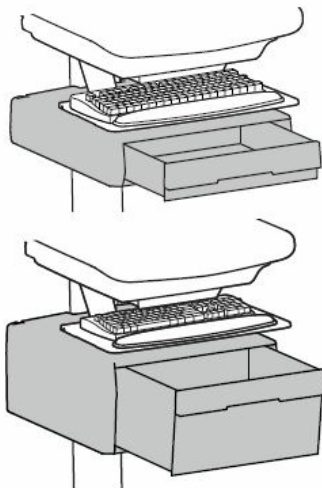


 $\leq 2 \text{ lbs (1 kg)}$
 Total weight capacity per drawer.

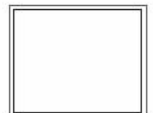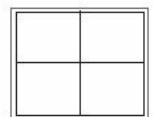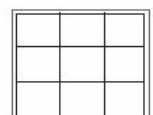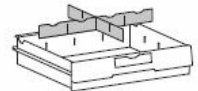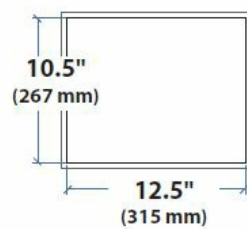

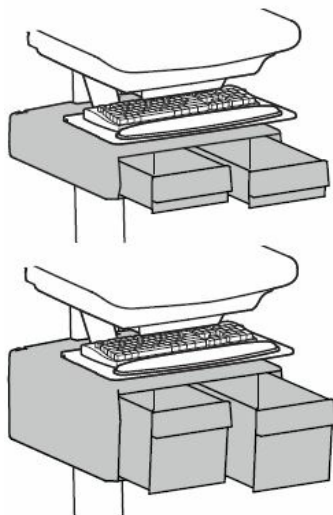



 $\leq 1 \text{ lbs (0.5 kg)}$
 Total weight capacity per drawer.

10.5" (267 mm)
 6" (152 mm)


2.5" (64 mm)

6" (152 mm)

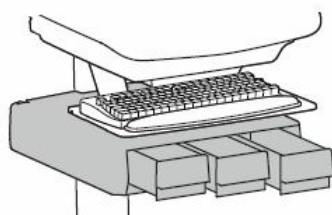



 $\leq 0.25 \text{ lbs (0.1 kg)}$
 Total weight capacity per drawer.


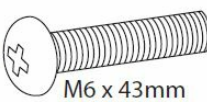

10.5" (267 mm)
 3.75" (96 mm)

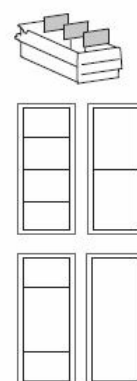








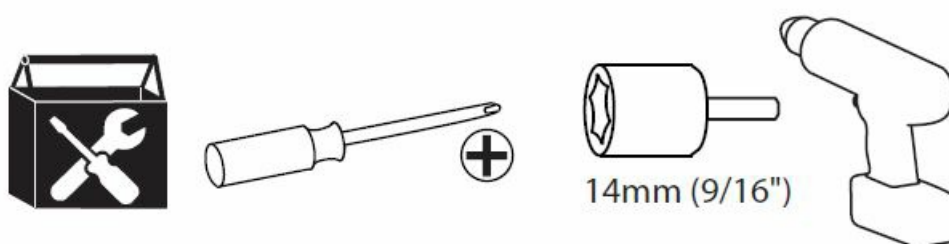


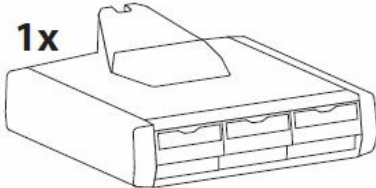
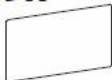
Tools Needed




Components

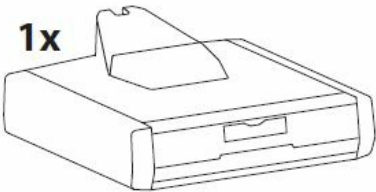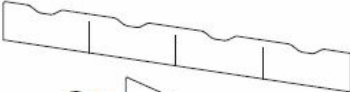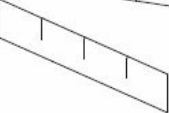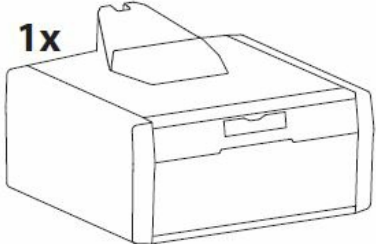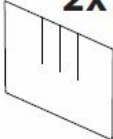1	A	B	C
	1x 	2x  M6 x 43mm	2x  M6 x 15mm





- This manual covers the installation of 5 possible drawers.
- There is only 1 drawer provided in this box.

	A		
2	1 Drawer	1x 	3x  3x 
3	1 Drawer	1x 	2x  2x 
4	2 Drawers	1x 	6x 
5	2 Drawers	1x 	6x 
6	3 Drawers	1x 	9x 



1. Remove the top cover.

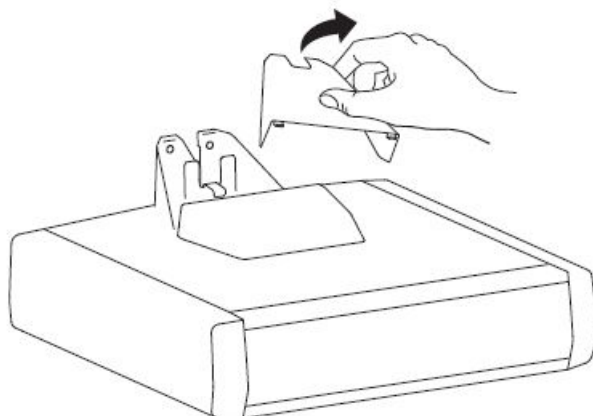



2. Hang the drawer tabs onto the brackets on the cart.

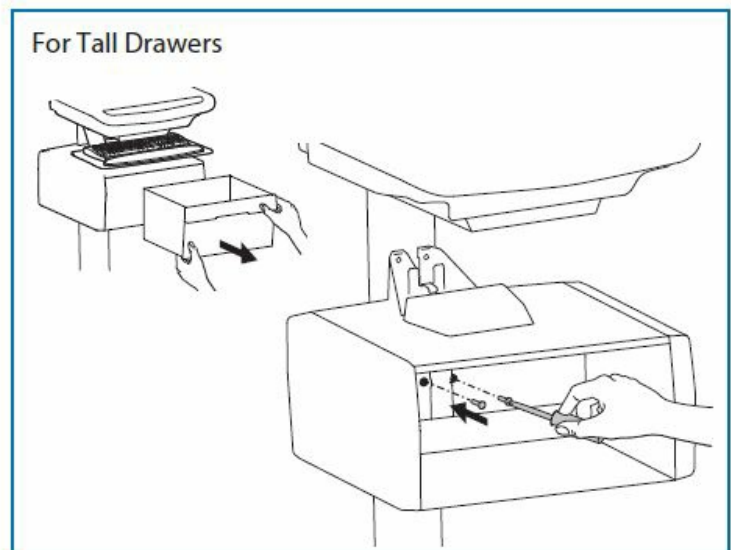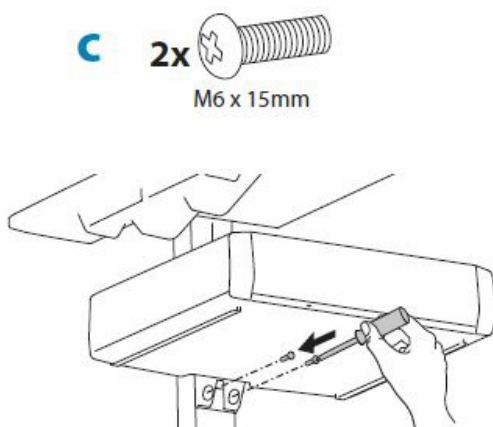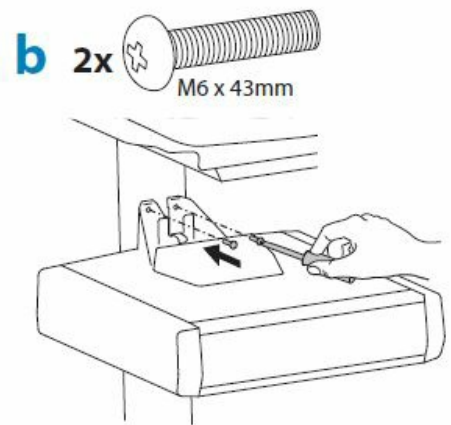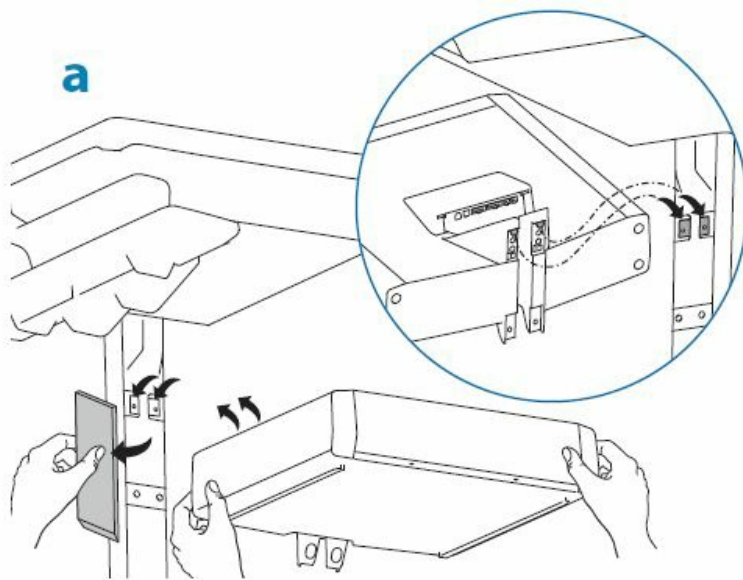

3. Replace top cover on drawer.

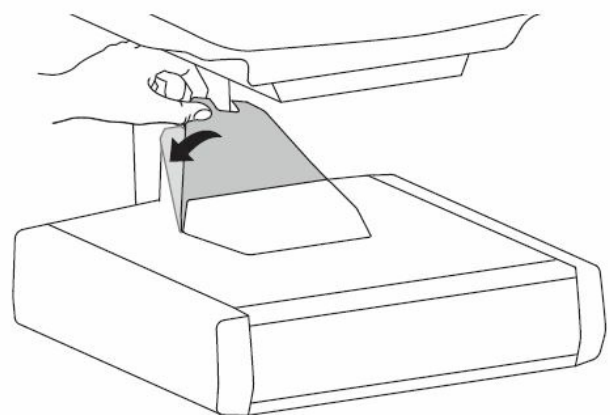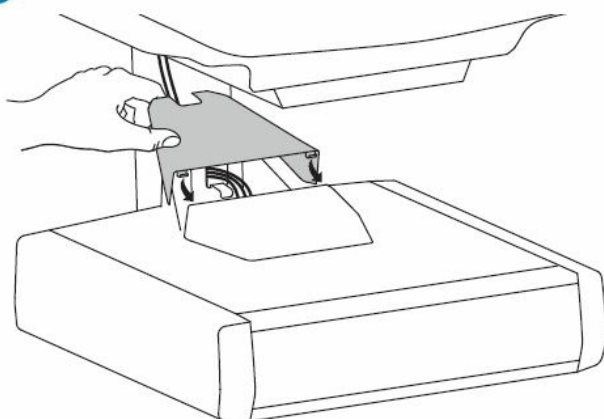


4. To unlock the storage drawer, turn-key counterclockwise 1/4 turn.

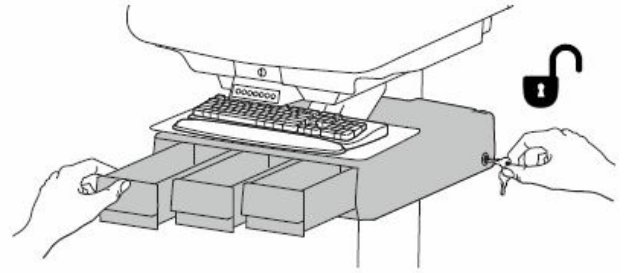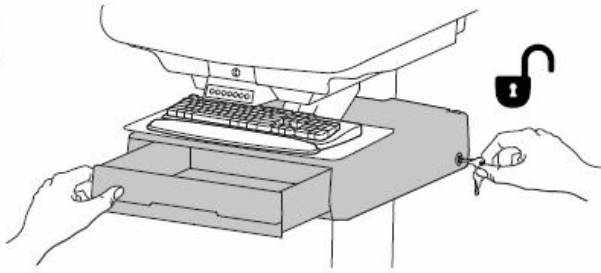



5. Adjustment




Adjust your mounting solution to handle the added weight. Adjustments should move smoothly and easily through the full range of motion and stay where you set it. If adjustments are difficult and do not stay in the desired position, follow the instructions to loosen or tighten the tension to create a smooth, easy adjustment motion. Depending on your product and the adjustment, it may take several turns to notice a difference.

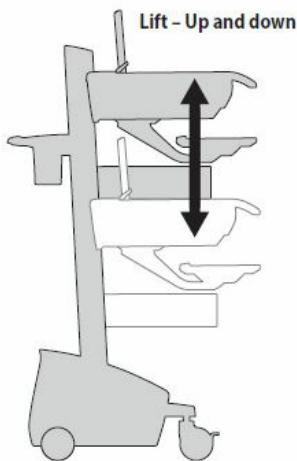

Release Brake to move riser.

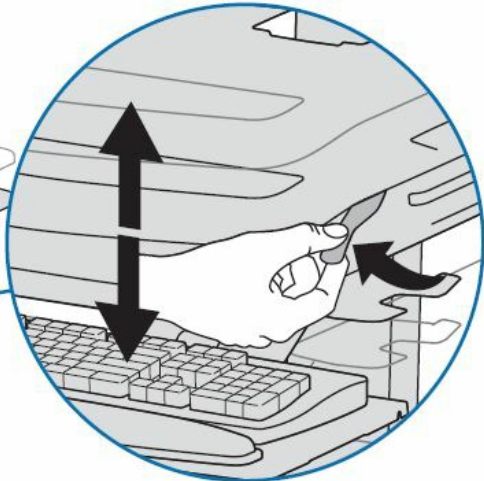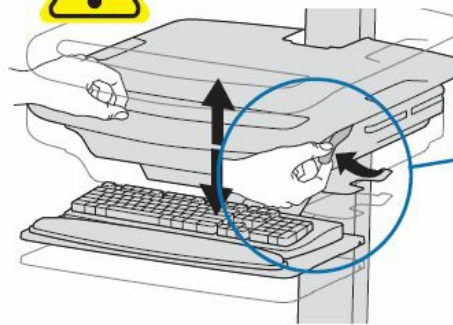


Follow these instructions to tighten or loosen tension.

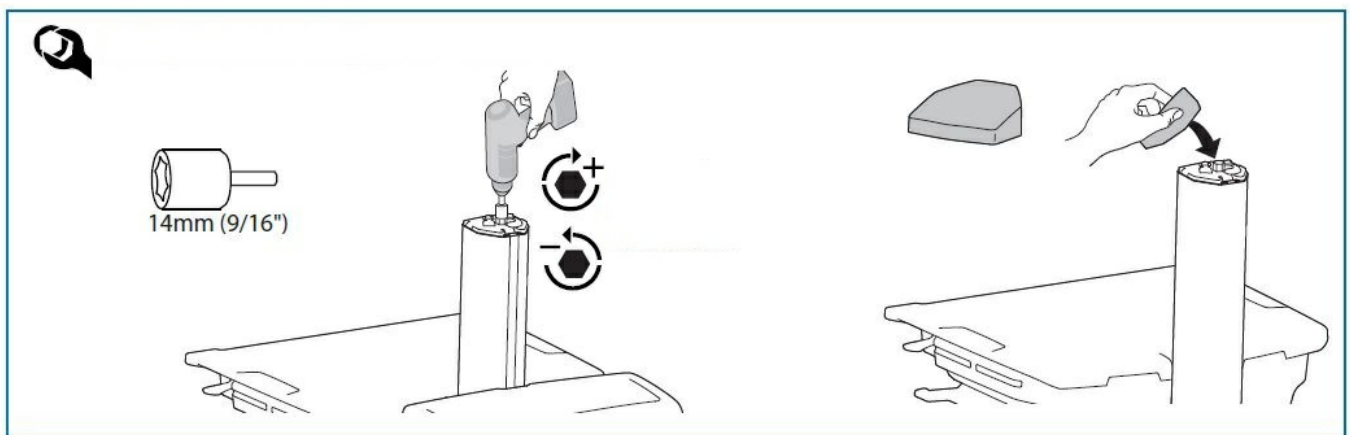

NOTE: Adjustment may require 40 – 60 revolutions.

For Warranty visit: www.ergotron.com/warranty For Service visit: www.ergotron.com For local customer care phone numbers visit: <http://contact.ergotron.com> www.ergotron.com | USA: 1-800-888-8458 | Europe: +31 (0)33-45 45 600 | China: 400-120-3051 | Japan: japansupport@ergotron.com © 2015 Ergotron, Inc. All rights reserved. While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use Of its products, it Will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users, For the most current product information, or to know if this document is available in languages Other than those herein, please contact Ergotron. NO part Of this publication may be reproduced, stored in a retrieval System, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA patents pending and patented US. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.

Documents / Resources

	<p>ergotron StyleView Primary Storage Drawer [pdf] User Guide 97-989-622, StyleView, StyleView Primary Storage Drawer, Primary Storage Drawer, Storage Drawer, Drawer</p>
---	---

References

- © [Monitor Mounts, Computer Carts, Sit to Stand Desks | Ergotron](#)