

eversense E3 Continuous Glucose Monitoring System Instructions

[Home](#) » [eversense](#) » eversense E3 Continuous Glucose Monitoring System Instructions 

Contents

- [1 eversense E3 Continuous Glucose Monitoring System](#)
- [2 Eversense E3 Next Steps](#)
- [3 INSTRUCTIONS](#)
- [4 Documents / Resources](#)
 - [4.1 References](#)
- [5 Related Posts](#)



eversense E3 Continuous Glucose Monitoring System



Eversense E3 Next Steps

Incision care for proper healing

- Do not swim or soak in a tub for five days.
- Avoid strenuous activities that may pull at the incision or cause a lot of sweating around the insertion area while the incision heals.
- Replace Tegaderm™ if it becomes saturated; otherwise, leave it on over the Steri-Strips™.
- Leave Steri-Strips™ on until they fall off.
- Trim the edges of the Steri-Strips™ if they start to curl; do not remove them when doing so.

Notify your doctor if:

- Steri-Strips™ come off before incision is fully closed.
- You develop a fever, or experience pain, redness, swelling, warmth or drainage at the incision site.

Note: If you need to replace your Tegaderm™, take extra care to ensure the Steri-Strips™ do not get pulled off.

Thoroughly review the Benefits and Risks section of your Eversense E3 CGM System User Guide.

Helpful training videos are available at www.eversensedidiabetes.com.

INSTRUCTIONS

Day One: Get Started

- Read the enclosed Eversense E3 User Guide and Quick Reference Guide.
- Download the Eversense App, and pair your transmitter to your mobile device.
- Link your sensor and smart transmitter and begin the 24-hour Warm-Up Phase.

- Enter the settings provided by your diabetes care team.
- Turn off your transmitter, there is no need to wear it until you begin the Initialization Phase.

Day Two: System Initialization Phase Begins

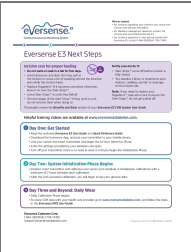

- Position smart transmitter with adhesive over sensor and complete 4 Initialization calibrations with a minimum of 2 hours between each calibration.
- After the 2nd successful calibration, you will begin to see your glucose data.

Day Three and Beyond: Daily Wear

- Daily Calibration Phase begins.
- To share CGM data with your health care provider, go to www.eversensedidiabetes.com, and follow the steps in the Eversense DMS User Guide.

Eversense Customer Care:
 1-844-SENSE4U (736-7348) Support@eversensedidiabetes.com

Documents / Resources

	eversense E3 Continuous Glucose Monitoring System [pdf] Instructions E3, Continuous Glucose Monitoring System, E3 Continuous Glucose Monitoring System, Glucose Monitoring System, Monitoring System
	eversense E3 Continuous Glucose Monitoring System [pdf] User Guide E3, Continuous Glucose Monitoring System, E3 Continuous Glucose Monitoring System, Glucose Monitoring System, Monitoring System

References

- [Introducing the Eversense® E3 CGM System | Ascensia Diabetes Care](#)
- [Introducing the Eversense® E3 CGM System | Ascensia Diabetes Care](#)