



ExamVision PowerGo Total Pure Total Intense LED Light User Guide

[Home](#) » [ExamVision](#) » ExamVision PowerGo Total Pure Total Intense LED Light User Guide 

ExamVision PowerGo Total Pure Total Intense LED Light User Guide



Contents

- [1 Attachment of Total Intense/Pure LED](#)
- [2 Operation of PowerGo control unit](#)
- [3 Video](#)
- [4 Additional information](#)
- [5 Documents / Resources](#)
 - [5.1 References](#)
- [6 Related Posts](#)

Attachment of Total Intense/Pure LED

1. Attach the lamp to the frame part



2. Connect the USB-C port to the lamp



3. Fixate the cable on the right or left temple



4. Connect the USB-C port to the PowerGo Control Unit



Operation of PowerGo control unit



Power on

Press the Power button once



Changing light intensity

One click increases the light intensity, and additional single clicks will cycle the unit through High, Medium and Low intensity settings



Power off

Press and hold for two seconds

Video

You can find the instructional video on our YouTube channel ([www.youtube.com/@ examvision](https://www.youtube.com/@examvision)) or by scanning

the QR code.



Additional information

Read the full Instruction for use for PowerGo here:

www.examvision.com/support

Read the full Instruction for use for Total Pure

Total Intense LED Light here:

www.examvision.com/support

Or just scan the QR code.



facebook.com/ExamVisionDK



instagram.com/examvision/




linkedin.com/company/examvision-aps



youtube.com/@ExamVisionChannel

www.examvision.com

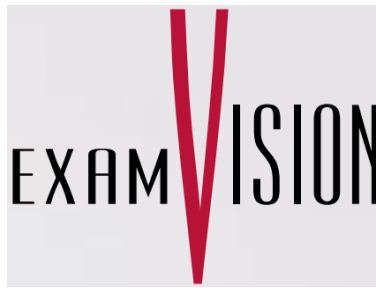

Documents / Resources



[ExamVision PowerGo Total Pure Total Intense LED Light \[pdf\] User Guide](#)
PowerGo Total Pure Total Intense LED Light, PowerGo, Total Pure Total Intense LED Light, Total Intense LED Light, Intense LED Light, LED Light

References

- [Instagram \(@examvision\) • Instagram photos and videos](#)
- [ExamVision: Custom-Crafted Dental Loupes & Surgical Loupes](#)
- [Support • Instructions for Use • Technical Specifications](#)
- [Support • Instructions for Use • Technical Specifications](#)
- [User Manual](#)