FERROX 260308 Parallel Bars with Platform Instruction Manual

[Home](#) » [FERROX](#) » FERROX 260308 Parallel Bars with Platform Instruction Manual 

Art.-Nr. / Prod. Code 3932202

FERROX[®]
Physiotherapy Quality Solutions

Contents

- [1 FERROX 260308 Parallel Bars with Platform Instruction Manual](#)
- [2 ASSEMBLY OF PARALLEL BARS](#)
- [3 IDENTIFICATION LABEL](#)
- [4 DATA SHEET](#)
- [5 CARRYING CAPACITY OF THE DEVICE](#)
- [6 Documents / Resources](#)
 - [6.1 References](#)
- [7 Related Posts](#)

FERROX 260308 Parallel Bars with Platform Instruction Manual

Read the instructions and use the product as directed according to its function.

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

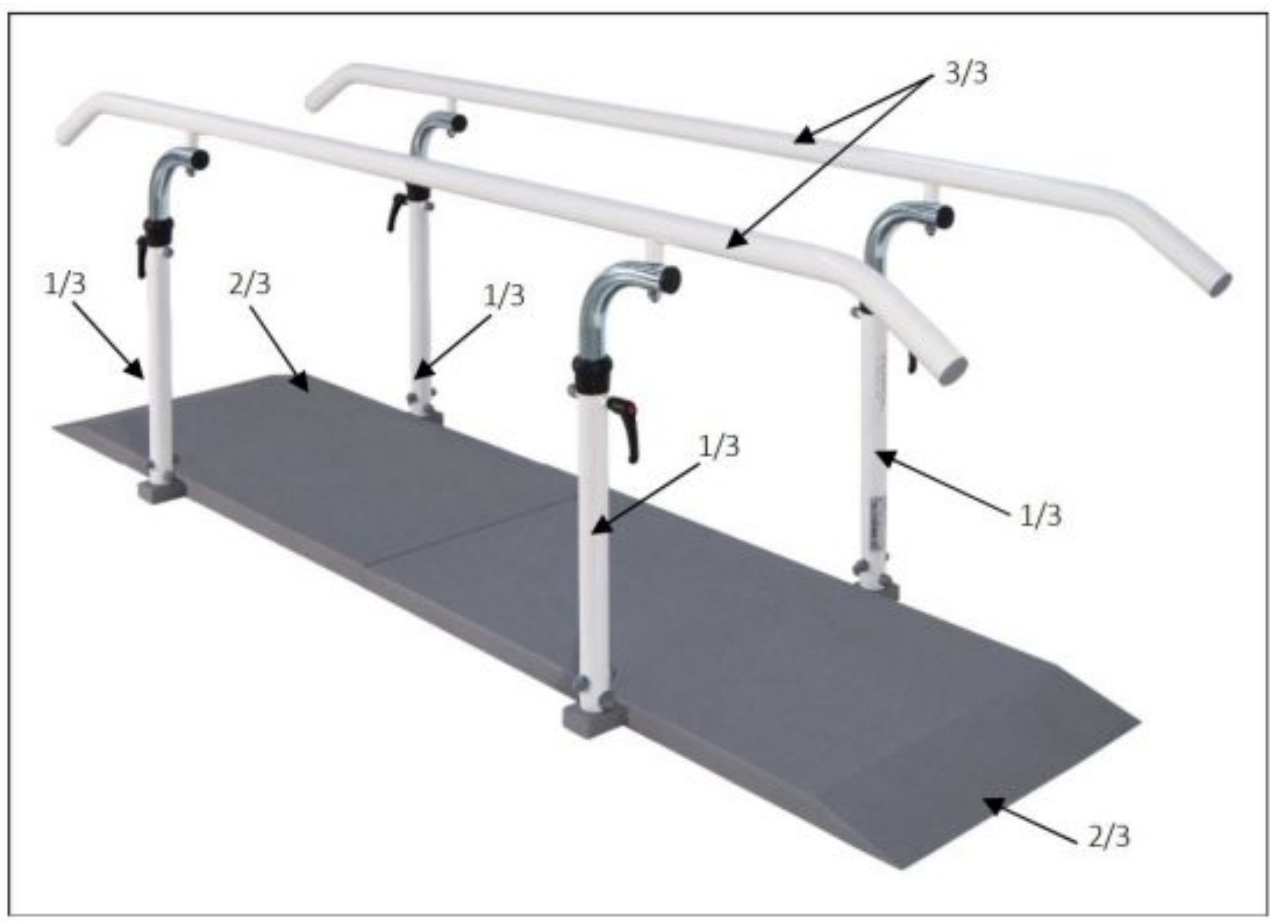
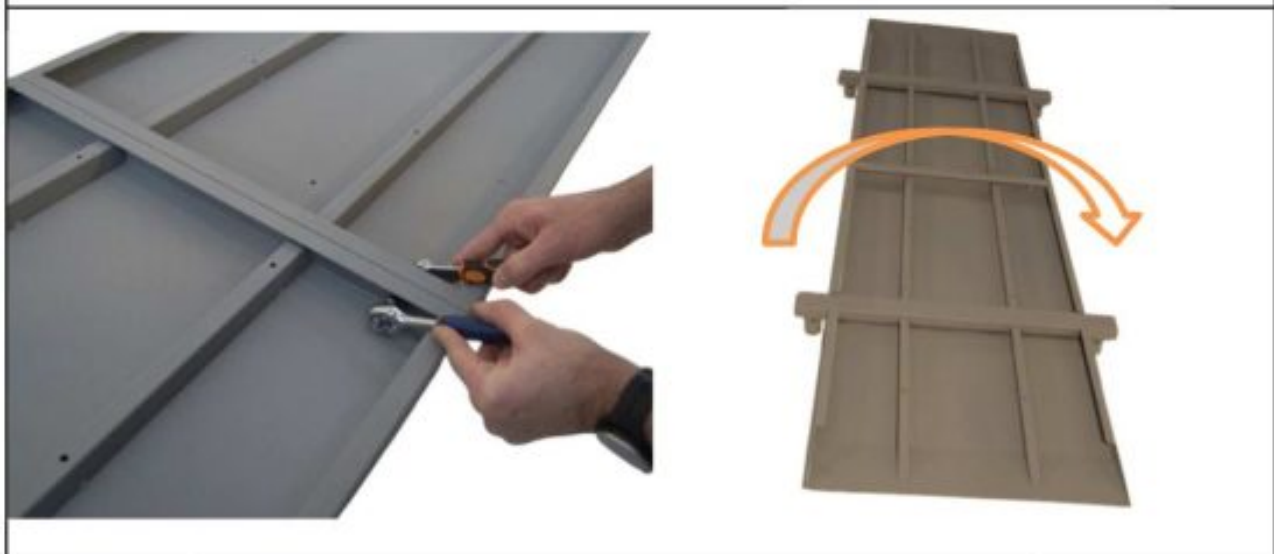
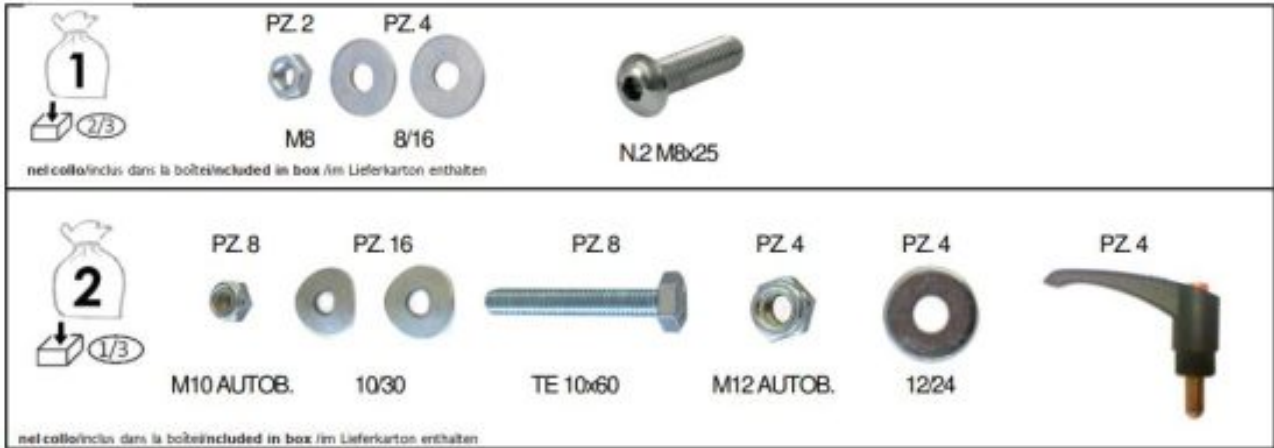
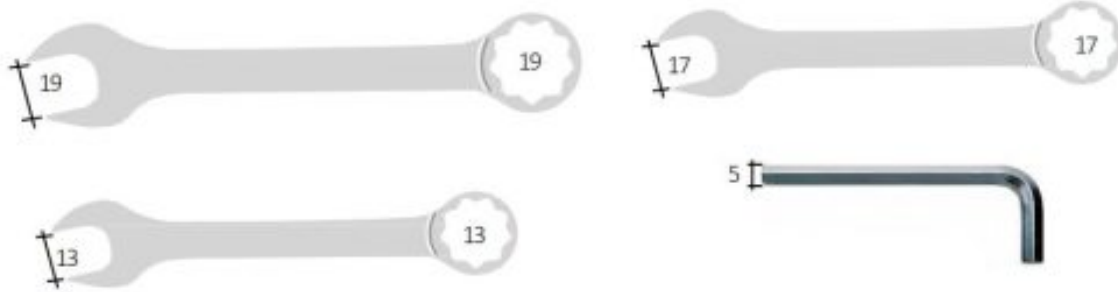
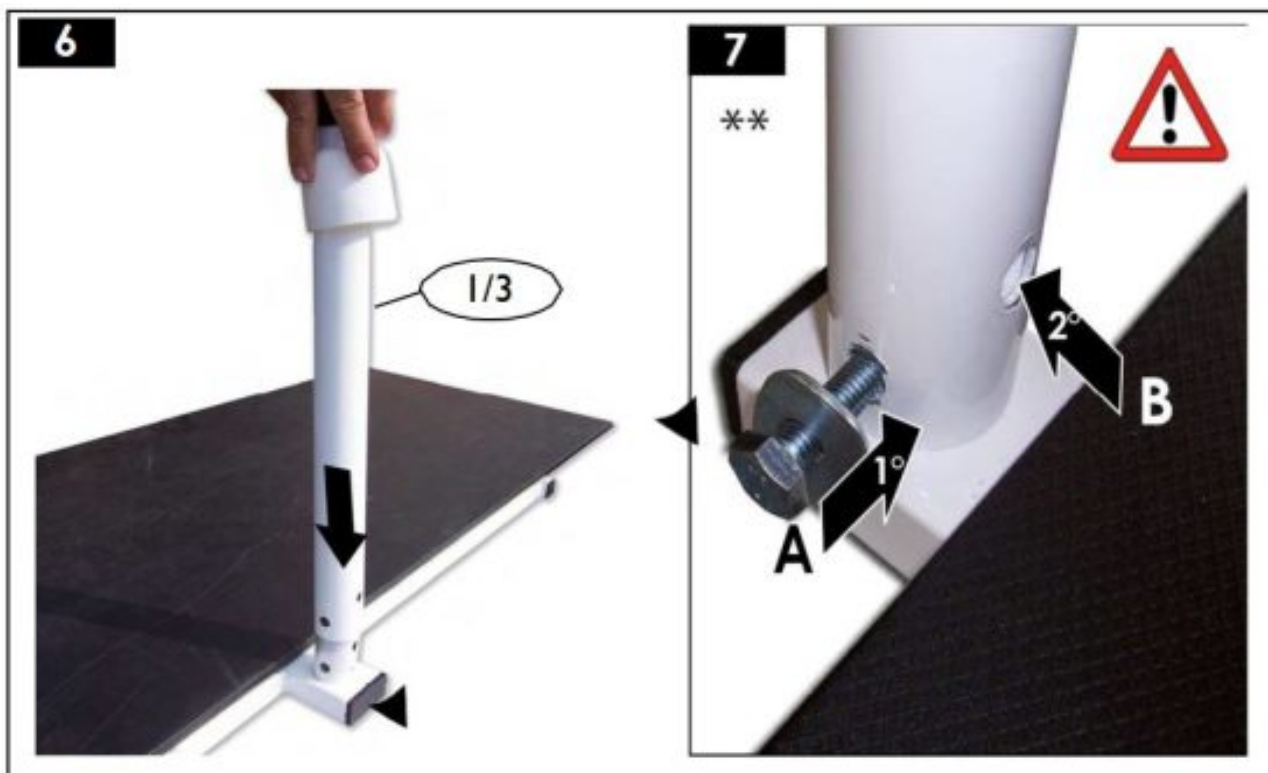
NOTE: TO ENSURE CORRECT ASSEMBLY OF THE ITEM IT IS NECESSARY TO START AT THE BOTTOM HOLE AND PROCEED FROM THERE UPWARDS

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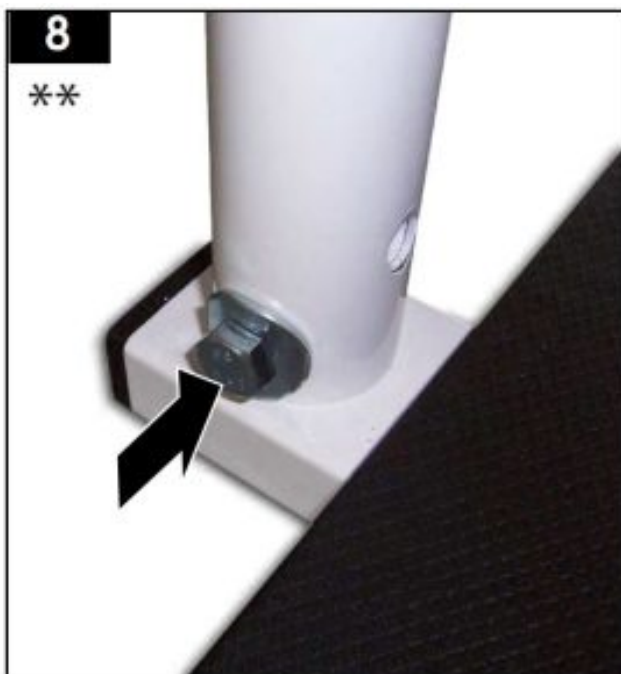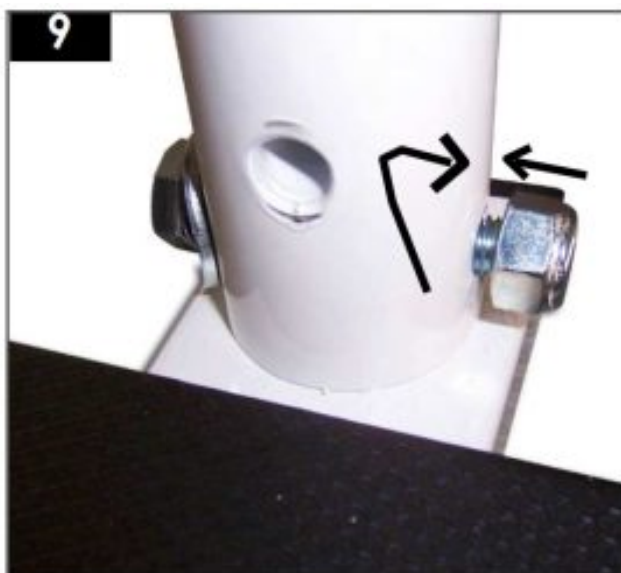
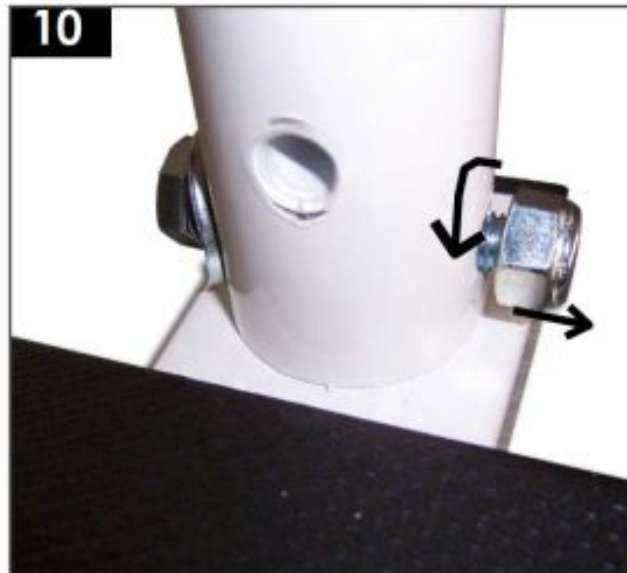
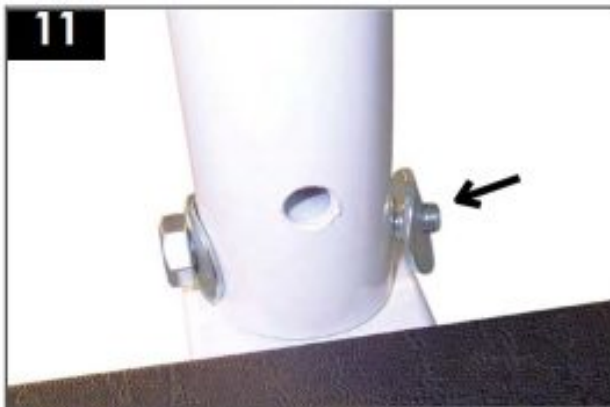
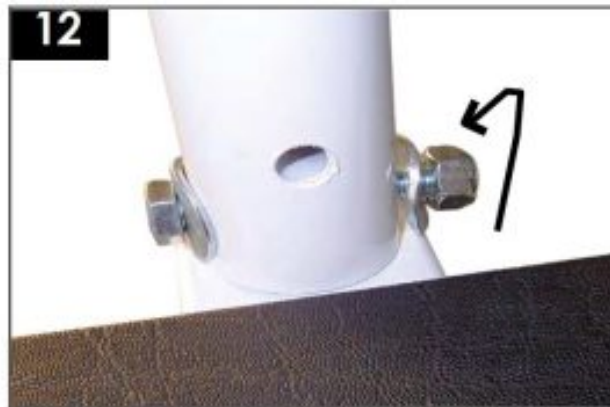
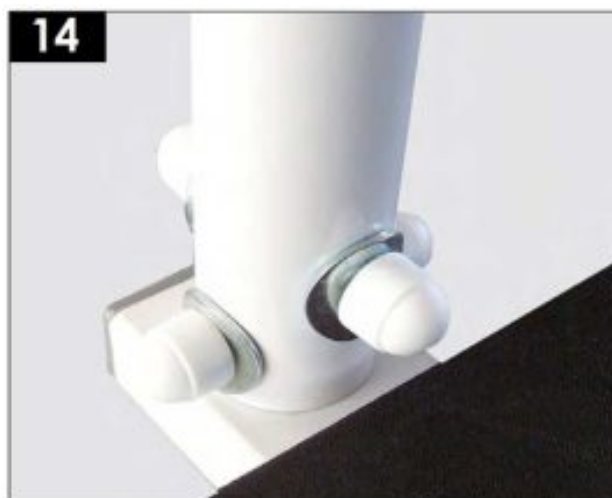
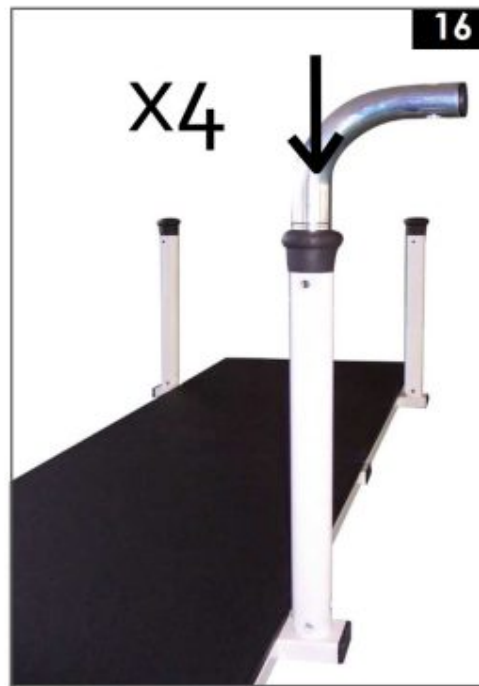
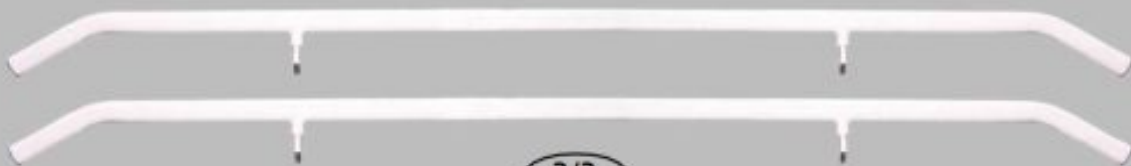
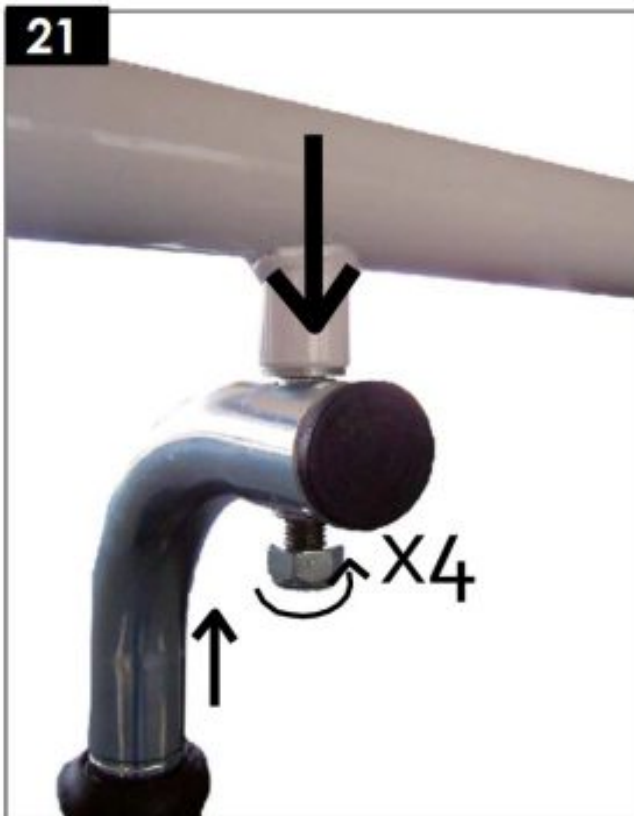
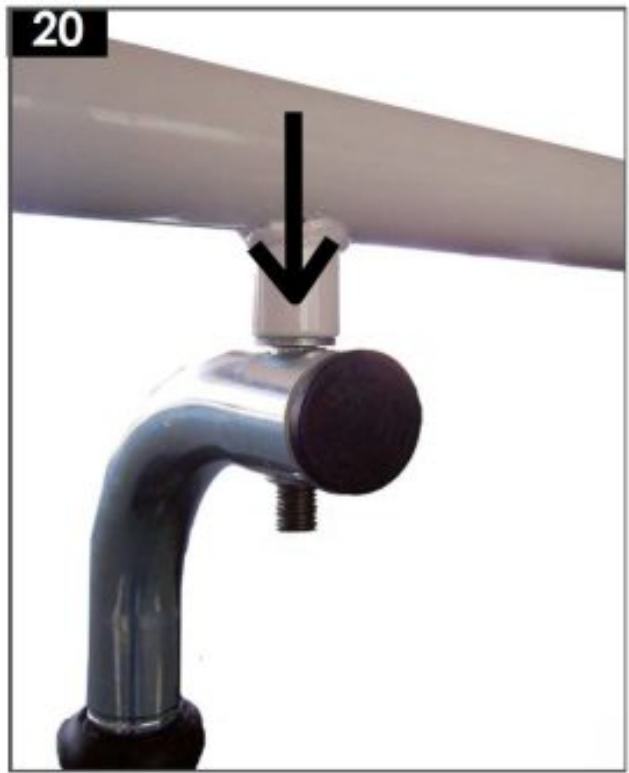
• FIX BOTH HANDRAILS IN THE INTENDED POSITION USING THE NUTS INCLUDED IN DELIVERY

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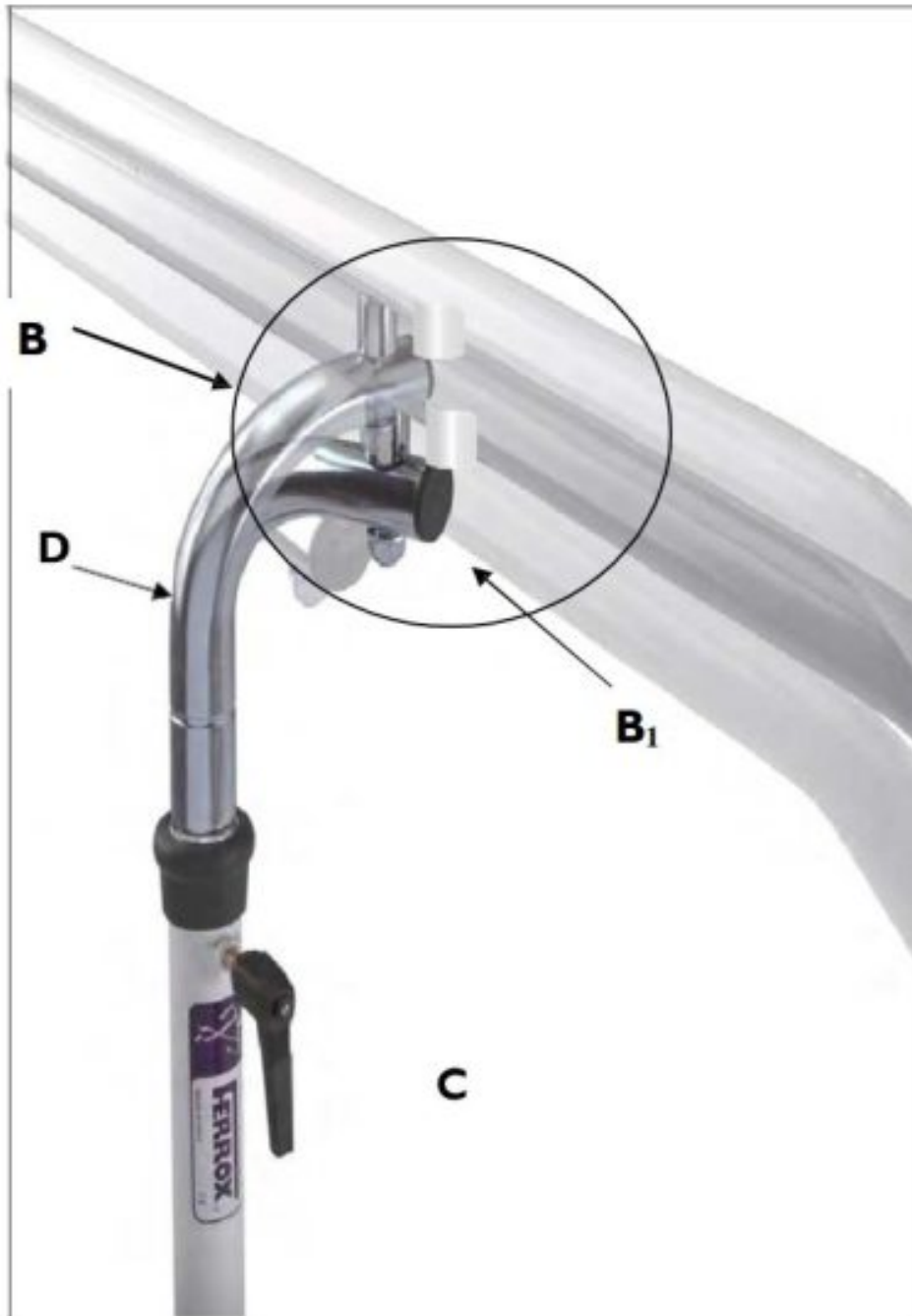
ASSEMBLY OF PARALLEL BARS


- THE ASSEMBLED PARTS (B) PROVIDE FOR THE FIXATION OF THE HANDRAIL TO ITS SUPPORTING STAND. SCREW-NUT (B1) HAS TO BE TIGHTENED WITHOUT IMPEDING THE ROTATION OF THE HANDRAIL, TO ALLOW FOR ITS

ADJUSTMENT IN BREADTH.

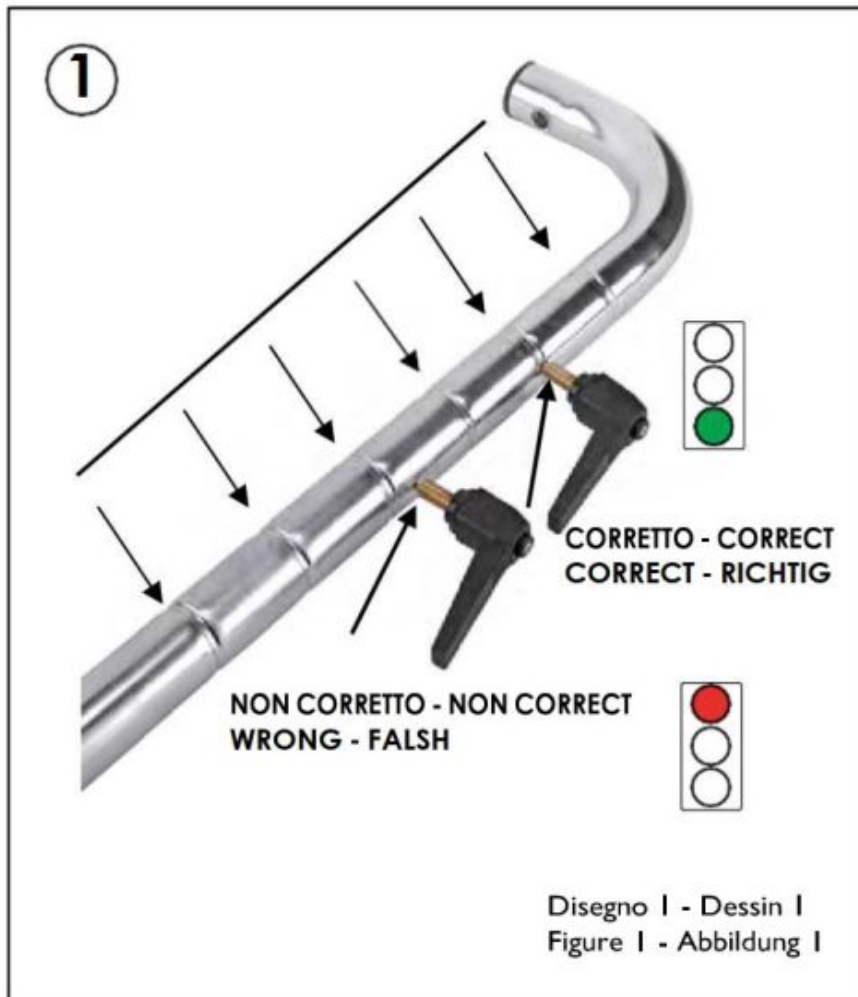
FOR ADJUSTMENT IN BREADTH, RELEASE LEVER (C) AND GENTLY PUSH AND TURN (TO THE LEFT/RIGHT) CHROMED SUPPORT (D).

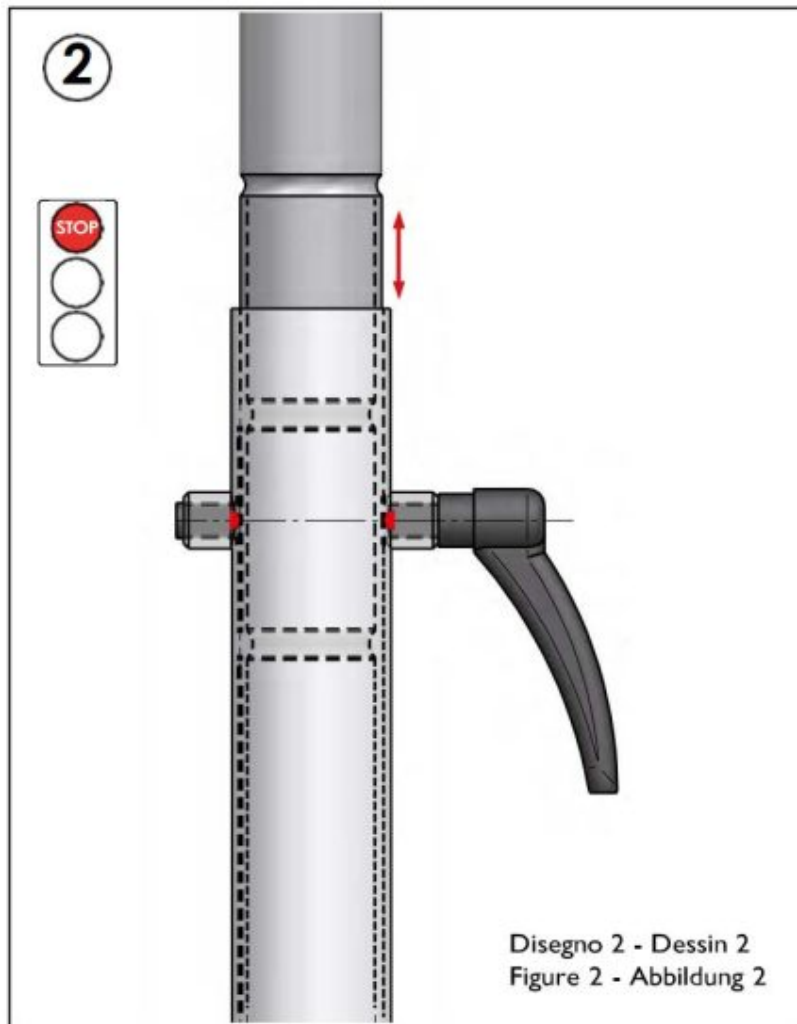THE HEIGHT OF THE PARALLEL BARS IS CORRECTLY AND SECURELY ADJUSTED, ONCE THE FASTENER TIGHTLY SITS IN ONE OF THE SPECIFICALLY DESIGNED NOTCHES

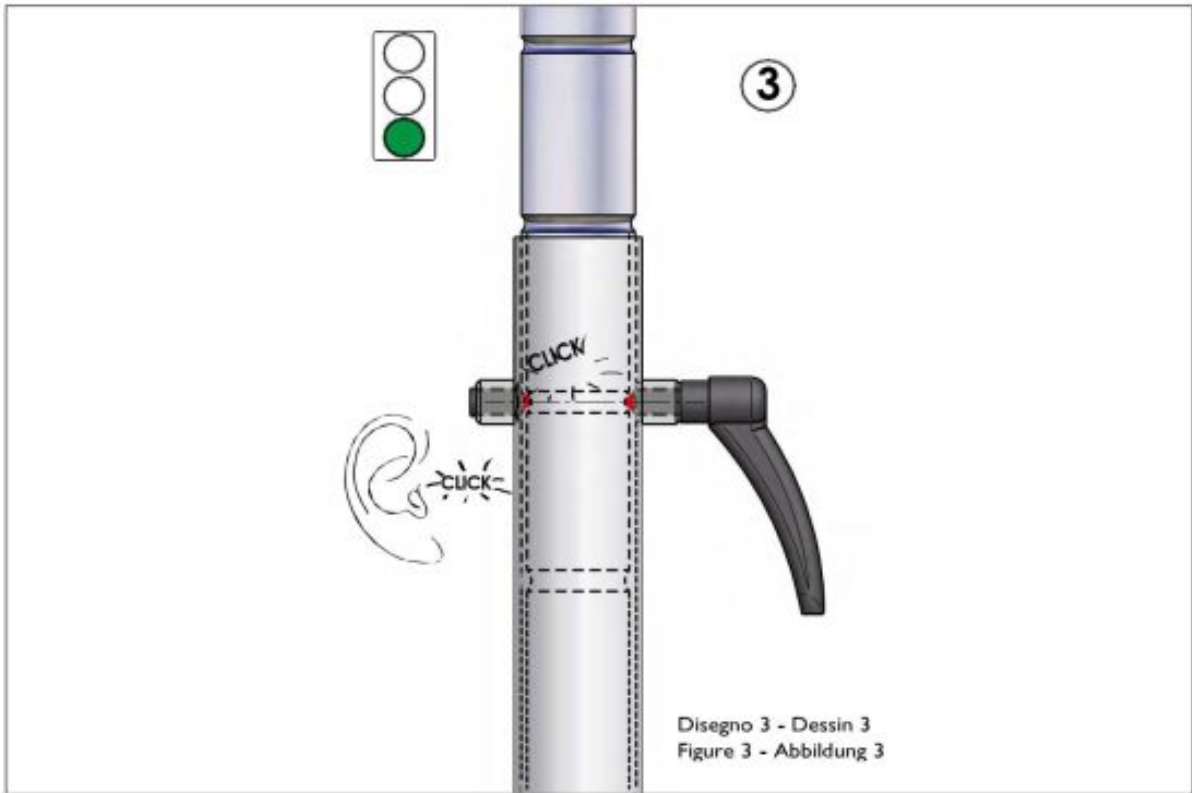THE NOTCH POSITIONING FOR HEIGHT ADJUSTMENT IS: WRONG.
REFER TO FIGURE 3 FOR CORRECT
ALIGNMENT.

THE NOTCH POSITIONING FOR HEIGHT ADJUSTMENT IS: CORRECT.
IN THIS ALIGNMENT THE FASTENER (A) IS CORRECTLY POSITIONED TO SNAP IN THE DESIGNATED NOTCH.

22

- TERMINARE L'ASSEMBLAGGIO RIPETENDO GLI STESSI PASSAGGI ESEGUITI ANCHE PER IL CORRIMANO OPPOSTO
- POUR COMPLÉTER L'INSTALLATION RÉPÉTER LA MÊME PROCÉDURE POUR LA MAIN COURANTE OPPOSÉE
- TO FINISH THE INSTALLATION OF THE PARALLEL BARS, ASSEMBLE THE OPPOSITE HANDRAIL BY REPEATING THE ABOVE PROCEDURE
- UM DIE INSTALLATION ABZUSCHLIEßEN, MONTIEREN SIE DEN GEGENÜBERLIEGENDEN HANDLAUF NACH DER OBEN BESCHRIEBENEN VORGEHENSWEISE

23

- ASPETTO CHE AVRÁ ASSUNTO LA PARALLELA DOPO CHE L'ASSEMBLAGGIO DEL CORRIMANO SARÁ STATO ESEGUITO
- VUE DE LA BARRE PARALLELE APRES QUE LES MAINS COURANTES AURONT ÉTÉ CORRECTEMENT INSTALLÉES
- VIEW OF THE PARALLEL BARS WITH THE HANDRAILS CORRECTLY INSTALLED
- ANSICHT DES GEHBARRENS MIT KORREKT INSTALLIERTEN HANDLÄUFEN

**IDENTIFICATION LABEL**

FERROX[®]
 FERROX s.r.l.
 Via del Lavoro, 9 Z.I.
 31013 Codognè (TV) - Italia
 Tel. +39 0438 777099
 Fax +39 0438 777091
 www.ferrox.it - info@ferrox.it

MD MODEL
REF YYYY
SN XXXXXX
 YY/MM/DD

 **CE**
 MADE IN ITALY

UDI

 (01)0805347043YYYY(11)YYMMDD(21)XXXXXX Rev. 02 del 03/20

CH
REP

Device & Care Sagl – CHE456306571
 Via Mulino 3, 6855 Stabio SWITZERLAND
 TEL.: +41 762642520
 EMAIL: info@deviceandcare.com

 Simbolo o Sigla	 Descrizione	 Description	 Beschreibung	
MD	Dispositivo Medico	Dispositif Médical	Medical Device	Dies ist ein Medizinprodukt
REF	Identificativo catalogo	Identifiant catalogue	Catalogue Number	Katalognummer
SN	Numero di serie	Numéro de série	Serial Number	Seriennummer
	Identificazione fabbricante	Identification du fabricant	Manufacturers name	Names des Herstellers
	Data di fabbricazione	Date de fabrication	Date of manufacture	Herstellungsdatum
	Carico di lavoro in sicurezza	Charge de travail en sécurité	Safety work load	Sichere Arbeitslast
	Consultare le istruzioni per l'uso	Consulter le mode d'emploi	Read the manual	Anteilung lesen
CE	Marchio di conformità CE MDR 2017/745	Marque de conformité CE MDR 2017/745	CE mark MDR 2017/745	CE – Kennzeichen MDR 2017/745
UDI	Identificazione unica del dispositivo	Identification Unique des Dispositifs Médicaux	Unique Device identification	Einmalige Produktkennung
CH REP	Mandatario Svizzero	Mandataire Suisse	Swiss Representative	Schweizer Vertreter

WARNINGS

CLASSIFICATION AND USE

PARALLEL BARS WITH PLATFORM: 260308

Read the instructions and use the product as directed according to its function.
Do not use the product in a way other than its intended purpose.

INTENDED USE

Parallel bars with base for continuous use in manned hospital environment.
Immobilised patients with simple fractures or residents in aged-care facilities, who require a firm support, adjustable in height and width for a greater feeling of security.

TYPE OF PATIENT

Adults who need to learn how to walk again or simply to move around safely inside a room and along a path.

PLACE OF USE

Hospitals, specialised clinics, aged-care facilities

OPERATORS QUALIFIED TO USE THE PRODUCT

All healthcare professionals, physiotherapists, and related industries.
The instructions given to the patient by the healthcare professional.

CONTRAINDICATIONS

Follow the safety instructions in user manual.
Not suitable for sports use.

OTHER INDICATIONS

The device does not have essential levels of performance.
Report all serious incidents that occur due to the device. Report the incident to the manufacturer and the competent authority

At the end of its life, the product must be disposed of in accordance with local regulations and taken to specific waste collection facilities.

DATA SHEET

PARALLEL BARS WITH PLATFORM: 260308

Dimensions: 250 X 36 Δ 87 X 78 Δ 104 cm

Weight: 57,5 kg

Storage: dry, dark place, -20°+60°

Cleaning: it is generally recommended to clean the device after each use and to carry out the disinfection cycle periodically. To clean the device, wipe its parts with a cloth moistened with warm water. Do not use any agents that could damage the surface of the device through abrasion or overly aggressive treatment.

Operating temperature: 0° / +40°

Expected lifetime: it is estimated that the device can be used for 10 years.

It is mandatory for the user and the patient to use disposable gloves and shoes with clean rubber soles with disposable overshoes

CARRYING CAPACITY OF THE DEVICE

DECLARATION OF CONFORMITY UE

MDR 2017/745

Device & Care Sagl – CHE456306571
Via Mulino 3, 6855 Stabio SWITZERLAND
TEL.: +41 762642520 – EMAIL: info@deviceandcare.com

Manufacturer:

FERROX S.r.l.
Via del Lavoro, 9 Z. I. - 31013 Codogné (TV) - ITALIA
☎ +39 0438 777099 - 📠 +39 0438 777091
C. F. - P. IVA - R. I. IT 03613220262

info@ferrox.it

SRN: IT-MF-000011243

Under its sole responsibility state that the products:

Name	Code	UDI-DI Basic:
PARALLEL BARS WITH PLATFORM 250 CM.	260308	805347043PARALLELAD2

meet all the applicable essential requirements, of Annex I, implemented by the
(UE) REGULATION 2017/745 OF THE EUROPEAN PARLAMENT AND COUNCIL on Apr 5 th 2017.

Classified (Annex VIII, MDR 2017/745):

Class I (Rule 1)

Procedure for the assessment of conformity:

Technical documentation as per annex II and III.

Date: 10/01/2025

Responsible FERROX s.r.l.

Signature: _____

FERROX®

Physiotherapy Quality Solutions

Management System
ISO 13485:2016
ISO 9001:2015
www.tuv.com
ID 300004946

Stabilimento e sede legale:
via del Lavoro Z.I.
31013 Codogné (TV) Italia
Tel. +39 0438 777099 - 778370
Fax +39 0438 777091
P.IVA: IT 03613220262

e-mail: info@ferrox.it
www.ferrox.it

Read More About This Manual & Download PDF:

Documents / Resources

	<p>FERROX 260308 Parallel Bars with Platform [pdf] Instruction Manual 260308, 3932202, 260308 Parallel Bars with Platform, 260308, Parallel Bars with Platform, Bars with Platform</p>
--	--

References

- [User Manual](#)

[Manuals+](#), [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.