



FreeStyle LibreLink App User Guide

[Home](#) » [FreeStyle Libre](#) » FreeStyle LibreLink App User Guide 

Free Style Libre Link App User Guide



With the FreeStyle LibreLink app,¹ you can scan your FreeStyle Libre 2 sensor with an Android smartphone

1. Go to Play Store
2. Search Free Style Libre Link
3. Look for Free Style Libre Link GB and click
4. Click Install
5. Once downloaded open the app and follow the on-screen instructions to get set-up

SCANNING THE FREESTYLE LIBRE 2 SENSOR WITH AN ANDROID SMARTPHONE

1. Open the FreeStyle LibreLink app on your Android smartphone



2. Hold the back of the Android smartphone near the sensor and position the Near Field Communication (NFC) antenna over the sensor



3. Continue holding your smartphone near the sensor until you hear a second tone and/or feel a vibration. This completes the scan




ALARM. SCAN. ACT.

- Remove bulky or metallic phone cases as they can interfere with the NFC signal.
- Do not move the smartphone until you hear the first tone and/or feel a vibration. This indicates your smartphone and sensor have established an NFC connection.
- 60-minute warm-up required when applying the sensor.


The FreeStyle LibreLink app and your Android smartphone

It's easy to set alarms on your FreeStyle LibreLink app


1. Tap Alarms in the menu

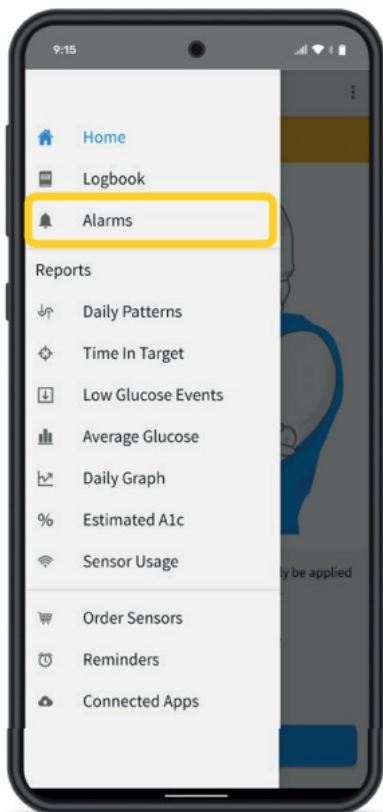


2. Touch Low Glucose Alarm and turn on alarm (alarms are off by default) Signal Loss Alarm is automatically turned on when glucose alarm is turned ON

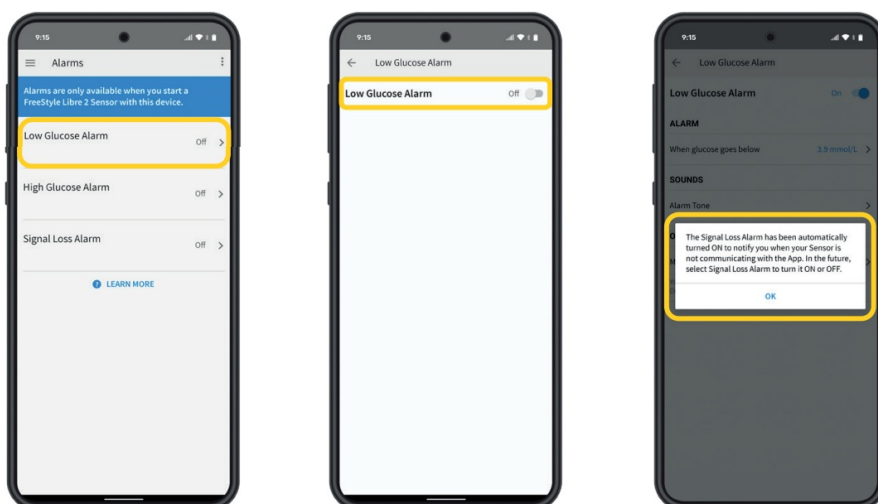


3. Scroll to select Low Glucose Value

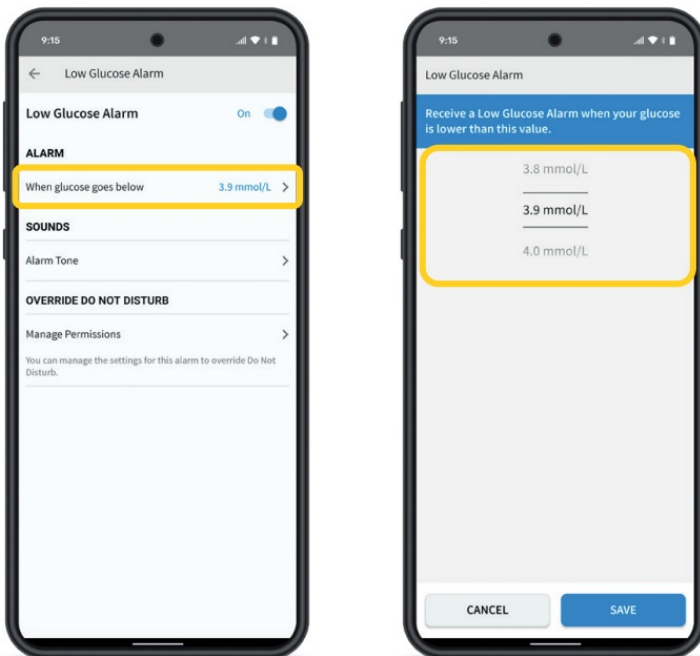

4. Touch Alarm Tone and make your tone choice

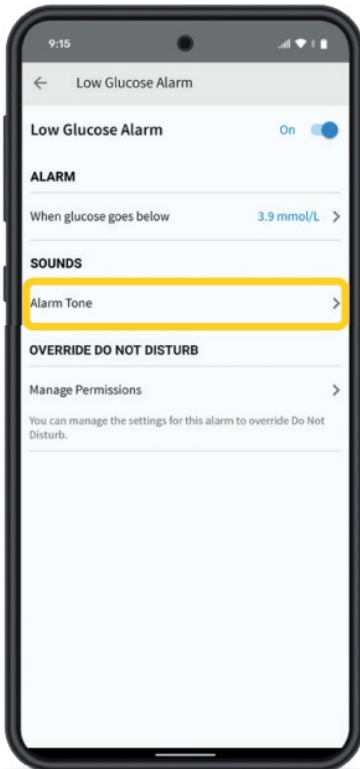

5. Set your High Glucose Alarm using the same steps

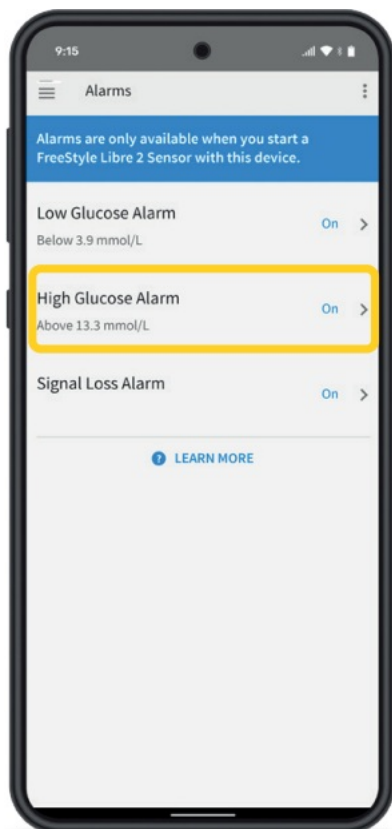

Images are for illustrative purposes only. Not actual patient data.

1. The FreeStyle LibreLink app is only compatible with certain mobile devices and operating systems. Please check the website for more information about device compatibility before using the app. Use of FreeStyle LibreLink requires registration with LibreView.
2. Scanning the sensor does not require lancets.
3. The mobile app is free to download and use. Internet connection is required to download the app, charges may apply.
4. To receive alarms on your phone, you should start your FreeStyle Libre 2 sensor with your compatible phone. Please see the FreeStyle LibreLink User's Manual for complete instructions:
www.diabetescare.abbott/support/manuals/uk.html.
5. 3.9 mmol/L is the default Low Glucose Alarm level and can be set between 3.3-5.6 mmol/L.
6. 13.3 mmol/L is the default High Glucose Alarm level and can be set between 6.7-22.2 mmol/L. © 2021 Abbott. FreeStyle, Libre, and related brand marks are marks of Abbott. ADC-31516 v2.0 11/21.







Contents

[1 Documents / Resources](#)

[1.1 References](#)

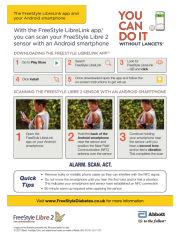[2 Related Posts](#)

Documents / Resources

 <p>The FreeStyle LibreLink app and your phone</p> <p>YOU CAN DO IT</p> <p>With the FreeStyle LibreLink app, you can scan your FreeStyle Libre 2 sensor with an Android phone to get the FreeStyle LibreLink app.</p> <p>DOWNLOADING THE FREESTYLE LIBRELINK APP</p> <ol style="list-style-type: none"> 1. Tap the Play Store icon. 2. Search for the FreeStyle LibreLink app. 3. Tap the Get button. 4. Tap the Open button. 5. Tap the Allow button. 6. Tap the Open button. <p>SCANNING THE FREESTYLE LIBRE 2 SENSOR WITH AN ANDROID PHONE</p> <ol style="list-style-type: none"> 1. Tap the Scan button. 2. Hold the phone over the sensor. 3. Tap the Scan button. <p>ALWAYS SCAN ACT.</p> <p>Quick Tips</p> <p>• To help you get started, the app will guide you through the setup process.</p> <p>• For more information, visit www.FreeStyleLibre.com or call 1-800-828-8888.</p> <p>FreeStyle Libre 2  </p>	<p>FreeStyle Libre FreeStyle LibreLink App [pdf] User Guide</p> <p>FreeStyle LibreLink App, FreeStyle LibreLink, App</p>
 <p>The FreeStyle LibreLink app and your phone</p> <p>YOU CAN DO IT</p> <p>With the FreeStyle LibreLink app, you can scan your FreeStyle Libre 2 sensor with an iPhone.</p> <p>DOWNLOADING THE FREESTYLE LIBRELINK APP</p> <ol style="list-style-type: none"> 1. Tap the App Store icon. 2. Search for the FreeStyle LibreLink app. 3. Tap the Get button. 4. Tap the Open button. 5. Tap the Allow button. 6. Tap the Open button. <p>SCANNING THE FREESTYLE LIBRE 2 SENSOR WITH AN IPHONE</p> <ol style="list-style-type: none"> 1. Tap the Scan button. 2. Hold the phone over the sensor. 3. Tap the Scan button. <p>ALWAYS SCAN ACT.</p> <p>Quick Tips</p> <p>• To help you get started, the app will guide you through the setup process.</p> <p>• For more information, visit www.FreeStyleLibre.com or call 1-800-828-8888.</p> <p>FreeStyle Libre 2  </p>	<p>FreeStyle Libre FreeStyle LibreLink App [pdf] User Guide</p> <p>FreeStyle Libre 2, FreeStyle LibreLink App, FreeStyle LibreLink, App</p>

References

-  [Home - FreeStyle Libre | Abbott](#)