



## geemarc AMPLIDECT295 Additional Handset User Guide

[Home](#) » [geemarc](#) » geemarc AMPLIDECT295 Additional Handset User Guide 



### USER GUIDE



**!!! Please refer to our website: [www.geemarc.com](http://www.geemarc.com) for an up to date user guide, as there may be important updates and changes you need to be aware of !!!**

## Contents

- 1 **AMPLIDECT295 Additional Handset**
- 2 **PACKAGE CONTENTS**
- 3 **INSTALLATION**
- 4 **OVERVIEW**
- 5 **HANDSET REGISTRATION**
- 6 **BOOSTER MODE**
- 7 **GUARANTEE**
- 8 **Documents / Resources**
  - 8.1 **References**
- 9 **Related Posts**

## AMPLIDECT295 Additional Handset

Congratulations on purchasing your Geemarc PhotoDECT295 handset, offering all the advantages of cordless technology

(compliant with Digitally Enhanced Cordless Telecommunications – DECT standard). which can be used as additional handset with our AMPLIDECT295 or AMPLIDECT COMBI 295 phones.

The Photodect 295 is hearing aid compatible.

**CAUTION:** Use only the battery pack provided. There may be a risk of explosion if a wrong type of battery is used

for the handset. Do not subject the battery to extreme high or low temperatures, or low air pressure at high altitude during use, storage or transportation. Disposal of battery into fire or a hot oven, or mechanically crushing or cutting or the battery can result in an explosion. Subjecting the battery pack to an extremely high temperature environment, or extremely low air pressure can result in an explosion, or leakage of flammable liquid or gas.

Normal operating temperature for the product (e.g. 0°C~40°C). Use only the adapters and battery pack listed below.




Power adapter information: output: 6V DC 0.45A, 2.7W. Vtech Telecommunications Limited.

Model: VT05EEU06045 (EU), VT05EUK06045 (UK)




Rechargeable Battery Pack: Model: NI-MH AAA500x 2. 2.4V, 500mAh. Corun.


## Handset

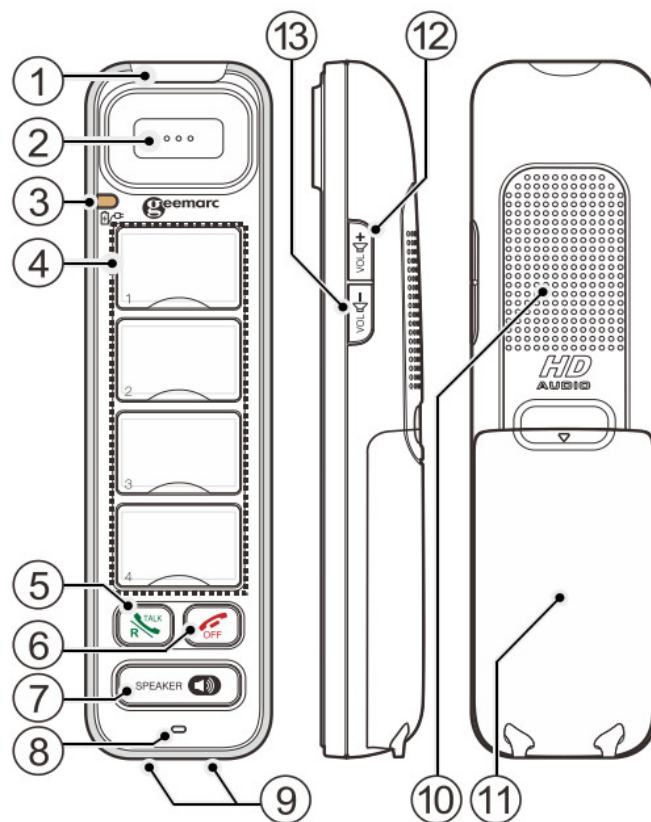


## Insert Photos for speed dial keys

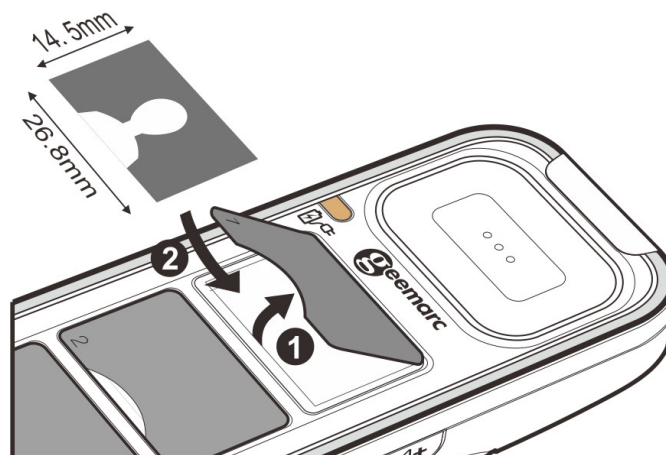

## Pod Connection

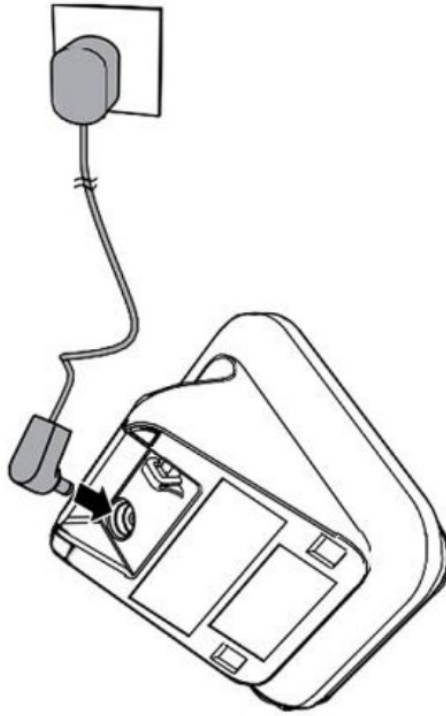

## Battery charging or Handset Registration

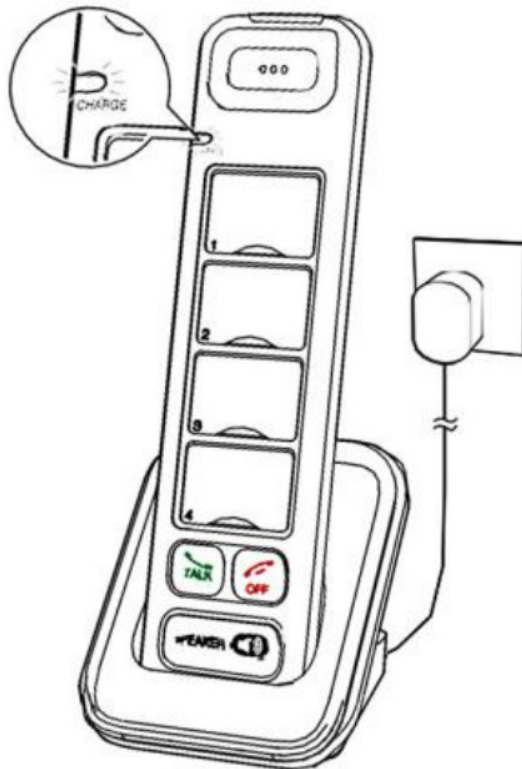

## PACKAGE CONTENTS

1 cordless handset, 1 Power adapter, 1 Charger, 1 NiMh battery pack, 1 User guide.

## INSTALLATION

Connect the cable from the power adapter to the small round socket on the base of the pod. Then plug the power

adapter into a mains outlet socket.  
Only use the power adapter supplied with this phone!

### Connect the battery pack

Connect the short lead from the battery pack to the socket inside the battery compartment. Carefully fit the battery pack as shown.



Align the battery cover flat against the battery compartment, then slide it towards the center of the handset until it clicks into place

### Charge the battery pack

Charge the batteries for at least 16 hours when charging for the first time.

**Note :** YOU WILL NEED TO REGISTER THIS HANDSET WITH A BASE STATION FROM ONE OF THE FOLLOWING PRODUCTS: DECT BASE (AMPLIDECT COMBI 295 or AMPLIDECT295).

## OVERVIEW

### 1. Incoming Call light

Flashes when receiving an incoming call.

### 2. Earpiece

### 3. Charger/Booster LED ( )

- Steady red when the handset battery is charging.
- Long flashes in red when the handset battery is low.
- Flashes red quickly when handset registration is in progress.
- Flashes red slowly when the handset is not registered or in charger.
- Steady blue when the booster function is activated.

### 4. Photo speed dial keys

This handset has 4 photo speed dial keys. These retrieve the same numbers stored in the photo and Direct Memory keys of the AMPLIDECT295 or AMPLIDECT COMBI 295.

Press to dial a pre-stored number.

### 5. Talk ON / Flash ( )

- Make or answer a call.
- Green light shows when handset is in earpiece mode.
- During a call, Press to access Recall feature (R).

### 6. TALK OFF ( )

- Hang up a call.

- Silence the ringer temporarily while the phone is ringing.
- Mute the paging tone.

## 7. **SPEAKERPHONE** ()

- Make or answer a call using the speakerphone.
- Switch between the speakerphone and the earpiece.
- Light Orange when on speakerphone mode.

8. Microphone

9. Charging contacts

10. Speakerphone

11. Battery compartment cover

## 12. **VOL+** ()

- Increase listening volume up to 30dB amplification during a call, (or increase ringer volume in idle mode).
- Press and hold the key during a call to activate or no the receiver Booster function (LED bleu).

## 13. **VOL-** ()

Decrease listening volume during a call. (or decrease ringer volume in idle mode (until ringer turn OFF)).

## **HANDSET REGISTRATION**

Base: First place the base of the AMPLIDECT295 or AMPLIDECT COMBI 295 in Registration mode.

Handset: Place the handset in the Pod. The red LED flashes quickly to show registration in process.

Registration can take up to 60 seconds. A confirmation tone is heard when registration is complete.

Note: Repeat this procedure if the handset remains unregistered. Quickly press [VOL+] [SPK] [VOL-] [SPK] [VOL+] [VOL-] [SPK] in idle mode, then HS will have a “Bi,Bi” tone to confirm that the procedure is a success.

## **BOOSTER MODE**

Booster mode will increase the listening volume (by up to 30dB amplification).

- During a call, press and hold the VOL+ button for 2 seconds to turn on BOOSTER MODE. (LED will be steady blue).
- To switch off BOOSTER MODE, press and hold VOL+ again.

## **GUARANTEE**

From the moment your Geemarc PhotoDECT295 is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit our website at [www.geemarc.com](http://www.geemarc.com).

The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorized Geemarc representative.

The Geemarc guarantee in no way limits your legal rights.

**IMPORTANT:** YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

Please note: The guarantee applies to the United Kingdom only.

## **DECLARATIONS:**

### **UKCA Directives**


Geemarc Telecom SA hereby declares that this PhotoDECT295 is in compliance with the essential requirements and other relevant provisions of the Radio Equipment Directive 2014/53/UE.

The UKCA declaration of conformity may be consulted at [www.geemarc.com](http://www.geemarc.com)

## RF Parameter

Frequency	1.88 GHz to 1.9 GHz
NB Channels	10
RF power	< 250mW

## Power Supply

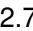Charger	Input 100-240V 50/60Hz; Output DC6V  0.45A, 2.7W adapter
Handset	2.4V,500mAH rechargeable NiMh battery pack

Electrical connection: The apparatus is designed to operate from a 230V 50Hz supply only. (Classified as hazardous voltage according to EN62368-1 standard).

The apparatus does not incorporate an integral power on/off switch. To switch off the apparatus, disconnect the power adaptor.

When installing the apparatus, ensure that the mains power socket is readily accessible.

## Recycling Directives

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way.

When this product is finished with, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip.  
Deposit the product in an appropriate WEEE skip.
- Or, hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.



**For product support and help visit our website at**

[www.geemarc.com](http://www.geemarc.com)

E-mail : [help@geemarc.com](mailto:help@geemarc.com)

telephone 01707 387602

lines are open 09h00 to 16h00 Mon to Fri

Parc de l'étoile, 2 rue Galilée, 59760 Grande-Synthe, France.



---

## Documents / Resources



[geemarc AMPLIDECT295 Additional Handset](#) [pdf] User Guide  
AMPLIDECT295 Additional Handset, AMPLIDECT295, Additional Handset, Handset

## References

- [🏠 Home - Geemarc UK](#)
- [User Manual](#)