GETINGE 6036003601 Universal Versatile Washer Disinfectors with Efficient





GETINGE 6036003601 Universal Versatile Washer Disinfectors with Efficient Instructions

Home » GETINGE » GETINGE 6036003601 Universal Versatile Washer Disinfectors with Efficient Instructions



Contents

- 1 GETINGE 6036003601 Universal Versatile Washer Disinfectors with
- 2 Instructions for Use
- 3 Directions for Use in Automated Washing Equipment
- **4 FEATURED**
- **5 CONTACT**
- 6 Documents / Resources
 - **6.1 References**



GETINGE 6036003601 Universal Versatile Washer Disinfectors with Efficient



Instructions for Use

XEN Universal is a mild alkaline liquid detergent for use in automated washer applications, including washer-disinfectors and cart washers. Universal is low foaming to optimize the performance of the automated washer. It is safe to use on robotic instruments. Its three enzymes—protease, lipase, and amylase—break down blood and proteins, fats and oils (lipids), and carbohydrates, respectively.


Order no.	Description
6036003601	Universal 1 x 5 L.
6036003701	Universal 1 x 5 gal.
6036003801	Universal 1 x 15 gal.



Directions for Use in Automated Washing Equipment

The detergent should be automatically dosed into the wash cycle. Dosing is 0.13–0.51 oz./gal. (1–4 ml/l), depending on soil load and water quality. Higher doses will be required for higher soil loads and for poor water quality. Automated washers should be set to dose at 95°F (35°C). The wash temperature should be set at 122°F–140°F (50°C–60°C).

FEATURED

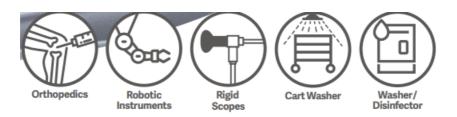

Caution

- · Do not mix with other chemicals.
- Wear correct PPE and follow applicable guidelines. Additional information on SDS.
- · Check material compatibility before use.


Danger: This may cause allergy or asthma symptoms or breathing difficulties if inhaled. Avoid breathing fumes, mists, vapours, or sprays. In case of inadequate ventilation, wear respiratory protection.

First Aid: IF INHALED: If breathing is difficult, remove the person to fresh air and keep at rest in a comfortable position for breathing. If experiencing respiratory symptoms, call a poison center or doctor.

Dispose of contents and containers in accordance with local, regional, and international regulations. Contains triethanolamine, protease, amylase, and lipase enzymes. For more ingredient information, visit www.getinge.com

Storage Information

Store in a cool, dry place protected from frost and away from acids and strong oxidizing agents. Store upright in original containers. Optimum storage temperature 41°F–77°F/5°C–25°C.

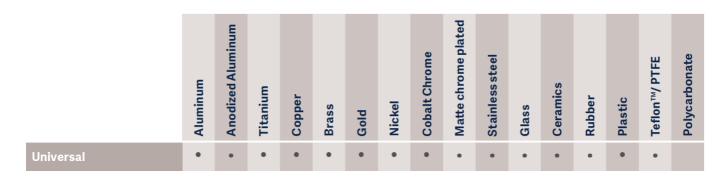

- The product is suitable for the material
- The product is suitable for the material and is a trademark of Chemours.

Getinge is a registered trademark of Getinge AB or its subsidiaries or affiliates in the United States or other countries. Copyright 2024 Getinge AB or its subsidiaries or affiliates. • All rights reserved.

CONTACT

- Manufacturer: Quadralene Ltd, Unit 3, St. Modwen Park, Andressey Way, Derby, DE21 6YH, United Kingdom.
- Sales Office, US: 1 Geoffrey Way, Wayne, NJ 07470, USA
- Sales Office, Canada: 90 Matheson Blvd West, Suite 300, Mississauga, Ontario L5R 3R3, Canada
- www.getinge.com

Documents / Resources



GETINGE 6036003601 Universal Versatile Washer Disinfectors with Efficient [pdf] Instructi

6036003601, 6036003601 Universal Versatile Washer Disinfectors with Efficient, Universal Versatile Washer Disinfectors with Efficient, Versatile Washer Disinfectors with Efficient, Washer Disinfectors with Efficient, Disinfectors with Efficient, Efficient

References

• User Manual

Manuals+, Privacy Policy

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.