




# GETINGE Servo-c Mechanical Ventilator User Guide

[Home](#) » [GETInGE](#) » GETINGE Servo-c Mechanical Ventilator User Guide 

## Contents

- [1 GETINGE Servo-c Mechanical Ventilator](#)
- [2 Product Usage Instructions](#)
- [3 Setting up the Servo-c](#)
- [4 Documents / Resources](#)
  - [4.1 References](#)
- [5 Related Posts](#)



**GETINGE Servo-c Mechanical Ventilator**



## Specifications

- **Product Name:** Servo-c
- **Power Source:** Plug-in power cord
- **Ventilation Types:** Invasive and non-invasive
- **Patient Categories:** Adult and pediatric
- **Weight Monitoring:** Tidal Volume in ml/kg
- **Alarm Priority Levels:** High, Medium, Low

## Product Usage Instructions

### Setting up the Servo-c

1. Plug in the power cord.
2. Switch ON the ventilator by pulling the ON/OFF switch downwards.
3. Connect the oxygen and air hoses.
4. Lock the wheels on the cart.
5. Start the PRE-USE CHECK by following the on-screen instructions. Note that you will need the test tube during this check. The patient circuit test is included but can also be selected separately.
6. Choose the PATIENT CATEGORY: adult or pediatric.
7. Choose the VENTILATION TYPE: invasive or non-invasive (optional).
8. Calculate Predicted Body Weight (PBW) automatically by tapping PBW and entering patient gender and height. For pediatric patients, enter the patient's weight. Tidal Volume in ml/kg will now be monitored.
9. Tap VENTILATION MODES. If you tap and hold on a mode, you will get more information.
10. Choose a mode and adjust the settings accordingly.
11. Some settings have dynamic images.

12. The safety scale can be expanded by tapping on it.
13. Change the alarm limits in the alarm window.
14. Tap START VENTILATION to begin.

For more detailed information, refer to the user's manual.

## FAQ

- **Q: What are the alarm priority levels?**

A: The alarms can be in one of three colors depending on priority: High priority, Medium priority, or Low priority.

- **Q: How can I adjust alarm limits in controlled modes?**

A: Tap on the active alarm and you can adjust alarm limits directly.

- **Q: How do I stop ventilation?**

A: Tap Standby and then tap and hold to stop ventilation.

- **Q: Where can I find additional values and settings?**

A: Additional values and settings can be found at [www.getinge.com](http://www.getinge.com).


## Setting up the Servo-c

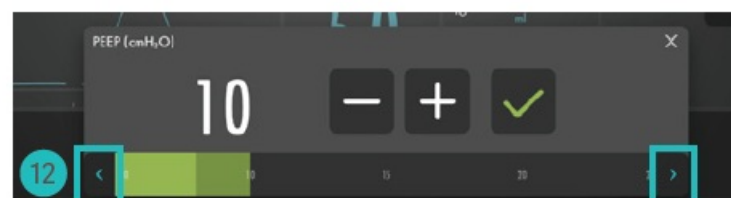
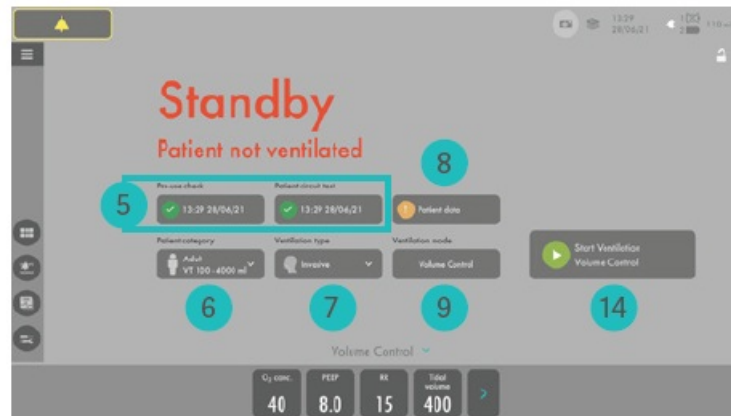
1. Plug in the power cord.
2. Switch ON the ventilator.

**NOTE:** When switching on the Servo-c, you need to first pull the ON/OFF switch downwards.






3. Connect the oxygen and air hoses.
4. Lock the wheels on the cart.
5. Start the PRE-USE CHECK. Follow on-screen instructions. (You need the test tube during the Pre-Use Check).  
The patient circuit test is included in the Pre-Use Check, but can also be selected separately.
6. Choose PATIENT CATEGORY: adult or pediatric.

7. Choose VENTILATION TYPE: invasive or non-invasive (optional).
8. Calculate Predicted Body Weight (PBW) automatically by tapping PBW and entering the patient's gender and height. (For the pediatric patient category it is the patient's weight). Tidal Volume in ml/kg will now be monitored.
9. Tap VENTILATION MODES. (If you tap and hold on to the mode you will get more information).
10. Choose mode and adjust the settings.
11. Some settings have dynamic images.
12. The safety scale can be expanded by tapping .
13. Change the alarm limits in the alarm window.
14. Tap START VENTILATION.



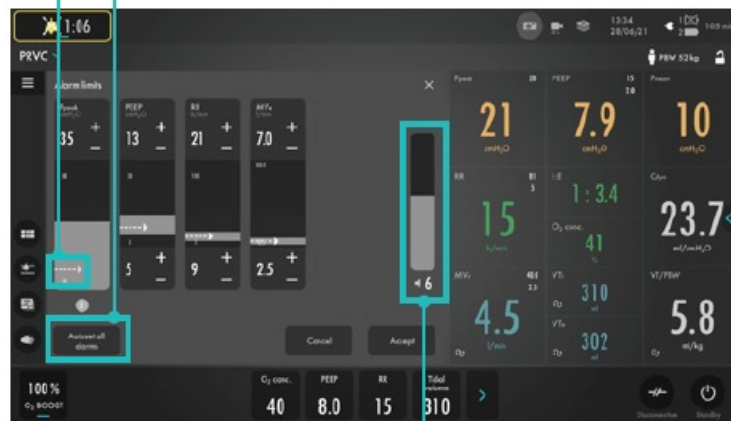
#### NOTE:

Alarms can be in one of three colors depending on priority:

-  High priority
-  Medium priority
-  Low priority

The arrow shows the actual measured value for the patient.

Autoset Alarm limits in controlled modes



Alarm volume



Alarms audio paused  
– Active alarms can be silence for two minutes by tapping.



Pre-silence alarms  
– To silence most alarms for two minutes, tap when no alarms are active.




Tap on the active alarm and you can adjust alarm limits directly.



This document is intended to provide information to an international audience outside of the US. Servo-c may be pending regulatory approvals to be marketed in your country. Contact your Getinge representative for more information. Manufacturer · Maquet Critical Care AB · Röntgenvägen 2 SE-171 54 Solna · Sweden · +46 (0)10 335 73 00 [www.getinge.com](http://www.getinge.com).

## Documents / Resources

 <p><b>Servo-c</b> — short guide</p>	<p><a href="#">GETINGE Servo-c Mechanical Ventilator [pdf] User Guide</a> Servo-c Mechanical Ventilator, Servo-c, Mechanical Ventilator, Ventilator</p>
---	---

## References

- [User Manual](#)