

GIMA Bandage Scissors Instruction Manual

Home » GIMA » GIMA Bandage Scissors Instruction Manual





All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

The blades of bandage scissors are made of 2Cr13 stainless steel, the handles of PP.

The blades of these scissors have been milled in order to cut clothes, gauzes, and bandages.

Bandage scissors are suitable for:

- Cutting medical gauze
- Cutting bandages away from the skin

Warning

Remember that scissors may cause harm if not handled with care.

Please pay due attention and care during use.

Choose a safe place for storing the scissors, out of reach of non-authorized personnel.

Clean the scissors after use and eliminate any residue. Keep the scissors closed when not in use.

Any equipment showing signs of wear should be discarded immediately.

Instructions for use

Firmly grip the scissors, cut starting from the front of the body. Do not cut laterally, pay attention in order not to damage the patient's skin.

*	Keep in a cool, dry place
茶	Keep away from sunlight
[]i	Consult instructions for use
REF	Product code
LOT	Lot number
NON	Non-sterile
***	Manufacturer
C€	Medical Device compliant with Regulation (EU) 2017/745
\triangle	Caution: read instructions (warnings) carefully
MD	Medical Device

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

Contents

1 Documents / Resources

1.1 References

2 Related Posts

Documents / Resources



GIMA Bandage Scissors [pdf] Instruction Manual GIMA, Bandage, Scissors

References

• © Gima, articoli medicali, apparecchi per medicina, elettromedicali, endoscopi, misuratori medicali, arredo ambulatorio, elettrobisturi, articoli per il pronto soccorso, articoli per la sterilizzazione

Manuals+,