



# GIMA FFP2 NR Particle Filtering Mask Instructions

[Home](#) » [GIMA](#) » GIMA FFP2 NR Particle Filtering Mask Instructions 

## Contents

- [1 GIMA FFP2 NR Particle Filtering Mask](#)
- [2 Wearing instructions](#)
- [3 Documents / Resources](#)
- [4 Related Posts](#)



**GIMA FFP2 NR Particle Filtering Mask**



It should be worn in order to block the exhalation or ejection of contaminating particles from the mouth and from the nasal cavity into common spaces – NON-STERILE, SINGLE-USE, DUSTPROOF – 42.8% nonwoven fabric, 28.6% hot-air cotton, 28.6% meltblown fabric – Do not press excessively during transport, in order to avoid damaging the package. 3-year validity, do not use beyond the expiration date.

### **Wearing instructions**

1. Place the nose piece on your nose and the mask on your chin.
2. Pull the elastic loops behind your ears and adjust the mask.
3. Adjust the nose piece to your nose.
4. Make sure that the mask fits close to your face.

### **WARNING**

- Risk may be expected if the product is used in a surgical setting or in a setting with significant exposure to bodily or other hazardous liquids or fluids; or used in a clinical setting where the risk of infection through inhalation is high; or used in the presence of a high-intensity heat source or flammable gas.
- The product is not designed for use in an anoxic or underwater environment.
- Check for integrity before use.
- This product is not a respirator.
- The mask does not eliminate the risk of contracting any disease or infection.
- Failure to properly use and maintain this product could result in illness.
- Do not use beyond expiration date.
- Do not use if the package is damaged.
- Do not use with oxygen and use only in ventilated areas with oxygen >19% and only for a single work shift. The device is less effective if it is altered or damaged or if hair and beard are present. If you feel sick, leave the working area and move to an aerated area. Do not use in highly contaminated or potentially explosive

areas.

## REFI LK-Z006 (20735)

Brand: XIQUE

Anhui Lekang Sanitary Materials Co ., Ltd- Qingcao Industrial Park, Tongcheng, Anqing City, Anhui Province

Made in China

Imported by:

Gima S.p.A. Via Marconi, 1 – 20060 Gessate (MI) Italy [gima@gimaitaly.com](mailto:gima@gimaitaly.com) – [export@gimaitaly.com](mailto:export@gimaitaly.com)  
[www.gimaitaly.com](http://www.gimaitaly.com)

## Documents / Resources

	<a href="#">GIMA FFP2 NR Particle Filtering Mask</a> [pdf] Instructions FFP2 NR, Particle Filtering Mask
---	---

Manuals+.